

audioscan[®]

Axiom[®] User's Guide Version 1.6

© August 2013

Table of Contents


1	About Axiom.....	5
	New in this software release.....	5
	Product description.....	5
	Associated items and supplies.....	5
	SAFETY WARNINGS and NOTICES.....	6
	Environmental safety.....	6
	Declaration of Electromagnetic Compatibility (EMC).....	6
	Warranty, Trademarks, Acknowledgments.....	7
	EC Declaration of Conformity.....	9
	Electronic User's Guide.....	10
	How to Avoid Undesirable Side Effects.....	10
2	Getting Started.....	12
	Unpacking and connecting.....	12
	General care instructions.....	14
	Microphone care.....	15
3	General Operation.....	16
	Switching ON/OFF.....	16
	Input device operation.....	16
	Barcode data input.....	17
	Screen messages and Help.....	18
	Software updating.....	18
4	General Setup.....	19
	Date and time setup.....	19
	Display settings.....	19
	Saving test setup.....	19
5	Networking.....	21
	Networking requirements.....	21
	Networking setup.....	21
	Remote Operation.....	23
	Changing the Remote Operation port.....	24
	NOAH Service Port.....	24
	Changing the NOAH service port.....	25
	Testing the NOAH service port.....	25
6	Single computer connection.....	27
	Automatic connection (recommended).....	27
	Static connection.....	27
7	Printing and Storing Results.....	29
	Barcodes, tabular data, headers and comments on printouts.....	29
	Local printer setup.....	30
	Printing to USB memory.....	30
	Network printer setup.....	31
	Printing to a network file.....	31
	Storing and restoring data – setup.....	32
	Storing and restoring data.....	33
	Data storage using NOAH.....	34
8	Test Box Measures - Setup.....	35
	Test box screen.....	35
	Format.....	35
	Scale.....	36
	Hide or Show test box curves.....	36
	1996 or 2003 ANSI standard.....	36
	ANSI test frequencies.....	36
	Test box calibration facts.....	37
	Calibrating test box reference microphone.....	38
	Calibration check for coupler microphone.....	38
	Coupling the hearing instrument.....	39

Positioning the hearing instrument.....	40
9 ANSI Hearing Aid Tests.....	41
ANSI S3.22-1996 facts.....	41
ANSI S3.22-2003 facts.....	41
ANSI 1996 Linear and AGC tests.....	42
ANSI 2003 Linear and AGC tests.....	42
ANSI test results.....	43
10 Other Test Box Measures.....	44
Multicurve procedure.....	44
Multicurve results.....	44
Spectral analysis in Multicurve.....	45
Manual control procedure.....	45
Sound level meter using manual control.....	46
11 Sensory loss simulator.....	48
Sensory loss simulator description.....	48
Sensory loss simulator operation.....	48
12 On-Ear Measures - Setup.....	50
On-ear calibration facts.....	50
Calibration of on-ear probe microphone.....	50
Calibration check for probe module.....	51
Max TM SPL setup.....	52
Positioning the client.....	53
Positioning the probe tube.....	53
13 On-Ear Measures Screen Setup.....	55
SPL or HL scale.....	55
Hide or show on-ear curves.....	55
14 On-Ear Instrument Measures.....	57
On-ear manual control.....	57
Sound level meter using on-ear microphones.....	57
15 RECD measurement.....	58
RECD facts.....	58
RECD coupler response.....	58
RECD on-ear response.....	59
RECD results.....	61
16 Insertion Gain.....	62
Insertion gain in SPL.....	62
Insertion gain in HL.....	63
Audiometric data entry.....	64
REUR measurement procedure.....	64
REAR measurement procedure.....	65
SII calculation in Insertion gain.....	66
CROS fitting using Insertion gain.....	66
17 Speechmap.....	68
Speechmap facts.....	68
DSL 5.0 in Speechmap.....	68
DSL 5.0 changes.....	69
NAL-NL1 in Speechmap.....	69
NAL-NL2 in Speechmap.....	69
Camfit in Speechmap.....	70
Using Speechmap.....	70
Speechmap Setup.....	71
Screen tour - unaided screen.....	73

Screen tour - aided screen.....	74
On-ear or Test box mode.....	74
SII calculation in Speechmap.....	75
Using Custom Stimuli in Speechmap.....	75
Creating WAV Files for Speechmap.....	77
18 Speechmap Fitting Procedures.....	79
Speechmap screen choices.....	79
Data entry.....	80
Fitting to targets for soft speech.....	80
Fitting to targets for average speech.....	81
Adjusting the Maximum Output Level.....	83
Open fittings in Speechmap.....	84
Verifying Frequency Compression/ Frequency-Lowering Hearing Instruments in Speechmap.....	84
FM fitting and verification.....	86
19 Speechmap Technical Details.....	88
Speechmap stimuli.....	88
Stimulus levels.....	89
Microphone location effects.....	89
Speech signal analysis.....	90
20 Troubleshooting.....	91
Self test failures.....	91
Initialize Function.....	91
Test box high distortion or noise.....	92
Test box curves inconsistent.....	92
Test box curves differ from specifications.....	92
Test box speaker overdriven.....	93
No test box reference mic. detected.....	93
Invalid test box calibration.....	93
No on-ear ref. mic. detected.....	93
Invalid on-ear calibration.....	94
Sound-field speaker overdriven.....	94
Barcode scanner malfunction.....	94
21 Technical Specifications.....	97
General.....	97
Test box.....	97
On-Ear.....	97
Sensory Loss Simulator.....	98
22 Glossary.....	100
23 References.....	105
24 Appendix 1.....	108
Manufacturer Disclosure Statement for Medical Device Security.....	108

1 About Axiom

This section describes the Audioscan Axiom, provides contact, warranty and trademark information, safety warnings and notices and instructions for accessing the electronic User's Guide.

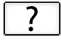Note that the User's Guide may be viewed on the Axiom at any time by clicking . (For long Help pages, use the mouse to switch between the Help index and the Help page and to scroll through the page).

The electronic User's Guide also contains a glossary and additional reference materials and is located on the updaters USB flash drive included with the Axiom.

New in this software release

Software 1.6 introduces the following significant additions and changes:

1. Remote Operation. For details, see 5. Remote Operation.
2. Wireless Networking. For details see 5. Networking.

Product description

The Axiom is a hearing aid analyzer intended to be used by hearing care professionals such as audiologists and hearing aid specialists to verify the electro-acoustic performance of a hearing aid connected to a standard earphone coupler or while worn on the ear of the end user. It consists of:

- a) an integrated acoustically-treated test chamber which houses a loudspeaker, a reference microphone for controlling the signal from the loudspeaker, and a standardized earphone coupler connected to a measuring microphone for the purpose of measuring the sound level produced in the standard coupler by a hearing aid.
- b) signal generation, measurement and control electronics.
- c) two real-ear measurement microphone assemblies, each housing a reference microphone for controlling the signal from the sound-field speakers and a probe microphone connected to a thin silicone tube which may be inserted into the ear canal for the purpose of measuring the sound level in the ear canal produced by a hearing aid.
- d) a miniature earphone used to measure the real-ear to coupler difference (RECD) useful in estimating the sound level produced in an individual ear from measurements in the standard earphone coupler.

There are 4 connectors for USB devices (flash drives, a QWERTY keyboard, mouse, printer, barcode scanner), a LAN port, and connectors for the external power supply (supplied), external loudspeaker (supplied) and 2 connectors for external HDMI video monitors (not supplied).

Electrical supply input requirement: 100 – 240 Vac 47 – 63 Hz 1.35A

A hospital grade grounded outlet is required.

Associated items and supplies

VA-120 Barcode scanner for reading barcoded audiometric data on printouts from Audioscan analyzers

VA-201 NOAH® module allows a networked PC running NOAH to exchange data with Audioscan analyzers








RE367-36 Probe microphone tubes for single patient use (36 per bag)

SAFETY WARNINGS and NOTICES


For purposes of IEC 60601-1, this product is Class I with Type BF applied part.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

This Class A digital apparatus complies with Canadian ICES-003

	WARNING: To avoid the risk of electric shock, this product must be used only with the power supply supplied with the product.
	WARNING: To avoid the risk of electric shock, this product must be connected only to a supply mains with protective earth (ground).
	WARNING: To avoid the risk of electrical shock, any line-powered peripheral equipment connected to this product must comply with UL/IEC 60601-1 OR comply with UL or IEC and ISO safety standards for such equipment AND a) be operated from an isolating transformer complying with UL/IEC 60601-1 OR b) be kept at least 6 feet (1.8 m) from the patient.
	WARNING: This equipment is not suitable for use in an oxygen-rich environment or in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
	WARNING: No modification of this equipment is permitted
	This symbol on the product is a WARNING that failure to follow instructions in this part of the User's and/or Quick Start Guides could place the operator or patient at risk.
	This symbol on the product means that the parts applied to the patient meet the safety requirements of IEC 60601-1 for type BF isolated (floating) applied parts

Environmental safety

	This symbol on the product means that this product is not to be disposed of in unsorted municipal waste because electrical and electronic waste may contain hazardous substances which could endanger the environment and human health.
This product and its associated items must be disposed of in accordance with local disposal regulations for electrical and electronic waste. Consult your local waste disposal authority regarding applicable regulations.	
The microphone probe tubes and the foam eartips used with the RECD transducer are for single patient use. After use, they may be disposed of in unsorted municipal waste or as required by your facility's waste management policy.	

Declaration of Electromagnetic Compatibility (EMC)

Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the following information:

- The Axiom should not be used adjacent to or stacked on other equipment. If this is necessary, its operation should be verified as normal in this configuration.
- Portable and mobile RF communications equipment can affect medical electrical equipment and may affect the performance of the Axiom.
- Performance degradation due to electromagnetic disturbances (including electrostatic discharge) is considered normal and acceptable

The compliances listed in the following table are met with the supplied RECD transducer, microphones, speaker and mouse connected, with 2 LCD monitors connected and with a, terminated Ethernet cable and unterminated USB cables (3) connected. The connection of other devices may result in increased emissions.

<i>Guidance and manufacturer's declaration - electromagnetic emissions</i>		
The Axiom is intended for use in the electromagnetic environment specified below. The user of the Axiom should assure that it is used in such an environment.		
<i>Emissions test</i>	<i>Compliance</i>	<i>Electromagnetic environment - guidance</i>
RF emissions CISPR 11	Group 1	The Axiom uses RF energy only for internal function. Therefore RF emissions are very low and not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The Axiom is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warranty, Trademarks, Acknowledgments

The Audioscan Axiom is manufactured by Etymonic Design Inc., 20 Ludwig St., Dorchester, Ontario, Canada, N0L 1G4. Web site www.audioscan.com.

Phone: 800-265-2093 (USA only); 519-268-3313 **Fax:** 519-268-3256

Email: info@audioscan.com or service@audioscan.com

The authorized representative for this product in the European Community is:

P.C. Werth, Audiology House, 45 Nightingale Lane, London, UK, SW12 8SP.

Phone: +44 (0) 181-675 5151 Fax +44 (0) 181-675 7577.

Warranty: The Axiom is warranted against defects for two years from date of purchase. Within this period, it will be repaired without charge for parts, labor or return shipping when returned prepaid to your authorized Audioscan service agent. This warranty does not apply to equipment that, in our sole judgment, has been subject to misuse, or unauthorized alteration or repair.

Trademarks:

Audioscan, Axiom, Speechmap, Verifit and Viewport, are registered trademarks of Etymonic Design Inc. DSL is a registered trademark of the University of Western Ontario. All rights reserved. HP LASERJET is a

registered trademark of Hewlett-Packard Company. NOAH is a registered trademark of the Hearing Instrument Manufacturer's Software Association. QUEST is a trademark of Quest Technologies Inc. PostScript is a registered trademark of Adobe Systems, Inc.

Acknowledgments:

DSL 5.0 is used under license from the University of Western Ontario (UWO) which is solely responsible for its content. We acknowledge the support received from past and present staff at the National Centre for Audiology at UWO in implementing the DSL method.

CAMFIT is used under license from Prof. Brian C.J. Moore, University of Cambridge, UK. We are indebted to the University of Memphis Hearing Aid Research Laboratory for permission to use some of their recorded speech material.

NAL-NL1 is used under license from the National Acoustics Laboratories, Australia.

NAL-NL2 is used under license from Hearworks Pty Ltd, Australia.

EC Declaration of Conformity

audioSCAN

A Division of Etymonic Design Incorporated

EC Declaration of Conformity

Manufacturer: Etymonic Design Incorporated
(operating as Audioscan)
20 Ludwig Street
Ontario, N0L 1G4
CANADA

European Authorized Representative: PC Werth Ltd.
Audiology House
45 Nightingale Lane
London SW 12 8SP
UK

CE-marked Products: Model **Verifit VF-1**, Real-Ear Hearing Aid Analyzer
(Commencing with Serial number 3879)

Model **RM500SL**, Real-Ear Hearing Aid Analyzer
(Commencing with Serial Number A1715)

Model **Axiom**, Real-Ear Hearing Aid Analyzer
(Commencing with Serial Number B1000)

Products Classification: Class IIa, after MDD Annex IX, Rule 5

Certification: EC Certificate CE 543241 issued by the British Standards Institution for compliance with Annex V, section 3.2 of European Union Council Directive 93/42 EEC, as amended by Directive 2007/47/EC, concerning medical devices.

Declaration: We hereby declare that the above mentioned devices comply with the legislation regulating medical devices in the United Kingdom transposing European Medical Devices Directive 93/42/EEC, namely Statutory Instruments 2002 No. 618 (Consolidated legislation), and the Medical Devices (Amendment) Regulations 2008 No. 2936 which transpose Directive 2007/47/EC into UK law

Date of Validity: February 2012

R.C. Boothroyd
General Manager,
Quality System Management Representative




CE
0086

audioSCAN®

20 Ludwig Street, Dorchester, Ontario, N0L 1G4 • www.audioscan.com • Tel: (519)268-3313 / Fax: (519)268-3256 / USA 1-800-265-2093

Electronic User's Guide

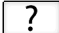You can download the current User's Guide directly from www.audioscan.com. A printable User's Guide is on the updater USB flash drive supplied with a new instrument. This User's Guide will be updated each time you download new software from www.audioscan.com to the updater USB flash drive. Except for some additional reference material, this same information is available to you at any time by clicking  on the Axiom.

A .pdf file viewer, such as Acrobat Reader (5.0 or higher) or Foxit Reader is required to view the User's Guide. To view the User's Guide:

1. Insert the updater USB flash drive into a USB port on your PC.
2. If your PC does not open the flash drive automatically, select My Computer, then the Removable Disk drive (usually E or F).
3. Double click the User_Guide folder to open it.
4. Double click the *english* folder and copy the Axiom Users Guide.pdf file to an appropriate location on your PC. Double click on the file to open it for viewing.

When you have finished copying the file from the USB flash drive, click on the *safely remove* icon on your PC and remove the flash drive when you are notified that it is safe to do so.

STORE THE UPDATER USB FLASH DRIVE IN A SAFE PLACE. YOU WILL REQUIRE IT TO INSTALL FUTURE SOFTWARE UPDATES.

How to Avoid Undesirable Side Effects

During the development of the hearing aid analyzer, Audioscan performed a rigorous Risk Assessment to identify any undesirable side effects that a user could be exposed to during the use of the Axiom, and incorporated numerous risk reduction design elements into the Axiom to minimize the risk to users and patients.

Following are the actions which a user should take to ensure that these risk control measures continue to be effective

Loud Sounds:

The Axiom is designed to produce sound pressure levels as high as 85 dB at the probe reference microphone. Exposure to these levels for more than 7 hours can produce hearing damage. When such levels are amplified by a hearing aid, the level in the ear canal will be determined by the settings of the hearing aid but may reach levels that can produce hearing damage in less than 30 seconds. To avoid this possibility,

1. hearing aids should be adjusted to limit sound pressure levels to safe levels
2. the maximum TM SPL setting (see Max TM SPL Setup) should be used to terminate tests if an unsafe level is detected in the ear canal
3. test levels should be limited to 70 dB SPL except when necessary to verify the limiting levels of the hearing aid, in which case, the test should not last longer than 15 seconds
4. be aware of the test signal and patient reaction during a test and be prepared to respond to any sign of discomfort by reducing the SPL setting, switching off the equipment or the hearing aid, or removing the patient from the area.

When using the equipment to measure the Real-Ear to Coupler Difference in small ear canals, it is possible to induce a hearing loss if the test is allowed to continue for more than 1 hour. Since accurate results can be

obtained in less than 10 seconds, this possibility should never occur in normal practice.

Power and Grounding:

This product contains numerous safety features to ensure that the probability of electrical shock is as low as reasonably practicable. In order to ensure that all of the safety features work optimally you must ensure that the power cord is plugged into a grounded outlet. Any line-powered peripherals connected to the Axiom must comply with UL/IEC 60601-1 OR comply with UL or IEC or ISO safety standards for such equipment, AND a) be operated from an isolating transformer complying with UL/IEC 60601-1, OR b) be kept at least 6 feet (1.8m) from the patient.

Ear Infection:

Probe tubes or RECD foam tips should not be re-used on another patient. There is a possibility of transferring an ear infection to the other patient. Probe tubes and RECD foam tips are for single-patient use only. Do not attempt to clean or re-use.

Ear Canal Discomfort:


An otoscopic examination should always be performed prior to inserting a probe tube into the ear canal to ensure that it is healthy and free of obstructions. Care is needed when inserting probe tubes into the ear canal. Although the probe tubes are made of soft, flexible material specially designed for this application, it is possible to scrape the ear canal or touch the eardrum causing brief discomfort. You should carefully follow the instructions in "Positioning the probe tube" section of this User's Guide.

2 Getting Started


This section provides instructions for unpacking the Axiom and connecting various components and associated items.

Unpacking and connecting

1. Carefully unpack the components of the Axiom from the shipping box and check the contents against the enclosed packing list.
2. Locate the Axiom on a solid surface in a quiet room and choose a location for the speaker and the client such that they will be about 2' (0.6 m) apart and both at least 4' (1.2 m) from any reflective surface. If the speaker is to be mounted on a desk, it should be at the edge facing outward. A room with sound-absorbing floor, ceiling and walls is recommended.
3. Turn the Axiom over and connect the on-ear probe microphones and the RECD transducer as shown.
4. Dress the leads out the sides, front or rear of the Axiom as appropriate for your work setup and turn the Axiom over, placing it on its rubber feet in the place where you will use it.

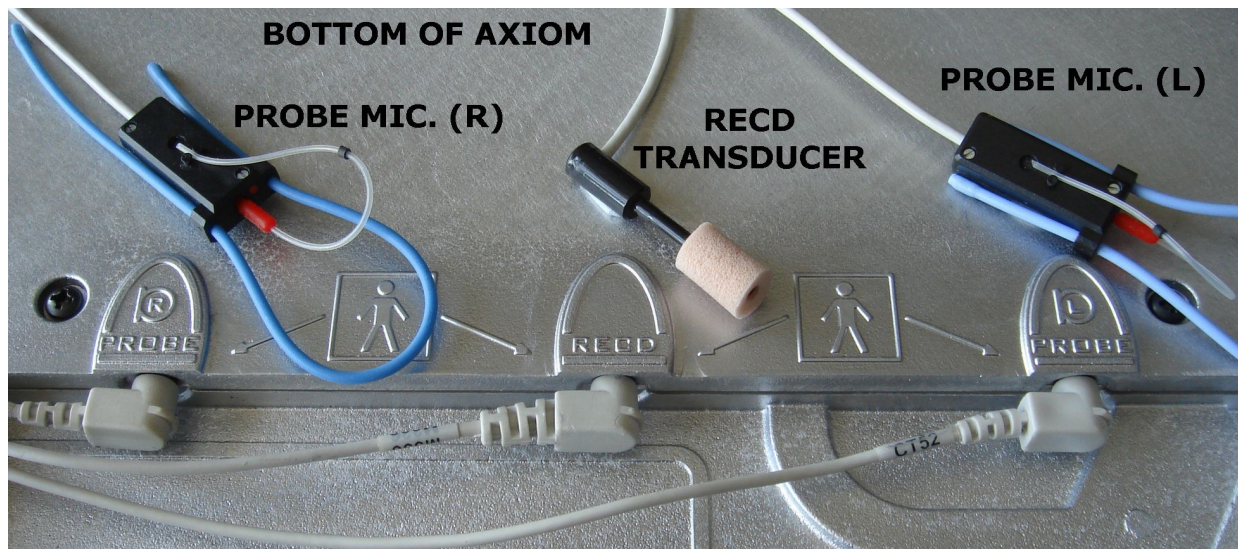

5. Open the lid of the test chamber by using your thumb to roll the rubber roller toward the front of the Axiom. This is made easier by placing your fingers under the front lip. When sealing the chamber in its closed position, place your fingers on top of the lid and use your thumb to roll the rubber roller toward the rear of the Axiom.

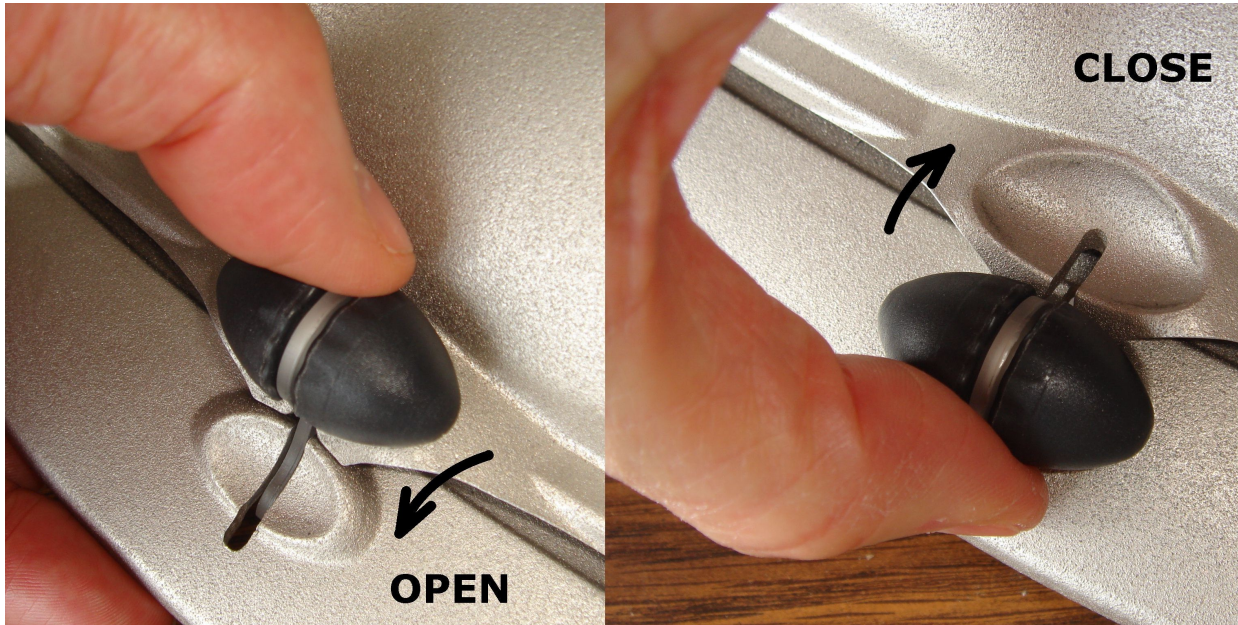

6. Plug the reference microphone and coupler microphone in to their respective connectors in the test chamber.



7. Connect the speaker to the speaker connector on the rear connector panel.
8. Connect a video monitor (HDMI or DVI with adapter) to the PRIMARY MONITOR connector on the rear

patient, but must not be used with multiple patients. Attempts at ultrasonic cleaning usually result in cerumen becoming lodged in the lateral end which causes irreparable damage to the probe microphones.

The case can normally be cleaned/dusted with a damp cloth, however if it is soiled we recommend wiping with a mild solution of water and detergent, or with alcohol-impregnated wipes deemed safe for use on electronic equipment. Parts that come into contact with patients (the probe module body, cable, and blue lanyard) should be regularly wiped down with alcohol wipes.

Hearing instruments should be cleaned prior to introduction to the test chamber due to the difficulty of sanitizing the acoustic foam. Custom hearing instruments must be cleaned with disinfectant towelettes (i.e. audiowipes) prior to placing in the test for this reason and to minimize contamination of the blue putty used with the HA-1 coupler. The acoustic foam in the floor of the test chamber is easily removed and should be replaced if badly worn or soiled.

Audioscan recommends periodic replacement of the blue putty used with the HA-1 coupler to ensure cleanliness. Both couplers and coupler mic can be wiped down with alcohol wipes as needed.

Ensure that all safety and usage recommendations on cleaning product packages are followed.

Microphone care

Coupler microphone:

1. **DO NOT** twist the cable when attaching a coupler to the coupler microphone. Turn only the coupler or unplug the coupler microphone before turning it.
2. **DO NOT** leave the coupler microphone plugged in when transporting.
3. **DO** make sure that the 2cc coupler is tightly screwed to the coupler microphone when performing hearing instrument tests. Coupler leakage can cause feedback and erratic response curves.
4. **DO** make sure the #13 tubing on the BTE (HA-2) coupler is free from any cracks/tears.
5. **DO** make sure that any replacement #13 tubing used on the BTE coupler is .38 inches in length.

Probe microphone:

Debris can alter the probe module reference microphone calibration and frequency response and can permanently clog the probe module port.


1. **DO NOT** reuse probe tubes. Probe tubes can be wiped with alcohol wipes for re-use with the same patient, but must not be used with multiple patients. Attempts at ultrasonic cleaning usually result in cerumen becoming lodged in the lateral end which causes irreparable damage to the probe microphones.
2. **DO NOT** attempt to open or repair the probe microphone. Attempting to repair the probe module may lead to damage or alteration of the factory calibration.
3. **DO** keep the probe module and probe tubes in a clean area.
4. **DO** stow the probe module when not in use or when transporting the Axiom.

3 General Operation


This section describes the use of a mouse or QWERTY keyboard to control the Axiom and to enter data, and the use of the (optional) barcode scanner to enter data from printouts produced by Audioscan analyzers. It also provides instructions for updating the Axiom operating software.

Switching ON/OFF

See Getting Started for instructions on connecting the Axiom to its external power supply module, the module to an electrical outlet and the location of the standby switch.


To start the Axiom, set the standby switch  to its "ON" position. The green light on the right rear corner will light and the instrument will begin its start-up sequence which will take a few seconds.

To shut down the Axiom, switch the standby switch  to its "OFF" position.








Note that the Axiom receives low voltage electrical power from the external power supply module. The standby switch  on the rear of the Axiom disconnects this electrical supply but the power supply module is always on as long as it is connected to a live electrical outlet. The green light on the power supply module indicates that it is receiving electrical power from the outlet.



Input device operation

The Axiom is operated by using a USB mouse (included) or a QWERTY keyboard (not included) to select icons and menu items, operate on-screen buttons and to enter audiometric data.

To activate an on-screen icon or button, or to select an item from a menu, use the mouse to place the screen pointer over the icon, button or menu item and left click. To enter a data point on a chart, such as an audiogram, place the screen pointer over the desired point on the chart and left click. To move the data point, place the screen pointer near the new location and left click. To delete a data point, place the screen pointer on the data point and left click. To summon a cursor on a graph, place the screen pointer on the graph and left click, then navigate to the point you want to identify; left click again to hide the cursor. The scroll wheel may be used to traverse the Help contents, multiple Help pages or long menu lists. *Note that mouse speed can be changed by clicking  and [Display].*

The functions of the screen icons and QWERTY keyboard keys are explained in the following table:


Screen icon	Function	QWERTY Key
	Continue. Proceed from current state	
	Cancel. Revert to previous state	[Esc]
	Show menu of available tests	[F3]
	Switch between Left/Right on-ear data or switch between A/B test box data	[F5]
	Show on-line help	[F1]
	Show the setup menu	[F2]

	Print screen to an external printer or a file	[Print Screen]
	Show menu for erasing data or exchanging data with network files or USB memory	[F4]
	Position the screen pointer over the icon, button or menu item	[Arrow keys]
	Activate a screen icon or button, or select an item from a menu (mouse left click)	[Enter] on numeric keypad

Barcode data input

In Speechmap and Insertion gain tests, audiometric data in barcode form on an Audioscan analyzer printout may be entered by scanning the appropriate barcode. In Speechmap, air and bone threshold, UCL, RECD, audiometric transducer and age are encoded. In Insertion gain, threshold and audiometric transducer type are encoded. The type of data and the ear (left, right) is shown below the barcode. Only data for the displayed ear are imported. Barcodes may be scanned in any order and it does not matter if the barcode is 'upside down'.

1. Select Speechmap or Insertion Gain from the Tests menu.

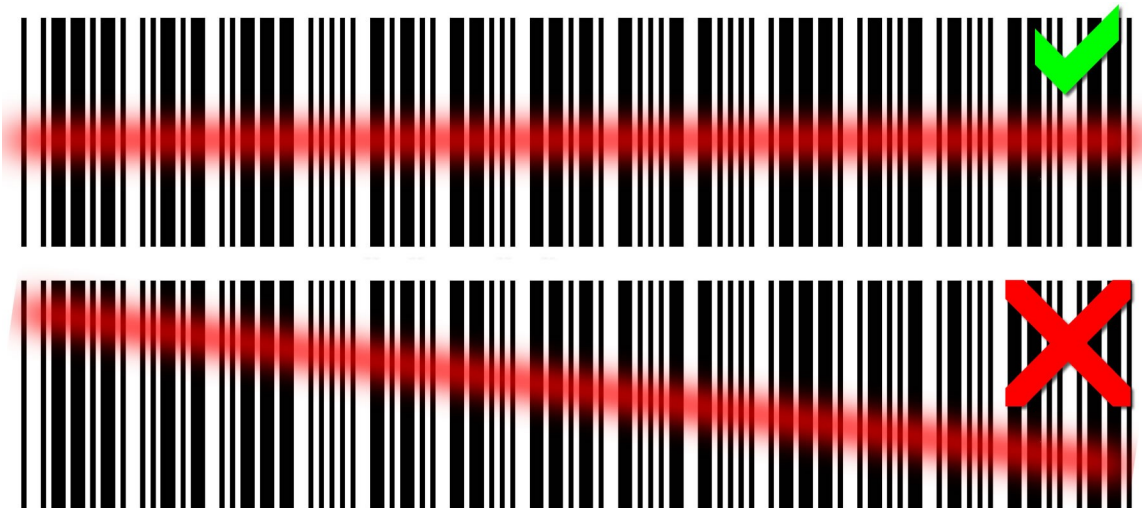


2. Hold the printout so that the barcode is flat. With the scanner 6 – 8 in. (15 – 20 cm) from the barcode, press the trigger on the scanner and center the illuminated red line along the length of the barcode.
3. When the scan is successful, the scanner will 'beep', the red line will extinguish and a Barcode Entry poster will appear on the screen. A green checkmark on the poster shows which data have been accepted. A message will advise if the barcode does not contain data for the screen you are viewing.
4. When all desired data have been accepted, select [Done] to apply the data.

See Barcodes, tabular data, headers and comments on printouts under [Printing and Storing Results](#).

Screen messages and Help

The Title bar (top line of the display screen) informs you of the selected Test (e.g., Speechmap).

The Message bar (bottom line of the display screen) suggests the next step in a test or informs you of the state of the instrument or conditions that could affect your data. For example, it will inform you if the microphones need to be calibrated.

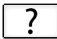Context-sensitive help is available by clicking on  or pressing F1 on a connected QWERTY keyboard. The Help index (left panel) or Help page (right panel) may be selected by left clicking the mouse on either panel or by pressing the F5 key on a QWERTY keyboard. The side sliders on each panel may be used to scroll through the Help index or long Help pages. This may also be accomplished using scroll wheel on the mouse or the arrows on a QWERTY keyboard.

Software updating



The software currently running on your Axiom is stored in internal memory. A USB flash drive was shipped with your Axiom. It will be required to transfer future software updates from a PC to the Axiom. **STORE IT IN A SAFE PLACE.** You may download the latest software from www.audioscan.com or request that a CD-ROM be mailed to you. Instructions for transferring the software update from your PC to the Axiom will be included with the software download or the CD-ROM.

4 General Setup

This section covers Date and Time setup, Display settings (including mouse pointer speed setting) and Saving test setup.



Date and time setup

To set the date and time that appears on printouts and calibration screens:


1. Left click on 
2. Left click on [Date & Time] on the Setup menu
3. To change the date, left click on the month, date or year windows and make your selection from the drop-down lists by left clicking on an item.
4. To change the time, left click on the hour or minute window and make your selection from the drop-down lists by left clicking on an item. Left click on am/pm window to toggle between am and pm.
5. To exit the Setup menu, click on 

Display settings

To change the screen language, screen saver timing and the background color for charts:


1. Left click on 
2. Left click on [Display] on the Setup menu
3. To change the language used on the screen, left click on the Language window and select from available languages (currently English only).
4. To change the idle time before the screen saver blanks the screen to prolong display life, left click on the Screen saver window and make your selection from the drop-down list by left clicking on a time.
5. To change the background color for charts, left click on the Color window to toggle between Black and White.
6. In this screen you can also click on the Mouse pointer speed button and select the response speed from the drop-down list.
7. To exit the Setup menu, left click on 

Saving test setup

This feature lets you retain the last-used stimulus type and level for most tests when the power is turned off and the last-used target method and transducer type in Speechmap and Insertion Gain tests. This feature also lets you determine how assessment parameters, test levels and stimuli are treated when you click on  and select [Erase data].

To enable/disable saving of test levels, stimuli and assessment parameters:

1. Left click on 

2. Left click on [Save option] on the Setup menu.
3. Left click on the Yes/No windows to enable/disable the save options indicated.
4. To exit the Setup menu, left click on 

5 Networking

Connection to a computer network allows screen images to be printed on a network printer or on a shared printer on a networked PC. Images and underlying data may also be saved to a shared folder on a networked PC. See [Printing and Storing Results](#) for details. The Axiom may also be connected to a single computer using a network router, switch or hub. Note that sharing data with NOAH is done over this network connection.



Failure to follow operating instructions could place the user or operator at risk.

Networking requirements

Connection of the Axiom to a wired computer network (LAN) requires a Cat 5, RJ-45 cable.

Connection of the Axiom to a wireless computer network (WLAN) requires a wireless router or other device that provides wireless service connected to the local area network. Connection speeds will be determined by the type of wireless connection provided and distance from the wireless router. The standard 802.11a, b, g, or n wireless networks can be connected to the Axiom.

The current supported forms of wireless encryption (network security) are as follows:


- WPA / WPA2 (using pass-phrase for authentication), most common
- WEP (10 or 26 digit hex key)
- None (open network)


The Axiom can connect to a local area network (LAN or WLAN) using either automatic or static internet address assignment. Most home and small-office networks use automatic addressing. These networks have a DHCP server (Dynamic Host Configuration Protocol). Special-purpose networks might use static address assignment provided by the network administrator.

Remote Operation of the product requires a valid network connection, whether it be wired or wireless the Axiom must have an active network connection .

Networking setup


Check [Networking requirements](#) and connect the Axiom to your network.

1. Click  and then choose [Network] from the Setup menu.
2. Turning networking off
 1. Select the “Off” option on the “Network” drop down.

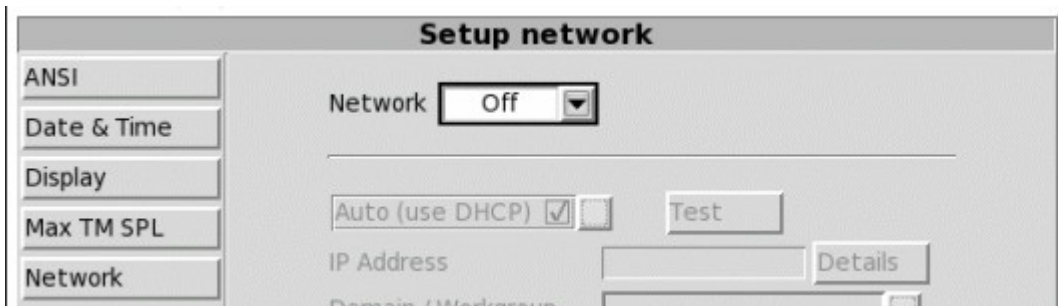

3. Connecting to a wired (LAN) network

1. Select the “Wired” option from the “Network” drop down.
2. In most cases, your network will support automatic address assignment (DHCP) and you need to only select [Auto] and click on [Test].
3. If the test is successful, your computer domain, address and port assignments will be reported.

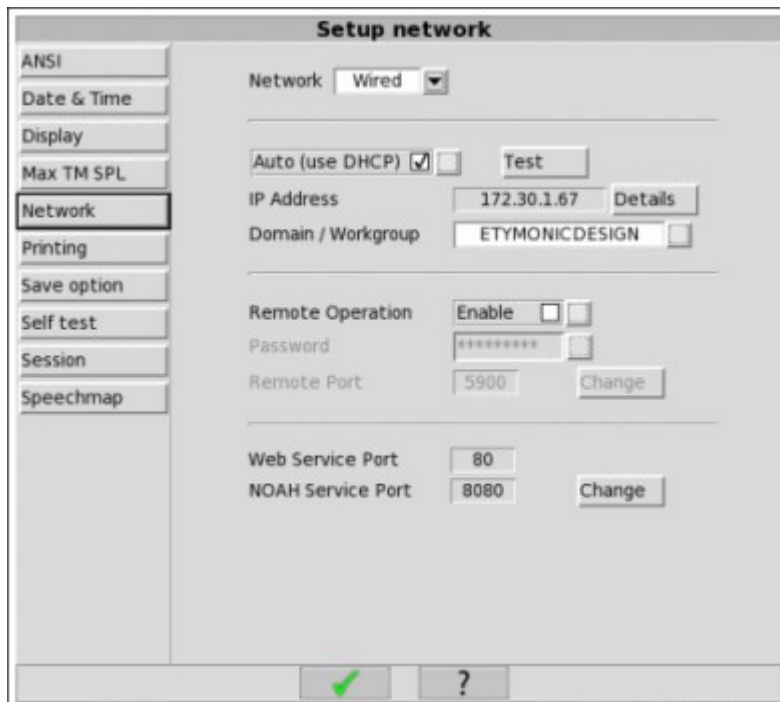


4. Connecting to a wireless (WLAN) network


1. Select the “Wireless” option from the “Network” drop down.
2. The Axiom will scan for wireless networks.
3. Once the scan is complete the “Name (SSID)” drop down will be populated. Select the appropriate wireless network, if your network is not visible you may type the name into this box as well.
4. Ensure the appropriate security type for your network is selected and input the key or pass phrase into the

corresponding text field. If you are unsure of the security type, please ask your network administrator.

5. In most cases, your wireless network will support automatic address assignment (DHCP) and you need to only select [Auto] and click on [Test].
6. If the test is successful, your computer domain, address and port assignments will be reported.

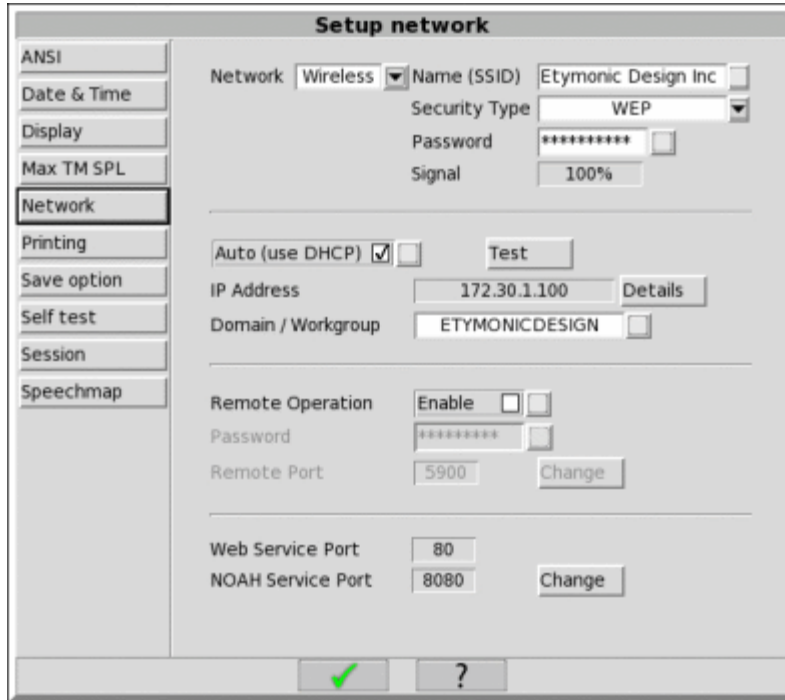


Note for network administrators: To manually change networking settings, uncheck Auto (DHCP) and [Change] button will be come active, click the [Change] button to set the network information statically.

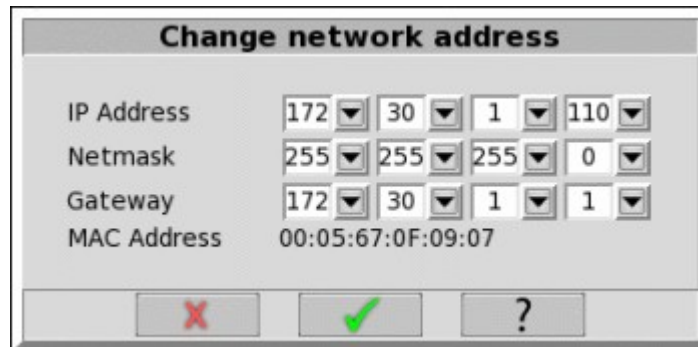


Remote Operation

Check [Networking requirements](#) and connect the Axiom to your network.

1. Click  and then choose [Network] from the Setup menu.

2. Select [Enable] to enable the Remote Operation network service
3. Input a password in the “Password” field
4. Click on the network setup screen to save the settings and start the remote operation server.

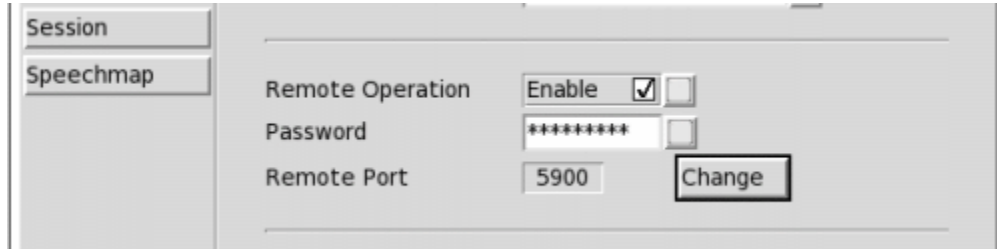


Note: “On top mode” and “Telehealth” applications can both be achieved by the “Remote Operation” feature.

Changing the Remote Operation port

Consult your Network Administrator before changing the Remote Operation port. Changing the port successfully requires understanding the security polices on your Windows PC and computer network.

The Audioscan Remote Operation application communicates with the Axiom through the Remote Operation network service. The Axiom provides the Remote Operation service on a TCP port. The Windows computer running the module must be configured to access the same TCP port, which is 5900 by default. Some Windows anti-virus software packages and some network firewalls prevent access to specific ports, including 5900. Using the Remote Operation application in the presence of such security settings may necessitate changing the Remote Operation service port.

1. Press <Change> next to the Remote Operation port.
2. Select the appropriate five digits to enter the desired port number. The software will prevent you from entering the same port as the Product web service (typically port 80).

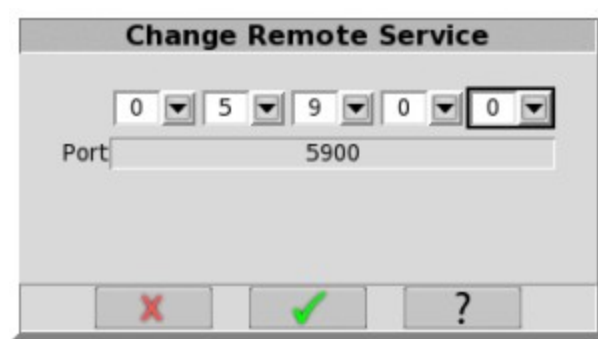

NOAH Service Port


Consult your Network Administrator before changing the NOAH service port. Changing the port successfully requires understanding the security polices on your Windows PC and computer network.

The Audioscan NOAH Module communicates with the Axiom through the NOAH network service. The Axiom provides the NOAH service on a TCP port. The Windows computer running the module must be configured to access the same TCP port, which is 8080 by default. Some Windows anti-virus software packages and some network firewalls prevent access to specific ports, including 8080. Using the NOAH module in the presence of such security settings may necessitate changing the NOAH service port.

Note that changing the NOAH service port requires restarting the Product software.

Changing the NOAH service port

1. Press <Change> next to the NOAH service port.
2. Select the appropriate five digits to enter the desired port number. The software will prevent you from entering the same port as the Product web service (typically port 80).

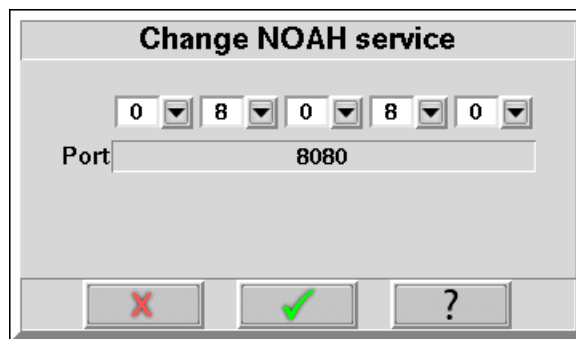

3. Click .
4. You will be prompted to restart the equipment. Click to restart, or to enter a different port number.



Testing the NOAH service port

The Axiom provides the NOAH service using the HTTP protocol. Communication between the Windows PC and the Axiom can be tested using an ordinary web browser.

1. On the Windows PC, enter the Axiom network address and port number into a web browser following this example:

`http://172.30.86:8080/noah`

2. If the communication is successful, the web browser will display NOAH data formatted in XML:

<noah> Example XML </noah>

6 Single computer connection

The Axiom can be networked to a single computer with two methods, depending on the network hardware available. The computer must be running Windows XP professional or higher (Vista, Windows 7) and have an available, functioning network port.

Automatic connection (recommended)

Automatic connection requires a small network router. It may be wireless or not, but must have at least two hardware Ethernet connections. Most routers have DHCP address assignment active by default and do not require installation of software on the PC to establish a simple connection with a Axiom.

1. Connect a Cat 5, RJ-45 cable from one router port to the Axiom network connector.
2. Connect a second cable from a router port to the network connector on the PC.
3. On the Axiom, press <Setup> and then choose [Network] from the Setup menu.
4. Select [Enable]. Ensure the Auto checkbox is checked, and choose [Test].

A network address will be provided by the router and should show up in the 'Network address' window. By default, Windows 7, Vista, and XP computers have networking and DHCP enabled so at this point your connection should be complete. To check your PC settings:

1. On Windows XP go to Control Panel / Network Connections. On Windows 7/Vista go to Control Panel / Network and Sharing Center then select 'Change Adapter Settings' on the left-hand panel.
2. Right-click on Local Area Connection and choose Properties. Ensure that Internet Protocol (TCP/IP or TCP/IPv4) is checked. Select this and select the Properties button. Ensure that 'Obtain an IP address automatically' is selected.

Static connection


A static connection requires an Ethernet hub or switch. The Network addresses must be manually entered into the Axiom and PC.

1. Connect a Cat 5, RJ-45 cable from one port on the switch or hub to the Axiom network connector.
2. Connect a second cable from a switch/hub port to the network connector on the PC.
3. On Windows XP go to Control Panel / Network Connections. On Windows 7/Vista go to Control Panel / Network and Sharing Center then select 'Change Adapter Settings' on the left-hand panel.
4. Right-click on Local Area Connection and choose Properties. Ensure that Internet Protocol (TCP/IP or TCP/IPv4) is checked. Select this and select the Properties button.
5. Check 'Use the following IP address' and enter the following in the IP address box: 172.30.1.1. Enter 255.255.255.0 in the Subnet mask box.
6. On the Axiom, uncheck Auto on the Setup network screen and click the Change button. Then set the network addresses as shown below by clicking on the down arrow buttons and selecting from the drop-down lists.
7. Click on the Test button to check that a connection has been established.

Change network address

IP Address	172	30	1	2
Network	172	30	1	0
Broadcast	172	30	1	255
Netmask	255	255	255	0



7 Printing and Storing Results

Clicking on  will cause the current screen image to be printed on a connected (USB) local printer, on a network printer, on a Windows-shared printer on a networked PC, or saved to file on a USB memory stick, or a Windows-shared file on a networked PC. Date, time, barcoded audiometric data and fixed or variable headers and comments (via a connected keyboard), may be added to printouts (but not when saving to a file). Tabular data may be appended to printouts of graphic data. Session data may also be saved as an xml file to a USB memory stick or in a Windows-shared folder on a networked PC for further analysis or to be restored to an Axiom or other Audioscan analyzer.

NOTE: If a header, comment, or client ID field, or file name, identifies the printed or saved data as belonging to an individual, it becomes Individually Identifiable Health Information and must be protected under the HIPAA Security Rule. See Appendix 1: Manufacturer Disclosure Statement for Medical Device Security.

Barcodes, tabular data, headers and comments on printouts

The date and a header, such as a facility name, may be added to printouts. Barcodes representing hearing threshold, UCL and RECD, where applicable, may be added to printouts of Speechmap and Insertion gain tests. Tabular data, where applicable, may be appended to printouts. Lines for hand-written notes or typed comments may also be added.


1. Connect a computer keyboard to one of the USB ports.
 2. Click  then click [Printing].
 3. Click [Page setup] to display the page setup poster.
 4. To print the *date and time* on each printout, click [Date].
 5. To print *lines* for handwritten notes, click [Lines].
 6. To add *barcodes* for audiometric data, click [Barcode].
 7. To add *tabular data*, click [Graph with table].
 8. To print a *fixed header* on printouts, click [Header]. Enter the desired header (up to 35 characters) on the connected keyboard. Click [Header] (or Enter on the keyboard number pad) to complete the entry. To be prompted for a *new header* for each printout, click [Ask].
 9. To print *fixed comments* following a printout, click [Description]. Enter comments using the connected keyboard. Click [Description] (or Enter on the keyboard number pad) to complete the entry. To be prompted for *new comments* for each printout, click [Ask].
 10. To be prompted for a *new file name* each time you print to a USB memory stick or a network file, click [Ask]. *Do not use space \$ \ / or tab in the file name.* USB file names are limited to 8 characters. See [Printing to a USB memory stick](#) or [Printing to a network file](#).
 11. A print layout for the selected printer is displayed.
- Click  to exit page setup and again to exit the Setup menu.


Local printer setup

1. Connect an external printer to one of the USB ports. *Only PCL 3 (not PCL3GUI), PCL 5, PostScript compatible, HP Laserjet Pro 1660, Deskjet D2680, D4160 and 460 printers are supported.*



Failure to follow operating instructions could place the user or operator at risk.

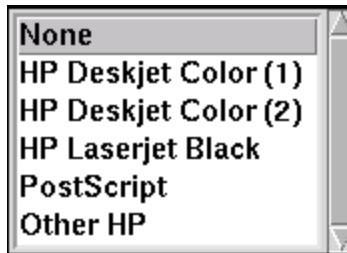

2. Click  then click [Printing].
3. Click [USB].
4. Click [Printer details] (automatic if printer type is set to None).
5. Click [Printer type]. Click your printer type. Select HP Deskjet Color(1) for a printer with a single ink



cartridge; HP Deskjet Color(2) for a printer with separate color and black ink cartridges; HP Laserjet Black. Check [Black only] for black printouts on a color printer. Select Other HP for a list of supported specific printer models.

6. Click . If Other HP was selected, this will summon the HP printer selection menu. Select a printer model from the drop-down list and click .
7. Click to exit the Setup menu.

Printing to USB memory


1. Plug a USB memory stick into one of the USB ports. The Axiom may be either on or off.
2. Click  then click [Printing].
3. Click [Print to USB file].
4. Click [Printer details] to change the file type [Bitmap (bmp)] or [Internet GIF (gif)] and the captured area [Full screen], [Main window] (full screen without the message line or [Active window] (foreground image only).
5. Click to save the settings and exit the setup menu.
6. To save a screen image to a file, click . *Do not unplug the USB stick until the 'Accessing USB stick' message disappears.*


NOTE: To input a file name from a connected keyboard each time you print, see [Barcodes, tabular data, headers and comments on printouts](#).

NOTE: To save all test data as an xml file, see [Storing and restoring data](#)


Network printer setup

To print to a network printer or to a Windows-shared printer on the network, you must first enable networking. See section [Networking](#). To print to a printer on a Windows PC connected to a network, you must also share the printer on the PC by selecting Start, Settings and Printers. Then right click on the printer you wish to share and select Sharing. *Only PCL 3 (not PCL3GUI), PCL 5, PostScript compatible, HP Laserjet Pro 1660, Deskjet D2680, D4160 and 460 printers are supported.* In Windows2003 Server you must add permission to allow Anonymous Logon to print to the shared printer.

1. Click  then click [Printing].
2. Click [Other network printer] or [Windows shared printer].
3. Click [Printer details] (automatic if printer type is set to None).
4. Click [Printer type], then select from HP Deskjet Color(1) (single ink cartridge), HP Deskjet Color(2) (both color and black ink cartridges), HP Laserjet Black, PostScript or Other HP (for a list of supported specific printer models). Check [Black only] for black printouts on a color printer.
5. Connect a computer keyboard to a USB port on the Axiom and fill in the boxes with the requested information. Click each box to open it for data entry and again to close it (or use the Enter key on the keyboard's number pad). Obtain needed information from your network administrator.

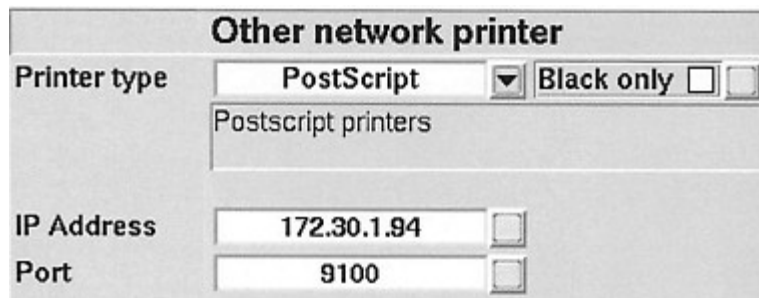



Other network printer	
Printer type	PostScript <input checked="" type="checkbox"/> Black only
	Postscript printers
IP Address	172.30.1.94
Port	9100

6. Click  to save the settings and exit the setup menu.

Printing to a network file


To save the screen image as a file on a network PC, you must first enable networking. See [Networking](#). You must also "share" a folder on the PC (e.g. c:\MyData) and ensure that Change permissions are enabled.

1. Click  then click [Printing].
2. Click [Print to file].
3. Click the file type [Bitmap (bmp)] or [Internet GIF (gif)].
4. Click [Full screen], [Main window] (full screen without the message line or [Active window] (foreground image only).
5. Connect a computer keyboard to a USB port on the Axiom and fill in [Computer], [Share name], [Username] and [Password] (not needed for Windows 95/98). Click each box to open it for data entry and again to close it (or use the Enter key on the keyboard's number pad).


6. Click [Test connection] and wait for a successful report.
7. Click to save the settings and exit the setup menu.
8. To save a screen image to a file, click . Do not unplug the USB stick until the 'Accessing USB stick' message disappears.

NOTE: To input a file name from a connected keyboard each time you print, see [Barcodes, tabular data, headers and comments on printouts](#).

NOTE: To save all test data as an xml file, see [Storing and restoring data](#)

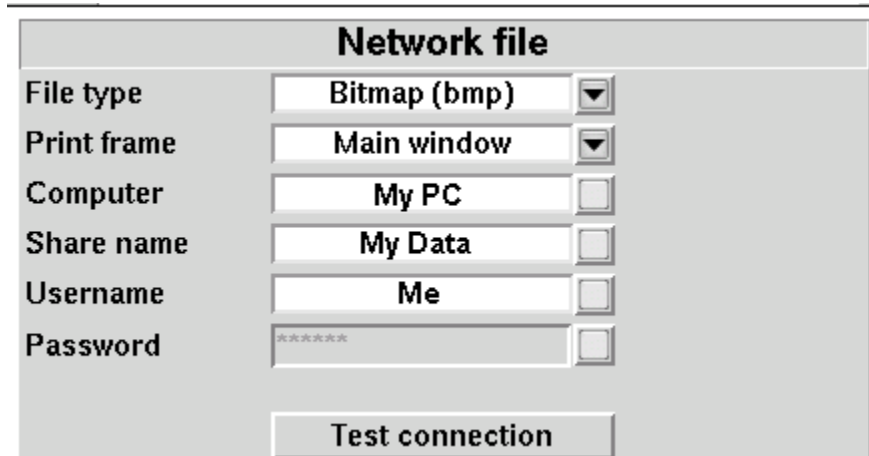


Storing and restoring data – setup

All measurement data (curves) displayed at the end of the test are stored until overwritten by repetition of the same test, invalidated by a related test (in such cases, the message bar will inform you of impending data loss), erased by clicking  and one of the [Erase data] choices or the power is turned off. (Setup data will be retained even after the power is turned off.)

The Axiom database may be stored and restored via xml files on a USB drive or a networked PC share. If the data exchange location is a network share, you must first enable networking. See [Networking](#). You must also "share" a folder on the PC (e.g. c:\MyData) and ensure that Change permissions are enabled.

To enable and configure file-based data exchange:

1. Click , then [Session].
2. Click [Enable file based data exchange].
3. Click the storage file location [USB drive or Network share].
4. For Network share connect a computer keyboard to a USB port on the Axiom and fill in [Computer], [Share name], [Username] and [Password] (not needed for Windows 95/98). Highlight and <PICK> each box to open it for data entry and again to close it (or use the Enter key on the keyboard's [number pad](#)). Click [Test connection] and wait for a successful report.
5. Click to save the settings and exit the setup menu.




Storing and restoring data

The Axiom database contains all the test and setup data for the Axiom at any point in time. The Session store features saves this database in xml format which may be viewed with a web browser or imported into an Excel spreadsheet or returned to the Axiom to recreate the session. The data is saved in a file which you can name using a connected computer keyboard. This name (the Client ID) may not contain ! ~ ` @ # \$ % ^ & * + = \ / ? < , .>. A number is automatically appended to the name you enter and this number is incremented each time you add a session to this Client ID (e.g. Andy.1.xml, Andy.2.xml, Andy.3.xml). If you do not enter a name, the file will be named session.#.xml (e.g. session.1.xml, session.2.xml, session.3.xml).

NOTE that Client IDs starting with numbers appear at the top of the list, followed by those that start with upper case letters while those starting with lower case letters will be at the bottom of the list. It is recommended that you use a consistent system when creating Client IDs (such as all upper case letters) to avoid duplicate entries.



If the Client ID links the data to an individual, the printout becomes ePHI (electronic Protected Health Information) which must be protected under the HIPAA Security Rule.

To store session data to a file:

1. Click , then click the top list button on the Session data poster.
2. Click [Store session to file] on the list.
3. Click the Client ID list button. This will display a list of Client IDs with the first entry highlighted.
4. To add a session to an existing file
 1. Use the mouse or the arrow keys on a connected keyboard to move the highlight in the list, **OR** start typing the file name on a connected keyboard. The highlight will move to the closest match as you type.
 2. When the desired file name is highlighted, left click on the Client ID button (you can always left click on any file in the list to select it without highlighting it). This will put the file name in the Client ID window and close the list.
 3. Click  to store the current session data to the file in the Client ID window with a new session number appended.
5. To add a new file to the Client ID list
 1. Display the Client ID list and start typing the new file name. As you type, the first matching entry in the list will be highlighted and this highlight will move as you type (the highlight will disappear when the file name you have typed does not duplicate an existing name).
 2. When you finish typing the Client ID, click on the Client ID button. This will close the drop-down list and leave the new file name in the Client ID window.
 3. Click  to store the current session data to the file in the Client ID window.

NOTE: A style sheet (audioscan.xsl) is automatically saved with the data file. To open the saved data file in Excel, right click on the file and select Open With.> Microsoft Excel. When prompted for a style sheet, select audioscan.xsl from the choices presented (keep the style sheet in the same folder as the data file).

To restore a session from a file:

1. Click , then click the top list button on the Session data poster.
2. Click [Restore session from file].
3. Click the Client ID list button. This will display a list of Client IDs with the first entry highlighted.
4. Use the mouse or the arrow keys on a connected keyboard to move the highlight in the list, OR start typing the file name on a connected keyboard. The highlight will move to the closest match as you type.
5. When the desired file name is highlighted, left click on the Client ID button (you can always left click on any file in the list to select it without highlighting it). This will put the file name in the Client ID window and close the list.
6. Click the Session list button to generate a list of previously stored sessions for the indicated Client ID.
7. Click a session.
8. Click  to load the session data.

Data storage using NOAH


Audioscan offers a module for storing data in the NOAH framework on a Windows PC. Contact Audioscan or visit www.audioscan.com for details.

8 Test Box Measures - Setup

For linear and AGC tests, two completely separate sets of test results, labeled A and B, may be retained and displayed. To switch between A and B test results, click on **A|B**. The appearance of these test box measures screens can also be changed to show results in either graphical or tabular format. To change the data format, click on the Format button.

Test box screen

For most tests, two completely separate sets of test results, labeled A and B, may be retained and displayed. To switch between A and B test results, press the <Left/Right> key.

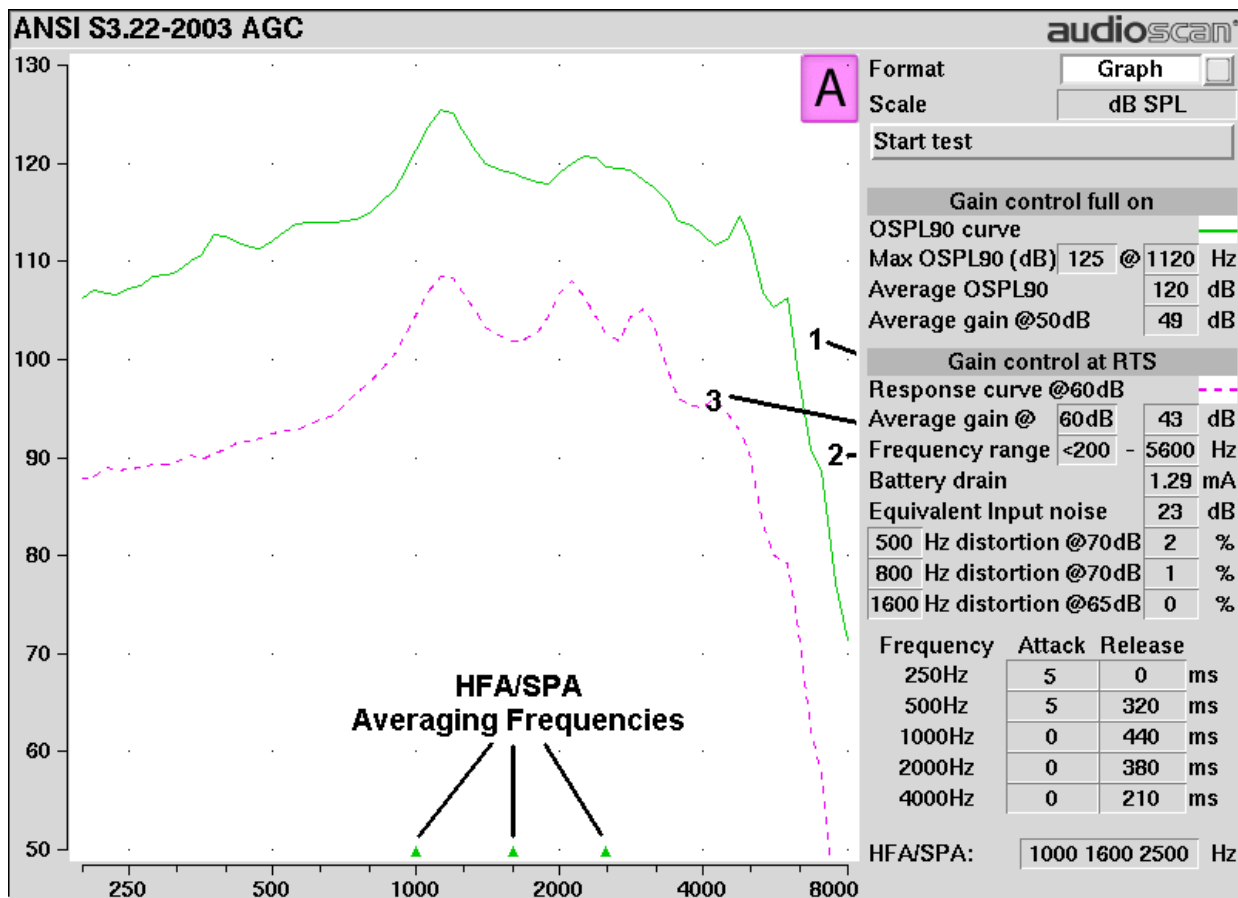

Format

Except for Speechmap and Manual control screens, you have the option of selecting either a graphic or tabular format.

To change the *data format*, highlight, then <PICK> [Graph] or [Table].

Scale

Test results are in SPL (dB) on all screens.


Hide or Show test box curves

To hide or show test curve(s):

On the Speechmap screen:

1. Click on the Hide/Show buttons located just below each of the Test identifier numbers as shown below. Note that the Hide/Show aided button has 3 states – Curves hidden, display LTASS only and display LTASS + speech envelope (i.e. the 'speech banana').





Hiding a curve does not erase it. To erase curves click on  and select one of the Erase options.

1996 or 2003 ANSI standard

Automated tests of linear and AGC hearing aids may be conducted according to either the ANSI S-3.22-1996 or ANSI S3.22-2003 test standards. For differences, see ANSI S3.22-1996 facts and ANSI S3.22-2003 facts


To select the standard to be used:

1. Click on  and then click on [ANSI].
2. Click on the version of the ANSI standard you wish to use.
3. Click on  to save your selections and exit the menu.

ANSI test frequencies

The Frequencies feature allows hearing instruments to be evaluated at the Special Purpose Average (SPA) frequencies defined by ANSI S3.22, rather than the default High Frequency Average (HFA) frequencies. The choice of SPA (or HFA) frequencies influences the average OSPL90, average full-on gain, reference test gain, bandwidth and equivalent input noise. The frequencies at which distortion is measured are one-half the selected SPA frequencies. You can also select the frequencies at which attack and release times are measured.

To change the averaging frequencies:

1. Click on  and then click on [ANSI]. From within the ANSI window, click on the 'HFA/SPA' pull-down menu

2. Highlight the desired HFA/SPA test frequencies and Click on it to select.

To change the frequencies for attack and release time measurements:


In the list of available frequencies, click on the frequencies to be used to measure attack and release time.

Click on to save your selections and exit the menu.

Test box calibration facts


Test box microphone calibration establishes a correction curve for the uncalibrated test box reference microphone by comparison with the factory calibrated coupler microphone. It may be performed as often as desired but will be invalidated at 12 am the following day or at 12 am the following Monday, depending on the choice of Daily or Weekly calibration interval. After the calibration has expired, the message window will turn blue and you will be prompted to calibrate whenever you attempt a test that uses the test box microphones.

Your calibration curve should be similar to the one shown but may be at a different level. It should remain stable over time. Irregularities in the calibration curve may be the result of not properly aligning the microphones. See [Calibrating test box reference microphone](#).

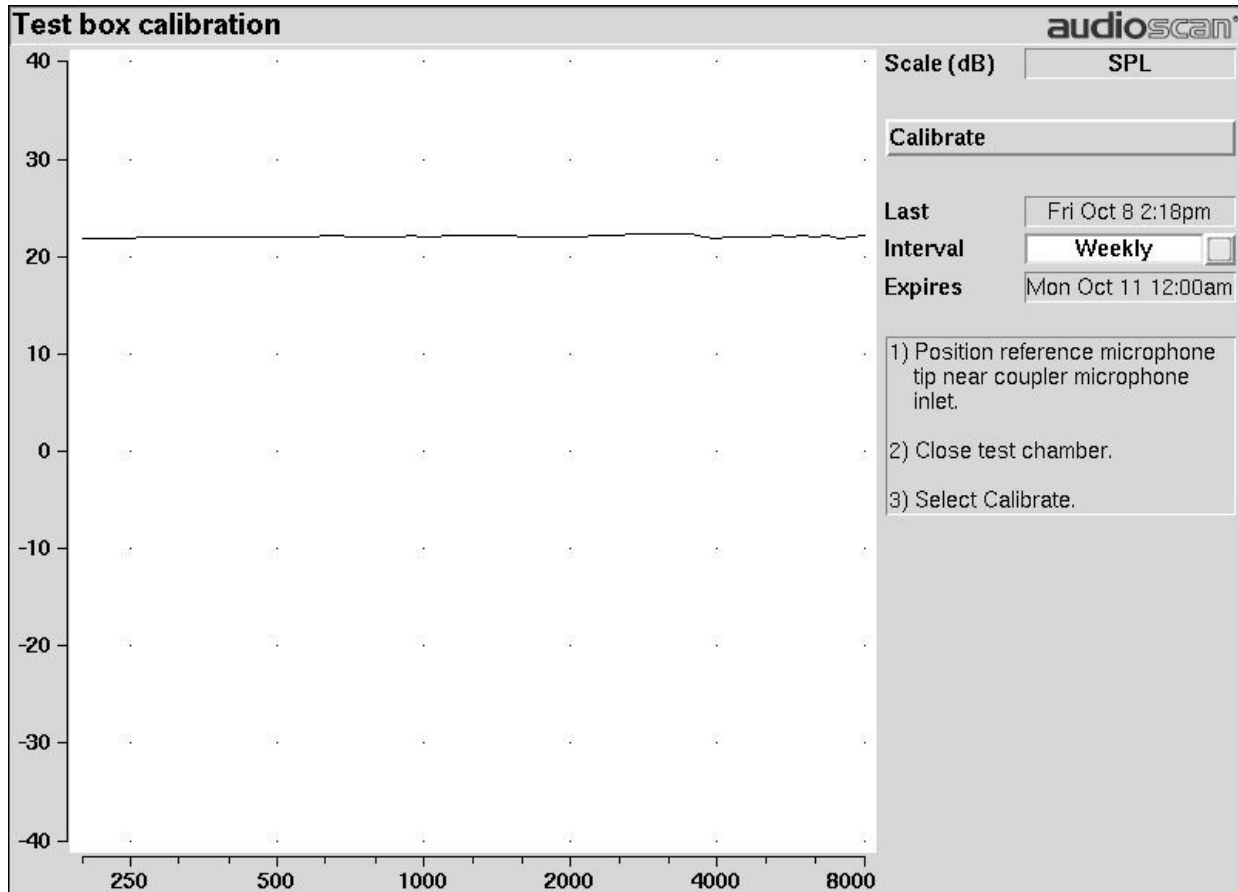

Calibrating test box reference microphone

Remove any coupler from the coupler microphone before calibration.


1. Position the test box reference microphone opening concentric with the coupler microphone opening and about 1 – 2 mm from as shown below.
2. Close and seal the test chamber by tightening the latch at the front of the lid.
3. Click on  , then click on the [Calibration] option within the Test box menu.
4. Click on [Daily] or [Weekly] to select the calibration interval.
5. Click on [Calibrate]. See [Test box calibration facts](#)

It is recommended that you print a copy of the initial calibration curve and compare it with your daily or weekly curve. Any change in this curve indicates a drift in either the coupler or the reference microphone which needs further investigation. See [Calibration check for coupler microphone](#) for an additional check.



Calibration check for coupler microphone

Accuracy of test box measurements depends on the calibration of the coupler microphone. This should be checked regularly using a microphone calibrator with an adapter ring. The RE780 (1 1/8 in.) and RE781 (1in.) rings are designed for Quest CA22 and QC-10/20 respectively. To verify the coupler microphone calibration:

1. Calibrate the test box reference microphone. See [Calibrating test box reference microphone](#)
2. Screw the appropriate calibrator ring onto the coupler microphone.
3. Insert the coupler microphone with the attached calibrator ring into the calibrator opening.
4. Turn the calibrator on and adjust it to present one of the available frequencies and levels (e.g., 94 dB SPL at 1000 Hz).
5. Click on  , and click on [Manual control] in the Test box menu.
6. Click the [Level] button and click on '0' on the drop-down list.
7. Click the [Frequency] button and highlight and click the frequency of the calibrator (e.g. 1000 Hz) on the drop-down list.
8. Click the [Filter] button and click on 1/12th octave from the drop-down list.
9. Click [Start test] (which changes to [Stop test]).
10. Check that the Coupler microphone level is within 1 dB of the calibrator level setting, +/- the calibrator accuracy. For a 94 dB calibrator with +/- 0.3 dB accuracy, the indicated coupler microphone level should be 94 +/- 1.3 dB.
11. To stop the measurement, click [Stop test].

Coupling the hearing instrument

The Axiom is supplied with 2 metal couplers, a type HA-1 (ITE) and a type HA-2 (BTE). *Note that an ANSI standard coupler for CICs does not exist.* These must be tested in the HA-1 coupler.

When threading the couplers onto the coupler microphone, *it is very important that they be firmly tightened* to ensure a good acoustic seal. Avoid twisting the coupler microphone cable – turn the coupler not the microphone or unplug the microphone while tightening the coupler.

BTE instruments are coupled to the HA-2 coupler by means of a 10 mm (3/8th in.) length of heavy wall #13 earmold tubing. The only purpose of this tubing is to seal the tip of the earhook to the coupler inlet. All of the tubing required by ANSI S3.22 is machined into the metal stem of the HA-2 coupler. The #13 tubing should be inspected regularly for cracks which will cause feedback.

Custom instruments are sealed to the HA-1 coupler with putty so that the end of the eartip is flush with the inside of the coupler opening. Putty should not extend into the coupler cavity or block the sound outlet of the instrument. *Vents should be sealed at the faceplate end.* It is very important that the instrument be well-sealed to the coupler.

Open fit instruments must use the coupler and coupling system specified by the manufacturer. This may involve a hook that replaces the open fit tubing or an adapter tube that may be sealed to the opening of the HA-1 coupler with putty.

RITE (Receiver in the ear) instruments are coupled to the HA-1 coupler using putty to seal the receiver module or soft tip to the coupler opening.

Excess low frequency output is usually due to open vents or a poor seal to the coupler.

For health reasons, hearing instruments should be clean before putty is applied and putty should be replaced frequently.

Positioning the hearing instrument

1. The hearing instrument should be coupled to the appropriate coupler. See [Coupling the hearing instrument](#).
2. Place the instrument and coupler in the test chamber so that the front microphone opening of the instrument is on the crossbar within the test zone.
3. Position the test box reference microphone as close as possible to the microphone opening of the hearing instrument without touching it.
4. Close and seal the test chamber.



9 ANSI Hearing Aid Tests

These tests follow the procedures of ANSI S3.22-1996 or ANSI S3.22-2003 as selected in Setup. See 1996 or 2003 ANSI standard :

Linear: An automated test sequence for linear hearing aids per ANSI S3.22.

AGC: An automated test sequence for AGC hearing aids per ANSI S3.22.

ANSI S3.22-1996 facts

The Axiom performs selected hearing instrument tests in accordance with ANSI S3.22-1996. This standard expresses some outputs and gains as High Frequency Average (HFA) values. The HFA is the average of values at 1000, 1600 and 2500 Hz. If a hearing aid meets certain conditions, the manufacturer may choose other averaging frequencies called Special Purpose Average (SPA) frequencies. The current averaging frequencies are indicated explicitly on test results where space permits and also appear as tick marks on the frequency axis of graphs. To change the averaging frequencies, see ANSI test frequencies.

This standard also requires some tests with the gain control full on and some with the gain control in the Reference Test Position (RTP). This requirement applies to both Linear and AGC aids that have a HFA (or SPA) full-on gain greater than the HFA (or SPA) OSPL90 minus 77 dB. Otherwise the RTP is full on. If required, the Axiom will pause in its testing and guide you in setting the gain control to the RTP.

All other controls on the hearing instrument should be set to provide the widest frequency response, the greatest gain and the highest maximum output. AGC controls and/or programmable parameters should be set as designated by the manufacturer.

At full-on gain the [Linear] and [AGC] test sequences provide a curve of OSPL90 (Output Sound Pressure Level for a 90 dB input SPL), the HFA (or SPA) and peak value of this curve, the frequency of the peak and the HFA (or SPA) gain for a 50 dB input SPL.

With the gain control at RTP, these sequences also provide a frequency response curve with an input SPL of 60 dB (Linear) or 50 dB (AGC), the HFA (or SPA) gain for a 60 dB input SPL, the frequency range, battery drain, equivalent input noise, harmonic distortion, attack and release time (AGC only).

ANSI S3.22-2003 facts

ANSI S3.22-2003 has been designated a recognized standard by the FDA and is mandatory for reporting test data.

The most significant change from the 1996 version is the requirement for the hearing aid to be set in its most linear mode for the setting of the gain control to Reference Test Setting (changed from Reference Test Position) and for all tests except attack and release and input-output (I/O) curves. These two tests are to be conducted with the AGC function set for maximum effect. The AGC test sequence will pause to allow AGC to be set prior to measuring attack and release time.


Full-on gain is determined with 50 dB input SPL (60 dB was formerly an option) and frequency response curves are run at 60 dB SPL for Linear and AGC aids.

The OSPL90 curve, the HFA (or SPA) and peak value of this curve, the frequency of the peak, the HFA (or SPA) full-on gain are determined at full-on gain setting. The response curve, frequency range, Reference Test Gain, battery drain, equivalent input noise, distortion, attack and release times are determined at RTS.

These changes result in more consistent values for reference test gain, equivalent input noise and attack and release time. See 1996 or 2003 ANSI standard, 8.2: ANSI S3.22-1996 facts and 8.5: ANSI 2003 Linear and


AGC tests.

ANSI 1996 Linear and AGC tests

5. Refer to Test Box Measures – Setup, Coupling the hearing instrument and Positioning the hearing instrument to prepare the instrument for testing.
6. Click , then click [Linear] or [AGC] on the Test box menu.
7. Click [Start test] and follow the test setup instructions which are presented.
8. *Turn the instrument ON.*
9. Set the gain control (manual or programmable) to maximum with all other controls set to provide the widest frequency response, greatest gain and highest maximum output. AGC controls and/or programmable parameters should be set as designated by the manufacturer.
10. Click to run the test or to abort the test. Note that, if required, the Axiom will pause during the test to request that the volume control be adjusted to the Reference Test Position (RTP). Open the test box if necessary and, without moving the hearing instrument, adjust the user gain control until the large vertical arrow on the screen prompt disappears. Click to resume the test.

If you are unsure which test sequence to run, try [AGC] first. If attack and release times are all near zero, the instrument is probably linear.

ANSI 2003 Linear and AGC tests

1. Refer to Test Box Measures – Setup, Coupling the hearing instrument and Positioning the hearing instrument to prepare the instrument for testing.
2. Click , then click [Linear] or [AGC] on the Test box menu.
3. Click [Start test] and follow the test setup instructions which are presented.
4. *Turn the instrument ON.*
5. Set the gain control (manual or programmable) to maximum with all other controls set to provide the widest frequency response, greatest gain and highest maximum output. AGC controls and/or programmable parameters should be set to minimize AGC action (AGC off or threshold high, compression ratio close to 1).
6. Click to run the test or to abort the test. Note that, if required, the Axiom will pause during the test to request that the gain control be adjusted to the Reference Test Setting (RTS). If necessary, open the HIT chamber and, without moving the hearing instrument, adjust the gain control until the large vertical arrow on the screen prompt disappears. Click to resume the test.

During the AGC test, the Axiom will pause to request that AGC controls or programmable parameters be set for *maximum* AGC effect. If the manufacturer does not provide settings to produce maximum AGC effect, use settings that result in the highest gain for soft sounds and the lowest gain for loud sounds. This will typically result in the lowest compression threshold, a flat I/O curve and the longest attack/release times. These are not 'typical' characteristics but indicate the adjustment range possible for these parameters.

If you are unsure which test sequence to run, try [AGC] first. If attack and release times are all near zero, the instrument is probably linear.

ANSI test results

In 1996 tests: Full-on average gain is measured at 50 dB SPL, average gain at RTP is usually (but not always) measured at 60 dB SPL and the frequency response is run at 60 dB SPL for linear aids or at 50 dB SPL for AGC aids.

In 2003 tests: Full-on average gain (1) is measured with a 50 dB input SPL, average gain at RTS (2) is always measured at 60 dB input SPL and the frequency response curve (3) is always run at 60 dB input SPL.




10 Other Test Box Measures

In addition to the tests of ANSI S3.22, the Axiom also provides the following tests in the test box:

Manual control: Coupler and test box reference mic. SPL and coupler mic. distortion at selected 1/12th octave frequencies and stimulus SPL of 0 and 40 – 90 dB in 5 dB steps. Can be used as a dual channel sound level meter with 1/12th octave, A and C filters.

Multicurve procedure


Multicurve generates 1 to 4 output or gain response curves using either pink noise or swept pure tones at input levels ranging from 40 – 90 dB SPL and 0 dB (off).

1. Click  then click [Multicurve] from the Test box column.
2. Refer to Coupling the hearing instrument, Positioning the hearing instrument use and care to prepare the instrument for testing.
3. Turn the instrument ON.
4. Click on button 1, 2, 3 or 4 in the Test column.
5. Select the Stimulus window, then either [Pink noise] or [Swept] from the drop-down list.
6. Click on the Level window for the selected Test, then select a stimulus level from the drop-down list.
7. Click on  to run a sweep or to capture a pink noise curve; press  to abort the test.
8. To measure another response curve, click on another test button; to overwrite a curve, click on its test button.

To change scale from dB SPL to dB Gain, highlight & <PICK> the Scale window.

Multicurve results

1. *Output* curves for the noise signal are in 1/12th octave bands. They will be approximately 18 dB lower than pure tone curves for a linear instrument.
2. *Gain* curves for a linear instrument will be the same for both swept and pink noise test signals.
3. For compression instruments, both gain and output curves may be expected to differ for the two test signals.
4. See Test Box Measures - Setup to change screen appearance.

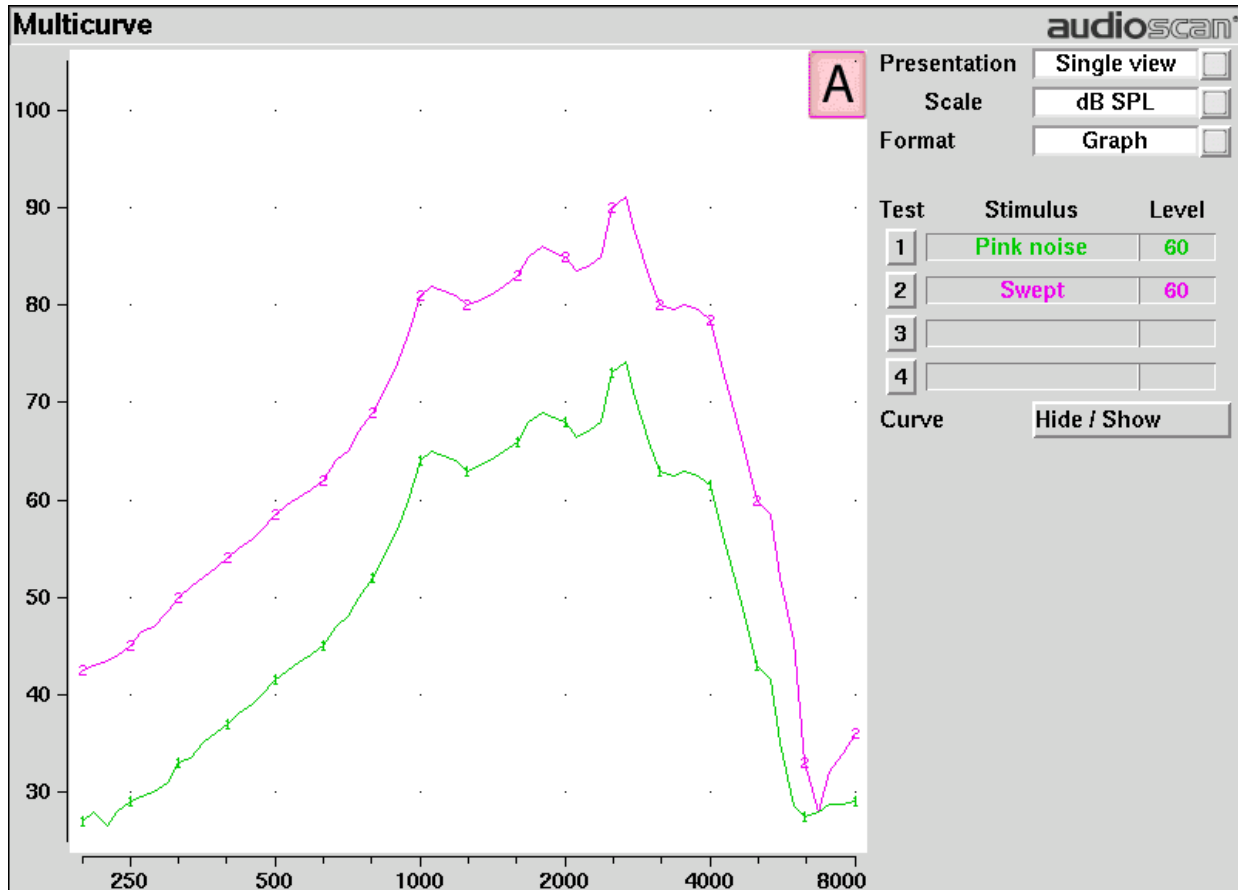

Spectral analysis in Multicurve


Selecting a stimulus level of 0 dB (off) in [Multicurve] presents a spectral analysis of the coupler microphone signal (Scale = dB SPL) or the difference between the coupler and reference microphone spectra (Scale = dB Gain).

If the stimulus type is [Pink noise], the analysis is performed in 1/12th octave bands and displayed in real time. Pressing <Continue> captures the displayed spectrum.

If the stimulus type is [Swept], a swept filter analysis is performed in 1/12th octave bands and displayed at 1/3rd octave frequencies. Pressing <Continue> causes the swept filter analysis to be performed in 1/12th octave bands and displayed at 1/12th octave frequencies.

Manual control procedure

Manual control displays the reference microphone SPL, the coupler microphone SPL, their difference in dB (gain) and coupler microphone signal distortion at any selected 1/12th octave frequency and at a selected input level from 40 to 90 dB (in 5 dB steps) and at 0 dB (off).


1. Click on  and select [Manual control] in the [Test box] menu.
2. Click on the [Level] button and select the desired stimulus level from the drop-down list.
3. Click on the [Frequency] button and select the desired frequency from the drop-down list. (Distortion


measurement data is available only from 200-4000 Hz.)

4. Click on the [Start test] bar to start the test. [Start test] changes to [Stop Test]. To stop the measurement, click on [Stop test]. You may change Level and Frequency without stopping the test.

Sound level meter using manual control

When set for a stimulus level of 0 dB (off), the Manual control mode allows the Axiom to be used as a sound level meter. Sound at the test box reference microphone and the coupler microphone may be analyzed by 1/12th octave, A-weighted or C-weighted filters. A and C-weighted measurements are limited to frequencies above 100 Hz and the averaging time and update rate are 384 ms.

1. Click on  and select [Manual control] in the [Test box] menu.
2. Click on the [Level] button and select 0 dB stimulus level from the drop-down list.
3. Click on the [Filter] button and select 1/12th octave, A-weighted, or C-weighted from the drop-down list.
4. For 1/12th octave filter, click on the [Frequency] button and select the desired frequency from the drop-down list.
5. Click on the [Start test] bar to start the test. [Start test] changes to [Stop Test]. To stop the measurement, click on [Stop test]. You may change Level and Frequency without stopping the test.

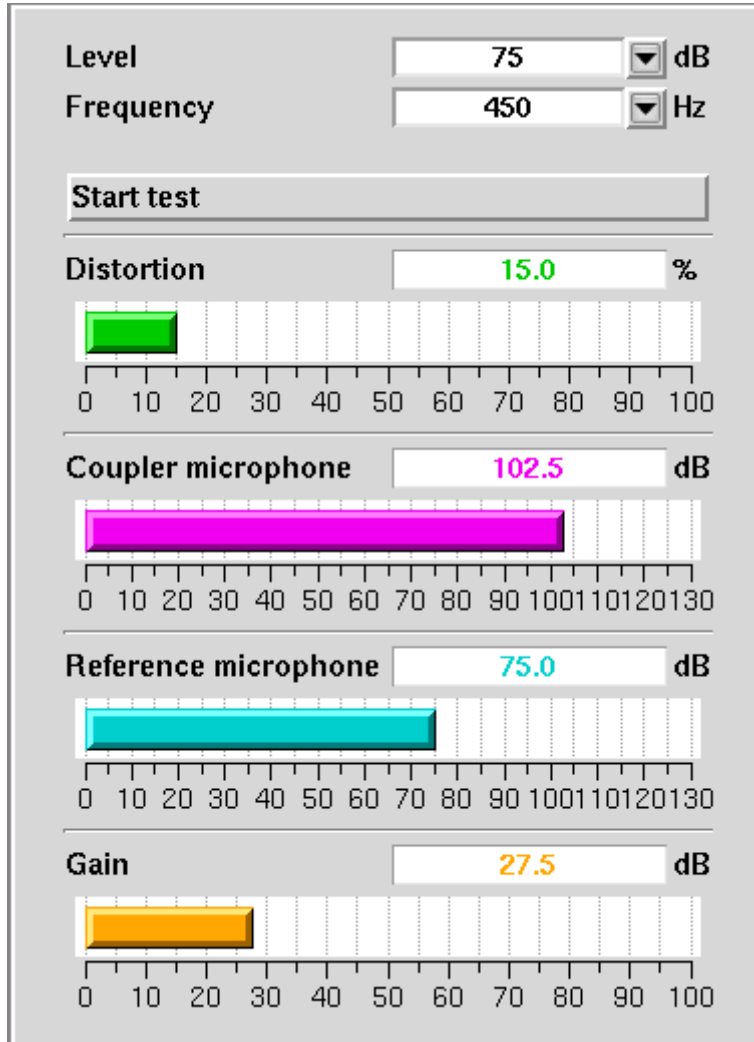

NOTE: Microphone extension cables (VA-131 & VA-133) are available from Audioscan. Standard audio extension cables should not be used. They will substantially increase noise levels.

11 Sensory loss simulator

Sensory loss simulator description

A pure conductive loss attenuates loud and soft sounds equally, and is easily simulated by just turning down the volume or inserting an earplug. Losses of a sensory nature are more complex and more difficult to simulate.

The most common sensory loss is caused by *outer hair cell* damage and results in loss of audibility for soft sounds but near-normal loudness for loud sounds. *Inner hair cell* damage elevates threshold and reduces loudness for both soft and loud sounds, possibly somewhat more for soft sounds.

This sensory loss simulator (SLS) is intended to allow normal-hearing listeners to hear sounds as if they had the elevated threshold and altered loudness perception caused by outer and inner hair cell damage. It is based on the cochlear hearing loss model described in Moore & Glasberg (2004).



In this simulation, losses less than 58 dB SPL are considered to be due entirely to outer hair cell damage. Any loss greater than this is considered inner hair cell loss.


This SLS does not simulate broadening of the auditory filters or other distortions that may accompany cochlear hearing loss.


To properly experience the simulation, listeners should be within 1m of the sound-field speaker and in a quiet room.




Sensory loss simulator operation

Listeners should be within 1m of the sound-field speaker and in a quiet room.

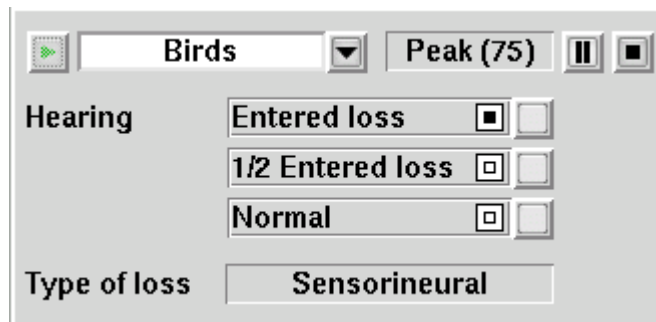1. Click on  and select [Speechmap] in either the [Test box] menu or the [On-ear] menu.
2. If an audiogram has previously been entered in Speechmap or Insertion gain, go to step 5. Otherwise, click on the [Audiometry] button.
3. Enter HL threshold values using the mouse. Left click the mouse on the audiogram form to enter a point. Click again to delete the point.
4. Click on  to return to the [Speechmap] screen when all points have been entered.
5. Click on the [Loss simulator] button.



6. Click on . Sound, processed to simulate the entered hearing loss, will be presented via the sound-field speaker.
7. Click on list button next to the play button for a list of sounds to play and click on one of them.

8. Click on one of the Hearing buttons to simulate the Entered loss, 1/2 Entered loss, or Normal hearing..
9. Click on  or  to stop the demonstration and freeze the stimulus speech banana on the screen.
10. Click on  to exit from the Sensory Loss Simulation screen.

12 On-Ear Measures - Setup

For accurate, repeatable measurements, the Axiom must be properly set up and calibrated, the client must be properly positioned and the probe tube must be properly positioned in the ear canal. Some common terms:

Real-Ear Unaided Response (*REUR*): SPL, or band SPL, in the open (unaided) ear canal for a given stimulus.

Real-Ear Unaided Gain (*REUG*): Difference, in dB, between the REUR and the stimulus SPL, or stimulus band SPL.

Real-Ear Aided Response (*REAR*): SPL, or band SPL, in the ear canal for a given stimulus, with a working hearing aid in place.

Real-Ear Aided Gain (*REAG*): Difference, in dB, between the REAR and the stimulus SPL, or stimulus band SPL.

Real-Ear Saturation Response (*RESR*): The REAR obtained using a narrow-band signal at a level high enough to saturate the hearing aid (usually 85 - 90 dB SPL).

Real-Ear-to-Coupler Difference (*RECD*): Difference, in dB, between the SPL produced in the ear canal and the SPL produced in a coupler by the same sound source.

On-ear calibration facts

The probe microphone and its associated tubing does not have a flat frequency response. The on-ear calibration process compares this response with the flat, factory-calibrated response of the on-ear reference microphone and compensates all subsequent probe-tube measurements for this difference. It may be performed as often as desired but will be invalidated at 12 am the following day or at 12 am the following Monday, depending on the choice of Daily or Weekly calibration interval. After the calibration has expired, you will be prompted to calibrate whenever you attempt a test that uses the on-ear measurement microphones.


It is suggested that a copy of the calibration curves for the probe modules when new be posted near the Axiom for comparison with daily or weekly curves. If there is a drift in calibration with time, the module may need replacement. See Calibration check for probe module.


Calibration should be repeated if a probe module is replaced or if a probe-tube with different dimensions is installed.


Switching left and right probe modules after calibration will result in the wrong calibration curve being applied.


Calibration of on-ear probe microphone

Ensure the probe microphone assembly to be calibrated is plugged into the appropriate jack. See Microphone connection and On-ear calibration facts

1. Click on  and then click on [Calibration] within the [On-ear] menu.
2. Press the enlarged end of a probe tube as far as it will go, into the recessed opening at the top of the probe module.
3. Position the open end of the probe tube in front of the reference microphone inlet and press it between the posts as shown.

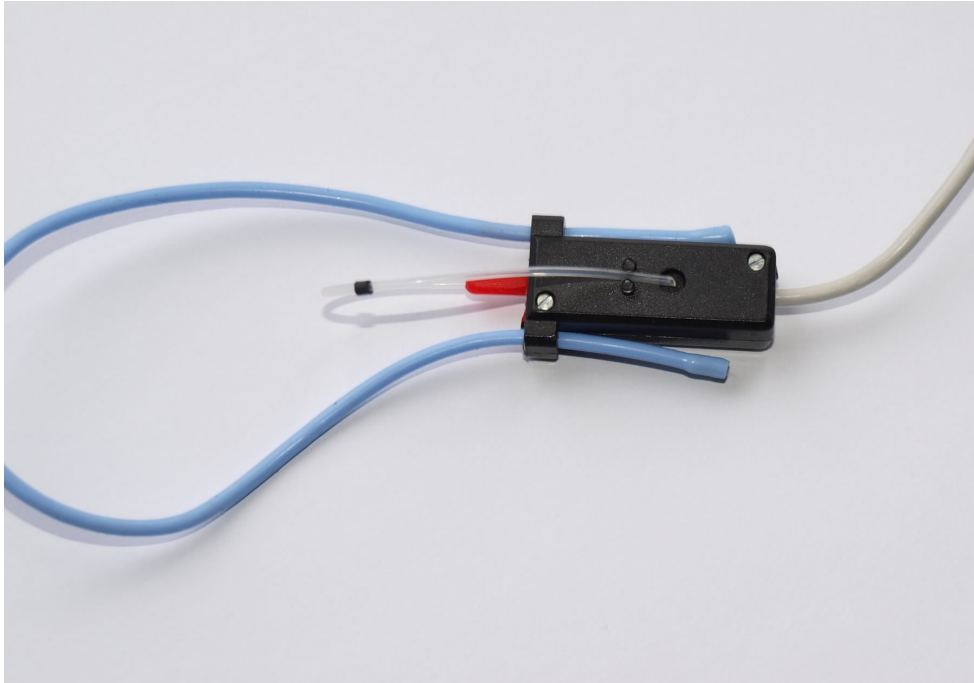


4. Hold the probe module 15 - 90 cm (6 – 36 inches) away from and directly in front of the sound-field loudspeaker. Orientation of the probe module is not critical.
5. Click on the [Calibrate] button. The resulting curve should be similar in shape to that shown but may be shifted +/- 5 dB.
6. Click on  and repeat steps 2 - 5 for the second probe module.
7. To change the calibration interval, click to choose [Daily] or [Weekly].

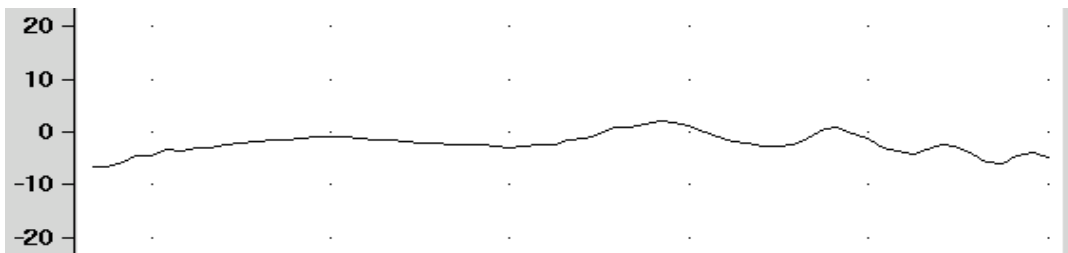


Calibration check for probe module

The calibration curve for the probe module is a comparison of the response of the probe microphone (with tube) to the response of the on-ear reference microphone. Any change in the curve indicates a change in one of these microphones (or the probe tube). See Calibration of on-ear probe microphone. As part of a regular calibration schedule, or to investigate changing calibration curves, the on-ear reference microphone may be checked against a high quality sound level meter.

1. Click on  then click on [Manual control] in the [On-ear] menu.
2. Click on the [Level] pull-down menu arrow and then select 70 from the drop-down list.
3. Click on the [Frequency] pull-down menu and then select 2000 Hz from the drop-down list.

4. Click on the [Start test] bar (which changes to [Stop test]).
5. Hold the probe module about 18 in. (0.5 m) in front of the sound-field speaker.
6. Hold the microphone of a calibrated sound level meter next to the on-ear reference microphone as shown. Set the sound level meter to C weighting and slow.
7. The sound level meter should read 70 +/- 2 dB.
8. Repeat with the frequency at 250 Hz.
9. To stop the measurement, click on [Stop test].

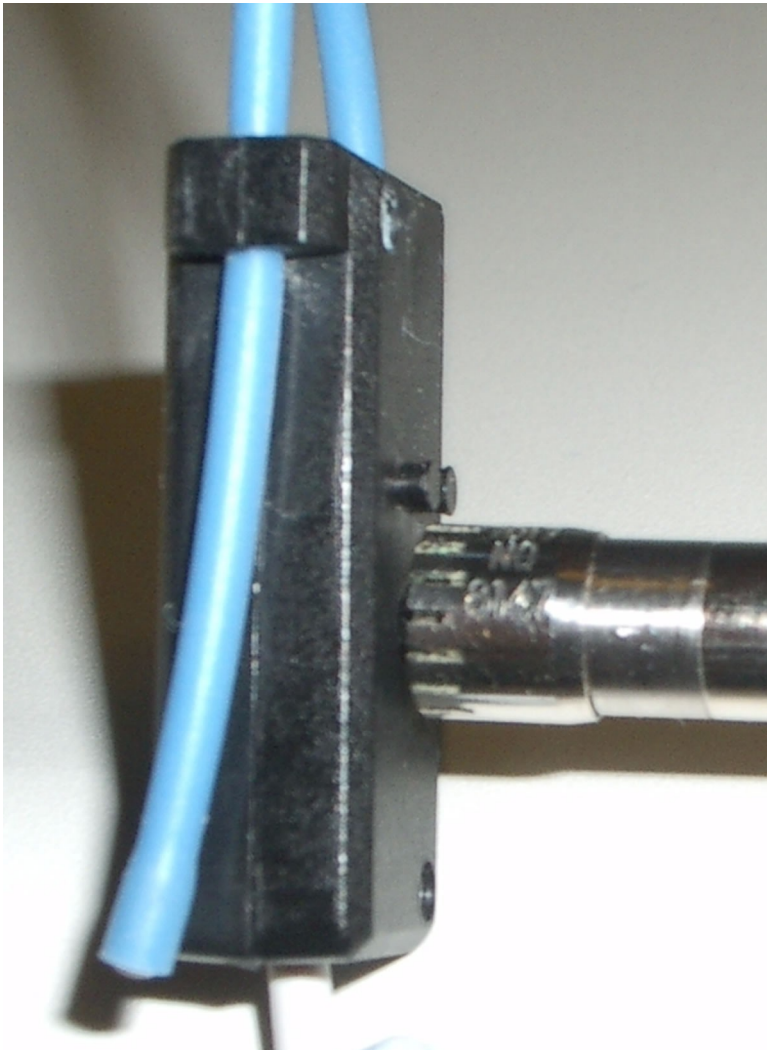



The probe module should be replaced if it fails this test.

Max TM SPL setup

The Max TM SPL safety feature allows you to set a limit on the SPL, measured by the probe microphone, above

which a test terminates. The MAX TM SPL limit is shown on all on-ear measurement screens. The default limit is 120 dB.

To choose a different Max TM SPL setting:

1. Click on  then click on [Max TM SPL] in the Setup menu.
2. Click on the Max TM SPL pull-down menu.
3. Click on the desired Max TM SPL setting from the drop-down list.
4. Click on  to exit Setup.

This Max TM SPL setting will be in effect until you change it.

If the limit is exceeded during an on-ear test, the Axiom will terminate the test and display the message 'MAX TM SPL EXCEEDED'. You must then decide to either reduce the maximum output limit of the hearing instrument or increase the Max TM SPL setting following the steps above.

If the MAX TM SPL limit has been disabled, a notice will appear on each real-ear screen. Also, a message reminding you that the MAX TM SPL limit has been disabled will appear when you select an on-ear test.

Positioning the client

Sound reflections from nearby surfaces (including the operator) can cause large measurement errors, especially if the client moves between REUR and REAR measurements. Room noise can increase test time, cause errors and alter the operation of the hearing instrument being evaluated.

1. Choose a quiet location and position the client and the sound-field speaker at least 1.5 m (5 feet) away from any hard surfaces.
2. Position the client directly in front of, and facing, the sound-field speaker at a distance of 45 to 90 cm (18 to 36 in.) from the center of the head.

Positioning the probe tube

1. Use an otoscope to ensure there is no ear canal obstruction (e.g. cerumen) and to get a sense of ear canal length.
2. Install a new probe tube on the probe module. Set the black marker ring approximately 28 mm from end of probe tube for adult females, approximately 30 mm from end of probe tube for adult males and approximately 20-25 mm from end of probe tube for children. These positions may be modified based on the otoscopic examination. (Tip - the probe module body is 28 mm long.)
3. Hang the probe module on the ear, as shown. *The reference microphone should face outwards.*
4. Adjust the blue cord until the probe module is snug against the head, directly below the earlobe (cheek level). To facilitate this, clip the probe module cable to clothing on the side opposite the test ear and draw the cable snug.
5. Hold the front blue cord so that it passes behind the ear canal entrance and pass the end of the probe tube in front of the blue cord and into the ear canal.
6. Carefully slide the probe tube into the ear canal until the marker ring approaches the intertragal notch. Then let the front blue cord move forward to hold the probe tube in place. Use an otoscope to verify that the tube is 2 to 5 mm from the eardrum.



Probe tube insertion: The blue lanyard holds the tube in place and the depth marker is at the intertragal notch

13 On-Ear Measures Screen Setup

To accommodate your working preferences, the appearance of On-ear measurement screens may be changed in several ways.

SPL or HL scale

In Insertion gain tests, you may view results in either dB HL or dB SPL.

Insertion gain tests view the hearing aid as improving the sound-field threshold. In Insertion gain, the HL scale presents aided results as improved sound-field thresholds while the speech region remains fixed.

To change the scale:


Click on the [Scale] toggle button to toggle between a dB SPL and dB HL scale.

Hide or show on-ear curves

Hide or show insertion gain curves

To hide or show insertion gain curves displayed on the insertion gain screen:

1. Click on the [Hide/Show] button located on the lower right side of the insertion gain screen. (NOTE: the REUG curve display defaults in the 'Show' position.
2. Click on the desired test result to toggle between the Hide or Show options. The curves will toggle on or off when selected.
3. Click within the Curve Display poster to return to the test screen.

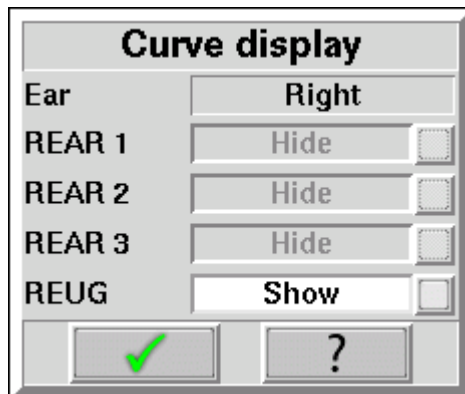


Hide or show Speechmap curves

To hide or show unaided and aided curves on the Speechmap screen:

1. Click on the Hide/Show buttons located just below each of the Test identifier numbers as shown below. Note that the Hide/Show aided button has 3 states – Curves hidden, display LTASS only and display LTASS + speech envelope (i.e. the 'speech banana').




Hiding a curve does not erase it. To erase curves click on  and select one of the Erase options.

14 On-Ear Instrument Measures


A number of the tests covered in Test box measures can also be performed with the hearing instrument coupled to the ear of a client rather than to a 2cc coupler. The on-ear results may be expected to differ from the test box results because of the different acoustic environment and the different acoustic load.

On-ear manual control

1. Refer to [On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.
2. Click on  and then click on [Manual control] in the [On-ear] menu.
3. Click on the [Level] button and select the desired stimulus level from the drop-down list.
4. Click on the [Frequency] button and select the desired frequency from the drop-down list. (Distortion measurement data is available only from 200-2500 Hz.)
5. Click on the [Start test] bar to start the test. [Start test] changes to [Stop Test]. To stop the measurement, click on [Stop test]. You may change Level and Frequency without stopping the test.

Sound level meter using on-ear microphones

The manual control feature enables the Axiom to be used as a sound level meter to perform C-weighted, A-weighted or 1/12th octave analysis of sound as measured by the probe and reference microphones. A and C-weighted measurements are limited to frequencies above 100 Hz and the averaging time and update rate are 384 ms.

1. Refer to [On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.
2. Click on  and then click on [Manual control] in the [On-ear] menu.
3. Click on the [Level] button and select 0 dB stimulus level from the drop-down list.
4. Click on the [Filter] button and select 1/12th octave, A-weighted, or C-weighted from the drop-down list.
5. For 1/12th octave filter, click on the [Frequency] button and select the desired frequency from the drop-down list.

Click on the [Start test] bar to start the test. [Start test] changes to [Stop Test]. To stop the measurement, click on [Stop test]. You may change Level and Frequency without stopping the test.

15 RECD measurement

The RECD (real-ear to coupler difference) may be individually measured to improve the accuracy of hearing aid fittings. The measurement procedure may be accessed by clicking on the [RECD] button located on the Speechmap screen.

RECD facts

Individual RECD values are derived by comparing the SPL produced by a sound source in the closed ear canal to that produced by the same sound source in a 2cc coupler. It is commonly assumed that the RECD so measured is a property of the individual ear and is independent of the sound source used to obtain it, making it useful for (1) converting HL threshold and UCL obtained using insert phones to SPL values, (2) simulating real-ear measurements from 2cc coupler measures (test box); and (3) deriving 2cc coupler targets from real-ear aided targets. However, this assumption is true, only if the acoustic impedance of the sound source is significantly higher than the acoustic impedance of the ear canal or the coupler. This is the case for ITE and ITC hearing aids and is approximately true for insert earphones with the supplied foam tips. The acoustic impedance of insert earphones drops if the foam tip is replaced by an earmold or the tubing that forms part of the HA-2 coupler. At some frequencies, for BTE hearing aids with their earhooks and earmold tubing (or the HA-2 coupler tubing), the acoustic impedance approaches that of an adult ear and falls below that of a child's ear. Each of these sound sources will produce a different RECD and, if the acoustic impedance of the sound source differs for the ear canal and 2cc coupler measurements, further RECD differences will result.

The Audioscan RECD measurement protocol uses the Audioscan RECD transducer and a foam tip as the sound source for ear canal SPL measurement and the same transducer and the HA-2 coupler for the 2cc coupler SPL measurement. Because the source impedance is known for both these cases and for insert earphones with a foam tip, adjustments can be made in software to reduce the impact of RECD differences on SPL thresholds, estimated real-ear SPL and 2cc targets. The RECD reported is for a foam tip and an HA-2 coupler. Other protocols may be used but the software adjustments may not be appropriate.

If you use insert ear phones for audiometry OR if you verify using the test box, you should measure RECD (preferred) or use age-related average RECD.



If you use headphones or sound field for audiometry AND you use on-ear tests for verification, RECD is not applied in any way and need not be measured.

If you use 2cc coupler targets, you should use measured or age-related RECD.


Insert phone audiometry with measured RECD and on-ear verification provides the best fitting accuracy, with test box verification being slightly less accurate. Insert phone HL data with age-related average RECD is next best.

RECD coupler response

Prepare the system for test box tests. See [Microphone connection](#), [Calibrating test box reference microphone](#). Connect the RECD transducer to the RECD jack on the underside of the test chamber.

1. Click on  and select [Speechmap] from either the [Test box] or [On-ear] menus.
2. Click on the [RECD] button to access the RECD setup poster.
3. Click on the [RECD] list button and select [Measure] from the menu. Then, click on .


4. If a valid coupler response exists, go to RECD on-ear response.

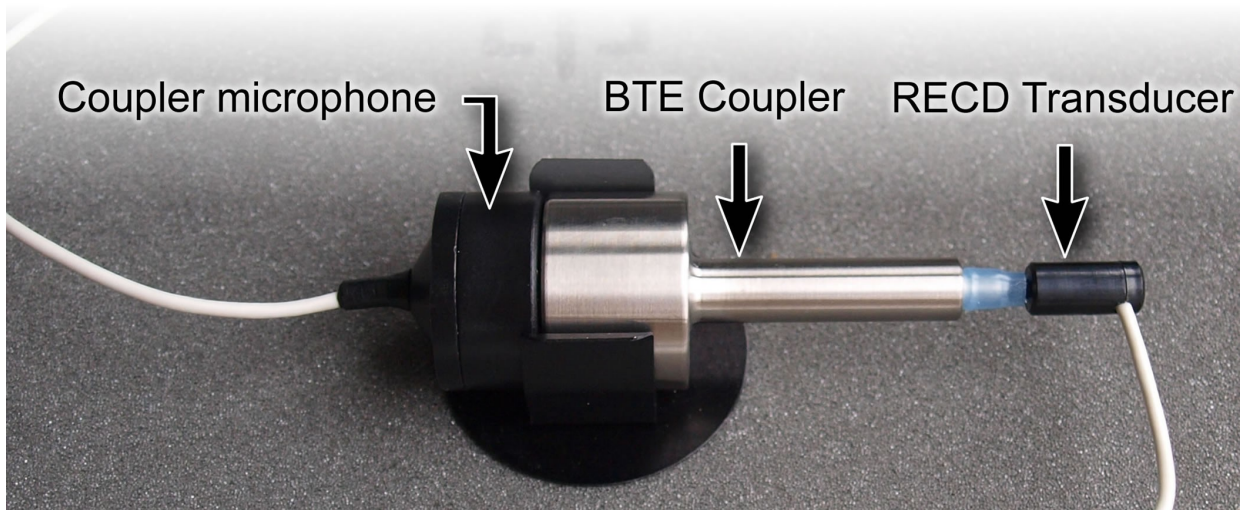




5. If no coupler response exists or the measurement interval has been exceeded, connect the RECD transducer to the BTE coupler.
6. Click on [Measure coupler] and follow the setup instructions.
7. Click on to generate the coupler response curve (green). When this curve is stable, click on to save it.
8. Click on [Daily] or [Weekly] to toggle the measurement interval for the coupler response.

RECD on-ear response

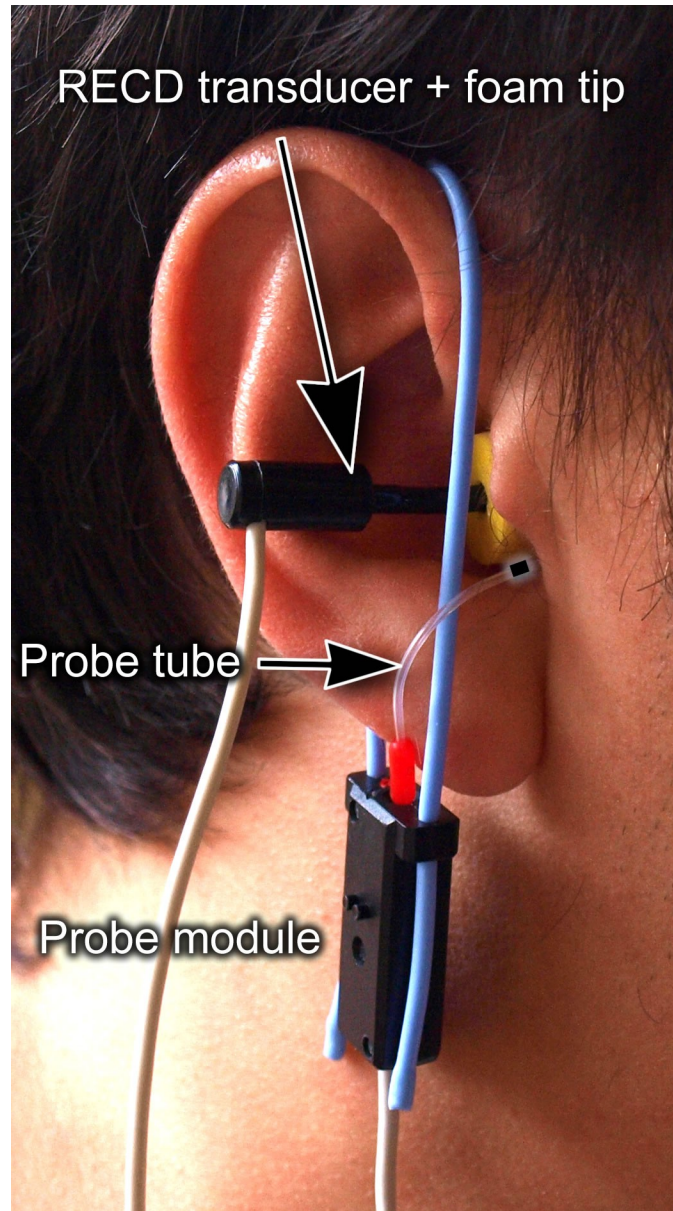Connect the RECD transducer to the RECD jack on the underside of the test chamber. Prepare the system for on-ear tests. See [Calibration of on-ear probe microphone](#).



1. Click on  and select [Speechmap] from either the [Test box] or [On-ear] menus.
2. Click on the [RECD] bar to access the RECD setup poster.
3. Click on the [RECD] list button and select [Measure] from the menu. Then, click on .
4. If no coupler response exists or the measurement interval has been exceeded, go to *RECD coupler response*. If a valid coupler response exists, then click on [Measure real ear].
5. Refer to Positioning the probe tube and insert the probe tube into the ear canal to within 2 - 5 mm of the eardrum.
6. Insert the foam tip into the ear, being careful not to advance the probe tube further into the ear canal. Insertion depth should be as used with insert phones. *Allow the foam tip to fully expand in the ear.*
7. Click on  to generate the real-ear response curve (pink) and the RECD curve (cyan). If the RECD curve


is *negative and unstable* in the low frequencies, check the seal of the foam tip. Increase the tip size and/or apply a lubricant to the foam tip to improve the seal.

- When these curves are stable, click on to save them.

For comparison purposes, the DSL 5.0 age-related average RECD is shown as a dotted curve on the screen. You may change this by clicking the Age list button on the RECD setup poster (step 2 above) and selecting a different age from the drop-down list.

RECD results

The RECD measurement screen shows the response of the RECD transducer in the real ear (pink) and in the 2cc coupler (green). The difference between these curves is the RECD (cyan). The DSL 5.0 age-appropriate average RECD is shown for reference (dotted). If the RECD curve deviates more than 10 dB from the average in the 4 - 6 kHz region, check for proper probe placement and possible blockage of the tube by the foam tip or cerumen. See RECD facts, RECD coupler response, RECD on-ear response.

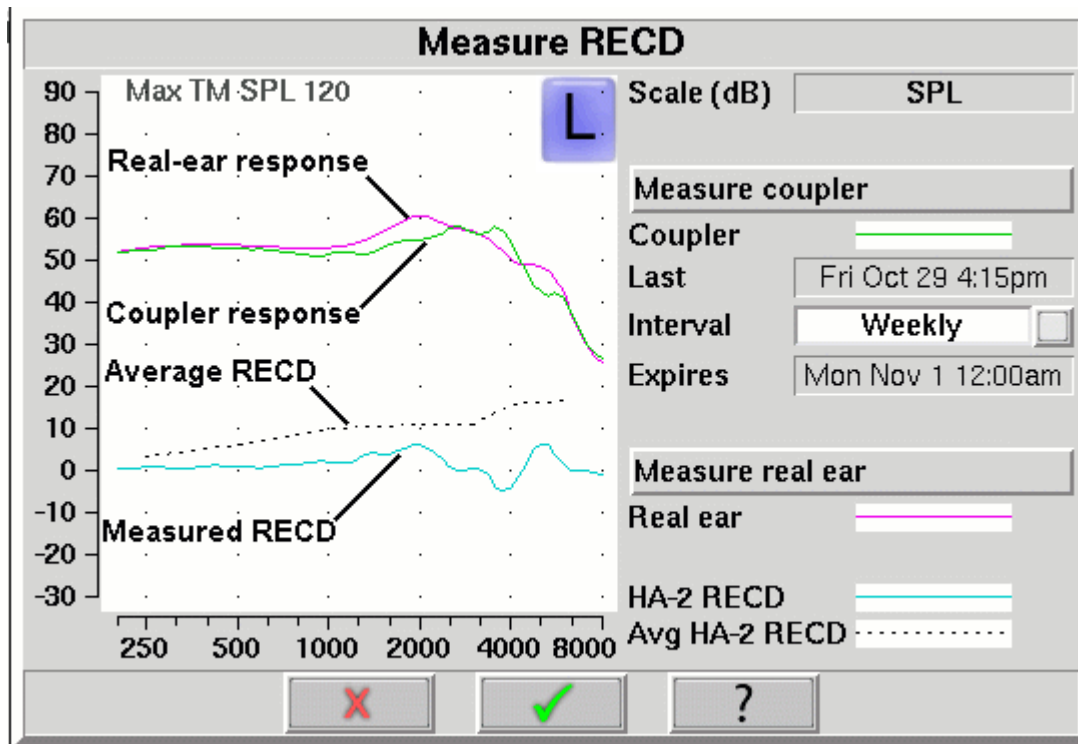


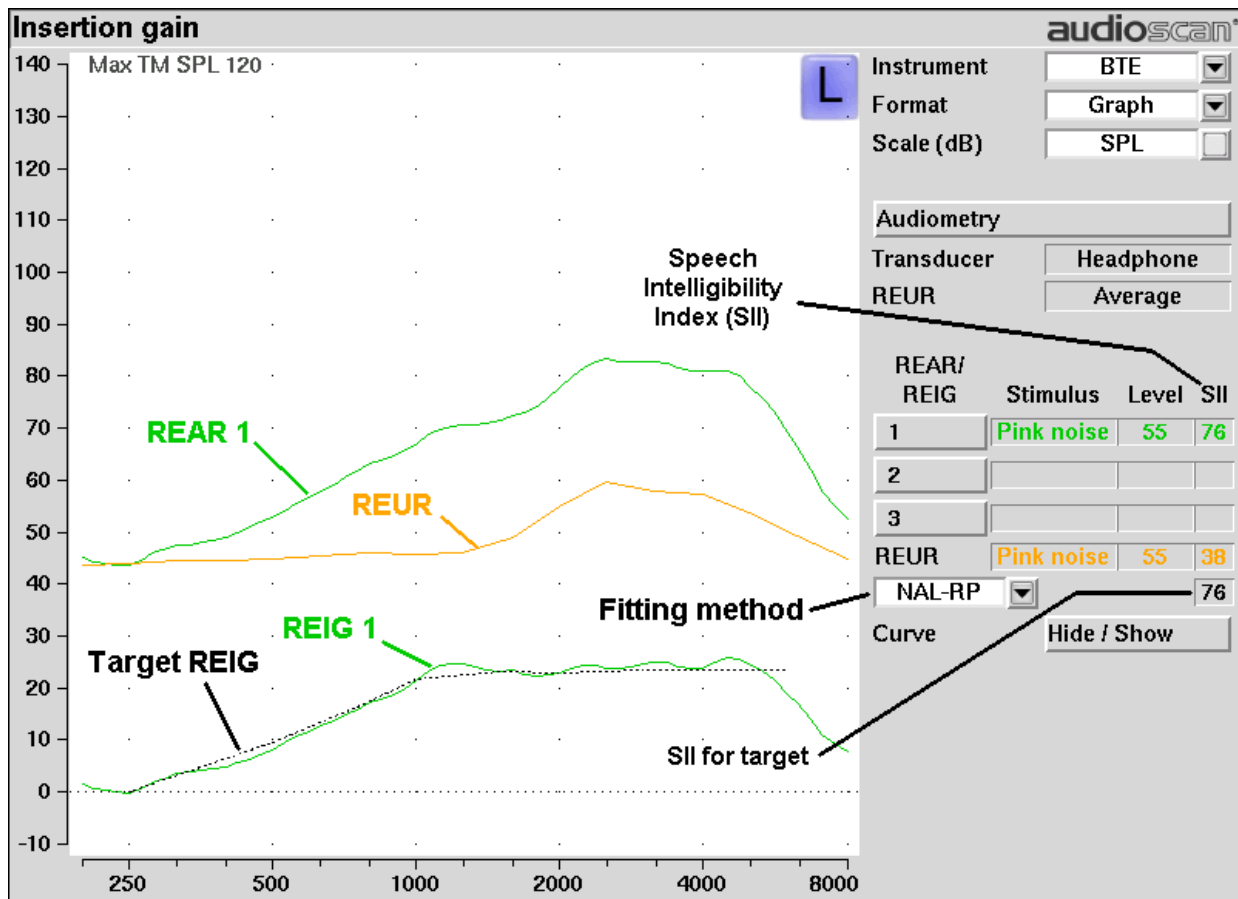
16 Insertion Gain

Insertion gain is the difference between aided and unaided ear canal SPL ($REIG = REAG - REUG$). It inherently assumes a sound-field audiogram. For non-linear hearing aids, REIG is stimulus-dependent and the dependency is specific to each hearing aid. Insertion gain fitting methods ignore these facts. **Speechmap is recommended for fitting all linear and non-linear hearing instruments.**

Refer to [On-Ear Measures - Setup](#) and [On-Ear Measures Screen Setup](#) for terminology and information on preparing the system for use and for proper positioning of the client and the probe tube.


Insertion gain in SPL

1. Click on . Click on [On-ear], then on [Insertion gain].
2. Click on the [Instrument] button to chose from BTE, ITE, ITC, CIC, CROS, Body. This changes the CORFIG used for 2cc targets.
3. Click on the fitting method button to chose from NAL-NL1, FIG6, NAL-RP, POGO II, Berger, Libby, User, None




Insertion gain in HL

See [On-Ear Measures - Setup](#) and [On-Ear Measures Screen Setup](#)

1. Click on  and then click on [Insertion gain] within the On-ear menu.
2. Click on the Instrument button to chose from BTE, ITE, ITC, CIC, CROS, Body. This changes the CORFIG used for 2cc targets.
3. Click on the fitting method button and chose from FIG6, NAL-NL1, NAL-R, POGO II, BERGER, LIBBY, USER, NONE.

An estimated aided threshold curve is calculated by shifting the unaided threshold curve upward by the REIG. This estimate assumes a) the *unaided threshold is a sound-field threshold* and b) a *linear aid* (i.e. REIG doesn't depend on level).

The REUR curve uses the right-hand SPL scale and is shown for reference only.

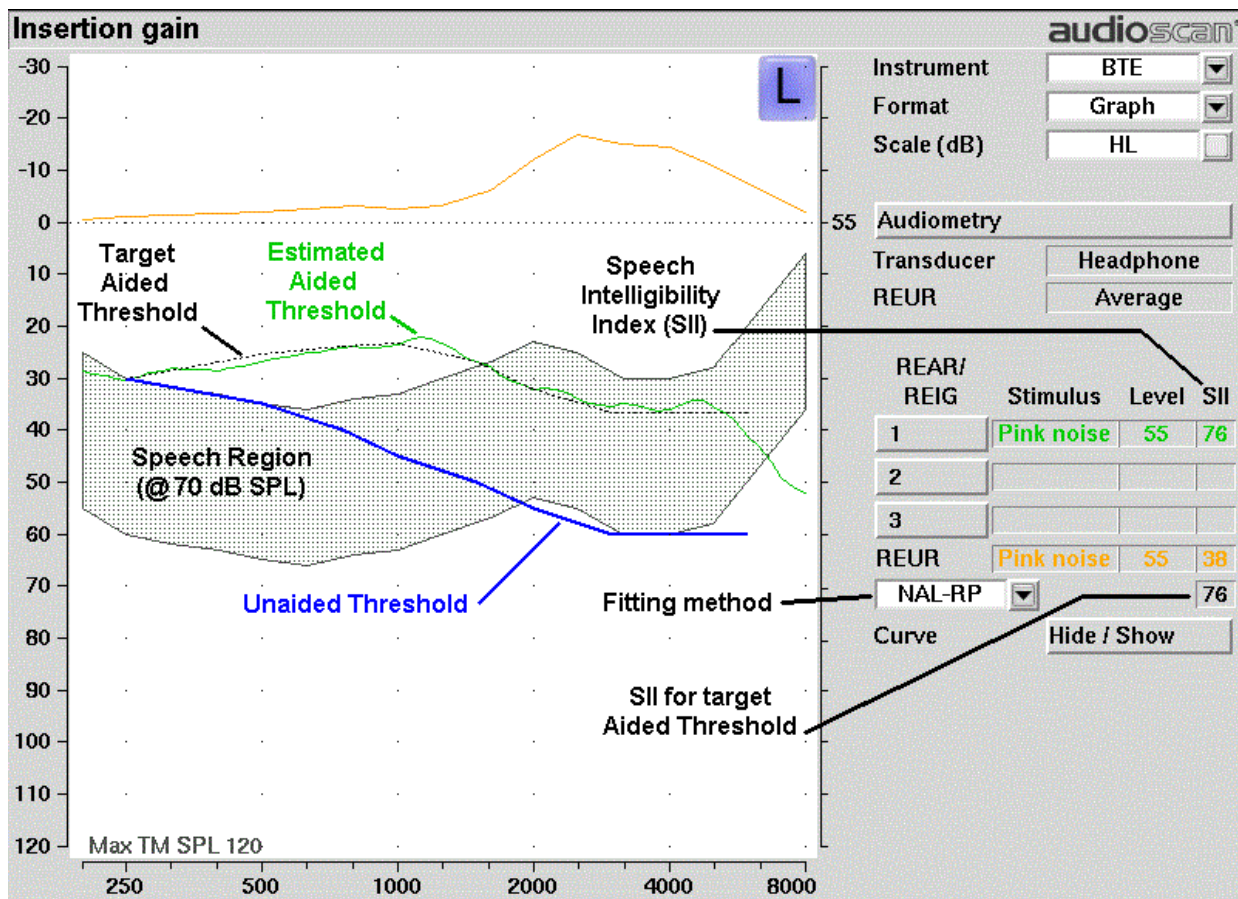

Audiometric data entry

Threshold data and parameters may be entered by scanning a barcode on an Audioscan printout, manually using a mouse or the numeric keypad on an external keyboard. See [Mouse, keyboard, barcode scanner](#) and [Input device operation](#).

To enter data from a barcode see Barcode data input and:

1. Press the barcode trigger and center the red line on the barcode from a distance of 6 – 8 in. (15 – 20 cm.). The scanner will beep and the Barcode entry poster opens.
2. Click on [Done] to accept the data indicated on the poster.

To enter assessment data and parameters manually:

1. Click on the [Audiometry] button
2. Click on the [Transducer] screen button and then select a choice from the list.
3. Click on the REUR screen button to toggle between [Average] and [Measure].

Measured REUR should be used with a soundfield threshold because individual REUR is part of the threshold. *Average REUR should be used with headphone or insert phone thresholds* because individual REUR isn't part of these thresholds.

4. When all selections have been completed, click on to save these selections and continue. Click on to exit this selections menu without saving.
5. Enter HL threshold values using a mouse or keyboard. Left click the mouse on the audiogram form to enter a point. Click again to delete the point. On the keyboard use the arrow keys to change frequency and level and use the numeric keypad [ENTER] key to enter or delete a threshold point.
6. Click on when all points have been entered.

The HL for the highest frequency is used for aided targets at higher frequencies.


REUR measurement procedure

Performing a REUR test erases all existing REAR and REIG data associated with the current test ear. REUR should be measured only if Transducer = Soundfield. Otherwise it is more accurate to use average. See Audiometric data entry.

1. Click on and then click on [Insertion gain] within the On-ear menu.
2. Referring to [On-ear measures - Setup](#) and [On-ear measures screen setup](#), prepare the system, position the patient and insert the probe tube.

3. If REUR is set to Average, click on the [Audiometry] button, then toggle the [REUR] button to [Measure]. Click on to advance to the Threshold entry poster. You may enter threshold data at this time (see Audiometric data entry) or click on to proceed with the measurement.
4. Click on the REUR button (below REAR/REIG test 3).
5. Click on to accept the default 55 dB SPL Pink noise stimulus and capture the REUR curve OR click on the [Stimulus] and/or [Level] buttons to select other values. These choices have no impact on the resulting REIG.
6. If there is a notch in the 4000 to 8000 Hz region, try advancing the probe tube 2 mm and repeating step 3. If the notch moves to a higher frequency, repeat this step until the notch moves beyond the highest frequency of interest.

Because it is analyzed in 1/3 octave bands, the Pink noise stimulus produces a REUR 12 dB below that produced by the Swept stimulus at the same overall level.

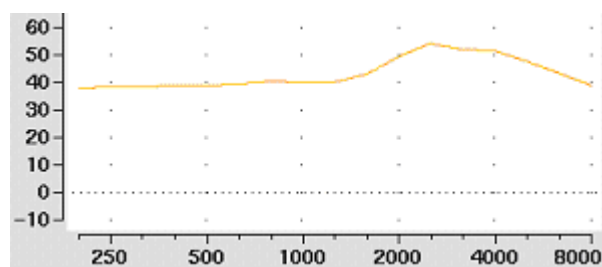

The Average REUR used is shown above. Measured REUR will vary from this.

For small children the peak may shift to the right and become sharper. Drainage tubes and perforated ear drums will cause large differences - usually around 1 kHz.

REAR measurement procedure

1. Click on and then click on [Insertion gain] in the On-ear menu
2. Referring to [On-Ear Measures - Setup](#) and [On-Ear Measures Screen Setup](#), prepare the system, position the patient and insert the probe tube. (For tight-fitting hearing aids or earmolds, the probe tube may be inserted through a vent.)
3. Without disturbing the probe tube, insert the hearing instrument or earmold.
4. Referring to Audiometric data entry, enter threshold data.
5. Click on button 1, 2, or 3 in the REAR/REIG column. This will present a signal from the sound-field speaker and display the REAR Setup box.
6. Click on the [Stimulus] button, then select either [Pink noise] or [Swept]. Use pink noise when using the NAL-NL1 fitting formula. Use Swept when testing maximum output capabilities.
7. Click on the [Level] button, then select a stimulus level from the list box. Use the level shown on the target REIG curves (if present). If no level is shown, use the lowest level required to overcome background noise.
8. Adjust the hearing instrument so that the REIG curve (SPL scale) or Aided threshold curve (HL scale) approximates the dotted target curve. For multi-level targets, match each curve using the stimulus level indicated.
9. Click on to complete the test and capture the data.

See [Insertion gain in SPL](#) and [Insertion gain in HL](#) for an explanation of screen curves. See SII calculation in Insertion gain


NOTE: Because it is analyzed in 1/3 octave bands, the Noise stimulus produces a REAR up to 12 dB below that produced by the Swept stimulus at the same overall level. The REIGs will be similar except at levels which cause non-linear operation of the hearing instrument.

SII calculation in Insertion gain

The Axiom calculates a Speech Intelligibility Index (SII) value for unaided speech at 70 dB SPL (REUR), the target REIG and for each aided test. The calculations use the 1/3 octave band method of ANSI S3.5 –1997 without the 160 Hz band and without masking effects with results expressed as a % by multiplying by 100. Level distortion effects are included but no hearing loss desensitization is applied. An idealized speech region of LTASS +/- 15 dB is used.

The speech recognition associated with a given SII is a function of the test material and the cognitive abilities of the listener. There is considerable individual variability in relating the SII to speech recognition, especially amongst impaired listeners. The following shows expected nominal recognition scores vs SII for normals on the Connected Speech Test (Sherbecoe and Studebaker 2003).

SII (%)	% Cor.	% Cor.	% Cor.
	Digits	IEEE Sentences	NU6 Words
0	0	0	0
5	30	4	4
10	80	9	8
15	96	13	11
20	98	19	19
25	99	43	27
30	100	60	36
35	100	74	45
40	100	89	56
45	100	94	63
50	100	96	70
55	100	98	76
60	100	99	81
70	100	99	87
80	100	100	93
90	100	100	96

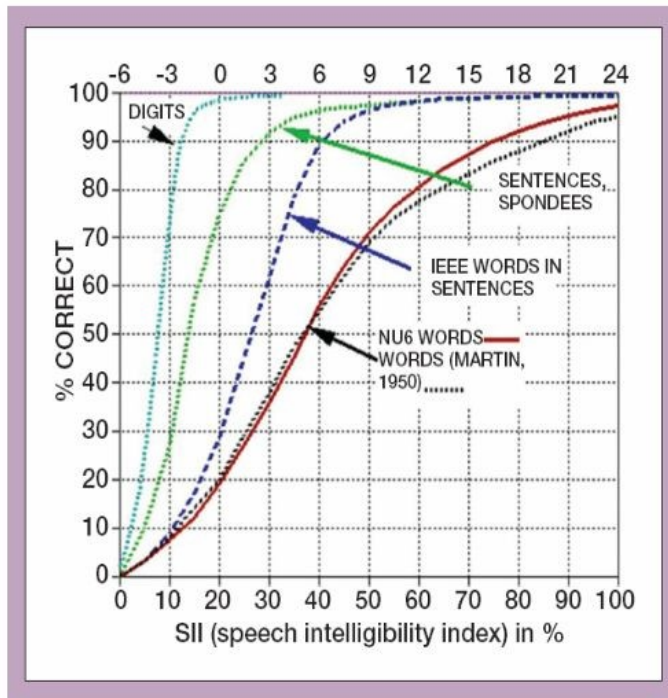


CROS fitting using Insertion gain

See [Insertion gain in SPL](#).


1. Place a probe module on each ear and insert the **probe tube into the good ear only**. Position the CROS instruments on (in) the ears and turn them ON.
2. Position the speaker at 45 degrees to the good ear and set Instrument to [BTE].

3. Select REAR 1 and record a response using [Pink noise] at 55. It should be like a normal REUR. If not, the ear may not be fully open.
4. Position the speaker at 45 degrees to the poor ear.
5. Set Instrument to [CROS].
6. Select REAR 2 and obtain a response using [Pink noise] at 55. Adjust the response to match that obtained in step 3.
7. Position the speaker directly in front of the client. Select REAR 3 and use [Pink noise] at 55 to check for a smooth response. Repeat with Instrument set to [BTE]. An irregular response may be the result of reflections from nearby objects (including the operator) or it may indicate phasing problems in the aids.

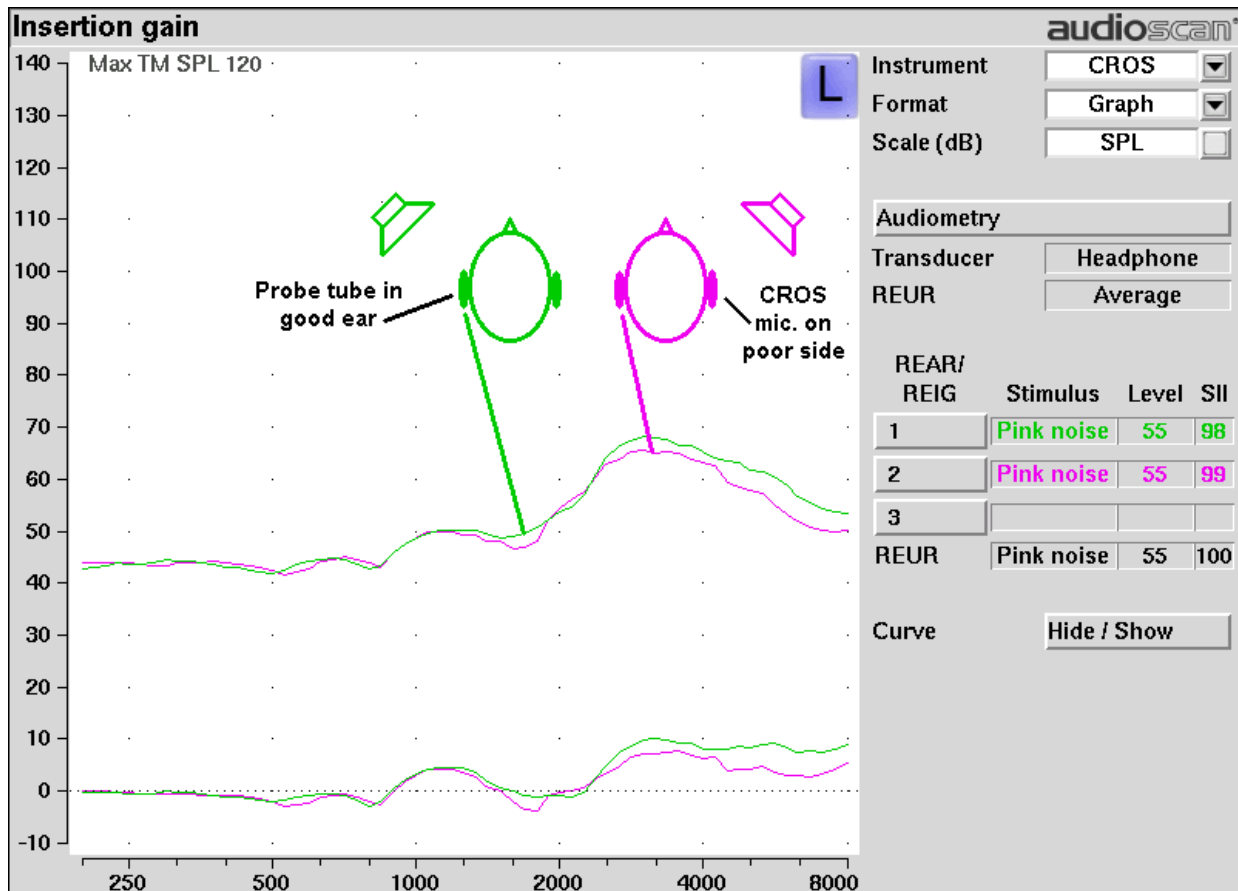

17 Speechmap

Speechmap presents hearing threshold, UCL and amplified speech spectra as SPL in the ear canal. The fitting goal is to make amplified speech audible and comfortable while avoiding discomfort for loud sounds.

Speechmap may be accessed from  as either a Test box measure or an On-ear measure. See [On-ear or Test box mode](#) for details.

Speechmap facts

Speechmap is a trademarked hearing instrument fitting environment introduced by Audioscan in 1992. It presents hearing threshold, UCL and amplified speech spectra as SPL in the ear canal. The fitting goal is to make amplified speech audible and comfortable while avoiding discomfort for loud sounds. Originally speech was simulated using amplitude-modulated tonal signals. The Axiom replaces these simulated speech signals with real speech, both recorded and live. For details of the test signals and analysis methods, see [Speechmap Technical Details](#).

Speechmap departs from insertion gain target methods in presenting amplification goals in the context of the residual auditory area, not as an isolated target line, and in the use of real speech, not tones or broad-band noise. This is extremely important for non-linear hearing aids, analog or digital, because their processing is signal-dependent. It also makes provision to correct audiometric data and 2cc coupler measurements for age-related or individually-measured acoustic differences between real ears and the couplers used for calibration and testing.

The Speechmap environment provides two hearing instrument verification options: *on-ear measures* and *test box measures*. See [On-ear or Test box mode](#) for details.

DSL 5.0 in Speechmap



Copyright DSL © 2004. Version 5.0

The *Desired Sensation Level (DSL)* method is a systematic eardrum sound pressure level (SPLogram) approach to hearing instrument fitting that seeks to ensure amplified speech will be audible and comfortable while loud sounds will not be uncomfortable. It takes into account individual acoustic factors in audiometric and electroacoustic data. DSL was developed by the National Centre for Audiology (NCA), London, Ontario. Audioscan has implemented DSL within its Speechmap fitting system since 1994.

In 2005, DSL underwent an extensive revision resulting in DSL 5.0. It was further revised in 2007 to version 5.0a. See [DSL 5.0 changes](#) for details of the changes.

The Audioscan version of DSL 5.0 may differ from others in the following ways:

1. DSL 5.0 uses RECD values for ages > 119 months from a different source than for ages up to and including 119 months; Audioscan has used the RECD values for a 119 month old for ages > 119 months so all values come from the same study.
2. Input-output curve targets and compression threshold targets are not provided. Such steady-state parameters are of little value in estimating amplified speech levels. Matching amplified speech to the LTASS targets at levels from 50 to 75 dB SPL provides much better assurance that fitting goals are being met.

3. There is no provision to input the number of compression channels. This may be useful when the signal used for verification is not the signal for which the targets were developed. It is irrelevant when using real-speech signals to match speech targets and when using narrow-band signals to match narrow-band maximum output targets.
4. The term Uncomfortable Level (UCL) is used rather than Upper Limit of Comfort (ULC) in referring to the ear canal SPL that should never be exceeded.
5. Broadband output limiting targets are not provided.

DSL 5.0 changes

The Audioscan version of DSL 5.0 includes provision for *bone conduction* and *ABR* threshold, *binaural* targets, targets for speech levels from *50 to 75 dB SPL*, *new targets* for children, *targets for adults* and *new RECD* default values for children (1 month to 10 years) and adults. The *UWO child's spectrum* has been removed and an *'own voice'* spectrum has been added. As a result of a re-analysis of published data, the *vocal effort effects* for *Soft and Average speech* levels have been made the same. In addition, the target symbol has been elongated to indicate that the fit-to *target is a range, not a point*.

The children's LTASSa targets for 70 dB speech are lower than those in previous versions of DSL. These differences are functions of the hearing loss and age (where average RECDs are used). Revisions made in version 5.0a reduce these differences at 250 Hz and for more severe losses. The differences will be more significant when the individual RECD values have not been measured for the child being fitted.

Audioscan assumes no responsibility for the validity of these changes; that responsibility rests with the National Centre for Audiology.

NAL-NL1 in Speechmap

The *NAL-NL1* procedure seeks to amplify speech such that all bands of speech are perceived with equal loudness while maximizing speech intelligibility and ensuring that the wearer perceives speech to be no louder than that which a normal hearing person would perceive. Although *NAL-NL1* states its goals for speech, it derives insertion gain targets assuming noise as a verification signal. Because it is more accurate to verify a non-linear fitting using the signal for which the procedure was developed, Audioscan converts the *NAL-NL1* insertion gain targets to *LTASS* targets for amplified speech in the *Speechmap* environment, resulting in *Speechmap/NAL-NL1*. The conversions use the same adult average RECD and REUG as used in *DSL*. The *RESR* targets are from *Hearing Aids* (Dillon, Thieme Publishing, 2001, p274).

An insertion gain version is available by selecting [Insertion gain] from the on-ear measures test menu. The pink noise stimulus should be used.

NAL-NL2 in Speechmap

Fitting targets, default age-related RECD and REUG are provided by a software library used under license from Hearworks Pty Ltd. which is solely responsible for their derivation.

This revision of *NAL-NL1* has many more input parameters and output target formats than its predecessor. In implementing *NAL-NL2*, Audioscan has made the following simplifications and clarifications:

1. *NAL-NL2* targets are provided as *REAR* targets in *Speechmap*. Insertion Gain targets are not provided.
2. There is no provision to select Insert earphones + earmold as the audiometric transducer since this invalidates the audiometer calibration and correction for this error is problematic.
3. Input-output curve targets and compression threshold targets are not provided. Such steady-state parameters

are of little value in estimating amplified speech levels. Matching amplified speech to the LTASS targets at levels from 50 to 75 dB SPL provides much better assurance that fitting goals are being met.

4. There is no requirement to input client gender. The impact of gender on REAR targets is less than +/- 1dB.
5. There is no requirement to input hearing aid experience. An "Experienced" user is assumed. The reduction in NAL-NL2 REAR targets for an inexperienced user is less than 4.5 dB.
6. There is no provision to generate targets for tonal languages. The REAR targets for tonal language differ less than 2.5 dB from those for non-tonal languages.
7. There is no requirement to input the number of compression channels. This may be useful when the signal used for verification is not the signal for which the targets were developed. It is irrelevant when using real-speech signals to match speech targets and when using narrow-band signals to match narrow-band maximum output targets.
8. There is no requirement to input compression speed. It has no impact on REAR targets.
9. There is no requirement to input earmold tubing type. It has no impact on REAR targets.
10. There is no requirement to input vent size or fitting depth. They have no impact on REAR targets.
11. For sound-field threshold the speaker is assumed to be at 45 degrees. For on-ear measurements, the speaker is assumed to be at 0 degrees and the reference microphone is assumed to be on the head surface.
12. The SII calculation is per ANSI S3.5-1997 and is the same for all fitting fitting methods.





Camfit in Speechmap

Camfit is an approach to fitting hearing aids developed by Brian Moore at the University of Cambridge, UK. It specifies the level-dependent gain required to correct abnormal loudness as modelled by his cochlear hearing loss simulation. It provides two approaches to correcting for the hearing loss:


Camfit restoration aims to amplify sounds that are soft, comfortable and loud for a person with normal hearing so that they are soft, comfortable and loud for a hearing aid wearer. This is the stated goal of the IHAF (International Hearing Aid Fitting Forum) fitting method.







Camfit equalisation aims to amplify speech to produce the same loudness in each critical band. It has been argued that this is likely to give the highest intelligibility for a given overall loudness. This is the rationale behind the NAL fitting methods and will generally produce similar targets.

Using Speechmap

The Speechmap fitting environment provides a number of distinctly different stimuli. Speech and noise are analyzed in 1/3 octave bands and for most speech stimuli data are presented as 3 curves. The top curve is the level exceeded 1% of the time (speech peaks), the lower curve is the level exceeded 70% of the time (speech valleys) and the middle curve is the average. Combined, these three curves depict the aided speech 'banana'. Starting a test by clicking on  causes the signal to be presented continuously – the recorded speech signals run in a loop - and the curves are updated frequently to show any changes you make to the settings of the hearing instrument. Clicking on  restarts the speech passage and the curves are then calculated for the entire passage, producing an accurate display of the speech region and LTASS, and an accurate SII calculation. When using live speech,  changes to  which may be selected to capture short-term speech features. See [SII calculation in Speechmap](#) and [Speechmap Technical Details](#).

Before starting, please see [On-Ear Measures - Setup](#), [Speechmap Fitting](#), [Speechmap Technical Details](#) and [On-Ear or Test Box Mode](#).


- **To access Speechmap:** click on , then click on [Speechmap] in either the On-ear menu or in the Test box menu.
- **To change screen setup:** see [On-Ear Measures Screen Setup](#), and [On-ear or Test box mode](#).
- **To set up hearing instrument type:** click on [Verification], then click on the Fitting list button and select from BTE, ITE, ITC, CIC, RITE (receiver in the ear – test box mode), Open (On-ear mode), Body, CROS (On-ear mode) or FM. This changes microphone location effects and the RECD used in test box measures and 2cc targets, and the reference microphone used for FM and CROS.
- **To enter audiometric data:** see [Assessment data entry](#).

To run a test, click on  or  or . Then, click on  to start the test. Once the test is started, click on the [Stimulus] list button to select the stimulus type. Click on the [Level] list button to select stimulus level. Click on  to run the long term average test and display the data. Click on  to cancel the test. See, [Screen tour & Speechmap Fitting Procedures](#).

Speechmap Setup

Speechmap provides a number of different test stimuli for fitting hearing aids and testing various aspects of their performance. Each time you run a test, you have the opportunity to select the stimulus from a list. You can manage which stimuli appear on this list by doing the following:

Click  and click on [Speechmap] in the Setup menu.

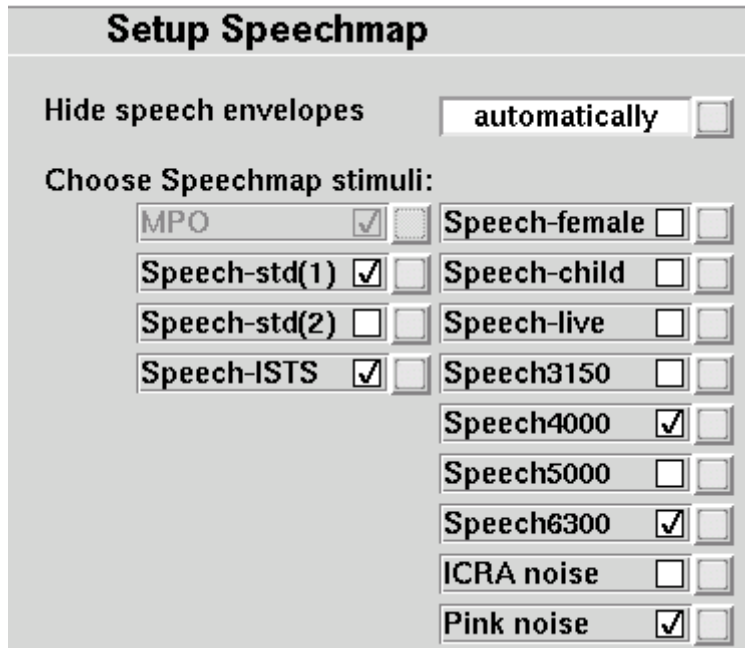


The available Speechmap stimuli are in two columns. The left-hand column contains the MPO stimulus and speech stimuli that have the specific Long-Term Average Speech Spectrum (LTASS) required when matching prescription fitting targets such as DSL or NAL-NL1. The MPO stimulus (used to verify that the maximum output SPL of the hearing aid does not exceed the uncomfortable level) is always selected. You must select at least one of the defined LTASS speech stimuli.

1. To select a stimulus, click the stimulus you want to add to the list. A check mark will be placed next to the selected stimulus.
2. To remove a checked stimulus from the list, click the stimulus you want to remove.


You can also choose to have the envelopes of the speech test results automatically hidden (leaving only the LTASS) when you start another test. Click the Hide speech envelopes button to toggle between automatically and manually.

When you have finished making your selections, click to save your selections and exit.

Screen tour - unaided screen

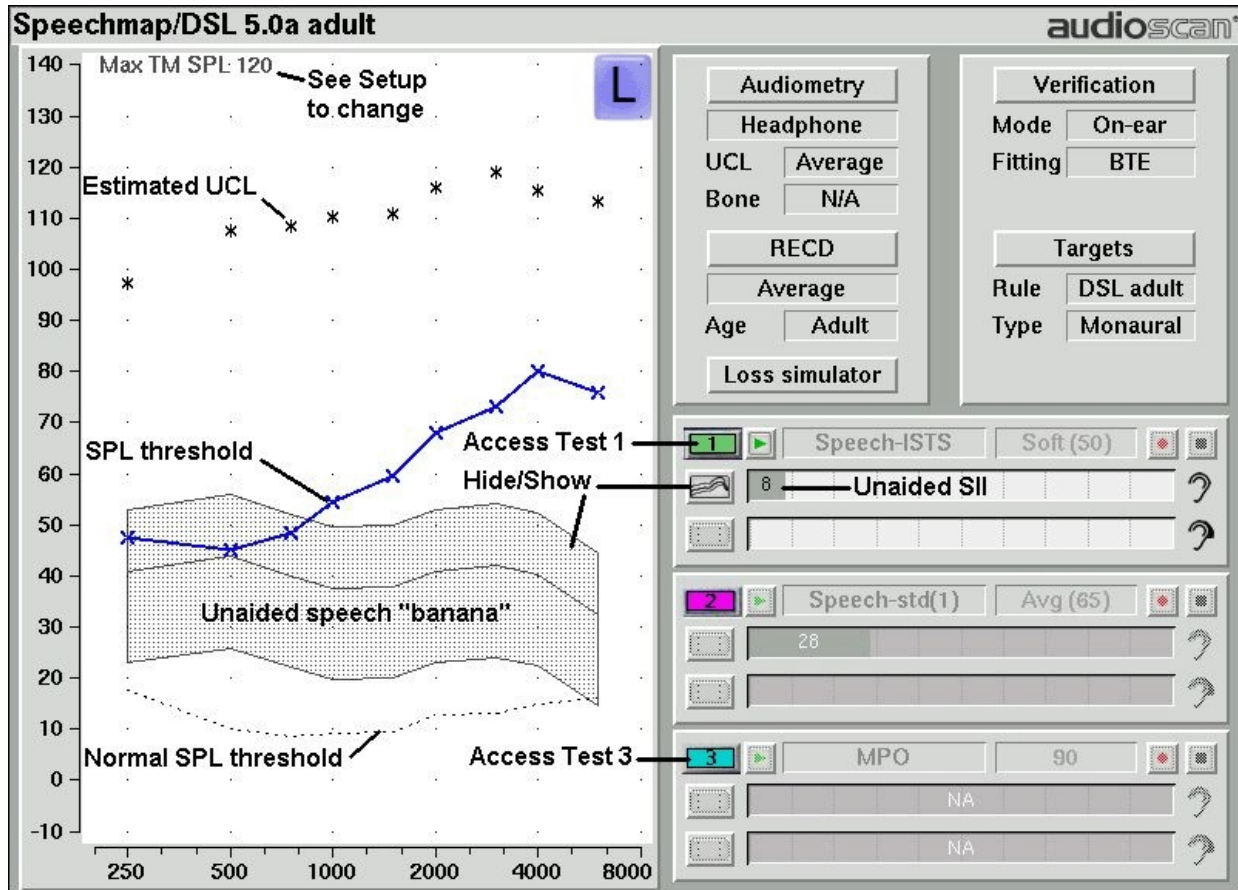

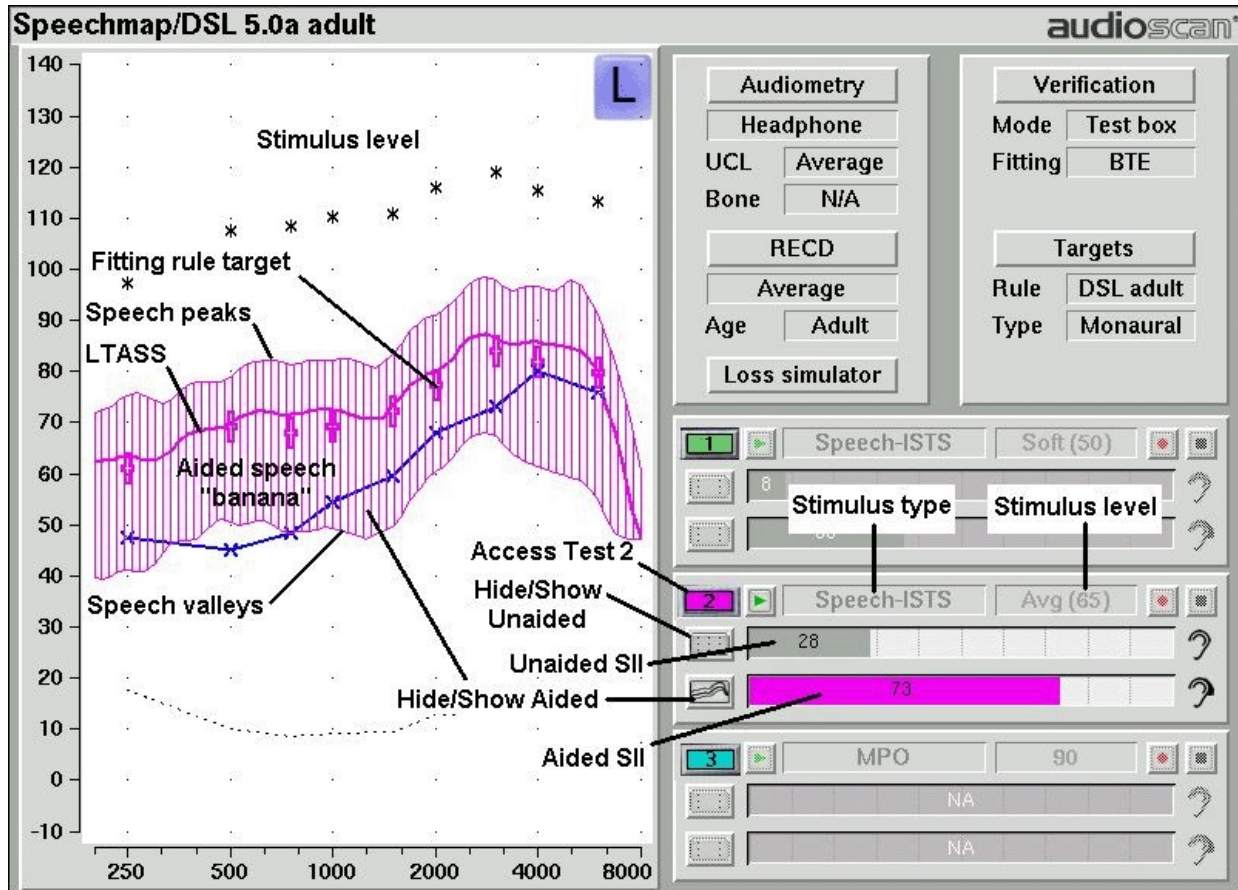

Screen tour - aided screen



On-ear or Test box mode

The Speechmap environment provides two hearing instrument verification options: *On-ear Speechmap* and *Test box Speechmap*. Using Speechmap in the On-ear menu, the stimulus is controlled by the on-ear reference microphone below the ear and SPL in the ear canal is measured by the probe microphone. Using Speechmap in the Test box menu, levels measured in the 2cc coupler are converted to estimated ear canal levels using the RECD values (age-appropriate average, entered, or measured). In addition, the stimulus delivered to the hearing instrument microphone is modified to include average microphone location effects for the hearing instrument type indicated. The screen display for both modes is the same, showing SPL, measured or simulated, in the ear canal.

For On-ear Speechmap: Click  and select [Speechmap] in the On-ear menu.

For Test box Speechmap: Click  and select [Speechmap] in the Test box menu. Review Calibrating test box reference microphone, Coupling the hearing instrument, Positioning the hearing instrument.

It is suggested that hearing instruments be preset in test box mode with fine tuning done in on-ear mode.


Note that on-ear mode should be used for vented or open fittings. Vent effects depend on the vent, the ear and the hearing aid settings and cannot be reliably predicted from 2cc coupler measurements.

SII calculation in Speechmap

The Axiom calculates a Speech Intelligibility Index (SII) value (in %) for unaided speech and for each aided test. The calculations use the 1/3 octave band method of ANSI S3.5 –1997 without the 160 Hz band and without masking effects with results expressed as a % by multiplying by 100. Level distortion effects are included but no hearing loss desensitization is applied. An idealized speech region of LTASS +/- 15 dB is used.

The speech recognition associated with a given SII is a function of the test material and the cognitive abilities of the listener. There is considerable individual variability in relating the SII to speech recognition, especially amongst impaired listeners. The following shows expected nominal recognition scores vs SII for normals when using a variety of speech stimulus conditions. (Killion and Mueller, Hearing Journal, January, 2010 pp 10-15)

SII (%)	% Cor.	% Cor.	% Cor.
	Digits	IEEE Sentences	NU6 Words
0	0	0	0
5	30	4	4
10	80	9	8
15	96	13	11
20	98	19	19
25	99	43	27
30	100	60	36
35	100	74	45
40	100	89	56
45	100	94	63
50	100	96	70
55	100	98	76
60	100	99	81
70	100	99	87
80	100	100	93
90	100	100	96

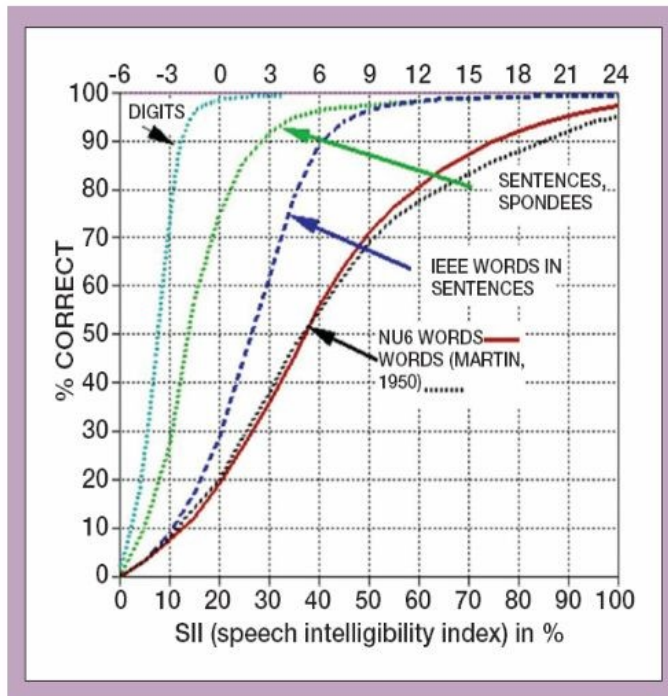

Using Custom Stimuli in Speechmap


Custom digital sound files can be used as stimuli in Speechmap. If you connect a USB stick with WAV files on it to the Axiom, Speechmap will make those files available as stimulus selections. This feature provides the freedom of using any stimulus in Speechmap with all the features that Speechmap offers. Please note that fitting targets are not available for custom stimuli.

A USB stick is required for this feature. It is highly recommended that the USB stick is formatted for data using the Audioscan Updater prior to using this feature. The Audioscan Updater comes with any current Audioscan software download or Cd. Visit www.audioscan.com to obtain software updates.

To use digital sound files as stimuli in Speechmap:

1. Connect a USB stick to your PC.
2. Format the USB stick for data using the Audioscan Updater.
3. Create digital sound files that are compatible with Speechmap (see [Creating WAV files for Speechmap](#)).

4. Copy the WAV files from the PC to the top level directory (i.e. not in any folder) of the USB stick.
5. Disconnect the USB stick from the PC and connect it to the USB port on the Axiom.
6. Run a test in Speechmap (see [Using Speechmap](#)).
7. Click on the [Stimulus] list button. Compatible WAV files will appear as selectable options in the stimulus list below the default stimuli. You may need to scroll down to see all the stimuli.
8. Select your custom stimulus. After a short delay, your stimulus will begin playing.
9. Use your new stimulus with all the features of Speechmap.

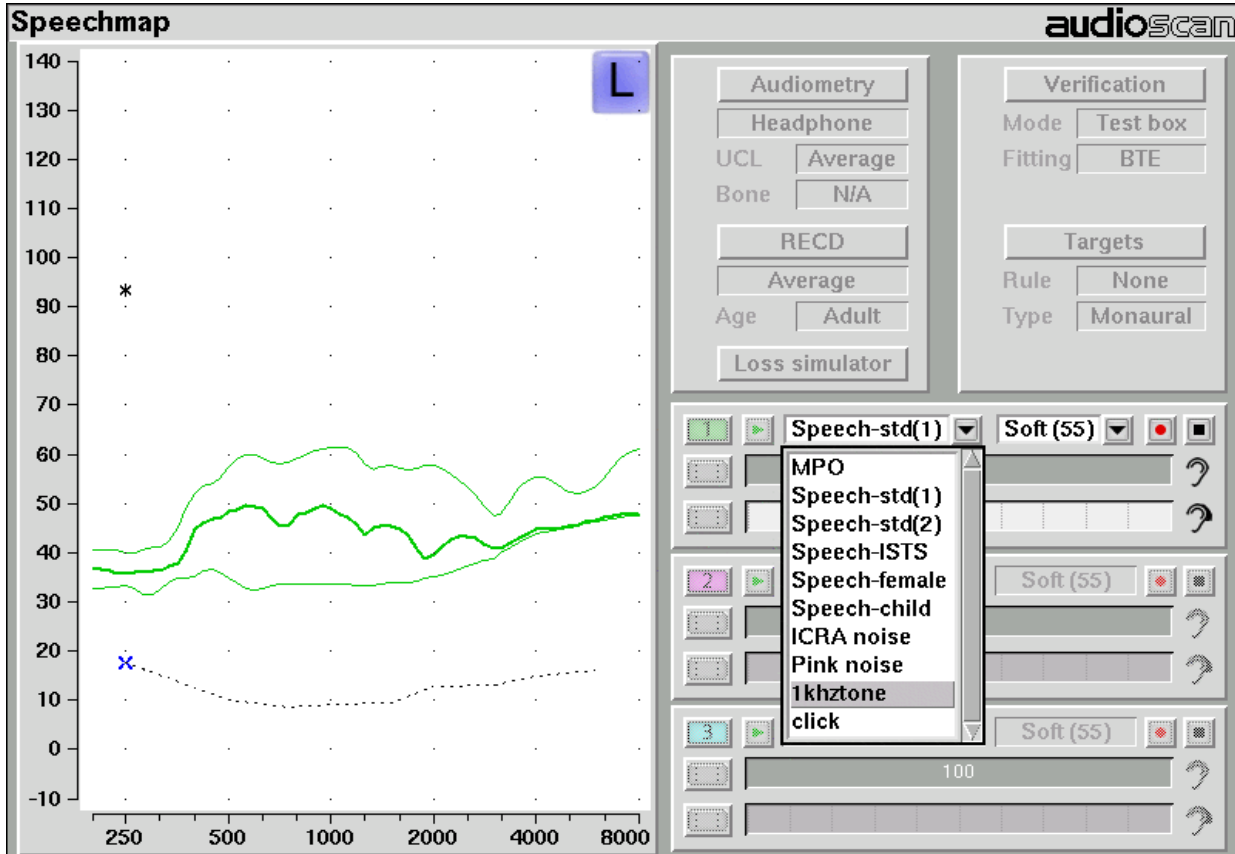

Example of two custom files, “click.wav” and “1khztone.wav”, in the stimulus list.

To remove custom stimuli from Speechmap: close any Test prompts and disconnect the USB stick from the Axiom.

Notes:

- Files that are greater than 20 seconds in duration will be truncated to 20 seconds.
- Speechmap will show up to 10 compatible WAV files. If you have more than 10, we recommend that you use multiple USB sticks or organize your files into folders, moving them into the top level directory of the USB stick as needed.
- Removing the USB stick while a test is running may cause unexpected behavior. Stop the test before removing the USB stick from the Axiom.

Creating WAV Files for Speechmap

WAV is a commonly used format for digital audio and can be created/edited by nearly all audio editing and recording software. A WAV file has many properties that defines the format of the digital audio. When a USB stick is connected to the Axiom, Speechmap looks for files in the top level directory that have the following properties:

File format: WAV

Data format: PCM (no compression)

Sampling Rate: 32 kHz

Bit precision: 16 or 32 bit

Channels: Mono

File name: 15 characters or less, not including .wav extension

Existing WAV files can easily be converted to match these specifications using your preferred audio editing software.

Leveling Options


By default, Speechmap will automatically level the WAV file so that the Long Term Spectrum matches the level selected for the test.

The automatic leveling of the WAV file can be modified by inserting a tag in the filename. This feature allows you to fine tune the output level of your stimulus. The tag must be added at the end of the filename but before the extension.

Example: **filename_+2.5.wav**

The tag must consist of an underscore, followed by a plus or minus sign, followed by the modifier in dB. In the example above, the tag “_+2.5” will level the file to be 2.5 dB above nominal.

There are no restrictions on the magnitude of the modifier; there is built-in protection against speaker overdrive. Precision is limited only by the 15 character limit of the filename.

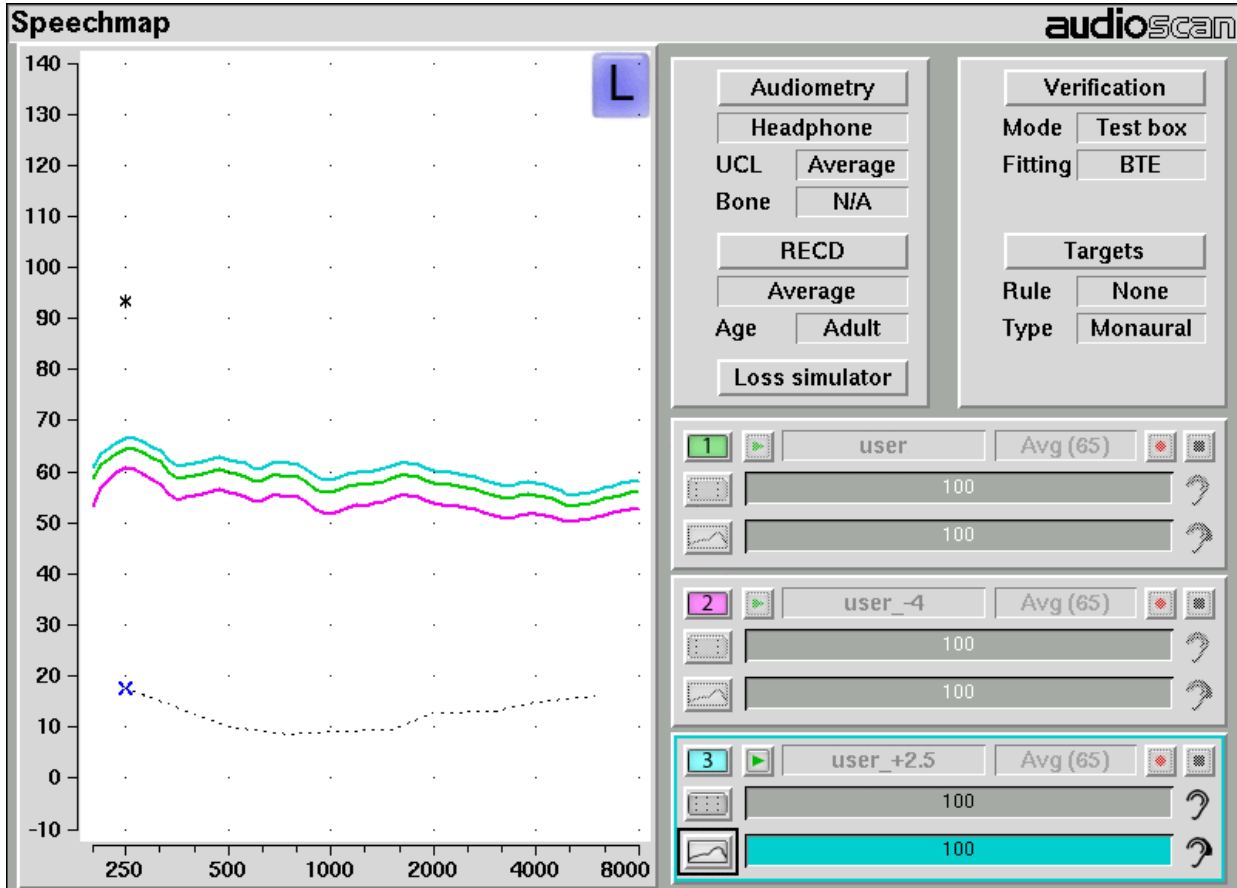

Example of user-supplied files, “user.wav”, “user_-4.wav” and “user_+2.5.wav”.

18 Speechmap Fitting Procedures

The Speechmap fitting screen provides a number of choices for inputting and applying audiometric data for the verification of hearing instrument fittings. The procedures in this section assume a familiarity with [Test Box Measures - Setup](#), [On-Ear Measures - Setup](#), [On-Ear Measures Screen Setup](#) and [Speechmap](#).

Speechmap screen choices

Speechmap choices are accessed by clicking on [Audiometry], [RECD], [Verification] or [Targets], which will cause a related selection poster to be presented. Changes made on the poster are applied and the poster closed by clicking on . The poster may be closed without applying the changes by clicking .

Defaults are the last-used choices if test setup memory has been turned on. See [Saving test setup](#).

Audiometry

Transducer:

Bone conduction:

UCL:

X ?

RECD


RECD:

Age:

Rule:

X ?

1. **Transducer:** Sound source used to obtain the audiogram. Select from [Headphone, Insert + Foam, or Soundfield]. If Headphone is selected, average adult real-ear to dial difference (REDD) is used to convert entered HL values to ear canal SPL. If *Insert + Foam* is selected, the appropriate real-ear to coupler difference (RECD) is used to convert entered HL values to ear canal SPL values. If *Soundfield* is selected appropriate real-ear unaided gain (REUG) values @ 45 degrees azimuth, will be used to convert HL values to ear canal SPL values.
2. **Bone conduction:** Select from [N/A or Enter]. If [Enter] is selected, an entry screen will be provided after the air conduction thresholds have been entered.
3. **UCL:** Uncomfortable Level. Select from [Enter or Average]. If Average is selected, UCL will be estimated from entered thresholds. If [Enter] is selected, an entry screen will be provided after the air (and bone) conduction thresholds have been entered. If measured UCL values are entered, these values will be converted to real-ear SPL and used as real-ear saturation response (RESR) targets. Note that DSL 5.0 has changed this term to Upper Limit of Comfort (ULC). Regardless of the acronym, it is a narrow-band SPL that should never be exceeded.
4. **RECD:** Real-ear to coupler difference. Select from DSL Average, Enter, Measure if the target rule is DSL child, NL2 Average, Enter, Measure if the target rule is NAL-NL2 or Average, Enter, Measure otherwise. The RECD is used to convert entered HL data (thresholds and UCL's) to SPL values on the Speechmap screen (when Insert +Foam is selected as Transducer) and to convert coupler SPL measured in the test box to estimated ear canal SPL. The DSL child and NAL-NL2 fitting rules provide differing age-appropriate average RECDs which will result in differing SPL thresholds and differing estimated ear canal SPL. If Enter is selected, an entry screen will be provided when the poster is closed. If Measure is selected, the Measure RECD screen is provided when the poster is closed. See RECD measurement.
5. **Age:** Select from Adult, 6 – 10 years, 1 – 60 months. This changes the RECD applied if DSL Average or NL2 Average has been selected when using these target rules.

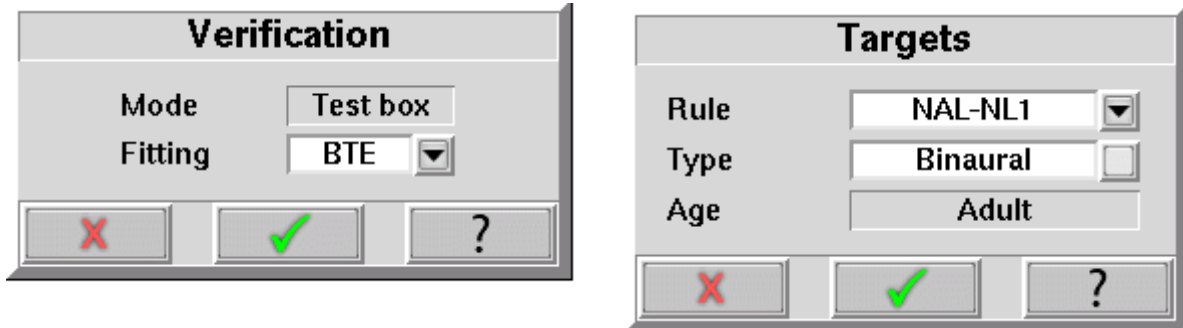

6. **Fitting:** Type of instrument or fitting to be verified. In Test box mode, select from BTE, ITE, ITC, CIC, RITE (receiver in the ear), Body or FM. In On-ear mode, select from BTE, ITE, ITC, CIC, Open, Body or FM.
7. **Rule:** Fitting rule. Select from DSL Adult, DSL Child, NAL-NL1, NAL-NL2 or None.
8. **Type:** Select from Binaural or Monaural fitting targets if DSL Adult, NAL-NL1 or NAL-NL2 has been selected. For DSL child, only targets for a monaural fitting are provided.
9. **Loss simulator:** An electro-acoustic simulation intended to allow normal-hearing listeners to hear sounds as if they had the elevated threshold and altered loudness perception caused by outer and inner hair cell damage. See Sensory loss simulator.



Data entry

Threshold data and other audiometric parameters may be entered by scanning the barcode(s) on a previous Audioscan printout or manually using a mouse or an external keyboard. See Mouse, keyboard, barcode scanner, and Input device operation.

To enter data from a barcode see Barcode data input and:


1. Press the barcode trigger and center the red line on each barcode from a distance of 6 – 8 in. (15 – 20 cm). The scanner will beep and the Barcode Entry poster will open. A green check mark indicates which data have been entered. It does not matter in which order the barcodes are scanned or if each is scanned more than once.
2. Click on [Done] to accept the data checked on the poster.






To enter audiometric data manually:


1. On the Threshold, UCL or RECD entry screens, enter each data point using a mouse or an external keyboard. Left click the mouse to enter a point. Click again to delete the point. On the keyboard, the arrow keys change frequency and level and the ENTER key on the numeric keypad enters or deletes a point. The keyboard's numeric keypad can also be used to type in data.
2. Click on  to save the data and exit the entry screen. Click on  to exit the entry screen without saving the data.



Fitting to targets for soft speech

See [Speechmap screen choices](#) for setup and audiometric data entry. See [Using Speechmap](#).

1. Follow the instructions in Speechmap screen choices to enter audiometric data and select the fitting rule.
2. Click on  to select the first Speechmap test. The Test 1 box is outlined to indicate it is active.

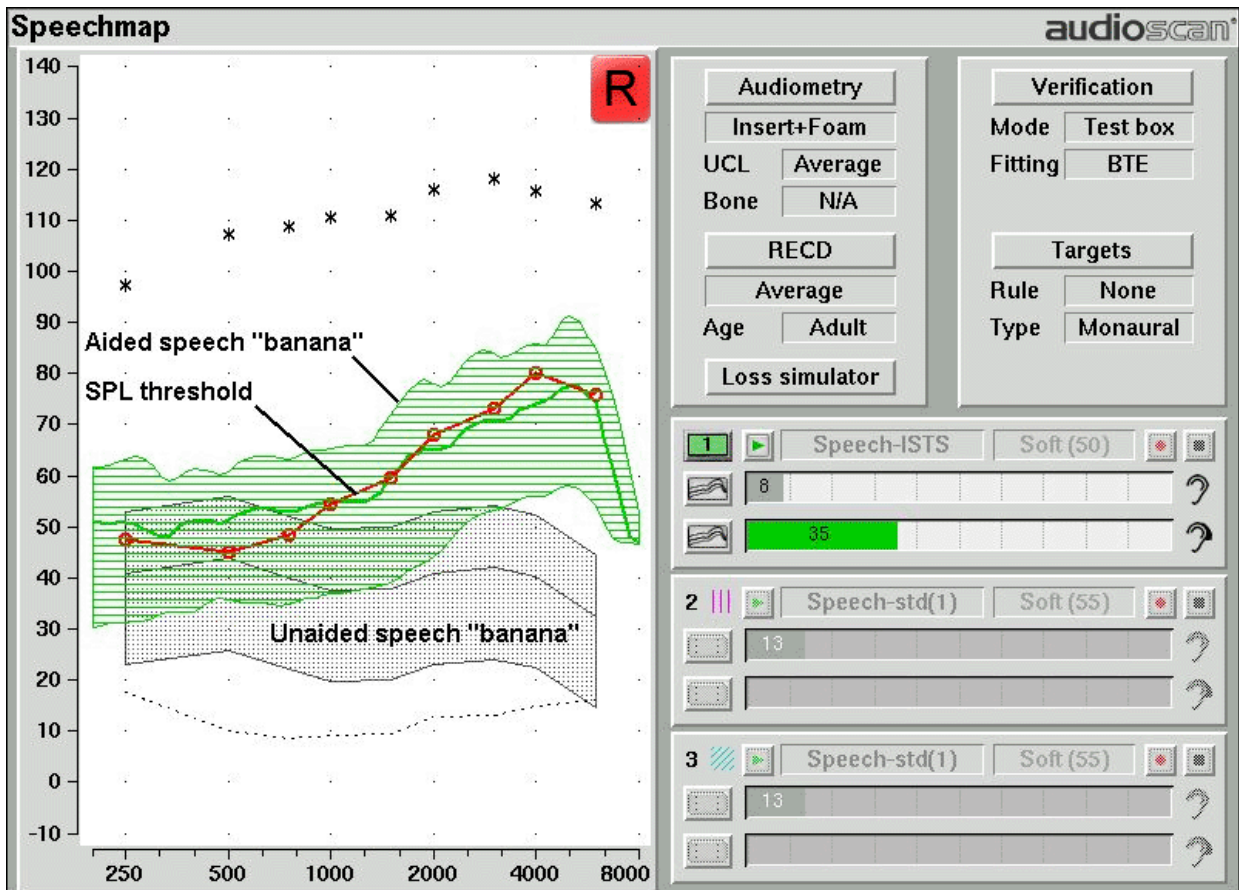3. Click on  to start this test. Once the test is started, click on the [Stimulus] list button to select the stimulus type as Speech-std(1), Speech-std(2) or Speech-ISTS. Click on the [Level] list button and select a stimulus level of 50 dB. The bar beside the  shows the SII for the unaided stimulus at 50 dB SPL (the Unaided Audibar). *To change the default stimulus type and level see [Saving test setup](#).*
4. While the passage is being presented, adjust the hearing instrument's frequency shaping and gain for soft sounds so that the middle curve (the LTASS) falls within the target range, shown by the  symbols, especially between 500 - 4000 Hz. These target symbols are automatically adjusted for the stimulus level you select.
5. Click on  to signal average and store a complete passage. Repeat as necessary.
6. Click on  to stop the test without recording any test data.




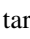





Upon completion of a test recording, a colored Aided Audibar beside the  shows the SII for the aided soft speech signal. The difference between the unaided and aided SII score (and the associated length of their respective 'Audibars') visually quantifies the speech intelligibility improvement for soft speech likely provided by the hearing instrument being fit. To further demonstrate the improvement, the unaided speech 'banana' may be displayed by clicking  beside the Unaided Audibar.



Fitting to targets for average speech

See [Speechmap screen choices](#) for setup and audiometric data entry. See [Using Speechmap](#).

1. Follow the instructions in [Speechmap screen choices](#) to enter audiometric data and select the fitting rule.

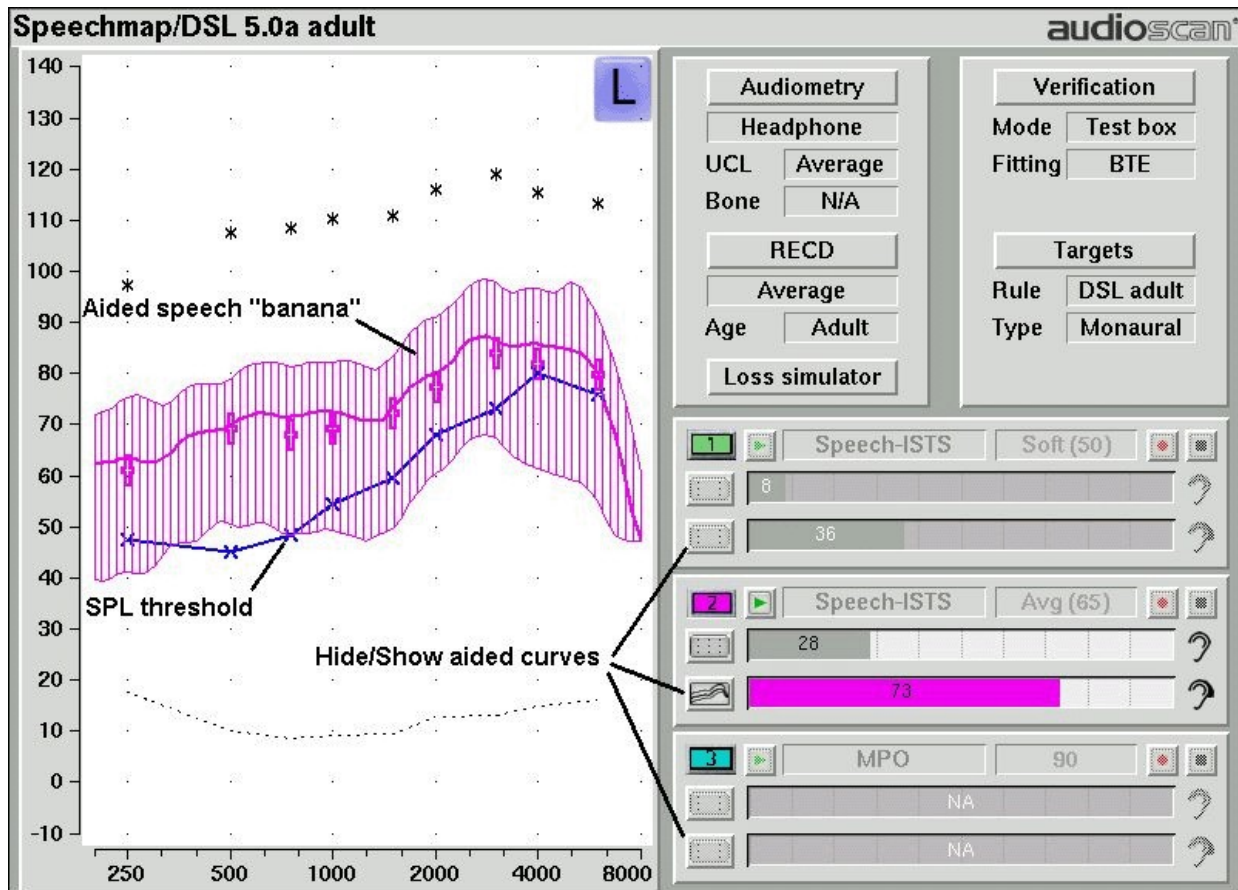


2. Click on  to select the second Speechmap test. The Test 2 box is outlined to indicate it is active.
3. Click on  to start this test. Once the test is started, click on the [Stimulus] list button to select the stimulus type as Speech-std(1), Speech-std(2) or Speech-ISTS. Click on the [Level] list button and select a stimulus level of 65 dB. The bar beside the  shows the SII for the unaided stimulus at 65 dB SPL (the Unaided Audibar). *When you start a new test, the speech envelope of the previous test may be automatically hidden. See [Speechmap Setup](#) to change this behaviour. To change the default stimulus type and level see [Saving test setup](#).*
4. While the passage is being presented, adjust the hearing instrument's frequency shaping and gain for average sounds so that the middle curve (the LTASS) falls within the target range, shown by the  symbols, especially between 500 - 4000 Hz. These target symbols are automatically adjusted for the stimulus level you select.
5. Click on  to signal average and store a complete passage. Repeat as necessary.
6. Click on  to stop the test without recording any test data.




Upon completion of a test recording, a colored Aided Audibar beside the  shows the SII for the aided soft speech signal. The difference between the unaided and aided SII score (and the associated length of their respective 'Audibars') visually quantifies the speech intelligibility improvement for soft speech likely provided by the hearing instrument being fit. To further demonstrate the improvement, the unaided speech 'banana' may be displayed by clicking  beside the Unaided Audibar.

Adjusting the Maximum Output Level

See [Speechmap screen choices](#) for setup and audiometric data entry. See [Using Speechmap](#).

1. Follow the instructions in Speechmap screen choices to enter audiometric data and select the fitting rule.
2. Click on **3** to select the third Speechmap test. The Test 3 box is outlined to indicate it is active.
3. Click on  to start this test. Once the test is started, click on the [Stimulus] list button to select the stimulus type as MPO. *When you start a new test, the speech envelope of the previous test may be automatically hidden. See [Speechmap Setup](#) to change this behaviour. To change the default stimulus type and level see [Saving test setup](#).*
4. In Test box mode, while the 90 dB sweep is being presented, adjust the hearing instrument's high-level gain and/or output limiting (e.g. MPO, PC) so that the curve matches the + REAR90 targets (DSL) or approaches but does not exceed the UCL estimates (*) at all frequencies.
5. Click on  to run and store a complete sweep. Repeat as necessary.
6. In On-ear mode, advise the client to point to the screen if a sound becomes too loud during the test (see Max TM SPL setup). Click on  to run and record a single sweep. Adjust the gain and output for loud sounds as for test box measures, factoring in client response.

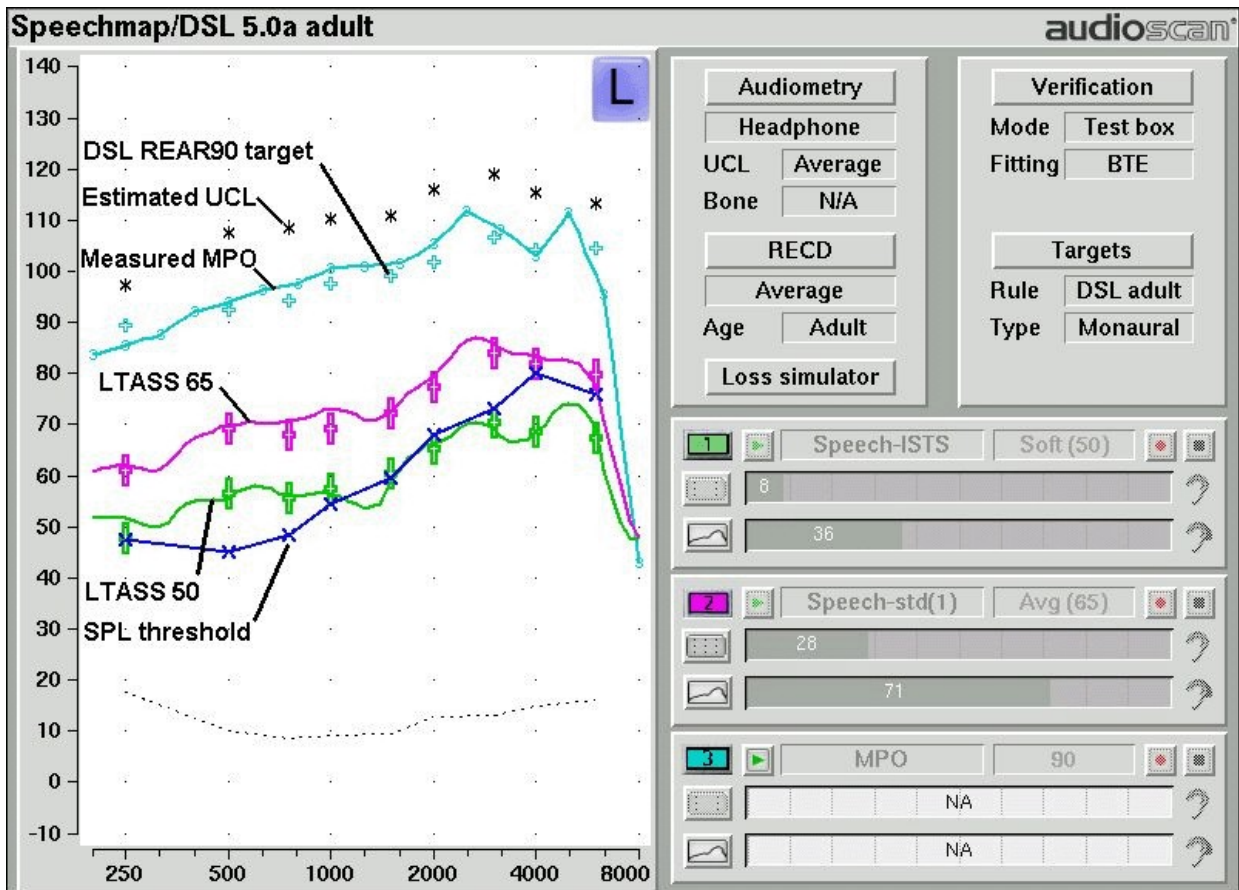

Open fittings in Speechmap


On-ear mode should be used for open and vented fittings. There are no special real-ear SPL targets for open fittings - it doesn't matter how the sound reaches the TM, the perception is the same. However, the sound that escapes from a vent or open fitting may be detected by the on-ear reference microphone, interfering with the sound-field equalization. For this reason, the hearing instrument must be muted during sound-field equalization.

1. With Speechmap in the [On-ear] mode, click on the [Verification] button and select [Open] from the Instrument pull-down menu.
2. Position the client and probe tube and enter audiometric data. See [On-Ear Measures - Setup, Speechmap, and Fitting to targets for soft and average speech](#) and [Adjusting the Maximum Output Level](#).
3. Place the open-fit hearing instrument on the ear with the probe tube in place, making sure to mute the hearing instrument.
4. Access Test 1, 2 or 3. If this is the first test since selecting Open, you will be prompted to mute or turn off the hearing instrument and click on the Equalize button on the setup poster. After equalization, unmute or turn the instrument on and follow the fitting procedures presented previously in this section. *It is necessary to repeat the Equalize procedure whenever the client or nearby objects are moved.*




Verifying Frequency Compression/ Frequency-Lowering Hearing Instruments in Speechmap

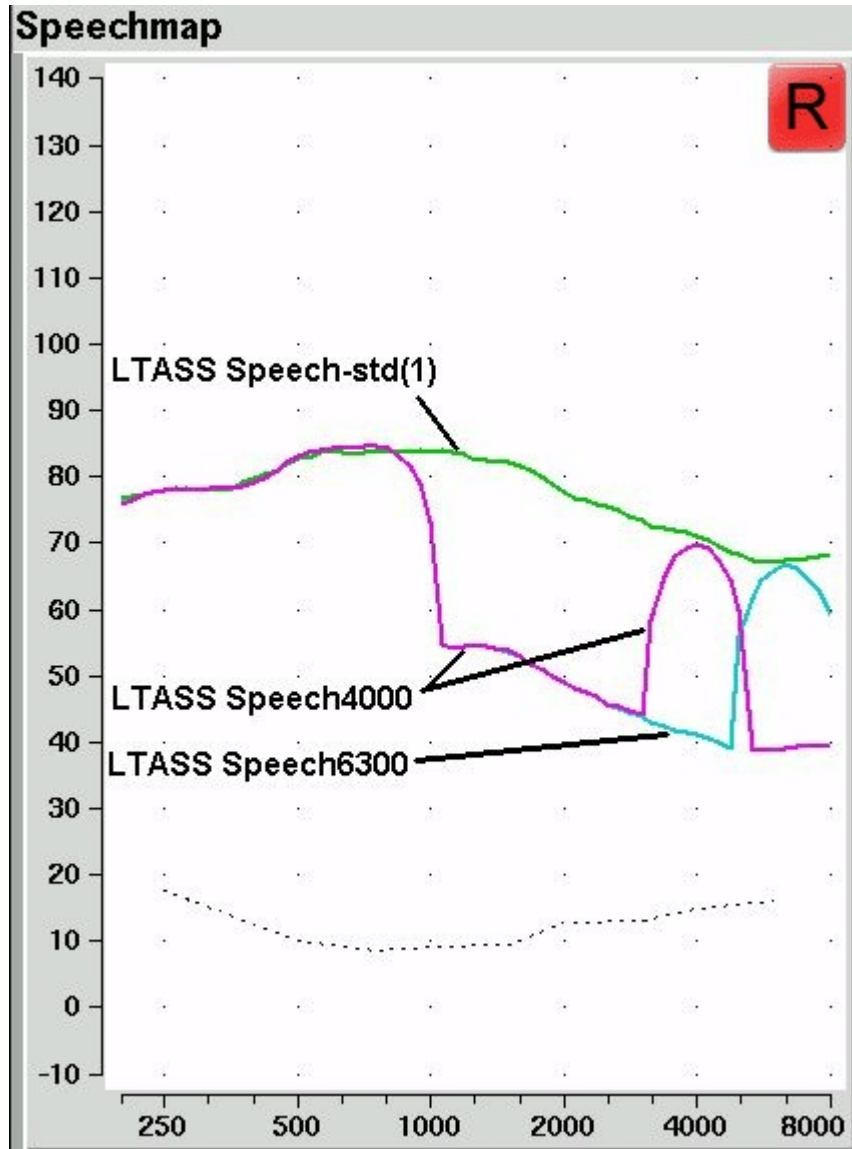

Frequency lowering is used when it is not possible to amplify the high frequency components of speech sufficiently to raise them above threshold. In this case, the high frequency components may be shifted to a lower frequency with a better hearing threshold where the available gain will render them audible.


Four modified versions of the Speech-std (1) test stimulus (Speech3150, Speech4000, Speech5000 and Speech6300) are provided in Speechmap to assist in verifying and adjusting frequency lowering devices. In each of these modified test stimuli, the 1/3 octave band levels above 1000Hz are reduced by 30dB, except for an isolated 1/3 octave band centered at the frequency indicated in the selected test stimulus' name. With these reduced band levels, the resulting LTASS produces a distinct 'cavity' between 1000Hz and the selected high frequency band, as seen below for the FM Boom test signal.



To test frequency-lowering:

1. Click on  and choose Speechmap from either the On-ear or Test box menus.
2. Program the hearing instrument so that the frequency-lowering feature is inactive.
3. Access Test 1, 2 or 3, then, click on  to start that test.
4. Choose Speech3150, Speech4000, Speech5000 or Speech6300 from the Stimulus pull-down menu. Choose 65dB SPL from the Level pull-down menu. Note that only the LTASS is shown for these stimuli and targets (if selected) have been suppressed.
5. Click on  to measure and store the long-term signal-averaged result. If the isolated band does not appear in the Test 1 curve, it may indicate that the hearing aid has no gain at the selected band frequency. This does not mean that the hearing aid will be unable to transpose the band to a lower frequency when frequency lowering is enabled.
6. Program the hearing instrument to activate the frequency-lowering feature.

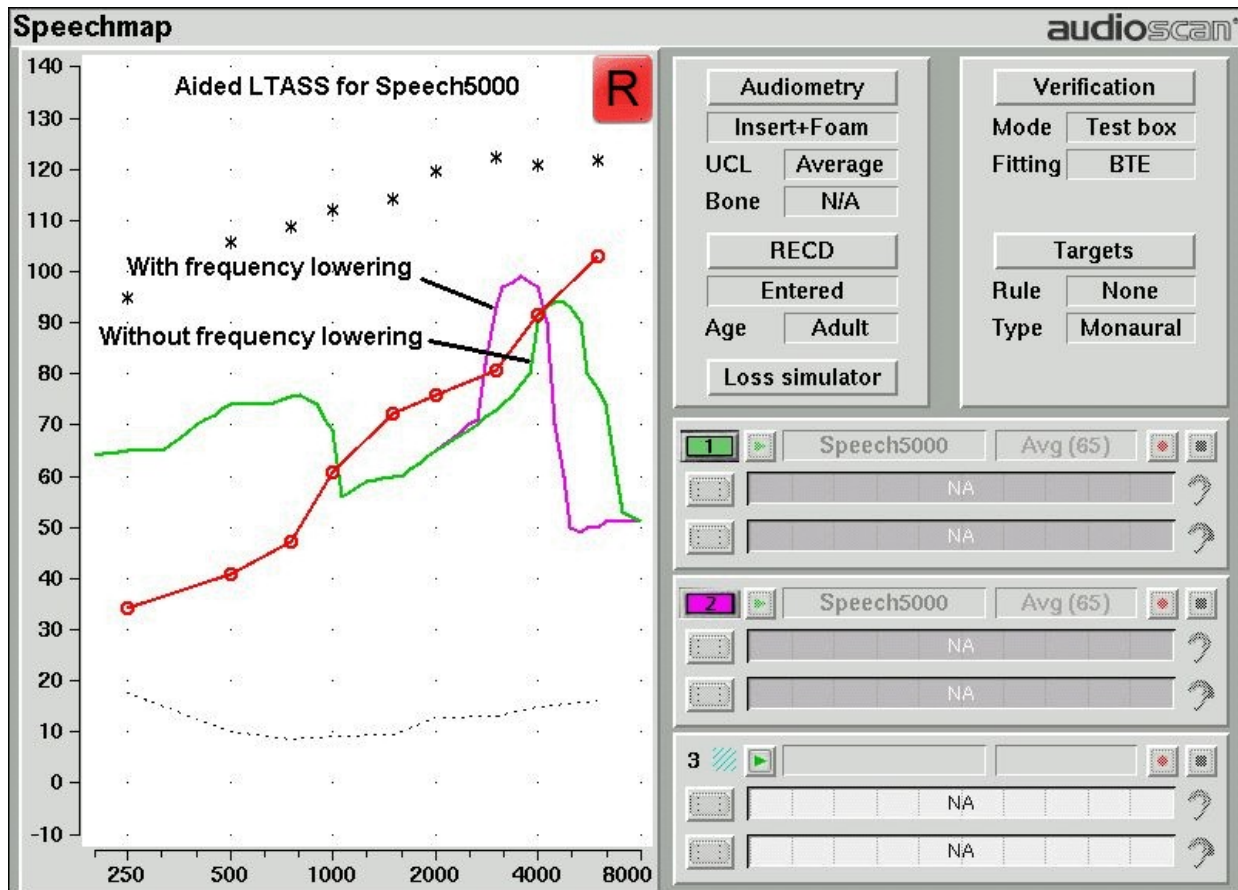

7. Click on another Test button, click on  and select the same stimulus and stimulus level used in Test 1.
8. Click on  to measure and store the long-term signal-averaged result.
9. The isolated band should now appear at a lower frequency and should reach or exceed threshold at its new location as shown below. Note that the 'cavity' between 1 kHz and the isolated band may be partially or completely filled in by noise within the hearing aid which will make the isolated band appear less distinct in the test curves.
10. Once the frequency-lowering properties of the hearing instrument have been verified, use one of the Speech-std stimuli to verify aided speech audibility or target match (as described elsewhere in this section) for frequencies below the isolated band.




FM fitting and verification

FM Offset or FM Transparency method in Speechmap (after Platz 2004)

It is assumed that the hearing aid has been set up for optimal audibility and maximum output. See [Speechmap screen choices](#) for setup and audiometric data entry. See [Using Speechmap](#)

1. Before attaching the FM receiver, couple the hearing aid to the 2cc coupler in the test box, click on  and select Speechmap from the Test box menu. Click the [Verification] button and select the appropriate Fitting type (typically BTE).
2. Click on the Test 1 button and , then choose Speech-std(1), (2) or the ISTS at a Level of 65 dB. Click on

 to run and record the speech passage.

3. Attach the FM receiver to the hearing aid and set the FM receiver to the default setting. The FM transmitter should be ON and set to MUTE. Repeat step 2 but use Test 2 instead of Test 1. If the results of Test 1 & Test 2 are not similar, the presence of the FM receiver has changed the intended performance of the hearing aid. Where possible, make programming adjustments to compensate -being careful not to make adjustments that will change Test 1 results.
4. Move the hearing aid, attached to the coupler and microphone, outside the test box at least 12' away from the FM transmitter*. Put the FM transmitter microphone, set to Omni, within 2mm of the reference microphone in the test box. Close the lid of the box.
5. With the FM transmitter unmuted, repeat step 2. Note that the hearing aid is outside the test box during this test - Quiet please! The results should be within +/- 2 dB of the results from step 3. If not, change FM settings and repeat.

* A coupler mic. extension cable is available. See *Microphone connection*.

Real-ear verification of FM-only open fittings in Speechmap (based on AAA Clinical Practice Guidelines: Remote Microphone Hearing Assistance Technologies for Children and Youth from Birth to 21 Years. April 2008)

Open fittings must be verified in On-ear mode. See [Speechmap screen choices](#) for setup and audiometric data entry. See [Using Speechmap](#)

1. Enter Audiometry data and select FM from the Fitting list box.
2. Position the transmitter microphone as in step 4 above. Place the probe tube in the ear canal and couple the FM-only device to the ear canal as in normal use.
3. At maximum volume setting, select the MPO stimulus. Adjust the receiver settings to meet targets. See [Adjusting the Maximum Output Level](#).
4. At use volume setting, select Speech-std (1), (2) or the ISTS and choose FM chest (84 dB SPL) or FM boom (93 dB SPL) for the level. Adjust the FM receiver gain/volume settings to meet targets in the 1000 – 4000 Hz range.
5. Repeat step 3 at use volume setting.

19 Speechmap Technical Details

Speechmap provides a variety of stimuli for the testing of non-linear hearing aids with a range of digital processing features. Additional information regarding the stimuli and their analysis is provided in Appendix 2 of the printable User's Guide.

Speechmap stimuli

The Speechmap fitting environment provides a number of well-controlled, distinctly different stimuli for both On-ear and Test box measurement modes. Four different digitized speech passages (2 male, 1 female and 1 child) are provided as well as the International Speech Test Signal (ISTS) and the single-talker International Collegium of Rehabilitative Audiology (ICRA) distorted speech signal. In addition, one of the speech passages has been filtered to provide 3 variations for evaluating frequency-lowering hearing aids. Pink noise at levels of 50 to 75 dB SPL in 5 dB steps and a tone burst at 85 or 90 dB SPL are also available.

The two 'Speech-std' signals are by the same male talker, filtered to provide the long-term average speech spectrum (LTASS) recommended by Cox & Moore (1988) for average vocal effort. This is the LTASS assumed in the Desired Sensation Level (DSL) method of hearing aid fitting. In addition, one of these passages (the Carrots passage) has been filtered to suppress the 1/3 octave bands above 1 kHz by 30 dB with the exception of a 1/3 octave band at 4 kHz (Speech4000), a band at 5kHz (Speech5000) and a band at 6.3 kHz (Speech6300). These latter three signals may be used to determine the amount of frequency shift provided by frequency-lowering hearing aids and the sensation level of the lowered components of the speech signal. The ICRA noise, female and child speech signals are presented as recorded with no spectral shaping.

The ISTS consists of 500 ms segments from recordings of 6 female talkers reading the same passage in American English, Arabic, Chinese, French, German and Spanish. These segments have been spliced together with appropriate pauses and filtered to match the average female spectrum from Byrne et al, An international comparison of long-term average speech spectra. *J. Acoust. Soc. Am.* 96 (1994), 2108-2120. The 15 second version of this signal has been provided.

Live speech may also be used.

As a result of reanalysis of published data, the previous distinction between 'soft' and 'average' vocal effort has been eliminated. Consequently, the speech, ISTS and ICRA signals are presented unfiltered for levels from 50 to 70 dB SPL. At 75 dB SPL, the spectrum is filtered to represent a 'loud' vocal effort. An 'own voice' filter is also available at the 75 dB SPL level. This is intended to produce a spectrum and level at a BTE microphone location similar to that produced by the voice of an average wearer.

A tone burst (MPO) stimulus provides 128 ms bursts with 128 ms gaps at 1/3 octaves with a level of 90 dB SPL in the test box and 85 dB SPL for on-ear mode.

In test box mode, all signals are filtered to include BTE, ITE, ITC, CIC, Body microphone location effects. The selected overall rms level is established prior to filtering.

For fitting FM systems, the stimuli may be modified to represent the level (84 dB SPL) and spectrum at a chest microphone location and the level (93 dB SPL) and spectrum at a boom microphone location. These are presented in the test box in both on-ear and test box modes.

The overall SPL, 1/3rd octave band SPL, vocal effort and microphone location effects are given in the following tables for the various broad-band signals available.

Stimulus levels

LTASS for standard speech is from Cox & Moore (1988). Loud spectrum is from ANSI S3.5-1997. Own voice spectrum is from Cornelisse et al (1991) adjusted to a 1 m reference.

<i>1/3 octave band SPL</i>					<i>Add to band SPL for</i>					
<i>Hz</i>	<i>Speech</i>				<i>ICRA Noise</i>	<i>Pink Noise</i>	<i>Loud</i>	<i>Own Voice</i>	<i>FM Chest</i>	<i>FM Boom</i>
	<i>Standard</i>	<i>ISTS</i>	<i>Female</i>	<i>Child</i>						
200	56.8	57.0	54.9	32.3	52.9	53	-2.2	.5	9.0	19.2
250	56.0	57.0	53.1	51.5	55.9	53	3.3	10.7	12.0	22.2
315	53.0	53.8	48.4	57.3	56.9	53	9.6	11.0	16.6	26.0
400	57.0	57.4	55.8	54.1	57.5	53	6.1	10.0	13.3	22.8
500	58.0	57.4	57.3	53.2	57.8	53	7.7	10.1	13.2	25.3
630	55.0	56.4	58.1	61.2	54.3	53	12.4	10.5	21.0	29.1
800	52.5	53.7	54.6	56.5	52.8	53	14.4	10.8	26.4	32.1
1000	51.0	50.0	52.0	52.7	50.4	53	15.0	10.5	26.4	32.7
1250	50.5	48.0	51.0	45.4	51.3	53	14.8	5.5	22.0	33.6
1600	48.0	47.4	51.1	41.1	48.5	53	15.5	4.0	22.1	33.9
2000	45.0	44.9	51.2	39.7	47.0	53	16.2	6.0	21.0	33.0
2500	44.0	42.8	49.3	35.5	44.7	53	14.8	6.0	18.2	31.9
3150	42.5	42.7	46.0	28.8	43.3	53	14.5	2.0	14.4	30.4
4000	42.0	41.0	40.0	31.2	41.1	53	13.2	-0.5	11.8	29.0
5000	40.0	40.3	36.1	30.6	39.6	53	9.2	-1.0	12.4	27.8
6300	41.5	41.3	43.5	32.6	33.5	53	4.2	0.0	11.4	26.2
8000	41.5	40.4	39.9	35.2	28.2	53	2.7	-4.0	11.0	25.8
Overall	65.0	65.0	65.0	65.0	65.0	65	75.0	75.0	84.0	93.0

Microphone location effects

<i>Microphone Location Effects</i>					
Add to all stimuli in test box mode					
	Hearing instrument type				
Frequency	BTE	ITE	ITC	CIC	BODY
200	0.5	0.3	0.0	0.9	3.0
250	0.5	0.5	0.3	0.8	3.0
315	0.8	0.8	0.3	0.7	3.0
400	1.1	1.0	0.7	0.5	3.0
500	1.2	1.8	0.0	0.4	4.0
630	1.1	2.0	0.1	0.4	3.0
800	0.9	2.0	0.4	0.4	2.0

1000	0.3	1.5	1.2	0.6	0.0
1250	0.6	0.3	-1.6	1.1	0.0
1600	2.5	-0.3	-1.9	2.0	-4.0
2000	4.1	3.8	2.1	3.3	-4.0
2500	3.5	5.0	4.8	5.0	-3.0
3150	2.8	3.3	3.5	6.9	-2.0
4000	3.7	4.3	6.4	8.3	0.0
5000	-1.2	4.3	6.6	7.6	0.0
6300	1.6	-0.4	-1.8	4.2	0.0
8000	3.3	1.0	-1.9	-5.6	0.0

Microphone location effects are added after the overall rms level has been set

Speech signal analysis

One of the most-used measures of a speech signal is the long-term average speech spectrum (LTASS). This is a 1/3 octave spectrum averaged over a sufficiently long portion of the speech material to provide a stable curve. In practice a 10 second average meets this requirement and, for this reason, all Verifit passages are at least 10 seconds long.

The dynamic nature of speech is often characterized by the distribution of short-term levels in each 1/3 octave band. These levels are determined by calculating a spectrum for each of a series of short time periods within the passage. Historically, time periods of 120, 125 or 128 ms have been used. The Verifit uses a 128 ms time period, resulting in 100 levels (or samples) in each 1/3 octave band for a 12.8 second passage. The level in each band that is exceeded by 1% of the samples (called L1 or 99th percentile) has historically been referred to as the speech peak for that band. The curve of the 99th percentile is approximately 12 dB above the LTASS. The level in each band that is exceeded by 70% of the samples (called L70, or 30th percentile) has historically been called the valley of speech for that band. The curve of the 30th percentiles is approximately 18 dB below the LTASS. The region between these two curves is often called the speech region, speech envelope or speech 'banana'. The speech envelope, when derived in this way, has significance in terms of both speech detection and speech understanding. Generally, speech will be detectable if the 99th percentile is at or near threshold. The Speech Intelligibility Index (SII) is maximized when the entire speech envelope (idealized as a 30 dB range) is above (masked) threshold. This will not be an SII of 100% (or 1) because of loudness distortion factors, but higher SII values will not produce significantly higher scores on most test material. The speech-reception threshold (SRT) is attained when the LTASS is at threshold (approximately - depending on test material and the individual)


20 Troubleshooting


This section covers common problems encountered when using the Axiom. If you require further assistance, note your model and serial number (located just behind the test box lid) and contact your local distributor, or Audioscan at 519-268-3313 800-265-2093 (USA only), 519-268-3256 (fax) or service@audioscan.com

PLEASE DO NOT SHIP YOUR UNIT BACK TO THE FACTORY. In most cases we can courier you a part that you can quickly and easily replace yourself.

Self test failures

Any failure during power-on self test (P.O.S.T.) is indicated on-screen and further use of the instrument is denied. Try the following steps to resolve the failure:

1. To access Self test, click on  and then click on [Self test].
2. If you received a Max out/in or a Routing failure, remove any hearing aids from the test chamber and rerun self test.
3. Unplug all microphones (On-ear & Test box) from the test chamber. Rerun self test. If self test passes, reconnect the microphones, one by one, running self test each time until a failure occurs. If the failure returns, display and print the test results and contact Audioscan for help.

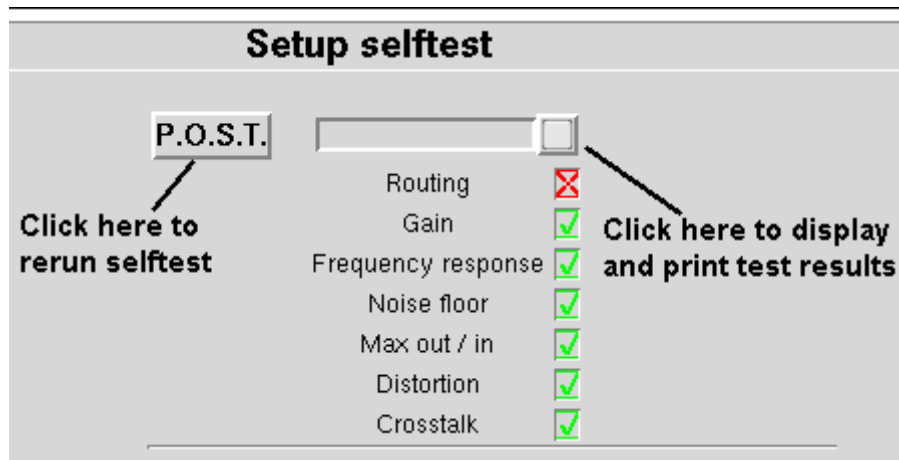

Initialize Function

Initializing your Audioscan unit will reset it to default factory settings. This erases the user preference files stored in internal memory. Any custom printer choice, network settings, test setups will need to be re-entered and the user calibrations will also be required.

1. To initialize, click on [Setup], then click on [Self test]
2. Click on [Initilize]. Unit will automatically restart.

Test box high distortion or noise

High distortion and/or noise numbers in an ANSI test

- There may be a leak between the hearing instrument under test and the coupler it is connected to. For BTE type instruments, check the integrity of the #13 tubing at the end of the coupler where it joins with the ear hook. Look for cracks. ITE type instruments need to have a good seal between the hearing instrument and the coupler. In addition, all vents need to be closed from the faceplate side of the hearing instrument.
- Ambient room noise may be leaking into the test chamber. Air conditioners, computer fans and similar noisemakers can yield artificially high noise figures. Check the sealing of the test chamber. Try turning off suspected noise sources to identify the cause.
- Vibration may be coupled into the test chamber from other equipment on the same table or wooden floor. Computers, fans and air conditioners may be the source of the vibration. Try turning off suspected sources to identify the cause. Relocate the source or the Axiom on a foam pad.

Test box curves inconsistent

The test box curves keep changing from test to test.

- The battery may be close to end of life or starved for air (zinc-air cells) because of a tight seal on the battery compartment. Try using a fresh battery.
- The test box reference microphone (the white one in the sound chamber) is incorrectly positioned. In order to control sound pressure levels precisely at the microphone inlet of the hearing instrument, the test box reference microphone must be positioned as close as possible to the hearing instrument's microphone without actually touching it.
- The noise or feedback reduction features are attempting to reject the test signal. These need to be disabled prior to tests with tones or noise.
- The instrument is on the verge of feedback due to a poor seal to the coupler, unplugged vent or a crack in the tubing on the BTE coupler.

Test box curves differ from specifications

The OSPL90 and response curves don't look like the manufacturer's.

- Check that all controls or program settings are the same as those used by the manufacturer.
- Check that vents on earmolds or ITE instruments are plugged at the faceplate.
- Check that ITE instruments are well sealed to the coupler.
- When putting a hearing instrument into the ITE coupler, be sure to keep the canal portion of the instrument flush with the inside of the 2cc volume.
- Ensure that the test box is sealed tightly.
- Ensure that the test box microphone calibration was performed correctly.
- Check that the coupler is screwed tightly onto the coupler microphone.

Test box speaker overdriven

Test box speaker overdriven!

- This message indicates that the test chamber speaker is unable to produce the SPL required for a test.
- Check that the test chamber lid is closed and sealed.
- Check that the test box calibration looks normal.
- Check that the hearing instrument under test is not feeding back.
- Try running an On-ear test.

No test box reference mic. detected

No test box reference mic. detected!

This message may be displayed when starting a hearing instrument test. It indicates that the Axiom is not measuring any sound at the test box reference microphone. The test box reference microphone is the white microphone located in the test chamber.

- Ensure that the test box reference microphone is secure. Unplug the white test box reference microphone. Plug the microphone back in again, ensuring that it 'clicks' into place. Try the calibration procedure again.
- Unplug the test box reference microphone and plug the on-ear probe microphone in its place. Try test box calibration using the reference microphone of the on-ear probe microphone in place of the test box reference microphone. If this is successful, the test box reference microphone is defective and must be replaced.

Invalid test box calibration

Invalid test box microphone calibration!

This message may be displayed during the test box calibration procedure. It indicates that the calibration curve is significantly different from what is expected. It may be accompanied by a missing test box reference microphone error.

- Check that there is no coupler on the coupler microphone and that the microphones are correctly positioned for test box calibration.
- Check that the coupler microphone is securely plugged into the correct socket.
- Follow the suggestions in No test box reference mic. detected.

No on-ear ref. mic. detected

No (right or left) on-ear reference mic. detected.

This message may be displayed when starting an On-ear test. It indicates that the Axiom is not measuring any sound at the on-ear probe reference microphone.

- Check that a probe microphone assembly is securely plugged into the PROBE socket underneath the test chamber and that the cable has not become damaged.

- Unplug the test box reference microphone in the test chamber and plug the on-ear probe microphone in its place. Try Test box calibration using the reference microphone of the on-ear probe microphone in place of the test box reference microphone. If this fails, the on-ear probe microphone is defective and must be replaced.

Invalid on-ear calibration

Invalid on-ear microphone calibration!

This message may be displayed during the on-ear calibration procedure. It indicates that the calibration curve is significantly different from what is expected. It may be accompanied by an on-ear reference microphone error.

- Check that a probe microphone assembly is securely plugged into the PROBE socket of the side (Right or Left) you are trying to calibrate, and that the cable has not become damaged.
- Try using a new probe tube. Ensure it is correctly attached and positioned.
- Try switching right and left probe microphone assemblies.

Sound-field speaker overdriven

External sound-field speaker overdriven!

This message may occur in on-ear unaided or aided tests if:


- The external sound-field speaker is improperly connected.
- The test environment is too reverberant for the distance between the client and the speaker. Move the on-ear speaker and client away from walls and hard objects. Move the client closer to the speaker.
- The speaker or connecting cable is defective. If this is the case, you should not hear any sound coming from external speaker during any test. Try a different speaker or cable.

Barcode scanner malfunction


If there is no red scanning beam when the trigger is pulled, tug on the cable where it enters the scanner. If it comes loose easily, push it back in until it clicks in place.

If the scanning beam stays on when the trigger is released or if the barcode scanner fails to read known good barcodes, try the following:

1. Unplug the scanner from the equipment, then reconnect it. If the problem persists...
2. Print this page on an external printer (see External display, printer, speaker) or from the electronic user's guide (see Electronic user's guide).
3. If your barcode scanner has a PSC logo, scan these two barcodes, starting with the top one. This will reset the scanner to its default settings.

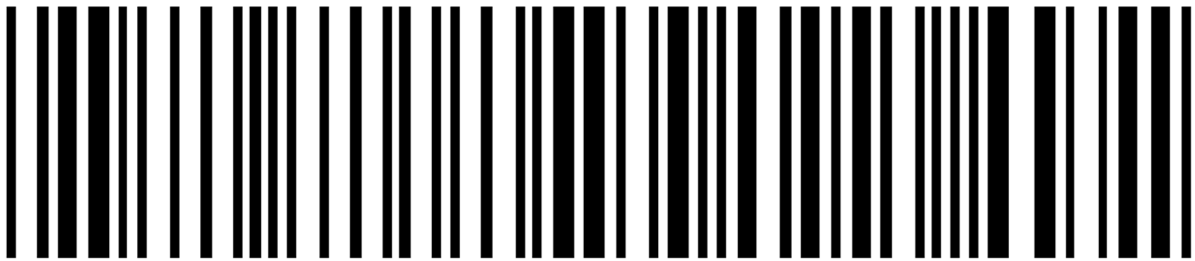

If your scanner has a Unitech logo, scan the following barcode to reset the scanner to its default settings.

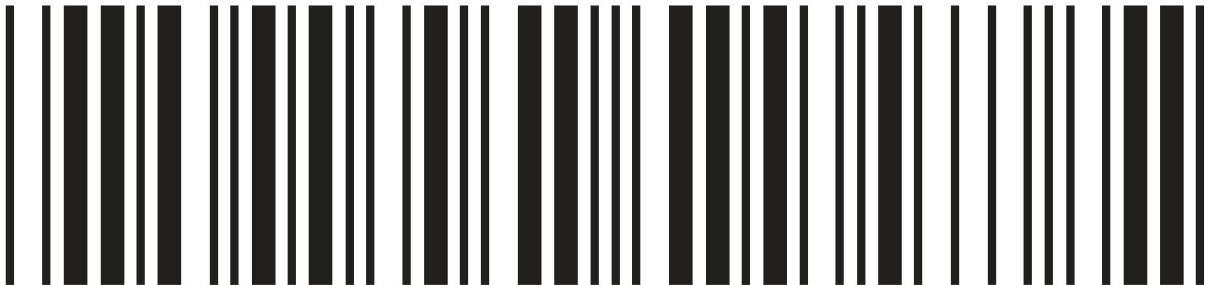

21 Technical Specifications

Specifications subject to change without notice

Environmental conditions

STORAGE & TRANSPORTATION

Temperature - 20° C to +60° C

Relative humidity (non-condensing).....5% to 95%
OPERATING
 Temperature.....+15° C to +30° C
 Relative Humidity.....20% - 50%
 Atmospheric Pressure.....81 - 106 kPa
 Altitude.....below 2000 m

General

Power source:.....100 – 240 V, 47 – 63Hz, 1.35A
 Overall dimensions.....12.5"x12.25"x4.25"
 (31.75cm x 31.2cm x 10.8cm)
 Weight.....7.6 lb (3.45kg)
 Display type.....User supplied
 Resolution.....800x600 (SVGA)
 Printer type.....User supplied
 Power amplifiers.....1
 Stimulus Channels.....1
 Measurement channels.....2
 Connectors.....4-USB
1 - Ethernet (RJ45)
2 - HDMI
1- RECD transducer(3.5mm st)
1 - test chamber ref. mic.(3.5mm st)
1 - coupler microphone(3.5mm st)
1 – External speaker (RCA)
2 - real-ear mic.(3.5mm st)

Test box

Working Space.....60in²
 Speaker.....1 – 30.5mm Round
 Test box isolation @ 1kHz.....>25 dB
 Frequency Range.....200 - 8000 Hz
 Coupler microphone noise floor.....(200 – 8000 Hz)
 <40 dB SPL
 Test Stimuli.....tone, tone burst,pink noise,
calibrated or live speech
 Test stimulus levels.....40 to 90 dB SPL in 5 dB steps
 Test stimulus distortion.....<2% at 90dB SPL
<0.5% at 70 dB SPL
 Test stimulus accuracy at reference mic. for tones

(200-2000 Hz).....±1.5 dB SPL
 (2000-8000 Hz).....±2.5 dB SPL
 Equalization method.....pressure method
 Analysis frequencies per octave.....12
 Analysis filter bandwidth.....1/12 octave
 Measurement accuracy at 1 kHz.....±1db
 Measurement accuracy re 1 kHz.±1 dB (200-5000 Hz)
±2.5 dB (5000-8000 Hz)
 Measurement range.....30 - 140 dB SPL
 Harmonic distortion measurement.....2nd, 3rd
or 2nd plus 3rd
 Harmonic distortion range.....200 to 4000 Hz
 Harmonic distortion accuracy.....±1%

ANSI S3.22 - 1996 and 2003 tests available
 OSPL90
 Full-on Gain
 Reference Test Gain
 Frequency Response
 Frequency Range
 Maximum OSPL90
 Harmonic Distortion
 Attack & Release time
 Equivalent Input Noise
 Coupler SPL

Other tests Available
 Speechmap ® real-speech audibility measures
 ... Manual measurement of output, gain and distortion

On-Ear

Speakers.....1 – 5.25"x3.75"x3.75"
 Probe microphone tube....Silicone 1.0 mm diameter x
 75 mm
 Probe microphone noise floor (200 – 8000 Hz).<45 dB
 SPL
 Frequency Range.....200 to 8000 Hz
 Test Stimuli.....tone, tone burst,
pink noise, calibrated or live speech
 Freq. modulation.....sawtooth ±3% over 128 ms
 Stimulus levels for tones.40 - 85 dB SPL in 5 dB steps
 Test stimulus accuracy at reference mic. for tones
 (2000- 8000 Hz).....±3 dB SPL
 Equalization Method.....pressure method
(stored for open fittings)
 Frequencies per octave (swept tones).....12
 Frequencies per octave (tone burst).....3
 Analysis bandwidth (speech, noise).....1/3 octave
 Measurement accuracy at 1 kHz.....±1 dB
 Measurement accuracy re 1 kHz.....
±1.5 dB (200-5000 Hz)
±2.5 dB (5000-8000Hz)
 Measurement Range...30-135 dB SPL (200-2500 Hz)
30-140 dB SPL (2500-8000Hz)
 ANSI S3.46 - 1997 tests available
 Real-Ear Unaided Response
 Real-Ear Aided Response
 Real-Ear Occluded Response

..... Real-Ear Insertion Gain

Other tests available

..... Speechmap ® real-speech audibility measures

..... On-ear harmonic distortion

..... On-ear spectral analysis

..... On-ear noise reduction verification

... Manual measurement of output, gain and distortion

Fitting methods available

..... Speechmap with DSL 5.0a, NAL-NL1, CAMFIT

.. Insertion gain with NAL-RP, NAL-NL1, Fig6, Pogoll,

.....Berger, Libby

Sensory Loss Simulator

Simulation types.....

.....Non-linear outer hair cell cochlear loss

Simulation bands.....65

22 Glossary

AGC, Automatic Gain Control, the means by which gain is automatically controlled by the level of the signal being amplified. (ANSI S3.22)

AGC-I, input-controlled AGC (see "input compression").

AGC-O, output controlled AGC (see "output compression").

AGRAM, Abbreviation for audiogram.

ANSI, American National Standards Institute.

Attack Time, the time required for HA output to reach +/- 3dB of its steady state level in response to an input step from 55 to 90dB SPL. (ANSI S3.22)

ASP, Automatic Signal Processing, In hearing aids, level dependent frequency response shaping.

AVC, Automatic Volume Control, slow-acting AGC that attempts to maintain a constant average signal level.

BSPL, Band Sound Pressure Level, the SPL within a restricted frequency band.

BTE, Behind-The-Ear (hearing aid).

Compression, a type of AGC in which an incremental change in the input level produces a smaller incremental change in the output level.

CR, Compression Ratio, the ratio of incremental change in input SPL to the resulting incremental change in output SPL. Used to characterize steady state AGC action; can be a function of input level, duration and frequency.

CROS, Contralateral Routing Of Signals; a type of hearing aid in which the microphone is at the ear opposite to the one receiving the amplified sound.

CORFIG, coupler response for flat insertion gain; the transformation added to real-ear insertion gain to obtain 2-cc coupler gain. Assumes linear amplification.

Coupler, a device used to acoustically couple an earphone or hearing aid to a measurement microphone and to provide an acoustic load.

DSL, Desired Sensation Level Method, a systematic eardrum (SPLogram) approach to the hearing aid fitting process that seeks to ensure amplified speech will be audible and comfortable while loud sounds will not be uncomfortable.

Directional hearing aid, a hearing aid for which the gain is dependent on the direction of sound incidence when measured under free-field conditions (ANSI S3.22).

Equalization, process of controlling the SPL (or band SPL) of the stimulus, as a function of frequency, at the field reference point (ANSI S3.46).

Expansion, a type of AGC in which an incremental change in the input level produces a larger incremental change in the output level.

FM, frequency modulation, a method of transmitting information in which the frequency of a radio wave (carrier) is varied (modulated) by the information (signal). Also, a method of reducing standing waves in a sound field in which the frequency of a test tone (carrier) is varied by a low frequency triangle or sine wave.

FM System, an assistive listening device consisting of a microphone, an fm radio transmitter and an fm receiver. The microphone and transmitter are worn by the speaker and the receiver is worn by the listener.

FOG, full on gain, gain for a 50 dB input SPL with the gain control set to maximum (ANSI S3.22)

Gain (in dB), output sound pressure relative to input sound pressure, expressed in decibels; $\text{gain} = 20\log_{10}(\text{output sound pressure} / \text{input sound pressure})$; also, $\text{gain} = (\text{output SPL}) - (\text{input SPL})$.

Gain control, a manually or electronically operated control for the adjustment of overall gain (ANSI S3.22).

HA-1 Coupler, a coupler having a volume of 2 cubic centimetres with direct access to the cavity. Used for testing in-the-ear and in-the-canal hearing aids; can also be used to test behind-the-ear hearing aids with a custom earmold attached. Aka the ITE coupler.

HA-2 Coupler, a coupler having a volume of 2 cubic centimetres with access through a rigid tube. When used for testing BTE hearing aids, the rigid tube consists of 18 mm of 3 mm tubing plus 25 mm of 2 mm tubing (ANSI S3.22). When used for calibrating insert earphones, the 2 mm tubing is not used (ANSI S3.6). Aka the BTE coupler.

Harmonic, a component of a tone complex that is an integer multiple of the lowest frequency component (fundamental).

Harmonic Distortion, the addition of harmonic components to a signal. The rms value of a harmonic component of a signal as a percentage of the rms value of the fundamental. If less than 20%, the rms value of the total signal may be used instead of the fundamental.

HFA, High Frequency Average, the average of values in dB at 1000, 1600, and 2500 Hz. (ANSI S3.22).

HL, Hearing Level, the hearing threshold referenced to the threshold of normal hearing, expressed in dB.

HI, Hearing instrument.

Input Compression, a form of AGC in which the signal level is regulated before the volume control.

Input / Output function, steady state, single-frequency plot of the coupler SPL on the ordinate as a function of input SPL on the abscissa with equal decibel scale divisions on each axis (ANSI S3.22).

ITE, In-The-Ear (hearing aid).

Kneepoint (compression threshold), the point on an input/output curve at which the slope digresses from unity indicating the signal level at which a non-linear process begins to take effect.¹

LCD, liquid crystal display. A thin, planar information display.

Linear Region, that portion of the input-output function which is a straight line at 45 degrees.

Linear Amplification, amplification having the same gain for all input levels until the maximum output of the device is reached.

LTASS, Long-term average speech spectrum, the rms level of a speech passage in 1/3rd octave bands, averaged over the entire passage.

Loop System, an assistive listening device that uses magnetic induction to carry a signal from a microphone/amplifier to a wire loop. The signal is picked up from the loop by a hearing aid set to the telecoil position or by a receiver with built-in telecoil, volume control, and earpiece. Loops are either worn around an individual's neck or can encircle a room, such as a classroom, providing an excellent signal-to-noise ratio.¹

MAF, minimum audible field, SPL of a tone at the threshold of audibility measured in a free sound field for a subject listening with both ears and facing the sound source. The SPL is measured with the subject removed from the field at the midpoint of an imaginary line joining the centers of the ear canal openings.

MAP, minimal audible pressure, SPL of a tone, measured or inferred at the tympanic membrane, at the threshold of audibility.

MPO, Maximum Power Output, the maximum SPL that a hearing aid can deliver in response to a steady narrow band input. The stimulus may be pulsed (a tone burst) to avoid discomfort and to approximate an aided UCL measurement stimulus.

Modulation, the process of varying a characteristic of one signal with another.

NAL, National Acoustic Laboratories (Australia)

NCA, National Centre for Audiology (Canada)

Nonlinear Region, that portion of the input-output function which is not a straight line at 45 degrees.

OSPL90 (output SPL for 90-dB input SPL), The SPL developed in a 2cc coupler when the input SPL is 90 dB, with the gain control of the hearing aid full-on (ANSI S3.22).

Oscillation, in audiology, refers to the undesired tonal output of an amplifier behaving unstably as a result of feedback; whistling.¹

Output Compression, a form of AGC in which the signal level is regulated at the output.

OLC, Output-limiting Compression (compression limiting), a form of AGC with a high kneepoint and high compression ratio.

PC, Peak-clipping, a method of limiting MPO by truncating the electrical signal supplied to the output transducer. The resulting acoustic signal may not be truncated but will be distorted.

PCR, Power Concentration Ratio, a measure of the degree to which a large amount of power is concentrated at a small number of frequencies in the output of a hearing aid.

Probe microphone, microphone (which may include an extension tube) having a sound inlet which is small enough to be acoustically unobtrusive when inserted into the ear canal (ANSI S3.46).

Pure tone, a signal containing one, and only one, frequency; a sinusoidal acoustic signal.¹

REAG, Real-Ear Aided Gain, difference, in dB as a function of frequency, between the SPL (or BSPL) at a specified measurement point in the ear canal and the SPL (or BSPL) at the field reference point, for a specified sound field, with the hearing aid in place and turned on (ANSI S3.46).

REAR, Real-ear Aided Response, the SPL (or BSPL) as a function of frequency, at a specified measurement point in the ear canal for a specified sound field, with the hearing aid in place and turned on (ANSI S3.46).

RECD, Real-Ear-to-Coupler Difference, difference, in dB as a function of frequency, between the SPL (or BSPL) produced by an insert transducer at the eardrum and in a 2cc coupler. HA-1 or HA-2 coupler should be specified.

REDD, Real-Ear-to-Dial Difference, difference, in dB as a function of frequency, between the SPL (or BSPL) measured at the eardrum and the audiometer dial setting that produced it. It is specific to the client, headphone and audiometer used to generate it.

REIG, Real-Ear Insertion Gain, difference, in dB as a function of frequency, between the REAG and the REUG taken with the same measurement point and the same sound field conditions. $REIG = REAG - REUG$. ANSI S3.46.

REOG, Real-Ear Occluded Gain, difference, in dB as a function of frequency, between the SPL (or BSPL) at a specified measurement point in the ear canal and the SPL (or BSPL) at the field reference point, for a specified sound field, with the hearing aid in place and turned off (ANSI S3.46).

REOR, Real-Ear Occluded Response, SPL (or BSPL), as a function of frequency, at a specified measurement point in the ear canal, for a specified sound field, with the hearing aid in place and turned off (ANSI S3.46). The

REOR is used to evaluate the seal of an earmold. It is not related to the required gain of a hearing aid.

REUG, Real-Ear Unaided Gain, difference, in dB as a function of frequency, between the SPL (or BSPL) at a specified measurement point in the ear canal and the SPL (or BSPL) at a field reference point, for a specified sound field, with the ear unoccluded (ANSI S3.46).

REUR, Real-ear Unaided Response, SPL (or BSPL), as a function of frequency, at a specified measurement point in the ear canal, for a specified sound field, with the ear canal unoccluded (ANSI S3.46).

Reference microphone, microphone used to measure the stimulus level in the measurement process or to control it in the equalization process.

Release Time, the time required for HA output to fall to within +/-4dB of its steady-state level for an input step from 90 to 55dB SPL. (ANSI S3.22)

REM, Real-ear Measurement.

RTG, Reference Test Gain, the HFA gain for a 60 dB input SPL with the gain control at RTS (ANSI S3.22).

Reference, Sound-field, the calibration point of a sound-field measurement.

RSETS, Relative Simulated Equivalent Telephone Sensitivity, difference in dB between the HFA-SPLITS (SPA-SPLITS) and the RTG plus 60 in the 'M' (microphone) mode. $RSETS = HFA-SPLITS - (RTG+60)$

RTS, Reference Test Setting of the gain control, for a 60 dB input SPL, the setting of the gain control required to produce an HFA gain within 1.5 dB of the HFA-OSPL90 minus 17 dB. If the full-on HFA gain is less than the HFA-OSPL90 minus 17 dB, the RTS is the full-on setting of the gain control (ANSI S3.22).

Saturation, in a hearing aid, the limiting of output caused by the output circuitry and transducer reaching their maximum capability.

SPA, Special Purpose Average, the average of values in dB at three third octave frequencies separated by 2/3 of an octave (ANSI S3.22). See also HFA.

Spectrum, the BSPL as a function of frequency for a broad-band signal.

SII, Speech Intelligibility Index, a quantity calculated from measures of speech, noise and hearing threshold that is highly correlated with the intelligibility of speech (ANSI S3.5).

Speech Region, a statistical range of short-term spectra present in a speech passage. The short-term spectra are typically 120 - 130 millisecond averages in 1/3 octaves. The statistical range is typically from the 70th to the 99th percentile.

SPL, Sound Pressure Level, rms sound pressure relative to 20 uPa, expressed in decibels. $SPL = 20\log_{10}(\text{rms sound pressure} / 20\text{uPa})$.

SPLITS, coupler SPL for an inductive telephone simulator, SPL developed in a 2cc coupler by a hearing aid with the gain control at the RTS when the input is the magnetic field generated by a TMFS (ANSI S3.22).

TMFS, Telephone magnetic-field simulator, a device for producing a magnetic field of consistent level and geometric shape when driven by a specified current (ANSI S3.22).

THD, Total Harmonic Distortion, the power in all harmonic components as a percentage of the power in the fundamental. If less than 20%, the power of the total signal may be used instead of the fundamental. For most hearing aids, the inclusion of harmonics above the third will not significantly alter the THD (ANSI S3.22).

Transducer, a device which transforms energy from one form to another. For example, a microphone or earphone.

UCL, Uncomfortable Listening Level, the level for a specified stimulus that is judged to be definitely

uncomfortable.

VGA, video graphics adapter.

VC, Volume Control, a user-operated gain control.

Warble Tone, a frequency-modulated sine wave. The modulating signal is often a low frequency triangle wave.

1. Valente, M: *Strategies for Selecting and Verifying Hearing Aid Fittings*. New York: Thieme Medical Publishers, Inc., 1994.

ANSI S3.5, *Methods for Calculation of the Speech Intelligibility Index*, 1997

ANSI S3.6, *Specification for Audiometers*, 1996

ANSI S3.22, *Specification of Hearing Aid Characteristics*, revised 2003.

ANSI S3.46, *Methods of Measurement of Real-Ear Performance Characteristics of Hearing Aids*, 1997.

23 References

- AAA Clinical Practice Guidelines: *Remote Microphone Hearing Assistance Technologies for Children and Youth Birth-21 Years*. Supplement A: Fitting and Verification Procedures for Ear-level FM. Draft 2007.
- American National Standards Institute. (1997). *American National Standard Methods of Measurement of Real-Ear Performance Characteristics of Hearing Aids*. ANSI S3.46-1997. New York: American National Standards Institute.
- American National Standards Institute. (1996). *American National Standard Specification of Hearing Aid Characteristics*. ANSI S3.22-1996. New York: American National Standards Institute.
- Bagatto, M., Moodie, S. T., Scollie, S., Seewald, R., Moodie, K.S., Pumford, J. & Liu, R. (2005). *Clinical protocols for hearing instrument fitting in the desired sensation level method*. Trends in Amplification, 9-4, 199-226.
- Bentler, R.A. & Pavlovic, C.V. (1989). *Transfer functions and correction factors used in hearing aid evaluation and research*. Ear and Hearing, 10(10),58-63.
- Byrne, D. (1977). *The speech spectrum - Some aspects of its significance for hearing aid selection and evaluation*. British Journal of Audiology, 11,40-46.
- Cole, W.A. & Sinclair, S.T. (1998). *The Audioscan RM500 Speechmap/DSL fitting system*. Trends in Amplification, 3(4):125-139.
- Corliss, E. L. R. (1990) *The ear as a mechanism of communication*. Journal of Audio Engineering Society, 38(9):640-652.
- Cornelisse, L.E., Gagne, J-P & Seewald, R.C. (1991). *Ear level recordings of the long-term average spectrum of speech*. Ear and Hearing, 12(1),47-54.
- Cornelisse, L.E., Gagne, J-P & Seewald, R.C. (1991). *Long-term average speech spectrum at chest-level microphone location*. Journal of Speech, Language, Pathology and Audiology, 15(3):7-12.
- Cornelisse, L.E., Seewald, R.C., & Jamieson, D.G. (1994). *Wide-dynamic-range compression hearing aids: The DSL[i/o] approach*. The Hearing Journal, 47(10), 23-29.
- Cornelisse L.E., Seewald R.C., & Jamieson D.G. (1995). *The input/output (I/O) formula: A theoretical approach to the fitting of personal amplification devices*. Journal of the Acoustical Society of America, 97(3), 1854-1684.
- Cox, R.M. & Moore, J.N. (1988). *Composite speech spectrum for hearing aid gain prescriptions*. Journal of Speech and Hearing Research, 31:102-107.
- Dunn, H. K., & White, D. S. (1940). *Statistical measurements on conversational speech*. Journal of the Acoustical Society of America, 11:278-288.
- Feigin, J.A., Kopun, J.G., Stelmachowicz, P.G. & Gorga, M.P. (1989). *Probe-tube microphone measures of ear-canal sound pressure levels in infants and children*. Ear and Hearing, 10(4), 254-258.
- Fikret-Pasa, S. & Revit, L.J. (1992). *Three experiments to validate new probe-measurement practices in prescribing customized hearing aids*. Journal of Speech and Hearing Research, 35:384-400.
- Freed, D.J., & Soli, S. D (2006). *An objective procedure for evaluation of adaptive antifeedback algorithms in hearing aids*. Ear & Hearing, 27(4).
- Hawkins, D.B., Cooper, W.A., & Thompson, D.J. (1990). *Comparisons among SPLs in Real Ears, 2cc and 6cc couplers*. Journal of the American Academy of Audiology 1:154-161.

- Hawkins, D.B., Walden, B.E., Montgomery, A.A., & Prosek, R.A., (1987). *Description and validation of an LDL procedure designed to select SSPL90*. *Ear and Hearing*, 8:162-169.
- Killion, M. C., & Monser, E. L. (1980). "Corfig: Coupler response for flat insertion gain." In G. A. Studebaker and I. Hochberg (Eds.), *Acoustical Factors Affecting Hearing Aid Performance*. Baltimore: University Park Press.
- Lewis, D.E., Feigin, J.A., Karasek, A.E., & Stelmachowicz, P.G. (1991). *Evaluation and assessment of FM systems*. *Ear and Hearing*, 12:268-280.
- Martin, H.C., Munro, K.J., & Langer, D.H. (1997). *Real-ear to coupler differences in children with grommets*. *British Journal of Audiology* 31:63-69.
- Moodie, K.S. (1996). *A practical approach to hearing aid selection*. *BSA News* 18:6-8.
- Moodie, K.S., Seewald, R.C. & Sinclair, S.T. (1994). *Procedure for predicting real-ear hearing aid performance in young children*. *American Journal of Audiology*, 3(1), 23-31.
- Moore, Brian C.J. & Glasberg, Brian R. (2004). *A revised model of loudness perception applied to cochlear hearing loss*. *Hearing Research*, 188 (2004), 70-88.
- Mueller, H.G. & Bright, K.E. (1994). Selection and verification of maximum output. In Valente M (ed.): *Strategies for Selecting and Verifying Hearing Aid Fittings*. New York: Thieme Medical Publishers, 38-63.
- Pavlovic, C. V. (1991). *Speech recognition and five articulation indexes* *Hearing Instruments* 42(9):20-23.
- Pearsons, K.S., Bennett, R.L., & Fidell, S. (1977). *Speech levels in various noise environments, Project Report On Contract 68 01-2466*. Washington, DC: US Environmental Protection Agency.
- Platz, Rainer, (2004). *SNR Advantage, FM Advantage and FM Fitting, Proceedings from the 1st International FM Conference*. www.phonak.com.
- Scollie, S.D., Seewald, R.C., Cornelisse, L.E. & Jenstad, L.M. (1998). *Validity and repeatability of level-independent HL to SPL transforms*. *Ear and Hearing* 19(5):407-413.
- Scollie, S.D., Seewald, R.C., Moodie, K.S., Dekok, K. (2000). *Preferred listening levels of children who use hearing aids: Comparison to prescriptive targets*. *JAAA* 11:230-238.
- Scollie, S., Seewald, R., Cornelisse, L., Moodie, S. T., Bagatto, M., Lurnagaray, D., Beaulac, S. & Pumford, J. (2005). *The Desired Sensation Level Multistage Input/Output Algorithm*. *Trends in Amplification*, 9-4, 159-197.
- Seewald, R.C. (1994a). Current issues in hearing aid fitting. In J-P Gagne and N. Tye-Murray (Eds.), *Research in Audiological Rehabilitation: Current Trends and Future Directions*. *Journal of the Academy of Rehabilitative Audiology Monograph*, XXVII.
- Seewald, R.C. (1994b). *Fitting children with the DSL method*. *The Hearing Journal*, 47(9), 10,48-51.
- Seewald, R.C., Moodie, K.S., Sinclair, S.T. & Cornelisse, L.E. (1996). *Traditional and theoretical approaches to selecting amplification for infants and young children*. In Bess, F.H., Gravel, J.S. &
- Tharpe, A.M. (eds.): *Amplification for Children with Auditory Deficits*, Nashville: Bill Wilkerson Center Press, 161-191.
- Seewald, R.C. & Ross, M. (1988). *Amplification for young hearing-impaired children*. In M.C. Pollack (Ed.), *Amplification for the Hearing-Impaired*, Third Edition (pp. 213-271). Orlando: Grune & Stratton.
- Seewald, R.C., Ross, M. & Spiro, M.K. (1985). *Selecting amplification characteristics for young hearing-impaired children*. *Ear and Hearing*, 6(1), 48-53.
- Seewald, R.C., Ross, M. & Stelmachowicz, P.G. (1987). *Selecting and verifying hearing aid performance*

- characteristics for young children.* Journal of the Academy of Rehabilitative Audiology, 20, 25-38.
- Seewald, R., Moodie, S. T., Scollie, S. & Bagatto, M. (2005). *The DSL method for pediatric hearing instrument fitting: Historical perspective and current issues.* Trends in Amplification, 9-4, 145-157.
- Seewald, R.C. & Moodie, K.S. (1992). Electroacoustic considerations. In M. Ross (ed), *FM Auditory Training Systems: Characteristics, Selection, & Use.* Timonium, MD: York Press.
- Sherbecoe, R.L & Studebaker, G.A. (2003) *Audibility-Index predictions of normal-hearing and hearing-impaired listeners' performance on the Connected Speech Test.* Ear and Hearing, 24-1, 71-88.
- Sinclair, S.T., Moodie, K.S. & Seewald, R.C. (1997). *Pediatric Hearing Instrument Fitting: Phonak Video Focus #2 Booklet,* Stofa, Switzerland: Phonak A/G.
- Sinclair, S.T. Seewald, R.C. & Cole, W.A. (1996). *A new real-ear fitting system: Because no two ears are alike.* Hearing Review 3(6):46-48.
- Skinner, M. W. (1988). *Hearing Aid Evaluation.* Englewood Cliffs: Prentice-Hall Inc.
- Smith, D. A. 1990. *Technological report: Acoustometry.* Hearing Instruments 41(11):40-44.
- Valente, M., Potts, L.G., Valente, M., Vass, W & Goebel, J. (1994). *Intersubject Variability of real-ear sound pressure level: Conventional and insert earphones.* Journal of the American Academy of Audiology, 5, 390-398.
- Valente, M: *Strategies for Selecting and Verifying Hearing Aid Fittings.* New York:Thieme Medical Publishers, Inc.,1994.
- Zelisko, D.L.C., Seewald, R.C. & Whiteside, S. (1992). *Comparing three procedures for predicting the ear canal SPL at LDL.* ASHA, 34(10).

24 Appendix 1

Manufacturer Disclosure Statement for Medical Device Security

Manufacturer Disclosure Statement for Medical Device Security--MDS ²			
Device Category *	Manufacturer *	Document ID	Document Release Date
Hearing Aid Analyzer	Etymonic Design Inc.		See page 1 of Users Guide
Device Model	Software Revision	Software Release Date	Other
Verifit / RMS00SL / Axiom	See Page 1 of Users Guide	See page 1 of Users Guide	
Manufacturer or Representative	Name	Title	Department
Contact Information	Chris Stokes-Rees	Technical Services Representative	
	Company Name	Telephone Number	E-mail
	Etymonic Design Inc.	800-265-2093	chris@audioscan.com
MANAGEMENT OF ELECTRONIC PROTECTED HEALTH INFORMATION (PHI) As defined by HIPAA Security Rule, 45 CFR Part 164			Yes No N/A
1. Can this device transmit or maintain <i>electronic Protected Health Information</i> (ePHI)?			Yes 1
2. Types of ePHI data elements that can be maintained by the device:			
a. Demographic (e.g., name, address, location, unique identification number)?			No
b. Medical record (e.g., medical record #, account #, test or treatment date, device identification number)?			No
c. Diagnostic/therapeutic (e.g., photo/radiograph, test results, or physiological data with identifying characteristics)?			No
d. Open, unstructured text entered by device user/operator?			Yes 1
3. Maintaining ePHI: <i>Can the device</i>			
a. Maintain ePHI temporarily in volatile memory (i.e., until cleared on by power-off or reset)?			Yes 2
b. Store ePHI persistently on local media?			No
c. Import/export ePHI with other systems?			Yes 3
4. Mechanisms used for the transmitting, importing/exporting of ePHI: <i>Can the device</i>			
a. Display ePHI (e.g., video display)?			Yes
b. Generate hardcopy reports or images containing ePHI?			Yes 1
c. Retrieve ePHI from or record ePHI to removable media (e.g., disk, DVD, CD_ROM, tape, CF/SD card, memory stick)?			Yes 1
d. Transmit/receive or import/export ePHI via dedicated cable connection (e.g., IEEE 1073, serial port, USB, FireWire)?			No
e. Transmit/receive ePHI via a network connection (e.g., LAN, WAN, VPN, intranet, Internet)?			Yes
f. Transmit/receive ePHI via an integrated wireless connection (e.g., WiFi, Bluetooth, infrared)?			No
g. Other			
ADMINISTRATIVE SAFEGUARDS			Yes No N/A
5. Does manufacturer offer operator and technical support training or documentation on device security features?			Yes 4
6. What underlying operating system(s) (including version number) are used by the device? Linux kernel 2.6.26			
PHYSICAL SAFEGUARDS			Yes No N/A
7. Are all device components maintaining ePHI (other than removable media) physically secure (i.e., cannot remove without tools)?			Yes
8. Does the device have an integral data backup capability (i.e., backup onto removable media such as tape, disk)?			No
9. Can the device boot from uncontrolled or removable media (i.e., a source other than an internal drive or memory component)?			Yes 5
TECHNICAL SAFEGUARDS			Yes No N/A
10. Can software or hardware not authorized by the device manufacturer be installed on the device?			No
11. Can the device be serviced remotely (i.e., maintenance activities performed by service person via network or remote connection)?			No
a. Can the device restrict remote access to specific devices or network locations (e.g., specific IP addresses)?			No
b. Can the device log provide an audit trail of remote-service activity?			No
c. Can security patches or other software be installed remotely?			No
12. Level of owner/operator service access to device operating system: <i>Can the device owner/operator?</i>			
a. Apply device manufacturer-validated security patches?			No
b. Install or update antivirus software?			No 6
c. Update virus definitions on manufacturer-installed antivirus software?			No 6
d. Obtain administrative privileges (e.g., access operating system or application via local root or admin account)?			No
13. Does the device support user/operator specific ID and password?			No
14. Are access sessions terminated after a predetermined length of inactivity (e.g., auto logoff)?			No
15. Events recorded in device audit log (e.g., user, date/time, action taken): <i>Can the audit log record</i>			
a. Login and logout by users/operators?			No 7
b. Viewing of ePHI?			No 7
c. Creation, modification or deletion of ePHI?			No 7
d. Import/export or transmittal/receipt of ePHI?			No 7
16. Does the device incorporate an emergency access ("break-glass") feature that logs each instance of use?			No
17. Can the device maintain ePHI (e.g., by internal battery) during power service interruptions?			No
18. Controls when exchanging ePHI with other devices:			
a. Transmitted only via a physically secure connection (e.g., dedicated cable)?			No
b. Encrypted prior to transmission via a network or removable media?			No 8
c. Restricted to a fixed list of network addresses (i.e., host-based access control list)?			No 9
19. Does the device ensure the integrity of the ePHI data with implicit or explicit error detection/correction technology?			No

Manufacturer Disclosure Statement for Medical Device Security--MDS²

RECOMMENDED SECURITY PRACTICES

If possible, do not create ePHI. ePHI can only be created if a QWERTY keyboard is connected to the device(s). Not providing a keyboard is an effective way of restricting the creation of ePHI.

In the event that ePHI *must* be produced, the device(s) are capable of outputting ePHI in several different forms as disclosed in this document. Any data containing ePHI should be de-identified before storing the data, whether it is to hard copy, network file, or USB. If de-identification is not possible physical access to the device or hard copies of data should be restricted to authorized personnel and saving to USB sticks should be prohibited.

EXPLANATORY NOTES (from questions 1 – 19)

IMPORTANT: Refer to Instructions for the Manufacturers Disclosure Statement for medical Device Security for the proper interpretation of information provided in this form.

1. The device(s), if equipped with a QWERTY keyboard, allows the operator to enter free-form text when saving or printing data. This data can be stored and retrieved from a USB stick or network file.
2. If an operator enters a header for printing, that header will remain on the machine until power is cycled or it is overwritten. Clearing a session does not erase header information an operator may have entered for printing.
3. The device(s) can import and export data and headers that may include ePHI, but only in the event that PHI was entered by the operator. The device(s) can import/export via network connection or transfer via USB stick.
4. This document along with available phone support constitutes our support for HIPAA compliance.
5. **The RM500SL device boots from a removable USB flash drive. The Verifit device (serial # 4670 and lower) boots from a removable CD-ROM. These devices will boot from removable media only if it contains software supplied by the manufacturer.**
6. There is no virus protection software available for the device(s).
7. The device(s) do not use any password protection and there is no device audit log.
8. The device(s) do not encrypt data and cannot save to encrypted media such as an encrypted USB. Any PHI produced by the device(s) should be de-identified appropriately.
9. The device(s) can connect to any networked computer to store and retrieve data, provided the user name and password is entered. Once the username and password is entered it is maintained by the device(s).

Verifit & RM500SL Test Signals & Analysis

Bill Cole BAsC, PEng



1 Introduction

The Verifit and RM500SL are full-duplex dual-channel audio measurement systems designed for the testing and fitting of all types of hearing instruments and many assistive listening devices. They provide an acoustic test chamber for the testing of devices coupled to standard 2 cm³ couplers as well as sound-field speaker(s) for on-ear measurement of hearing aid performance. The Verifit test chamber contains two loudspeakers for the functional testing of directional hearing instruments. Test signals are delivered to the device under test via the test chamber loudspeaker(s), the test chamber telephone magnetic-field simulator (TMFS), the test chamber magnetic field test loop (Verifit only) or the sound-field loudspeaker(s). Device output signals are measured in the 2 cm³ coupler via the coupler microphone or in a real ear via the probe microphone. Data from the 2 cm³ coupler may be viewed as coupler SPL, coupler gain or as estimated SPL in the ear canal (simulated real-ear measurements, S-REM). Data from the probe microphone may be viewed as ear canal SPL, as ear canal SPL re normal hearing threshold (i.e. dB HL), as insertion gain or as ear canal SPL in the context of an auditory area (Speechmap®).

2 The test signals

Sinusoidal, pseudo-random noise and digitized real speech signals are provided. Two signals derived from real speech, the International Speech Test Signal (ISTS) and the single-talker International Collegium of Rehabilitative Audiology (ICRA) signal, are also provided. Test signals are generated in real time by the digital signal processor (DSP) or played from 16 bit binary audio files stored in the on-board flash memory. In the Verifit, these audio files are up-loaded from the internal compact disc (CD) drive at power up. In order to provide a repeatable acoustic signal to the device under test, a reference microphone is used in conjunction with a digital control loop to maintain the desired band level at each frequency. Live audio (speech, music etc.) may also be used as a test signal but it is not controlled for spectrum or level.

Some useful facts about broad-band signals

Overall SPL is the SPL in a band containing all significant frequency components of the signal.

Spectrum level is the SPL in a band 1 Hz wide.

Band SPL is the SPL in a restricted frequency range. If the signal is uniform in the band, band SPL = Spectrum level + 10*log(bandwidth).

A spectrum is the band SPL, or spectrum level, in a series of adjacent bands.

For a broad-band signal, the overall SPL is greater than the band SPL and the band SPL is greater than the spectrum level. For a pure tone, the overall SPL, the band SPL and the spectrum level are the same.

A white noise signal has a spectrum level that is independent of frequency (i.e. constant SPL per Hertz).

A pink noise signal has a spectrum level that is inversely proportional to frequency (i.e. constant SPL per octave).

A fractional octave band (e.g. 1/3 octave) has a bandwidth that is proportional to frequency.

A pink noise signal has a flat spectrum when analyzed in fractional-octave bands.

A critical band is a band within which the loudness of a continuously-distributed signal of constant SPL is independent of bandwidth.

2.1 Narrow-band signals

2.1.1 Sinusoidal (pure tones)

Sinusoidal signals are used in the ANSI hearing aid tests, in Multicurve 2 cm³ frequency response tests (gain or output), in 2 cm³ and on-ear single frequency (manual) tests, in Insertion gain tests and in tone-burst maximum output (MPO) tests. The MPO stimulus consists of a series 128ms tone bursts with 128ms gaps at an SPL of 90 dB SPL in the test box and 85 dB in the sound field. Frequencies used are 1/12 octave except for the MPO test which uses 1/3 octave frequencies. Levels from 40 to 90 dB SPL are available at the reference microphone. Control of the level at each frequency is maintained by measuring the frequency response of the signal path to the reference microphone 256 ms before each test using a 256 ms, 50 – 55 dB SPL, multi-tone complex or a 256 ms chirp, depending on the test to follow. Drive levels at each frequency are then set to achieve the desired band levels at the reference microphone. For on-ear measurements, tones are frequency modulated (triangle, ± 3% over 128 ms).

2.2 Broad-band signals

2.2.1 Pink Noise signal

The pink noise signal is available for Multicurve 2 cm³ coupler gain and output frequency response measurements and for Insertion gain and Speechmap on-ear measurements. It is a pseudo-random signal composed of 1024 simultaneous tones summed to provide a crest factor of 12 dB. The spectrum of the pink noise signal is controlled by the reference microphone in conjunction with a digital feedback loop with a frequency resolution of 1/12 octave and a response time of about 1/3 of a second. Pink noise was selected as a test signal because it has equal energy per octave, producing a flat spectrum when analyzed in 1/12 or 1/3 octave bands. Figure 1 shows the 1/3 octave spectrum of the pink noise signal and the noise signal specified in ANSI S3.42-1997. Note that the ANSI S3.42-1997 spectrum represents speech peaks not the long-term average.


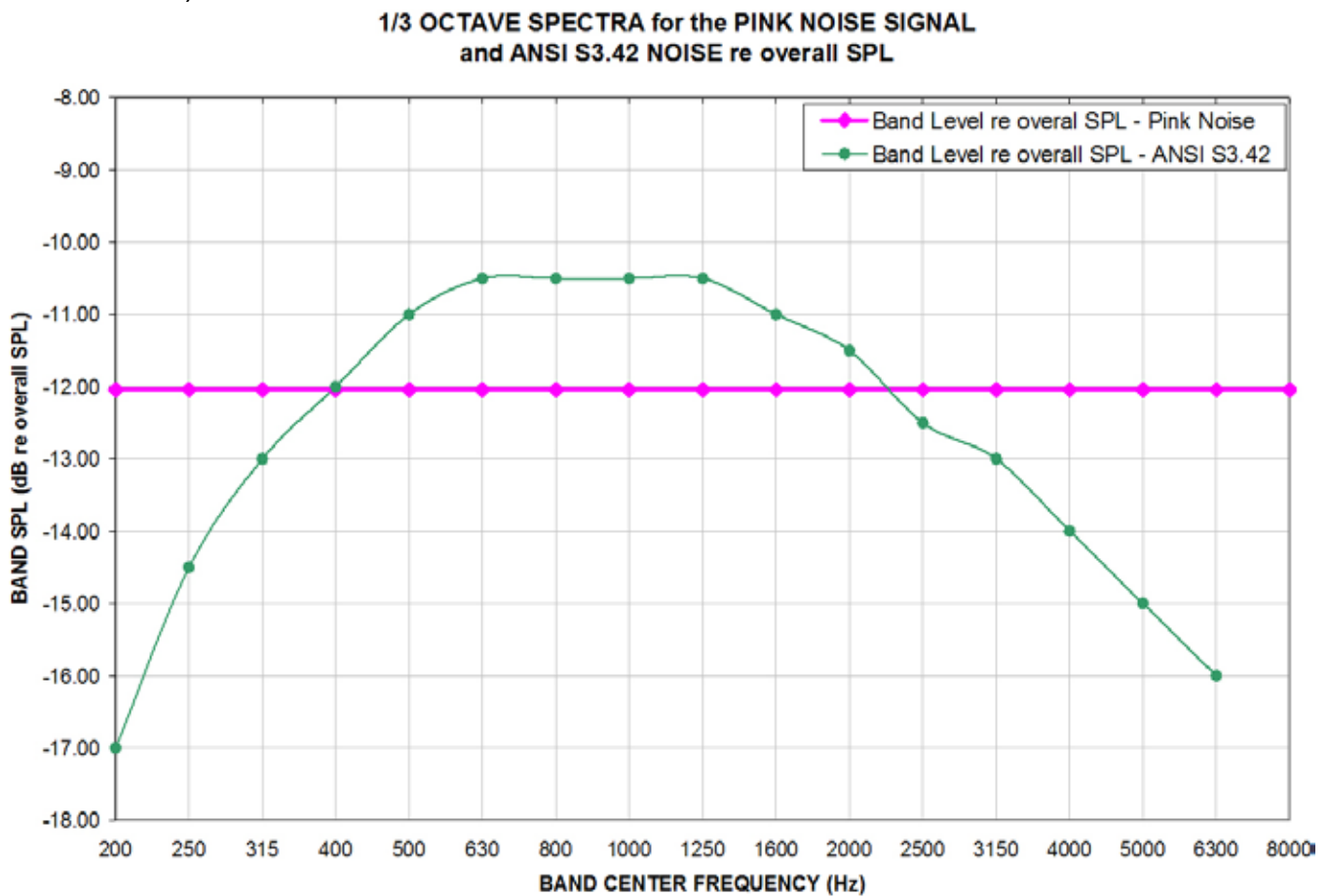


Figure 1: 1/3 octave spectra for the pink noise signal and the noise signal specified in ANSI S3.42-1997.

2.2.2 Dual-direction pink noise signal (Verifit only)

This unique test signal is available in the dual-source Verifit test chamber and for on-ear measurements by adding an additional sound-field speaker. It is similar to the pink noise signal described previously except that the component tones are presented simultaneously from two speakers, half from each. The spectrum of the tone complex from each speaker is independently controlled with 1/12 octave resolution

at the reference microphone. Two simultaneous response curves – one for each speaker – are derived from the measured output of the hearing aid. This provides a real-time measure of the functioning of the directional features of hearing aids that is independent of compression or noise reduction algorithms, unlike other methods that sequentially measure response from different directions and work only with these features disabled.

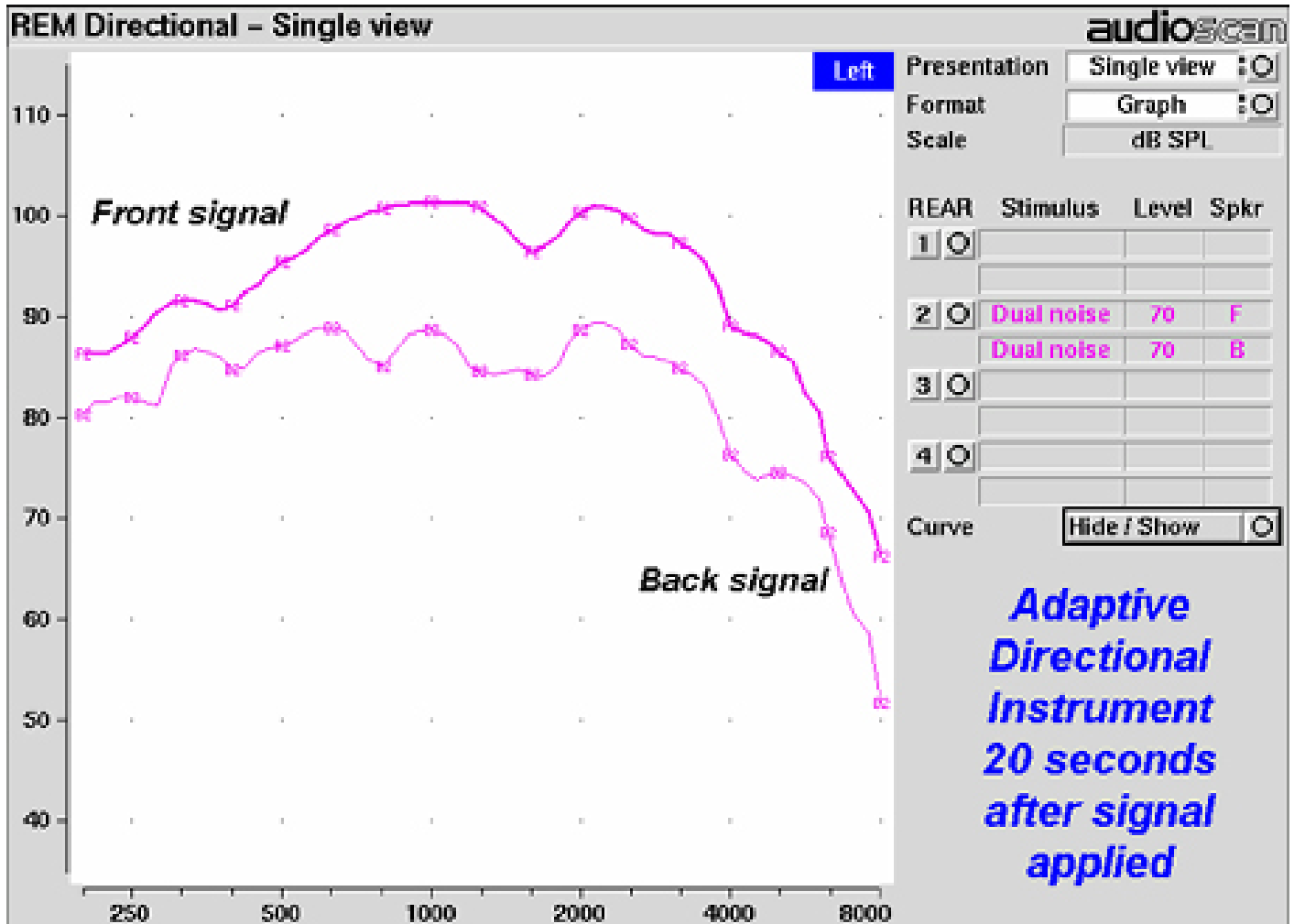

2.2.3 Real-speech signals

Real-speech signals are provided in Speechmap for both On-ear and Test box measurement modes. Four different digitized speech passages (2 male, 1 female and 1 child) are provided as well as the International Speech Test Signal (ISTS) and the single-talker International Collegium of Rehabilitative Audiology (ICRA) distorted speech signal. In addition, one of the speech passages has been filtered to provide 3 variations for evaluating frequency-lowering hearing aids. Each has a duration of 10 – 15 seconds and may be presented as a single passage or in a continuous loop. In order to provide a repeatable speech signal to the device under test, the signal path must be equalized prior to the presentation of the speech signal. This is accomplished by presenting a 896 ms pink noise burst at the selected speech level, 256 ms prior to the start of each speech passage and adjusting a digital filter to provide a flat response at the reference microphone.

The two “Speech-std” signals are by the same male talker, filtered to provide the long-term average speech spectrum (LTASS) recommended by Cox & Moore (1988) for average vocal effort. This is the LTASS assumed in the Desired Sensation Level (DSL) method of hearing aid fitting. In addition, four special versions of the Speech-std (1) test stimulus are provided in Speechmap to assist in the adjustment of frequency-lowering hearing aids. These are called Speech3150, Speech4000, Speech5000 and Speech6300. The Speech3150 stimulus has had the bands at 1000 Hz and above attenuated by 30 dB except for the 1/3 octave band at 3150 Hz which is unattenuated. Similarly, the Speech4000 stimulus has an unattenuated band at 4000 Hz, the Speech5000 stimulus has an unattenuated band at 5000 Hz and the Speech6300 stimulus has an unattenuated band at 6300 Hz (Figure 4). For these stimuli, the indicated level is for the unfiltered Speech-std (1) passage and the band levels in all unattenuated bands are the band levels present in the Speech-std (1) passage at that level. These three signals may be used to determine the amount of frequency shift provided by frequency-lowering hearing aids and the sensation level of the lowered components of the speech signal.

The “female” and “child” signals are presented “as recorded” without any processing and have been chosen to provide a range of spectra.

The ISTS was developed under the European Hearing Instrument Manufacturers Association (EHIMA) which holds the copyright. The sound file is available free of charge from the EHIMA website. The ISTS consists of 500 ms segments from recordings of 6 female talkers reading the same passage in American English, Arabic, Chinese, French, German and Spanish. These segments have been spliced together with appropriate pauses and filtered to match the average female spectrum from Byrne et al, An international comparison of long-term average speech spectra. *J. Acoust. Soc. Am.* 96 (1994), 2108-2120. For more details, consult the EHIMA website (www.ehima.com).

The ICRA distorted speech signal is a recording of an English-speaking talker that has been digitally modified to make the speech largely unintelligible. The resultant signal has many of the properties of real speech but has a harsh sound and lacks harmonic structure. The latter may be significant for hearing aids which use this property of speech to control noise reduction schemes. The LTASS is similar to the “Speech-std” signals up to 5 kHz.

Overall SPLs of 50, 55, 60, 65, 70 and 75 dB (at the reference microphone) are available. Soft and Average levels (50 - 70 dB SPL) have the same spectrum. A Loud vocal effort filter is applied to the 75 dB level signals. This is shown in Figure 5.

In the Test box mode in Speechmap (previously called S-REM) measurements made in the 2 cc coupler in the test chamber are converted to estimated real ear SPL. In this mode, the effects of hearing aid microphone location need to be included in the test signals so spectra are further altered to include the microphone location effects shown in Figure 6.

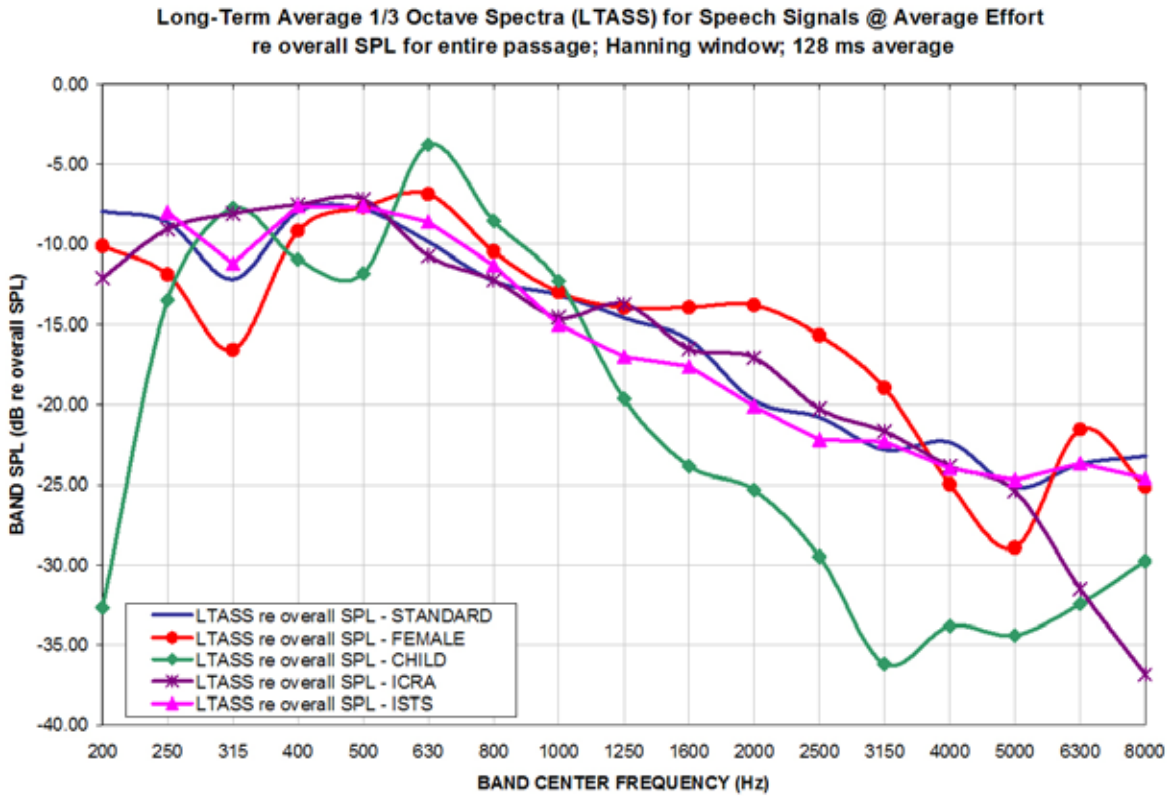


Figure 3: LTASS for Speechmap speech signals at average vocal effort

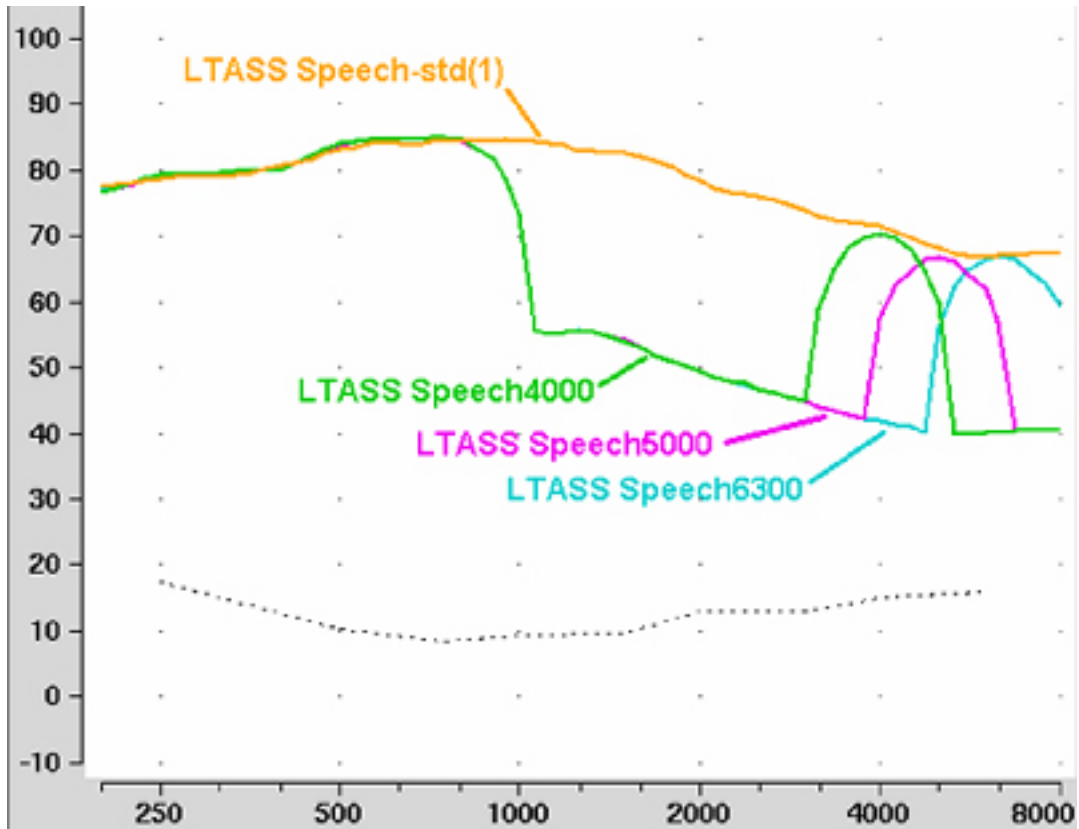


Figure 4: LTASS for Speech-std(1), Speech4000, Speech5000, Speech6300 for the FM boom location effects of Figure 6. Note that curves are 1/3 octave band levels at 1/12 octave intervals which causes the 1/3 octave bands at 4000, 5000 and 6300 Hz to appear broader than 1/3 octave. For clarity, the Speech3150 curve has been omitted.

LOUD VOCAL EFFORT EFFECT re nominal band SPL

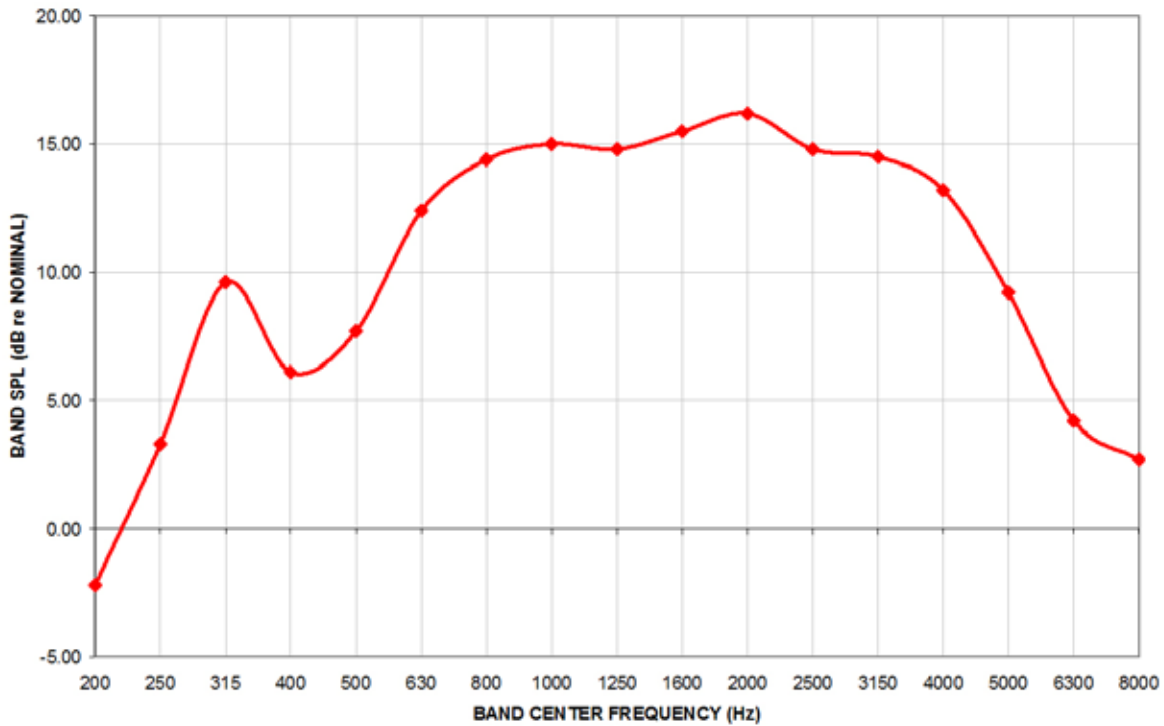


Figure 5: Loud vocal effort effect in dB re nominal band levels. Nominal band levels are band levels for an overall SPL of 65 dB. This shaping is applied to the 75 dB SPL speech and ICRA signals.

MICROPHONE LOCATION EFFECTS re nominal band SPL

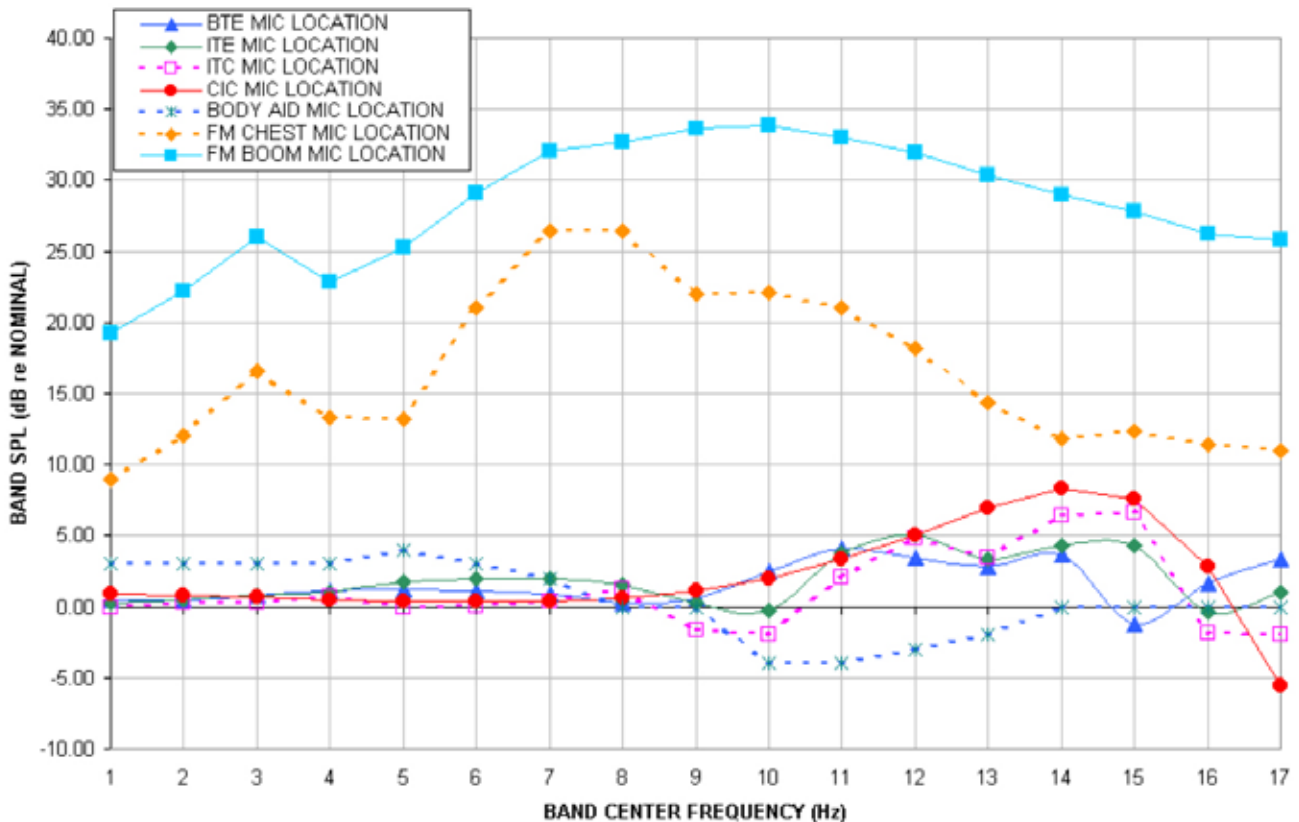


Figure 6: Microphone location effects in dB re nominal band SPL. Nominal band levels are band levels for an overall rms level of 65 dB SPL and average vocal effort in a free sound field.

In addition to these well-controlled and repeatable signals, live speech may be used as a test signal. It will, of course, be unequalized and at an uncontrolled level.

3 Analysis of broad-band signals

Broad-band signals contain energy at many frequencies simultaneously. Such signals are usually analyzed in a series of narrow frequency bands to produce a spectrum. The auditory system functions on a logarithmic frequency scale and analyzes broad-band signals in critical bands which approximate 1/3 octave bands (Figure 7). Using 1/3 octave bands for analysis of broad-band signals allows measured levels to be compared more readily to narrow-band behavioral measures, such as threshold. For this reason, analysis of broad-band signals in all On-ear measurements (including Speechmap Test box mode) is in 1/3 octave

bands. Other analyzers use narrower analysis bands, sometimes having constant bandwidth. As shown in Figure 7, this can result in significantly underestimating the audibility (or comfort or discomfort) of a complex signal.

It should be noted that, in all On-ear measurements, 1/3 octave band SPL is displayed (and reported when saving data to a file) at 1/12 octave frequencies which provides curve smoothing and causes the spectrum of narrow-band signals to appear broadened. When calculating overall rms from these data, the SPL at 225 Hz and subsequent 1/3 octave increments should be used.

Analysis of broad-band signals in all Test box tests, except Speechmap, is in 1/12 octave bands and the band SPL is displayed at 1/12 octave frequencies.

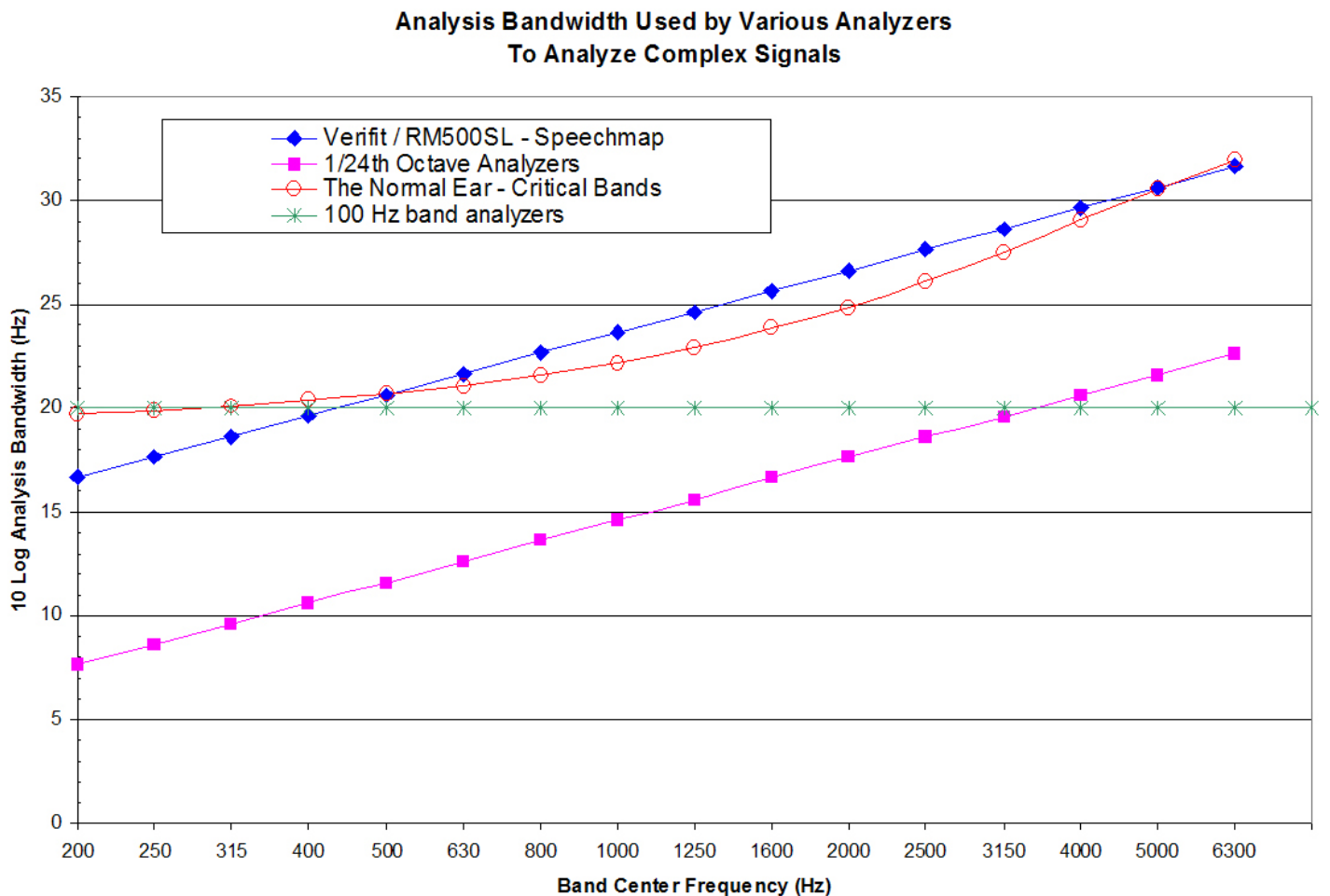


Figure 7: Analysis bandwidths of some real-ear analyzers and critical bands for a normal ear. If the analysis bandwidth is less than a critical band, aided response curves shown to be at threshold (or UCL) will actually be well above it.

3.1 Pink noise signal analysis

All band levels are calculated with an averaging time of 128 ms.

For a linear hearing aid, a pink noise stimulus will result in a response curve that has the same shape as one obtained using a swept pure tone. However, it must be remembered that, while a swept tone has the same band SPL as overall SPL, the band SPL for a noise signal is significantly lower than the overall SPL. Consequently, for a linear aid, output curves obtained using 1/12 octave analysis will be about 18 dB lower than the output curves obtained using a swept tone at the same overall SPL. As long as the hearing aid is linear, the gain obtained will be the same for both signals. Figure 8 shows output (A) and gain (B) for a linear hearing aid, obtained using a swept tone (1) and pink noise (2) with a 60 dB overall SPL.

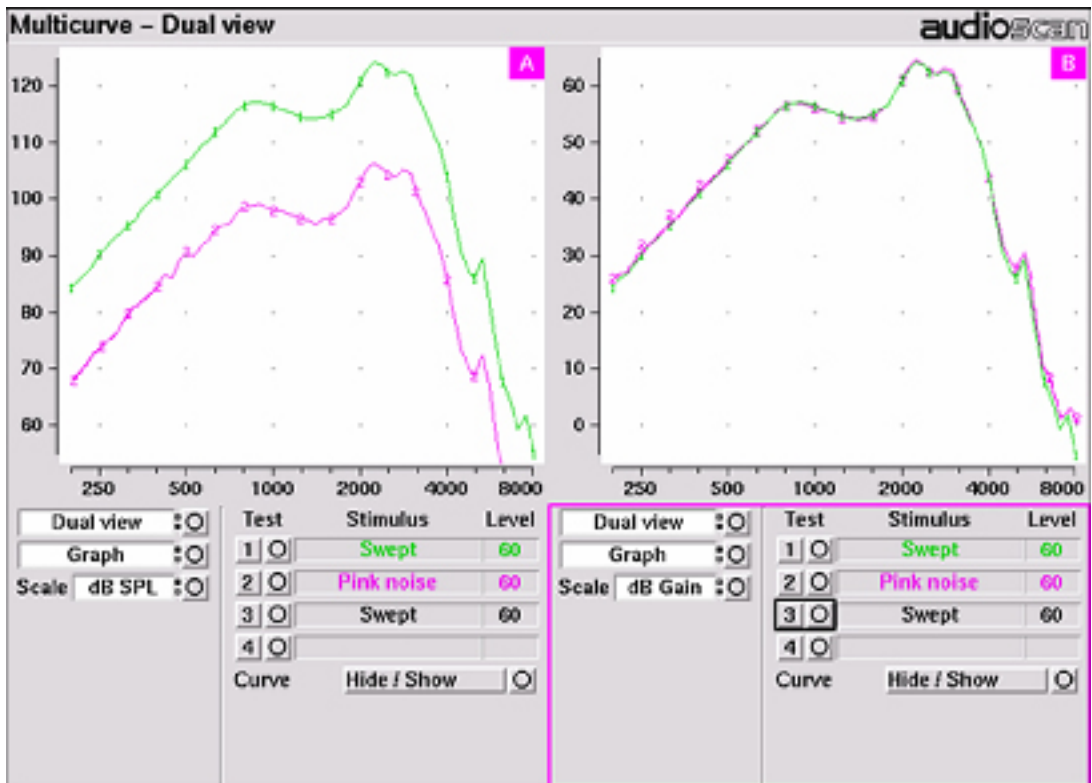


Figure 8: Output (left) and Gain (right) curves for a linear hearing aid, generated using a swept tone (1) and pink noise analyzed in 1/12 octave bands (2). (Dual view not available in RM500SL)

3.2 Real-speech signal analysis

One of the most-used measures of a speech signal is the long-term average speech spectrum (LTASS). This is a 1/3 octave spectrum averaged over a sufficiently long portion of the speech material to provide a stable curve. In practice a 10 second average meets this

requirement and, for this reason, all Speechmap passages are at least 10 seconds long. The LTASS curves displayed in Speechmap are 1/3 octave band levels at 1/12 octave intervals. It should be noted that this results in smoothing and the apparent broadening of the spectrum of a narrow-band signal. For example a 1/3 octave band of speech will exhibit a spectrum 2/3 octave wide.

The dynamic nature of speech is often characterized by the distribution of short-term levels in each 1/3 octave band. Historically, time periods of 120, 125 or 128 ms have been used. In Speechmap, 1/3 octave band levels at 1/12 octave intervals are derived every 128 ms. The level in each band that is exceeded by 1% of the samples (called L1, or 99th percentile) has historically been referred to as the speech peak for that band. The curve for these L1 levels is approximately 12 dB above the

LTASS. The level in each band that is exceeded by 70% of the samples (called L70, or 30th percentile) has historically been called the valley of speech for that band. The curve for these L70 levels is approximately 18 dB below the LTASS. The region between these two curves is often called the speech region, speech envelope or speech "banana". The speech envelope, when derived in this way, has significance in terms of both speech detection and speech understanding.

Generally, speech will be detectable if the L1 level is at or near threshold. The Speech Intelligibility Index (SII) is maximized when the entire speech envelope (idealized as a 30 dB range) is above (masked) threshold. This will not be an SII of 100% (or 1) because of loudness distortion factors, but higher SII values will not produce significantly higher scores on most test material.

The speech-reception threshold (SRT) is attained when the LTASS is at threshold (approximately - depending on test material and the individual). These scenarios are shown in Figures 10 - 12 which follow.

It should be noted that analysis methods which use shorter time periods produce higher peak levels and significantly different speech envelopes. In order to produce results that can be directly compared to measures of threshold (and UCL), the analysis time period needs to approximate the integration time of the ear. Although this varies with frequency and individuals, a value between 100 - 200 ms is likely. The Verifit and RM500SL use a 128 ms analysis time period as an approximation because it also has considerable historic support.

Because the spectrum of a speech signal varies with time, it is necessary to average measurements over

several seconds to obtain a stable, repeatable result. However, the effects of adjustments to hearing aid parameters need to be seen quickly in order to be useful for optimization. To resolve these conflicting needs, speech signals are first presented in a repeating loop with a sound-field equalization test prior to the start of each passage. A running calculation of the short-term speech envelope and average is performed on 2 - 3 seconds worth of data and is displayed in real time so that the effects of adjustments to hearing aid parameters are easily viewed. Pressing the Continue key causes the passage to restart and run in its entirety. The envelope and average are calculated over the full passage to provide stable and repeatable data.

When using live speech as a test signal, a "Freeze curve" function is available to capture the short-term spectra for examination and counseling purposes.

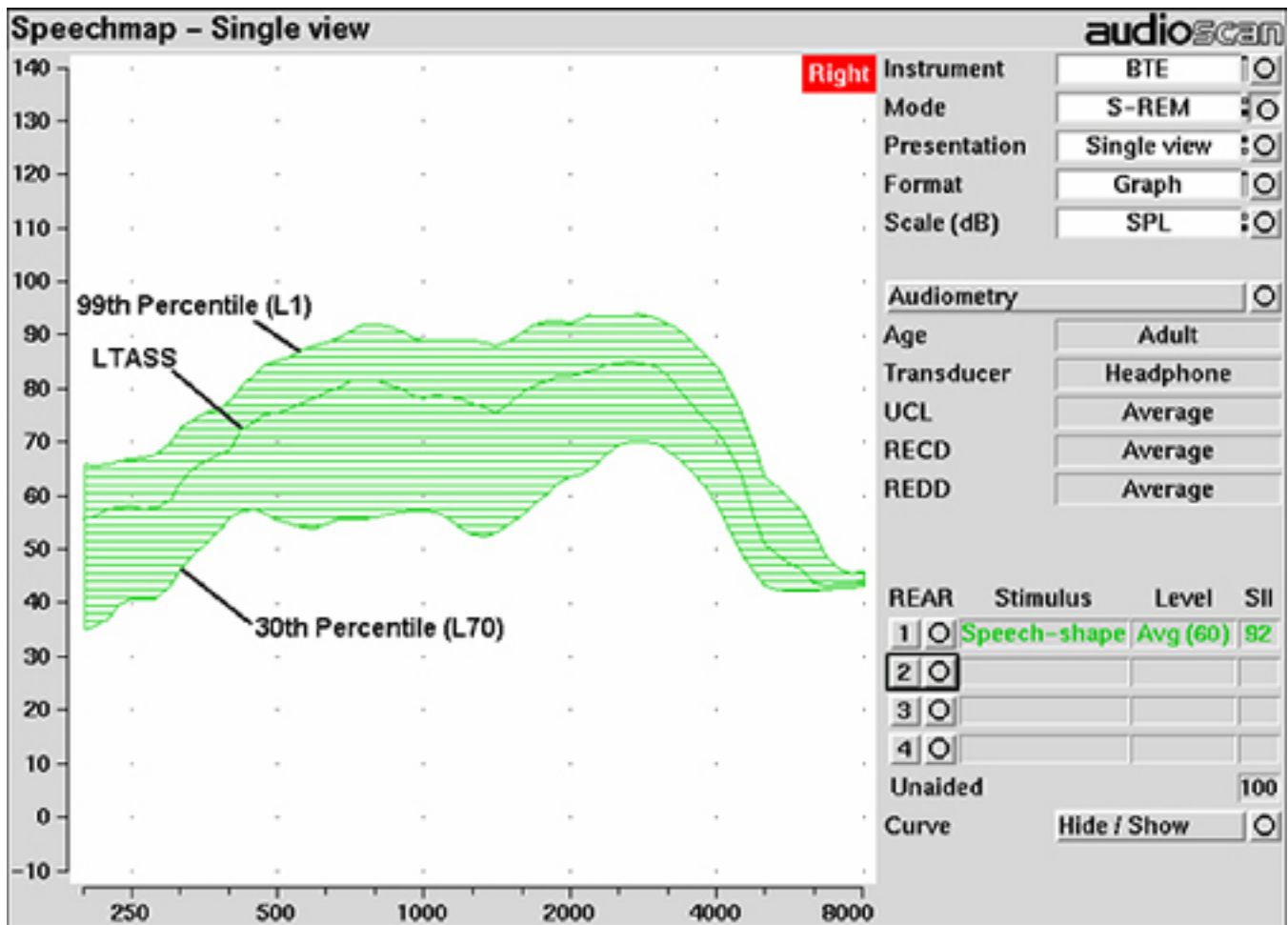


Figure 9: Example of speech envelope and LTASS

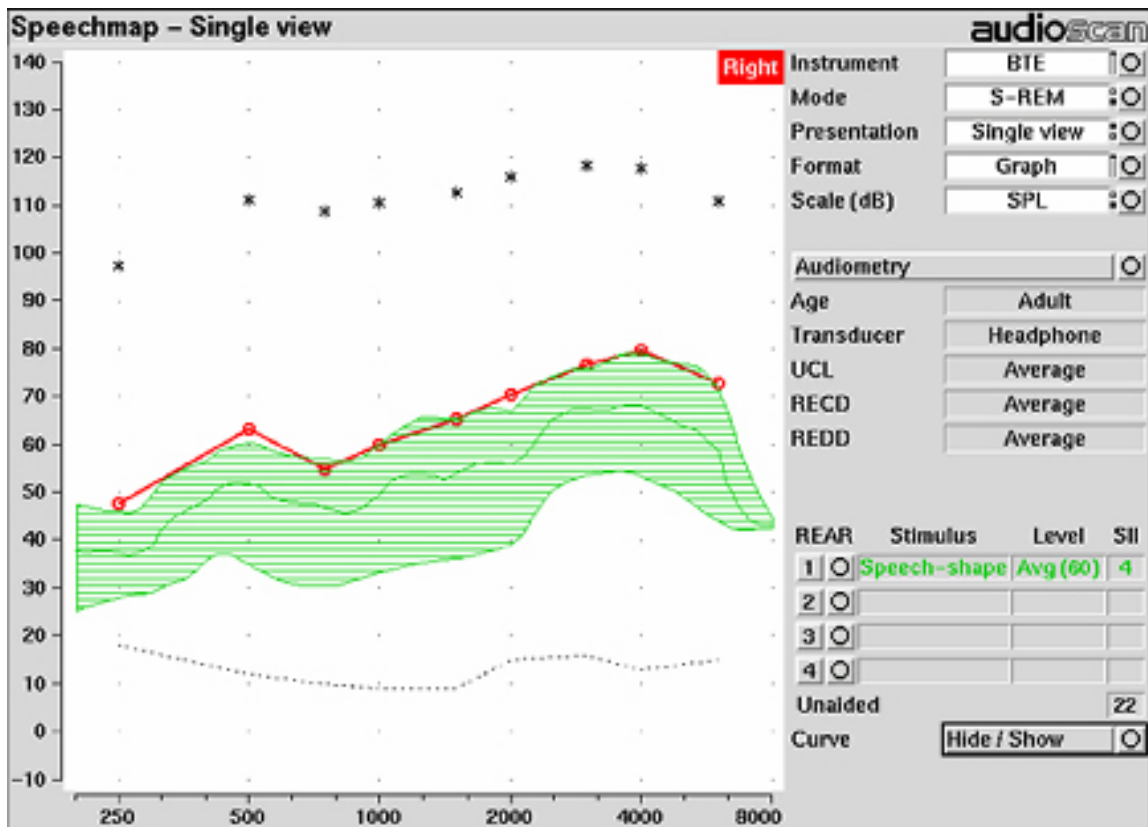


Figure 10: Speech is detectable but not understandable if the 99th percentile is at threshold

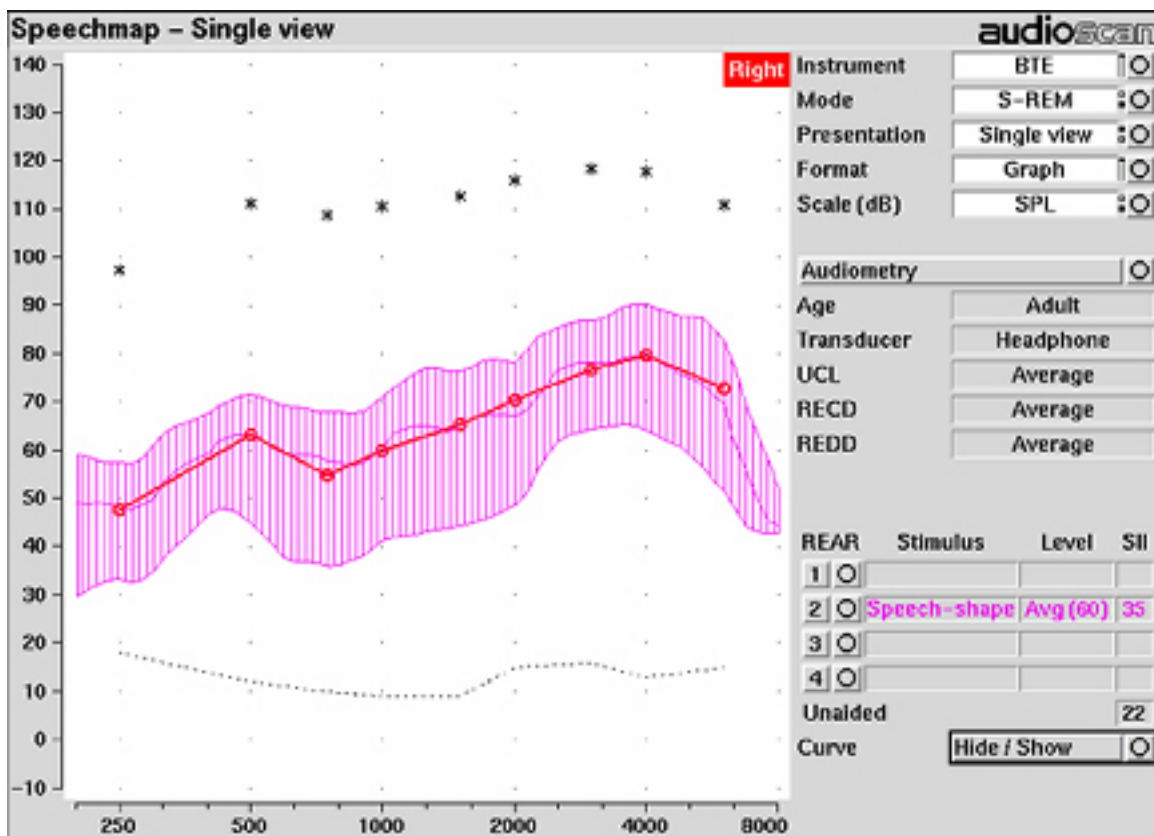


Figure 11: When LTASS is at threshold, SII \approx 33% which corresponds approximately to SRT

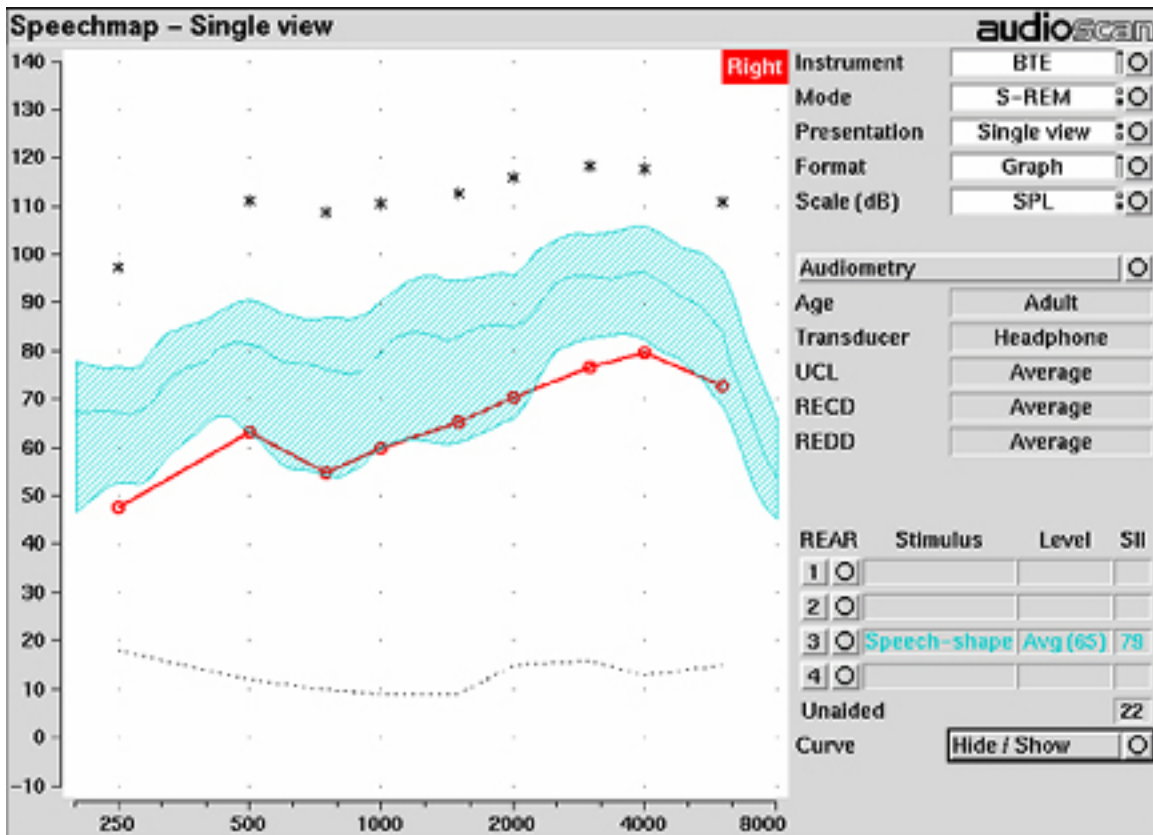


Figure 12: The highest SII is obtained when the entire speech envelope is above threshold

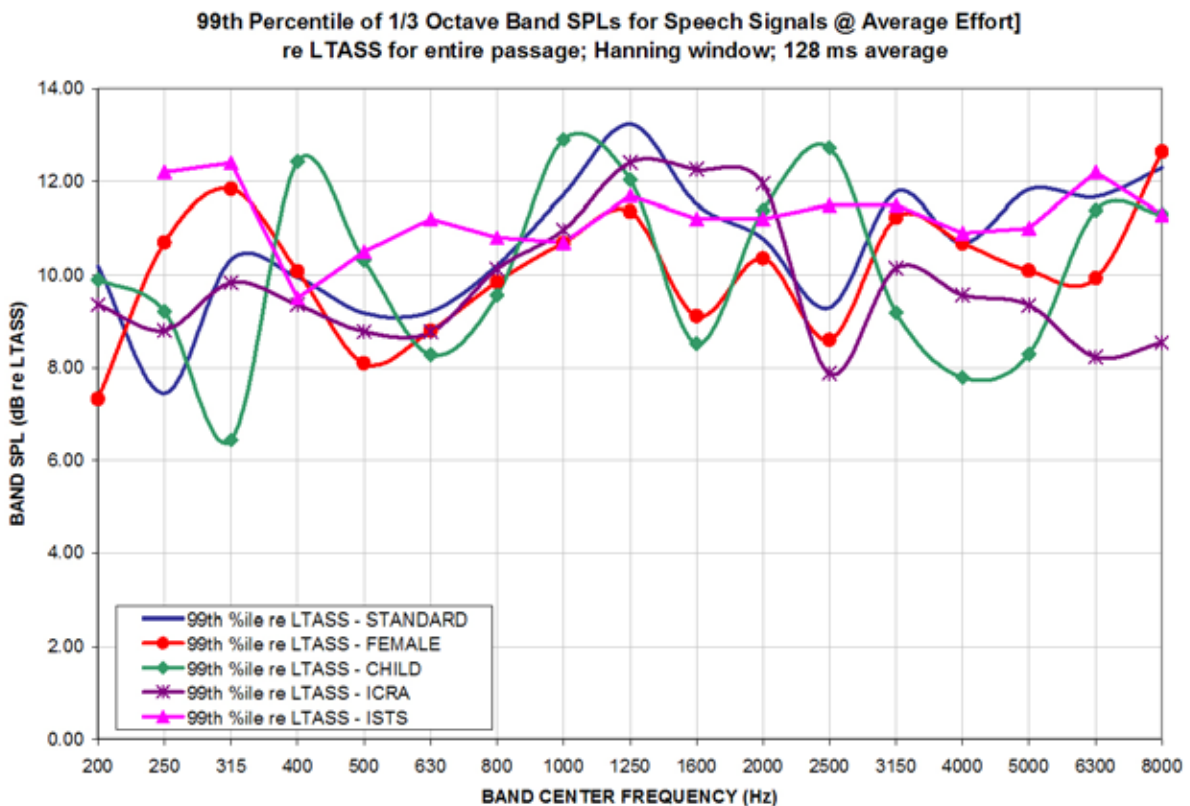


Figure 13: Level of 99th percentile for Speechmap speech signals at average vocal effort

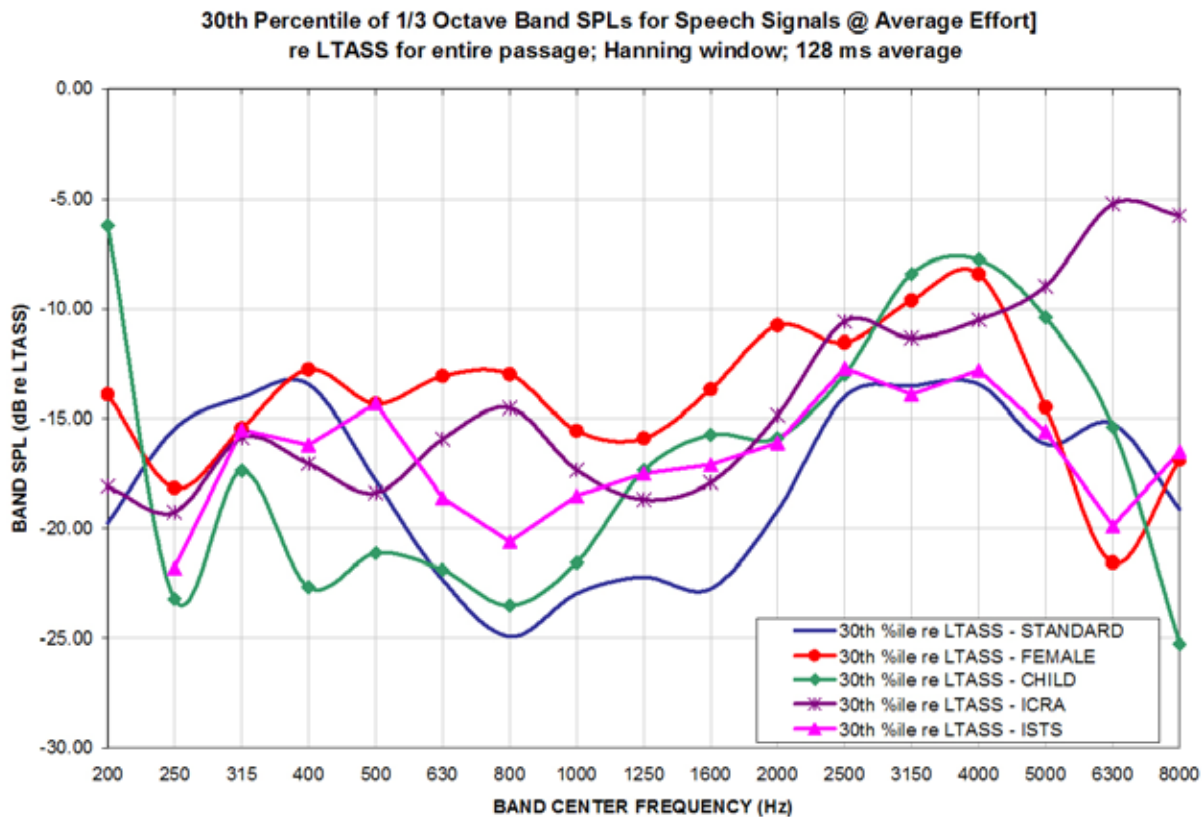


Figure 14: Level of 30th percentile for Speechmap speech signals at average vocal effort

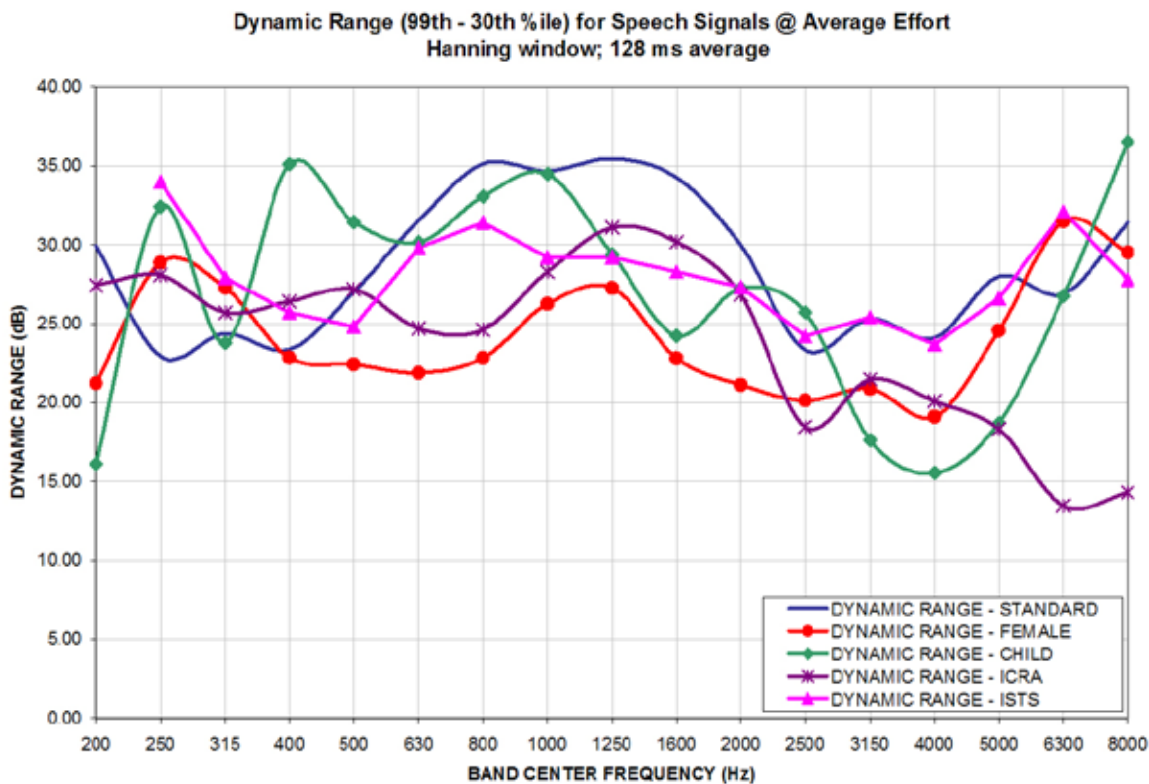


Figure 15: Dynamic range of Verifit speech signals at average vocal effort