

audioscan[®]

Verifit[®]

VERIFIT[®] HEARING INSTRUMENT FITTING SYSTEM

USER'S GUIDE

Version 3.0

ã Dec 2007

Table of Contents

1	About Verifit	7
1.1	Section overview.....	7
1.2	New in this software release.....	8
1.3	Electronic user's guide	8
1.4	Warranty, Trademarks, Acknowledgements	9
1.5	Notices for the European Community	9
1.6	Declaration of EMC compliance for the European Community	10
2	Getting Started	11
2.1	Section overview.....	11
2.2	Unpacking and connecting	11
2.3	CD-ROM installation	12
2.4	Microphone connection	13
2.5	Microphone care	14
2.6	Battery pill use and care.....	15
2.7	Mouse, keyboard, barcode scanner.....	15
2.8	External display, printer, speaker	16
3	General Operation	16
3.1	Section overview.....	16
3.2	Input device operation.....	17
3.3	Barcode data input.....	18
3.4	Keypad keys	19
3.5	Menus, lists and buttons.....	20
3.6	Screen messages and Help	20
3.7	CD-ROM removal and replacement.....	21
3.8	Software updating	21
4	General Setup	21
4.1	Section overview.....	21
4.2	Date and time setup	22
4.3	Display settings.....	22
4.4	Saving test setup	22
5	Networking	23
5.1	Section overview.....	23
5.2	Networking requirements	23
5.3	Networking setup	24
5.4	Single computer connection	24
6	Printing and Storing Results	25
6.1	Section overview.....	25
6.2	Internal printer paper loading.....	26
6.3	Barcodes, tabular data, headers and comments on printouts.....	27
6.4	Local printer setup	28
6.5	Printing to a USB memory stick.....	28


6.6	Network printer setup	29
6.7	Printing to a network file	30
6.8	Storing and restoring data - setup.....	31
6.9	Storing and restoring data	32
6.10	Data storage using NOAH.....	33
6.11	NOAH 3 module use	34
7	Test Box Measures - Setup	35
7.1	Section overview.....	35
7.2	Test box single or dual view	35
7.3	Graph or table format.....	36
7.4	Output or gain scale.....	36
7.5	Hide or Show test box curves.....	36
7.6	1996 or 2003 ANSI standard	36
7.7	ANSI test frequencies	37
7.8	Test box calibration facts.....	37
7.9	Calibrating test box reference microphone.....	38
7.10	Calibration check for coupler microphone	39
7.11	Coupling the hearing instrument.....	40
7.12	Positioning the hearing instrument.....	41
8	ANSI S3.22 Hearing Aid Tests.....	41
8.1	Section overview.....	41
8.2	ANSI S3.22-1996 facts.....	42
8.3	ANSI S3.22-2003 facts.....	42
8.4	ANSI 1996 Linear and AGC tests	43
8.5	ANSI 2003 Linear and AGC tests	44
8.6	ANSI test results	45
8.7	ANSI input-output curves	46
8.8	ANSI telecoil terminology	47
8.9	ANSI telephone simulator (TMFS) test	48
8.10	ANSI test loop test	49
8.11	Telecoil test results	50
9	Other Test Box Measures.....	51
9.1	Section overview.....	51
9.2	Harmonic distortion	52
9.3	Noise reduction.....	52
9.4	Feedback suppression test.....	53
9.5	Directional function test.....	54
9.6	Directional ITE positioning.....	55
9.7	Directional BTE positioning	56
9.8	Test box directional procedure	57
9.9	Multicurve procedure.....	58
9.10	Multicurve results	59

9.11	Spectral analysis in Multicurve	59
9.12	Battery drain test.....	60
9.13	Manual control procedure.....	61
9.14	Sound level meter using manual control	61
10	Sensory loss simulator	62
10.1	Sensory loss simulator description	62
10.2	Sensory loss simulator operation.....	62
11	On-Ear Measures - Setup.....	63
11.1	Section overview.....	63
11.2	External sound-field speaker setup.....	64
11.3	On-ear calibration facts	64
11.4	Calibration of on-ear probe microphone.....	65
11.5	Calibration check for probe module	66
11.6	Max TM SPL setup.....	67
11.7	ABR nHL to eHL setup.....	67
11.8	Positioning the client	68
11.9	Positioning the probe tube.....	68
12	On-Ear Measures Screen Setup	69
12.1	Section overview.....	69
12.2	On-ear single or dual view.....	69
12.3	Graph, table or 2cc target format.....	70
12.4	SPL or HL scale	70
12.5	Hide or show on-ear curves.....	71
13	On-Ear Instrument Measures	72
13.1	Section overview.....	72
13.2	On-ear directional test overview	72
13.3	On-ear directional testing	73
13.4	On-ear feedback test.....	74
13.5	On-ear noise reduction test	74
13.6	On-ear manual control.....	75
13.7	Sound level meter using on-ear microphones	75
14	Occlusion Effect Test.....	76
14.1	Section overview.....	76
14.2	Occlusion effect measurement	76
15	RECD (real-ear to coupler difference).....	77
15.1	Section overview.....	77
15.2	RECD facts.....	77
15.3	RECD coupler response.....	78
15.4	RECD on-ear response	79
15.5	RECD results	80
16	Insertion Gain	81
16.1	Section overview.....	81

16.2	Insertion gain in SPL.....	82
16.3	Insertion gain in HL.....	83
16.4	Audiometric data entry.....	84
16.5	REUR measurement procedure.....	85
16.6	REAR measurement procedure.....	86
16.7	SII calculation in Insertion gain.....	87
16.8	CROS fitting using Insertion gain.....	88
17	Speechmap ®.....	89
17.1	Section overview.....	89
17.2	Speechmap facts.....	89
17.3	DSL 5.0 in Speechmap.....	90
17.4	DSL 5.0 changes.....	91
17.5	NAL-NL1 in Speechmap.....	91
17.6	Camfit in Speechmap.....	92
17.7	Using Speechmap.....	92
17.8	Screen tour - unaided screen.....	93
17.9	Screen tour - aided screen.....	93
17.10	On-ear or Test box mode.....	94
17.11	SII calculation in Speechmap.....	95
18	Speechmap Fitting Procedures.....	96
18.1	Section overview.....	96
18.2	Assessment data entry.....	96
18.3	Assessment data choices - 1.....	97
18.4	Assessment data choices – 2.....	98
18.5	Fitting to targets for soft speech.....	99
18.6	Fitting to targets for loud sounds.....	100
18.7	Fitting to targets for mid-level speech.....	101
18.8	Open fittings in Speechmap.....	101
18.9	FM fitting and verification.....	102
18.10	CROS fitting in Speechmap.....	103
19	Speechmap Technical Details.....	104
19.1	Section overview.....	104
19.2	Speechmap stimuli.....	104
19.3	Stimulus levels.....	105
19.4	Microphone location effects.....	106
19.5	Speech signal analysis.....	107
20	Viewport.....	108
20.1	Section overview.....	108
20.2	Viewport setup.....	108
20.3	Accessing Viewport tests.....	109
20.4	Viewport Audibility test.....	109
20.5	Viewport Test box directional test.....	110

20.6	Viewport On-ear directional test	111
20.7	Viewport Noise reduction test	111
20.8	Viewport Test box feedback test.....	112
20.9	Viewport On-ear feedback test	112
21	Troubleshooting.....	113
21.1	Section overview.....	113
21.2	Self test failures	113
21.3	Test box high distortion or noise.....	114
21.4	Test box curves inconsistent	114
21.5	Test box curves differ from specifications	115
21.6	Test box speaker overdriven	115
21.7	No test box reference mic. detected	115
21.8	Invalid test box calibration	116
21.9	No on-ear ref. mic. detected	116
21.10	Invalid on-ear calibration	116
21.11	Sound-field speaker overdriven	117
21.12	Barcode scanner malfunction	117
22	Technical Specifications	118
23	Glossary.....	119
24	References.....	123
25	Appendix 1: Manufacturer Disclosure Statement for Medical Device Security	126
26	Appendix 2: AudioNote 2.1 VERIFIT and RM500SL Test Signals and Analysis.....	127


The Audioscan® Verifit® User's Guide Version 3.0

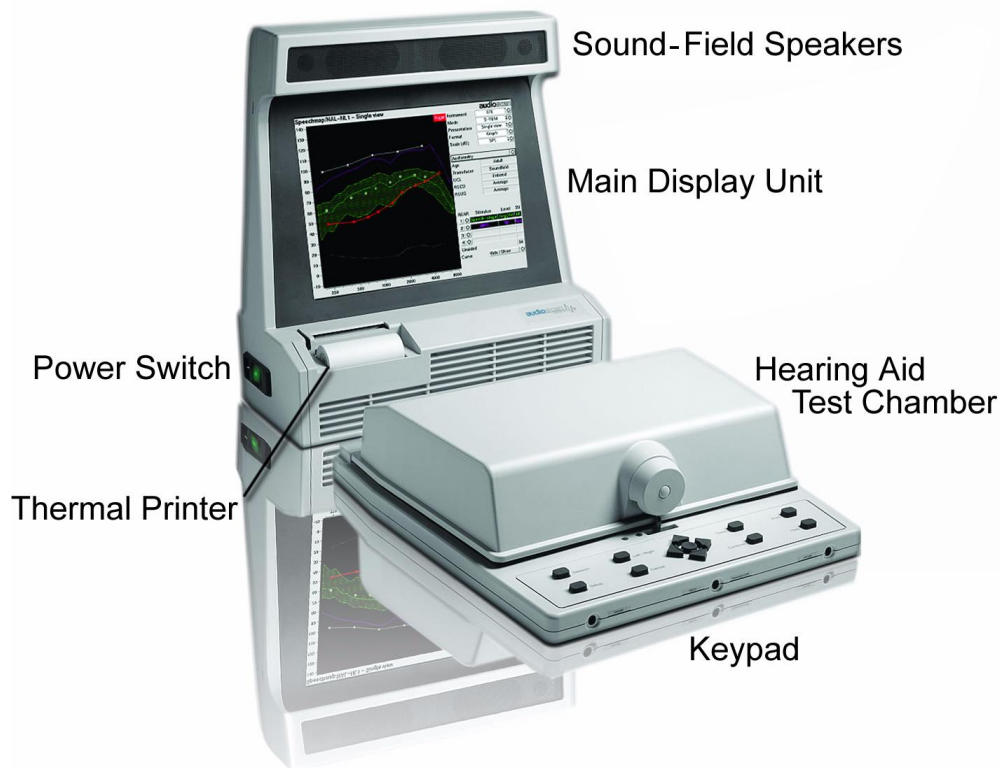

1 About Verifit

1.1 Section overview

This section provides a listing of features **new in this software release**, instructions for **accessing the electronic User's Guide**, contact, **warranty** and trademark information and **notices** for the **European Community**.

Help is available on the following topics by pressing <Help> (For long Help pages, use <Left/Right> (or mouse) to switch between the Help index and the Help page and  arrows (or mouse) to scroll through the page.)

- | | |
|---------------------------------|---|
| 2: Getting Started | 12: On-Ear Measures Screen Setup |
| 3: General Operation | 13: On-Ear Instrument Measures |
| 4: General Setup | 14: Occlusion Effect Test |
| 5: Networking | 15: RECD (real-ear to coupler difference) |
| 6: Printing and Storing Results | 16: Insertion Gain |
| 7: Test Box Measures - Setup | 17: Speechmap ® |
| 8: ANSI S3.22 Hearing Aid Tests | 18: Speechmap Fitting Procedures |
| 9: Other Test Box Measures | 19: Speechmap Technical Details |
| 10: Sensory loss simulator | 20: Viewport |
| 11: On-Ear Measures - Setup | 21: Troubleshooting |

In addition you will find a glossary, references and more information in the electronic user's guide on the Audioscan software CD.

FastFacts 1.1: Verifit Help topics

1.2 New in this software release

Software 3.0 introduces the following significant additions and changes:

- 1) **Viewport** – quick and easy digital instrument fitting and feature verification - all on one screen and documented on one-page.
- 2) **Noise reduction test** quickly demonstrates benefit for real-world noises
- 3) **Feedback suppression** test shows effectiveness in reducing feedback tendency and the impact of feedback suppression on speech
- 4) **Directional test** combining speech and noise to check speech-detecting adaptive directional systems
- 5) **DSL 5.0a** incorporating the latest revisions to the DSL 5.0 fitting method
- 6) **CAMFIT** – a fitting method for adults developed at the University of Cambridge
- 7) **Store complete sessions** to USB memory stick or a network file in XML format and return them to the Verifit at any time for viewing. **Please note that this new feature may result in the creation of Individually Identifiable Health Information which must be protected under the HIPAA Security Rule. See [6: Printing and Storing Results](#) .**
- 8) **Both graphic and tabular data** on one printout
- 9) **Receiver in the ear (RITE)** hearing instruments supported in Speechmap
- 10) **Speech envelope** can be hidden or displayed for each speech curve and is automatically hidden when a new test is run.
- 11) **Scrolling on-line help pages** – more help on each page

Please consult the relevant section of this User's Guide or Help for more details.

FastFacts 1.2: New in this software release

1.3 Electronic user's guide

A printable user's guide is on the Audioscan CD-ROM supplied with a new instrument or mailed as a software update. You may want to copy it to your PC before installing the CD-ROM in the Verifit. Once installed in the Verifit, the CD-ROM may be later removed by following the instructions in [3.7: CD-ROM removal and replacement](#).

A .pdf file viewer, such as Acrobat Reader (5.0 or higher) or Foxit Reader is required to view the User's Guide. Foxit Reader is supplied on the CD-ROM.

- 1) Insert the CD-ROM into the CD drive on your computer.
- 2) Select My Computer, then the CD drive and the User's Guide folder.
- 3) To install Foxit Reader, copy FoxitReader.exe to an appropriate location on your PC. To run FoxitReader.exe double click it.
- 4) To view the User's Guide, double-click the VerifitUsersGuide.pdf file.

The User's Guide may also be downloaded from www.audioscan.com.

FastFacts 1.3: Electronic users guide

1.4 Warranty, Trademarks, Acknowledgements

The **Audioscan Verifit model VF-1** is an electro-acoustic test system used to verify the performance of a hearing aid in an ear simulator (2cc coupler) or in the ear of a wearer. It is manufactured by Etymonic Design Inc., 41 Byron Ave., Dorchester, Ontario, Canada, N0L 1G0. www.audioscan.com
Phone: 800-265-2093 (USA only); 519-268-3313 **Fax:** 519-268-3256
Email: info@audioscan.com or service@audioscan.com

Warranty: The Verifit model VF-1 is warranted against defects for two years from date of purchase. Within this period, it will be repaired without charge for parts, labor or return shipping when returned prepaid to your authorized Audioscan service agent. **This warranty does not apply to battery substitutes (pills)**, which carry a 90 day warranty, or to equipment that, in our sole judgement, has been subject to misuse, or unauthorized alteration or repair.

Trademarks: Audioscan, Verifit, and Speechmap are registered trademarks of Etymonic Design Inc. DSL is a registered trademark of the University of Western Ontario. HP LASERJET is a registered trademark of Hewlett-Packard Company. IBM is a registered trademark of International Business Machines Corporation. K-AMP is a registered trademark of Etymotic Research Inc. KEMAR is a registered trademark of Knowles Electronics Inc., QUEST is a trademark of Quest Technologies Inc. PostScript is a registered trademark of Adobe Systems, Inc.

Acknowledgements: DSL 5.0 is used under license from the University of Western Ontario (UWO) which is solely responsible for its content. We acknowledge the support received from past and present staff at the National Centre for Audiology at UWO in implementing the DSL method. CAMFIT is used under license from Prof. Brian C.J. Moore, University of Cambridge, UK. We are indebted to the University of Memphis Hearing Aid Research Laboratory for permission to use some of their recorded speech material.

FastFacts 1.4: Warranty, trademarks, acknowledgements

1.5 Notices for the European Community

Authorised Audioscan representative within the European Community:
PC Werth Ltd., Audiology House, 45 Nightingale Lane, London SW12 8SP UK
Phone: +44 (0)20 8772 2700 www.pcwerth.co.uk

Electromagnetic Compatibility (EMC)

Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the following information:

- ✓ The Verifit should not be used adjacent to or stacked on other equipment. If this is necessary, its operation should be verified as normal in this configuration.
- ✓ Portable and mobile RF communications equipment can affect medical electrical equipment and may affect the performance of the Verifit.
- ✓ The Verifit has not been tested for immunity to electromagnetic disturbances. Performance degradation due to such disturbances (including electrostatic discharge) is considered normal and acceptable.

FastFacts 1.5: Notices for the European Community

1.6 Declaration of EMC compliance for the European Community

The compliances listed in the following table are met with the supplied RECD transducer, microphones, headphones, mouse and keyboard connected and with unterminated speaker cables (2), USB cables (2), ethernet cable, parallel printer cable, serial cable and VGA display cable connected. The connection of other devices may result in increased emissions.

Guidance and manufacturer's declaration - electromagnetic emissions

The Verifit is intended for use in the electromagnetic environment specified below. The user of the Verifit should assure that it is used in such an environment.

<u>Emissions test</u>	<u>Compliance</u>	<u>Electromagnetic environment - guidance</u>
RF emissions CISPR 11	Group 1	The Verifit uses RF energy only for its internal function. Therefore, its RF emissions are very low and not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The Verifit is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

FastFacts 1.6: Declaration of EMC compliance for the European Community

2 Getting Started

2.1 Section overview

This section provides help on unpacking the Verifit and connecting various components and accessories and on installing the CD-ROM.


The following topics are covered in this section:

- 2.2: Unpacking and connecting
- 2.3: CD-ROM installation
- 2.4: Microphone connection
- 2.5: Microphone care
- 2.6: Battery pill use and care
- 2.7: Mouse, keyboard, barcode scanner
- 2.8: External display, printer, speaker


FastFacts 2.1: Getting started

2.2 Unpacking and connecting

- 1) Carefully unpack the two parts of the Verifit and check the contents of the shipping box against the enclosed packing list. Note that some parts may be packed inside the test chamber.

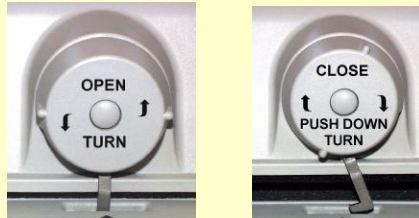

- 2) Connect the two 15 pin cables from the connectors on the rear of the test chamber to the connectors on the front of the main display unit as shown. The top side of each connector is labeled to assist you. **Take care not to force the connectors as the pins are easily bent.** Both short (as shown) and long cables are provided for flexibility in locating the two parts of the instrument.

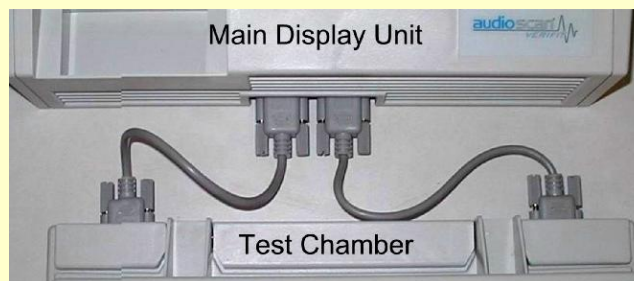

- 3) Connect the power cord to the socket next to the power switch on the left side of the main display unit. Plug the other end into a **grounded** 100 – 240 volt power outlet.

FastFacts 2.2: Unpacking and connecting

2.3 CD-ROM installation

Operating software, some test signals, the online Help files and a printable user's guide are all on the Verifit CD-ROM supplied with the instrument. **The CD-ROM must be present in the CD drive for normal instrument operation.** The CD drive door is located on the rear of the main display unit below the external speaker jacks. **The door of the CD drive will open only when power is first applied.** If necessary, to open the CD drive without power, insert the end of a small screwdriver (or a paper clip) in the hole near the light on the CD drive door.

- 1) Set the power switch to the ON position. The power switch should light and the display screen should become active.
- 2) Press the OPEN button on the door of the CD drive. The door should open.
- 3) Set the power switch to the OFF position.
- 4) Position the Verifit CD-ROM on the spindle, printed side up, and press gently downward until it snaps into place.
- 5) Gently push the CD drive door inward until it snaps closed.
- 6) Set the power switch to the ON position.



FastFacts 2.3: CD-ROM installation

2.4 Microphone connection

- 1) Plug the reference microphone and the coupler microphone into the connectors in the test chamber as shown.



- 2) Plug the probe microphone assemblies into the probe connectors located at the front edge of the built-in keypad as shown.



NOTE: A microphone extension cable (VA-130) is available from Audioscan. Standard audio extension cables should not be used. They will substantially increase noise levels.

FastFacts 2.4: Microphone connection

2.5 Microphone care

Coupler microphone:

- 1) **DO NOT** twist the cable when attaching a coupler to the coupler microphone. Turn only the coupler or unplug the coupler microphone before turning it.
- 2) **DO NOT** leave the coupler microphone plugged in when transporting.
- 3) **DO** make sure that the 2cc coupler is tightly screwed to the coupler microphone when performing hearing instrument tests. **Coupler leakage can cause feedback and erratic response curves.**
- 4) **DO** make sure the #13 tubing on the BTE (HA-2) coupler is free from any cracks/tears.
- 5) **DO** make sure that any replacement #13 tubing used on the BTE coupler is .38 inches in length.

Probe microphones:

Debris can alter the probe module reference microphone calibration and frequency response and can permanently clog the probe module port.

- 1) **DO NOT reuse probe tubes.** Debris and liquid from cleaning attempts can lodge in the enlarged base of the probe tube and subsequently clog the probe microphone port.
- 2) **DO NOT** attempt to open or repair the probe microphone. Attempting to repair the probe module may lead to damage or alteration of the factory calibration.
- 3) **DO** keep the probe module and probe tubes in a clean area.
- 4) **DO** stow the probe module when not in use or when transporting the Verifit.

FastFacts 2.5: Microphone care

2.6 Battery pill use and care

The thin connecting strip of **each battery pill is fragile**. When inserting pills into hearing instrument battery compartments, take care that this strip is not pinched or bent severely as the battery door is closed.

- 1) Select a battery pill that is appropriately sized for the hearing instrument that you are testing.
- 2) Insert the pill into the hearing instrument, carefully closing the battery door over the thin connecting strip.
- 3) Plug the pill's cable into the battery pill jack inside the Verifit test box (to the right of the coupler microphone connection).
- 4) Turn the hearing instrument on.



FastFacts 2.6: Battery pill use and care

2.7 Mouse, keyboard, barcode scanner

The Verifit may be operated from the **built-in keypad**, a **mouse** or a standard computer (QWERTY) **keyboard**, which may also be used to enter headers and comments on printouts. PS-2 connectors for these devices are provided on the rear of the main display unit. A USB mouse or keyboard may be plugged directly into either of the USB ports provided. See [3.2: Input device operation](#).

An optional **barcode scanner** may be connected to a USB port to enter threshold, UCL and RECD data directly from a Verifit or RM500SL printout. Barcoded threshold data printed by some Grason-Stadler audiometric equipment may also be scanned. Scanners other than that supplied by Audioscan may not work and are not supported by Audioscan.



In the European Community, if this device is connected to one or more other devices to make a system, the CE mark on this device is only valid if the other devices in the system carry a CE mark in respect of the Medical Device Directive and if the supplier has stated that the requirements of the Medical Device Directive will be met for this particular combination.

FastFacts 2.7: Mouse, keyboard, barcode scanner

2.8 External display, printer, speaker

An **external display monitor** may be connected to the 15-pin connector on the rear of the main display unit. It will automatically display what is on the internal screen. The monitor should be capable of 800 x 600 resolution.

An **external printer** may be connected to the parallel printer connector or to one of the USB ports. It may be color or black & white but it **must be PCL3, PCL 5 or Postscript compatible**. The external printer must first be selected in Setup. See [6: Printing and Storing Results](#)

Connectors for **external front and back sound-field speakers** are provided. Speakers are available from Audioscan but any small 4 - 8 ohm speaker having a smooth frequency response and good efficiency may be used. The rear external speaker is automatically selected when required by a test but the front speaker must first be selected in Setup. See [11.2: External sound-field speaker setup](#)



In the European Community, if this device is connected to one or more other devices to make a system, the CE mark on this device is only valid if the other devices in the system carry a CE mark in respect of the Medical Device Directive and if the supplier has stated that the requirements of the Medical Device Directive will be met for this particular combination.

FastFacts 2.8: External display, printer, speakers

3 General Operation

3.1 Section overview

The Verifit is operated by using the built-in keypad, a USB or PS-2 QWERTY keyboard (not included) or a USB or PS-2 computer mouse. Any of these pointing devices can be used to select items from menus, operate on-screen buttons and to enter audiometric data. In addition, audiometric data printed in barcode form by the Verifit or some Grason-Stadler products, may be scanned into the Verifit.

This section covers the following topics:

[3.2: Input device operation](#)

[3.3: Barcode data input](#)

[3.4: Keypad keys](#)

[3.5: Menus, lists and buttons](#)

[3.6: Screen messages and Help](#)

[3.7: CD-ROM removal and replacement](#)

[3.8: Software updating](#)

FastFacts 3.1: General operation

3.2 Input device operation

The Verifit may be operated by means of the **built-in keypad**, an external **mouse** or an external QWERTY **keyboard**. These devices are used to summon on-screen menus and select items from them, to operate on-screen buttons and to input data. The **scroll wheel** on this type of mouse will scroll through long lists in list boxes and in Help. **Clicking the right mouse button** generates an **image of the keypad** which may be operated by the mouse. **Clicking the left mouse button** when the mouse pointer is on a graph will display a **screen cursor** with a digital readout of X and Y co-ordinates. Clicking again will dismiss the cursor.


Function	Keypad	QWERTY	Mouse
Summon Help menu	<Help>	F1	Right click
Summon Setup menu	<Setup>	F2	Right click
Summon Test menu	<Tests>	F3	Right click
Summon Session menu	<Session>	F4	Right click
Switch between Ears or A/B Data or Help index and Help page	<Left/Right>	F5	Left click or Right click for keypad image
Print	<Print>	Print Scrn	Right click
Move across screen columns	↔ Arrows	↔ Arrows	Move mouse
Move within a screen column	⬆ Arrows	⬆ Arrows	Move mouse
Select a highlighted item or operate a screen button	Round key (<PICK> key)	Numeric Enter Key	Left click
Enter numeric data	Select point on chart	Numeric keypad	Left click on a point on chart
Enter text	Not possible	Any key	Not possible
Proceed from current state	<Continue>	Enter ↵	Click Continue
Revert to previous state	<Cancel>	Esc	Click Cancel

Also see [2.7: Mouse, keyboard, barcode scanner](#) and [3.3: Barcode data input](#)

FastFacts 3.2: Input device operation

3.3 Barcode data input

In **Speechmap**, **Viewport** and **Insertion gain** tests, audiometric data in barcode form on a Verifit or RM500SL printout may be entered by scanning the appropriate barcode. In Speechmap, threshold, UCL, RECD, transducer, age, and ABR nHL to eHL conversion factors are encoded. In Insertion gain, threshold and transducer type are encoded. The type of data and the ear (left, right or left/right) is shown below the barcode. Only data for the displayed ear are imported (data for both ears in dual view). Barcodes may be scanned in any order and it does not matter if the barcode is “upside down”.

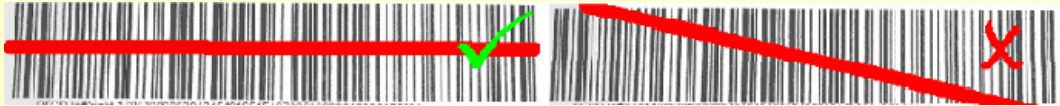

- 1) Select Speechmap, Viewport or Insertion gain from the Tests menu.
- 2) Hold the printout so that the barcode to be scanned is flat. With the scanner 6” - 8” from the barcode, press the trigger on the scanner and center the illuminated red line along the length of the barcode.
- 3) When the scan is successful, the scanner will “beep”, the red line will extinguish and a Barcode Entry poster will appear on the screen. A green checkmark on the poster shows which data have been accepted. A message will advise if the barcode does not contain data for the screen you are viewing.
- 4) When all desired data have been accepted, select [Done] to apply the data.


See [6.3: Barcodes, tabular data, headers and comments on printouts](#). Barcodes on some Grason-Stadler printouts may also be scanned into the Verifit.

Barcode Entry		
Ear	<input type="button" value="Left"/>	<input type="button" value="Right"/>
Age	<input type="button" value="Adult"/>	
Transducer	<input type="button" value="Headphone"/>	
Threshold	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
UCL	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
RECD	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input type="button" value="Done"/> <input type="radio"/>		
<input type="button" value="Cancel"/> <input type="checkbox"/>		

FastFacts 3.3: Barcode data input


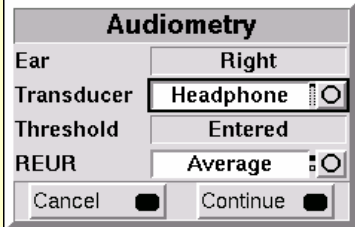

3.4 Keypad keys


Keypad Button	Function
<Help>	Summon context-sensitive Help Screen.
<Setup>	Summon Setup Menu containing a list of Verifit features that can be modified. Any modifications made will be maintained after power off.
<Tests>	Summon Test Selection Menu containing a list of the available test procedures.
<Session>	Summon Session Data Menu to erase or export data
<Left/Right>	Used in the Single View Mode to alternate between ears or A/B Data. Used in the Dual View Mode to navigate between ears or test panels. Used in Help to switch between Help index and Help page.
<Print>	Print on internal or external printer or print to file.
↔ Arrows	Move across screen columns.
↕ Arrows	Move within a screen column.
Round Key (<PICK> key)	Select a highlighted item or operate a screen button.
<Continue>	Proceed from current state.
<Cancel>	Revert to previous state.

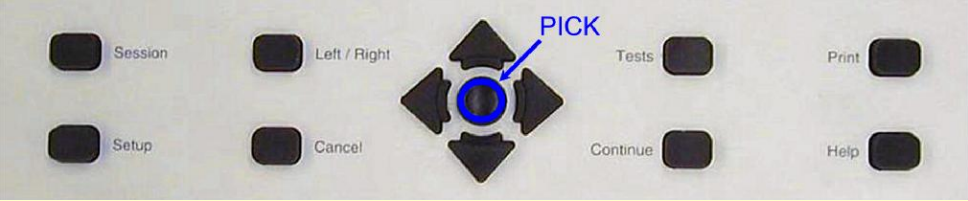

FastFacts 3.4: Keypad keys


3.5 Menus, lists and buttons

Menu	Setup Box	List Box
		


Screen Buttons are identified by a circle within a raised block  and are located at the right hand end of a line of screen text. A screen button highlighted using the keypad or a QWERTY keyboard is surrounded by a heavy line. A screen button highlighted by the mouse pointer changes color from light to dark gray. A highlighted screen button is operated by pressing the <PICK> (round) key on the keypad, the Enter key on a keyboard or by clicking the left mouse button. This process also selects the highlighted item in a list box.

There are three different types of screen buttons:


When selected, a **Menu Button** displays a menu, a setup box or a test screen.

 **Insertion Gain**

When selected, a **List Button** displays a drop-down list box.

 **BTE** The current list item is shown in the window.

When selected, a **Toggle Button** alternates between states.


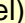
 **Single view** The current state is shown in the window.

FastFacts 3.5: Menus, lists and buttons

3.6 Screen messages and Help

The **Title bar** (top line of the display screen) informs you of the selected Test (e.g., Multicurve) and the current Presentation (Single view or Dual view).

The **Message bar** (bottom line of the display screen) suggests the next step in a test or informs you of the state of the instrument or conditions that could affect your data. For example, it will inform you if the microphones need to be calibrated.

Context-sensitive help is available by pressing <Help>. The  Arrows or a mouse may be used to scroll through the Help index (left panel). The Left/Right key or a mouse may be used to switch between the Help index and the Help page (right panel). The  Arrows or a mouse may be used to scroll through long help pages.

FastFacts 3.6: Screen messages and Help

3.7 CD-ROM removal and replacement

To remove the CD-ROM from the Verifit CD drive:

- 1) Press <Setup> then highlight and <PICK> [CD-ROM].
- 2) Highlight and <PICK> [Open CD drive].
- 3) Open the CD drive by pressing the OPEN button on the door of the CD drive.
- 4) Remove the CD-ROM by carefully snapping it off the spindle.
- 5) When replacing the CD-ROM, carefully snap it, label up, onto the spindle.
- 6) Gently push the CD drive door inward until it snaps into place.
- 7) Press <Continue> and follow any on-screen instructions.
- 8) Press <Continue> to exit Setup.

FastFacts 3.7: CD-ROM removal and replacement

3.8 Software updating

Software updates are provided on a CD-ROM which also contains a new User's Guide. You may want to copy the User's Guide to your PC before installing the CD-ROM into the Verifit. See [1.3: Electronic user's guide](#) for instructions on accessing the User's Guide.

- 2) Following the instructions in [3.7: CD-ROM removal and replacement](#) replace the old software CD-ROM with the new one.
- 3) Follow any on-screen instructions to complete the update

FastFacts 3.8: Software updating

4 General Setup

4.1 Section overview

This section covers the following topics:

- [4.2: Date and time setup](#)
- [4.3: Display settings](#)
- [4.4: Saving test setup](#)

FastFacts 4.1: General setup

4.2 Date and time setup

To set the date and time that appears on printouts and calibration screens:

- 1) Press <Setup>, then highlight and <PICK> [Date & Time] on the Setup poster.
- 2) To change the date, highlight and <PICK> the month window to display a drop-down list of months.
- 3) Highlight and <PICK> the desired month on the list.
- 4) Repeat the previous step to change the date, year and time in the appropriate windows.
- 5) When the new date and time information has been set, press <Continue> to exit.

FastFacts 4.2: Date and time setup

4.3 Display settings

This feature sets the screen saver timing and the background color used on the graphic test displays. The screen saver blanks the screen after 60, 90 or 120 minutes of inactivity to prolong the life of the display. The settings apply to both the internal display and to any connected external monitor.

To change display settings:

- 1) Press <Setup>, then highlight and <PICK> [Display].
- 2) To change the screen saver timing, highlight and <PICK> the current setting
- 3) Highlight and <PICK> the desired screen saver timing from the drop-down list.
- 4) To change the background color of the graphic display on the various test screens, highlight and <PICK> [Color] to toggle between black and white.
- 5) Press <Continue> to exit setup.

FastFacts 4.3: Display settings

4.4 Saving test setup

This feature lets you retain the last-used stimulus type and level for most tests when the power is turned off. It also lets you retain the last-used target method and transducer type in Viewport, Speechmap and Insertion Gain tests.

To enable/disable this feature:

- 1) Press <Setup>, then highlight and <PICK> [Save option]
- 2) Highlight and <PICK> [Save test setup] to toggle between Yes and No.

FastFacts 4.4: Saving test setup

5 Networking

5.1 Section overview

The Verifit may be connected to a Windows network by connecting to the rear connector panel using a Cat 5, RJ-45 cable. This allows screen images to be printed on a network printer or on a shared printer on a networked PC. Images and underlying data may also be saved to a shared folder on a networked PC. See [6: Printing and Storing Results](#) for details. The Verifit may also be connected to a single computer using a network hub. Note that sharing data with NOAH is done over the serial RS-232 port, not the network connector.

This section covers the following topics:

[5.2: Networking requirements](#)

[5.3: Networking setup](#)


[5.4: Single computer connection](#)

FastFacts 5.1: Networking

5.2 Networking requirements

The Verifit may be connected to a Windows network by connecting to the rear connector panel using a Cat 5, RJ-45 cable. For automatic address assignment, the network must have a Dynamic Host Configuration Protocol (DHCP) server.

For manual addressing, the remote computer must be running Windows 95 or higher, File and Printer Sharing for Microsoft Networks must be enabled and the TCP/IP Connection must be configured so that the IP address of the remote computer is on the same subnet as the Verifit. These settings should be obtained from your network administrator.

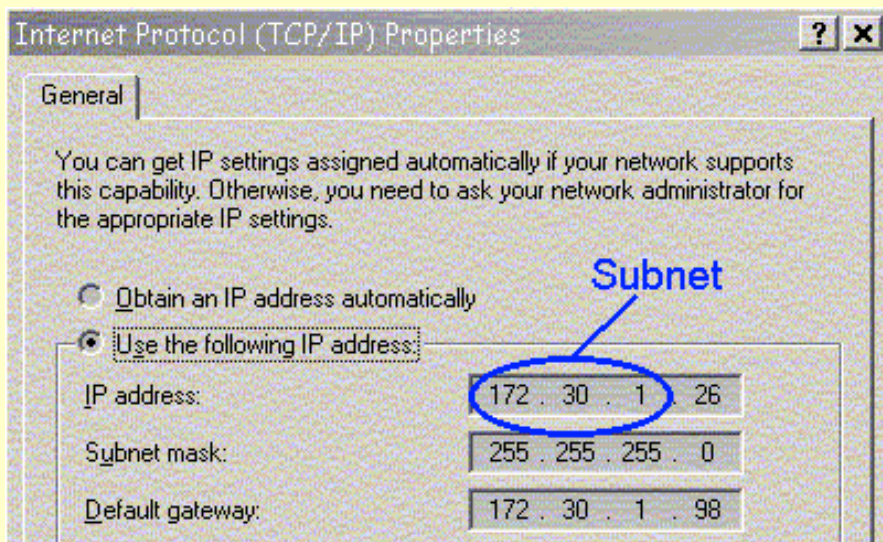


FastFacts 5.2: Networking requirements

5.3 Networking setup

Check [5.2: Networking requirements](#) and connect the Verifit to your network.

- 1) Press <Setup> then highlight and <PICK> [Network]
- 2) In the Network poster, highlight and <PICK> [Enable networking].
- 3) If your network supports automatic address assignment (DHCP), highlight and <PICK> [Obtain automatically]. Highlight and <PICK> [Test connection].
- 4) For manual configuration, highlight and <PICK> [Obtain manually]. Then highlight each of the 4 boxes in each row and highlight and <PICK> the numbers, provided by your network administrator, from the drop-down lists.
- 5) Highlight and <PICK> [Test connection]. The Verifit will prompt you for an IP address of a computer or router on your network. In a single PC-to-Verifit network, this is the IP address of the PC you are connecting to.

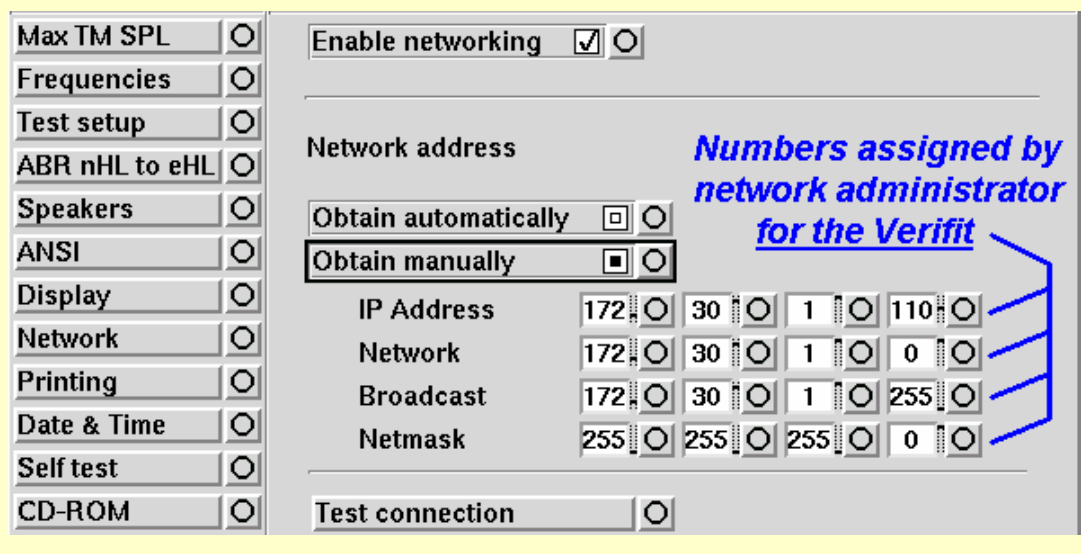

FastFacts 5.3: Networking setup

5.4 Single computer connection

The computer must be running Windows 95 or higher, with a functioning network card connected to one port on a Ethernet **network hub or switch**. Connect a Cat 5, RJ-45 cable from a hub port to the Verifit connector. **To set up the PC:**

- 1) Right click Network Neighborhood (or My Network Places) & select Properties.
- 2) Right click Local Area Network Connection & select Properties.
- 3) If Internet protocol TCP/IP is not installed, install it from your Windows disk.
- 4) Highlight Internet Protocol (TCP/IP) & select the Properties button.
- 5) Check "Use the following IP address" and enter the following in the IP address box: 172. 30. 1. 1. Enter 255. 255. 255. 0 in the Subnet mask box.

On the Verifit, see [5.3 Networking setup](#) and manually configure the following:



FastFacts 5.4: Single computer connection

6 Printing and Storing Results

6.1 Section overview

Pressing <Print> will cause the current screen image to be **printed** on the internal printer, on an external local printer, on a network printer, on a Windows-shared printer on a networked PC, or **saved** to file on a USB memory stick, or a Windows-shared file on a networked PC. Date, time, barcoded audiometric data and fixed or variable headers and comments (via a connected keyboard), may be added to printouts (but not when saving to a file). Tabular data may be appended to printouts of graphic data. Session data may also be saved as an xml file to a USB memory stick or in a Windows-shared folder on a networked PC for further analysis or to be restored to the Verifit for future use.

NOTE: If a header, comment, or client ID field, or file name, identifies the printed or saved data as belonging to an individual, it becomes Individually Identifiable Health Information and must be protected under the HIPAA Security Rule. See Appendix 1: Manufacturer Disclosure Statement for Medical Device Security.

The following topics are covered in this section:

- 6.2: Internal printer paper loading
- 6.3: Barcodes, tabular data, headers and comments on printouts
- 6.4: Local printer setup
- 6.5: Printing to a USB memory stick
- 6.6: Network printer setup
- 6.7: Printing to a network file
- 6.8: Storing and restoring data - setup
- 6.9: Storing and restoring data
- 6.10: Data storage using NOAH
- 6.11: NOAH 3 module use

FastFacts 6.1: Printing and storing results

6.2 Internal printer paper loading

On-screen instructions on paper replacement are provided if you try to print to the internal printer when it is out of paper or when the paper lever has been raised.

To load a new roll of paper:

- 1 Lift the printer head using the small black lever to the left of the printer.
- 2 Press the <Print> key.
- 3 Remove the empty spool from the paper tray and any remaining paper from the printer.
- 4 Hold a new roll in front of the paper tray with paper feeding from the top of roll.
- 5 Insert the paper's leading edge into the lower slot at rear of tray.
- 6 Feed the paper by highlighting the on-screen 'Paper Feed' button and pressing <PICK>.
- 7 Advance the paper until it comes out of the upper slot.
- 8 Place the roll in the paper tray and advance another foot of paper.
- 9 Lower the printer head using the small black lever. Press <Continue> to print or <Cancel> to cancel the print job.



If nothing shows on the paper when you press <Print> and the paper is coming out –check that the glossy side is up as it comes out of the printer.

Unused thermal paper may darken if stored for prolonged periods at elevated temperatures and lose contrast if subjected to light. Keep long-term supplies in a cool dry place out of direct light.

FastFacts 6.2: Internal printer paper loading

6.3 Barcodes, tabular data, headers and comments on printouts


The date and a header, such as a facility name, may be added to printouts. Barcodes representing hearing threshold, UCL and RECD, where applicable, may be added to printouts of Viewport, Speechmap, Insertion gain and RECD tests. Tabular data, where applicable, may be appended to printouts. Lines for handwritten notes or typed comments may also be added. **If a header or comment links the data to an individual, the printout becomes Individually Identifiable Health Information which must be protected under the HIPAA Security Rule.**

- 1) Connect a computer keyboard to the rear connector panel.
- 2) Press <Setup>, then highlight and <PICK> [Printing].
- 3) Highlight [Page setup] and press <PICK> to display the page setup poster.
- 4) To print the **date and time** on each printout, highlight and <PICK> [Date].
- 5) To print **lines** for handwritten notes, highlight and <PICK> [Lines].
- 6) To add **barcodes** for audiometric data, highlight and <PICK> [Barcode].
- 7) To add **tabular data**, highlight and <PICK> [Graph with table].
- 8) To print a **fixed header** on printouts, highlight and <PICK> [Header]. Connect a keyboard to the rear connector panel and enter the desired header (up to 35 characters). Press <PICK> (or Enter on the keyboard **number** pad) to complete the entry. To be prompted for a **new header** for each printout, highlight and <PICK> [Ask].
- 9) To print **fixed comments** following a printout, highlight and <PICK> [Description]. Connect a keyboard and enter comments. Press <PICK> (or Enter on the keyboard **number** pad) to complete the entry. To be prompted for **new comments** for each printout, highlight and <PICK> [Ask].
- 10) To be prompted for a **new file name** each time you print to a USB memory stick or a network file, highlight and <PICK> [Ask]. **Do not use space \$ \ / or tab in the file name.** USB file names are limited to 8 characters. See [6.5 Printing to a USB memory stick](#) or [6.7: Printing to a network file](#).
- 11) A print layout for the selected printer is displayed.
- 12) Press <Continue> to exit page setup and <Continue> to exit Setup.

FastFacts 6.3: Barcodes, tabular data, headers and comments on printouts

6.4 Local printer setup

- 1) Connect an external printer to the parallel or USB port on the rear connector panel. **Only PCL 3, PCL 5, PostScript compatible, HP Deskjet D4160 and 460 printers are supported.**
- 2) Press <Setup>, then highlight and <PICK> [Printing].
- 3) Highlight & <PICK> [Parallel] or [USB].
- 4) Highlight & <PICK> [Printer details] (automatic if printer type is set to None).
- 5) Highlight & <PICK> [Printer type].
- 6) Highlight & <PICK> your printer type. Select Color(1) for a printer with a single ink cartridge; color(2) for a printer with separate color and black ink cartridges; Other HP for a list of supported specific printer models (**only** on serial #1866 and higher). Check [Black only] for black printouts on a color printer. Press <Continue>.
- 7) If Other HP is selected, select a Printer model and press <Continue>.
- 8) Press <Continue> to exit Setup.

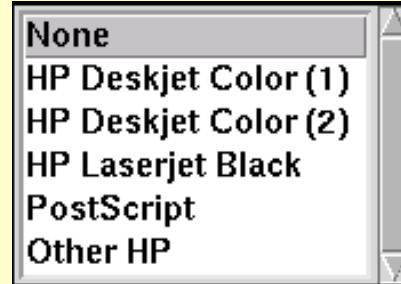

FastFacts 6.4: Local printer setup

6.5 Printing to a USB memory stick

- 1) Plug a USB memory stick into one of the USB ports on the rear connector panel. The Verifit may be either on or off.
- 2) Press <Setup>, then highlight and <PICK> [Printing].
- 3) Highlight and <PICK> [Print to USB file].
- 4) Highlight and <PICK> [Printer details] to change the file type [Bitmap (bmp)] or [Internet GIF (gif)] and the captured area [Full screen], [Main window] (full screen without the message line or [Active window] (foreground image only).
- 5) To save a screen image to a file, press <Print>. **Do not unplug the USB stick until the “Accessing USB stick” message disappears.**
- 6) To input a file name from a connected keyboard each time you print, highlight and <PICK> [Page setup], then highlight & <PICK> [Ask]. File names are limited to 8 characters plus .bmp or .gif. Do not use space \$ \ / or tab in the file name. **This may create electronic Protected Health Information (ePHI).** See [6.1: Section overview and Appendix 1 of the User’s Guide.](#)
- 7) To save all test data as an xml file, first press <Setup>, then highlight and <PICK> [Session]. Highlight and <PICK> [Enable file based data exchange] then highlight and <PICK> [USB drive] and <Continue>.
- 8) Press <Session>, then <PICK> the [Erase all data] button to show a list of options. Highlight and <PICK> [Store session to file] and press <Continue>.

A style sheet (audioscan.xsl) which allows the stored xml file to be viewed with a web browser or Excel (right click and select Open with Excel) is automatically saved with the data file. Select this style sheet when asked.

FastFacts 6.5: Printing to a USB memory stick

6.6 Network printer setup

To print to a network printer or to a Windows-shared printer on the network, you must first enable networking. See section [5: Networking](#). To print to a printer on a Windows PC connected to a network, you must also share the printer on the PC by selecting Start, Settings and Printers. Then right click on the printer you wish to share and select Sharing. **Only PCL 3, PCL 5, PostScript compatible, HP Deskjet D4160 and 460 printers are supported.** In Windows2003 Server you must add permission to allow Anonymous Logon to print to the shared printer.

- 1) Press <Setup>, then highlight and <PICK> [Printing].
- 2) Highlight & <PICK> [Other network printer] or [Windows shared printer].
- 3) Highlight & <PICK> [Printer details] (automatic if printer type is set to None).
- 4) Highlight & <PICK> [Printer type], then select from Deskjet Color(1) (single ink cartridge), Deskjet Color(2) (both color and black ink cartridges), Laserjet Black, PostScript or Other HP (a list of supported specific printer models). Check [Black only] for black printouts on a color printer.
- 5) Connect a computer keyboard to the rear connector panel and fill in the boxes with the requested information. Highlight and <PICK> each box to open it for data entry and again to close it (or use the Enter key on the keyboard's numeric keypad). Obtain needed information from your network administrator.

Windows network printer		Other network printer	
Printer type	HP Deskjet Color (2) <input type="radio"/> Black only <input type="checkbox"/>	Printer type	HP Laserjet Black <input type="radio"/>
	Hewlett-Packard Deskjet (tm) color printer with separate black and color cartridges		Hewlett-Packard Laserjet (tm) printer
Computer	MAD1 <input type="radio"/>	IP Address	172.30.1.99 <input type="radio"/>
Share name	ColorJet55 <input type="radio"/> Case Sensitive	Port	9100 <input type="radio"/>
Username	Dave <input type="radio"/>		
Password	***** <input type="radio"/>		
	Test connection <input type="radio"/>		Cancel <input type="checkbox"/> Continue <input type="checkbox"/>
	Cancel <input type="checkbox"/> Continue <input type="checkbox"/>		

FastFacts 6.6: Network printer setup

6.7 Printing to a network file

To save the screen image as a file on a network PC, you must first enable networking. See [5: Networking](#). You must also "share" a folder in the PC root directory (e.g. c:\MyData) and ensure that Change permissions are enabled.

- 1) Press <Setup>, then highlight and <PICK> [Printing].
- 2) Highlight and <PICK> [Print to file].
- 3) Highlight and <PICK> the file type [Bitmap (bmp)] or [Internet GIF (gif)].
- 4) Highlight and <PICK> [Full screen], [Main window] (full screen without the message line or [Active window] (foreground image only).
- 5) Connect a computer keyboard to the rear connector panel and fill in [Computer], [Share name], [Username] and [Password] (not needed for Windows 95/98). Highlight and <PICK> each box to open it for data entry and again to close it (or use the Enter key on the keyboard's numeric keypad).
- 6) Highlight and <PICK> [Test connection] and wait for a successful report.
- 7) To input a file name each time you <Print>, highlight and <PICK> [Page setup], then highlight & <PICK> [Ask]. **Do not use space \$ \ / or tab in the file name. This may create electronic Protected Health Information (ePHI).**

The screenshot shows a dialog box titled "Network file" with the following fields and controls:

File type	Bitmap (bmp)	<input type="radio"/>
Print frame	Main window	<input type="radio"/>
Computer	Alice1	<input type="radio"/>
Share name	MyData	<input type="radio"/>
Username	Alice	<input type="radio"/>
Password	*****	<input type="radio"/>
Test connection		<input type="radio"/>
Cancel		<input type="checkbox"/>
Continue		<input type="checkbox"/>

Annotations with blue arrows point to the right side of the dialog box:

- "Network name of your PC" points to the "Computer" field.
- "Shared folder on your PC (Case sensitive. May not contain space \$ \ / or tab)" points to the "Share name" field.
- "Network user name and password (if required)" points to the "Username" and "Password" fields.

FastFacts 6.7: Printing to a network file

6.8 Storing and restoring data - setup

All measurement data (curves) displayed at the end of the test are stored until overwritten by repetition of the same test, invalidated by a related test (in such cases, the message bar will inform you of impending data loss), erased by selecting <Session> and one of the [Erase data] choices or the power is turned off. (Setup data will be retained even after the power is turned off.)

The Verifit database may be stored and restored via xml files on a USB drive or a networked PC share. If the data exchange location is a network share, you must first enable networking. See [5: Networking](#). You must also "share" a folder in the PC root directory (e.g. c:\MyData) and ensure that Change permissions are enabled.

To enable and configure file-based data exchange:

- 1) Press <Setup>, then highlight and <PICK> [Session].
- 2) Highlight and <PICK> [Enable file based data exchange].
- 3) Highlight and <PICK> the storage file location [USB drive or Network share].
- 4) For Network share connect a computer keyboard to the rear connector panel and fill in [Computer], [Share name], [Username] and [Password] (not needed for Windows 95/98). Highlight and <PICK> each box to open it for data entry and again to close it (or use the Enter key on the keyboard's numeric keypad). Highlight and <PICK> [Test connection] and wait for a successful report.

FastFacts 6.8: Storing and restoring data - setup

6.9 Storing and restoring data

To store session data to a file:

- 1) Press the <Session> key, then highlight and <PICK> the top list button.
- 2) Highlight and <PICK> [Store session to file] from the list.
- 3) Highlight and <PICK> the Client ID list button.

Fill in Client ID with a computer keyboard (see [2.7: Mouse, keyboard, barcode scanner](#)) or leave it blank and the file will be named session.#.xml. **The Client ID may not contain ! ~ ` @ # \$ % ^ & * + = \ / ? < , . >** **If the Client ID links the data to an individual, the printout becomes ePHI (electronic Protected Health Information) which must be protected under the HIPAA Security Rule.**

- 4) Press <Continue>

To restore a session from a file:

- 1) Press the <Session> key, then highlight and <PICK> the top list button.
- 2) Highlight and <PICK> [Restore session from file].
- 3) Highlight and <PICK> the Client ID list button to generate a list of previously entered client identification codes.
- 4) Highlight and <PICK> a client identification code from the list.
- 5) Highlight and <PICK> the Session list button to generate a list of previously stored sessions for the indicated Client ID.
- 6) Highlight and <PICK> a session and press <Continue>.

A style sheet (audioscan.xsl) which allows the stored xml file to be viewed with a web browser or Excel (right click and select Open with Excel) is automatically saved with the data file.

FastFacts 6.9: Storing and restoring data

6.10 Data storage using NOAH

Overview: The Verifit NOAH module transfers test box, audiometric and collected on-ear data between the Verifit and the NOAH 3 framework via a proprietary serial cable. Data are stored as an action in the NOAH 3 client session list. Stored data may not be viewed or printed on the PC; it must be sent back to the Verifit for viewing or printing. Selected public data (HL, UCL, RECD) may be exchanged with other NOAH modules through the NOAH framework. The module software and proprietary cable are packaged as an optional accessory kit (VA-200).

Requirements: The NOAH Module works on any PC running Windows 2000 or Windows XP Pro and the NOAH 3 framework, and having a 9-pin RS232 serial (COM) port. The Verifit must be running software 2.2 or higher.

Connecting to the PC: Connect the 9-pin RS232 serial cable to the back of the Verifit. Then connect the other end of the cable to a dedicated RS232 connector on the PC. Note the number of the COM port (i.e. COM1, 2, 3 or 4); you will be asked for it during installation of the module software. It can be changed later by selecting Setup from the Verifit NOAH module menu bar.

Installing module software: Place the software CD in the CD drive of the PC that will be running the NOAH module and launch NOAH 3.



From the Main menu select **Tools/Setup/Install Module**.


Select **Install New Module**. Navigate to the CD drive containing the Verifit CD and click on Install.ini and then [Open]. Follow the on-screen instructions.



For instructions on using the module, see [6.11: NOAH 3 module use](#).



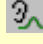

FastFacts 6.10: Data storage using NOAH

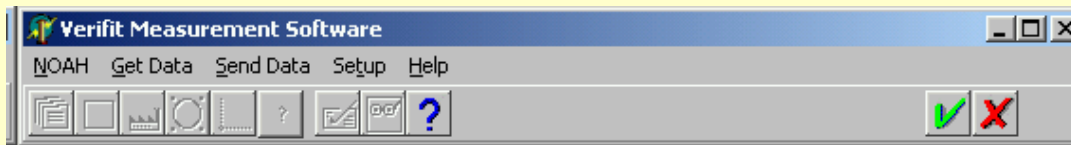6.11 NOAH 3 module use

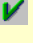To launch the module, if there is no Verifit action in the session list, click on  (if it has been set up) or click  and select VERIFIT under the tab for Measurement modules. If there is a previous Verifit action in the session folder, you may also launch the module by double-clicking the action. The menu bar will appear:



To get data from the Verifit, click **Get Data** on the menu bar. Select Hearing Instrument Test (HIT) data or Real Ear Measurement (REM) data and click OK. The transfer takes a few seconds during which time the hour glass cursor is displayed. Click  or . If data has been received, you will be prompted to **Add** which adds a new session or **Cancel** which exits without saving.

To send data (either Verifit or public data from other modules) to the Verifit, click **Send Data** on the menu bar. **The data sent, depends on how you launch the module.** If you launch the module by clicking a toolbar button , , , , NOAH sends the **most recent data** of each type (Audiometric, HIT or REM) in the session list to the Verifit. This may include data from a previous session and may not include data from the highlighted action. If you launch the module by **double-clicking** a previously-saved action, NOAH sends the **data in that action** to the Verifit. This is preferred since no unexpected data will be sent to the Verifit. The transfer may take a few seconds, during which time the hourglass cursor is shown. The Verifit displays a notice after it has received the data; press <Continue> to proceed.

To close the module, click  or  and select Cancel.

FastFacts 6.11: NOAH 3 module use

7 Test Box Measures - Setup

7.1 Section overview

The appearance of test box screens may be changed in several ways. Most screens may be viewed as a single, full-screen or dual, side by side panels. Most screens may be formatted to show results in graphical or tabular format. The Multicurve screen may be scaled in dB SPL or dB Gain. Individual curves may be hidden (but not erased).

This section covers the following topics:

- 7.2: Test box single or dual view
- 7.3: Graph or table format
- 7.4: Output or gain scale
- 7.5: Hide or Show test box curves
- 7.6: 1996 or 2003 ANSI standard
- 7.7: ANSI test frequencies
- 7.8: Test box calibration facts
- 7.9: Calibrating test box reference microphone
- 7.10: Calibration check for coupler microphone
- 7.11: Coupling the hearing instrument
- 7.12: Positioning the hearing instrument

FastFacts 7.1: Test box measures - setup


7.2 Test box single or dual view

Except for Viewport, Input/Output, Battery drain and Manual control screens, you have the option of selecting either a single, full-screen view or a dual, side by side view. **Some data may be dropped from the dual view.**

To change the presentation, highlight, then <PICK> [Single view] or [Dual view].

To switch between A and B test panel in single or dual view, press <Left/Right>.

In dual view, you may switch panels by left-clicking the mouse on a screen button in the inactive test panel. However, this also operates the button.

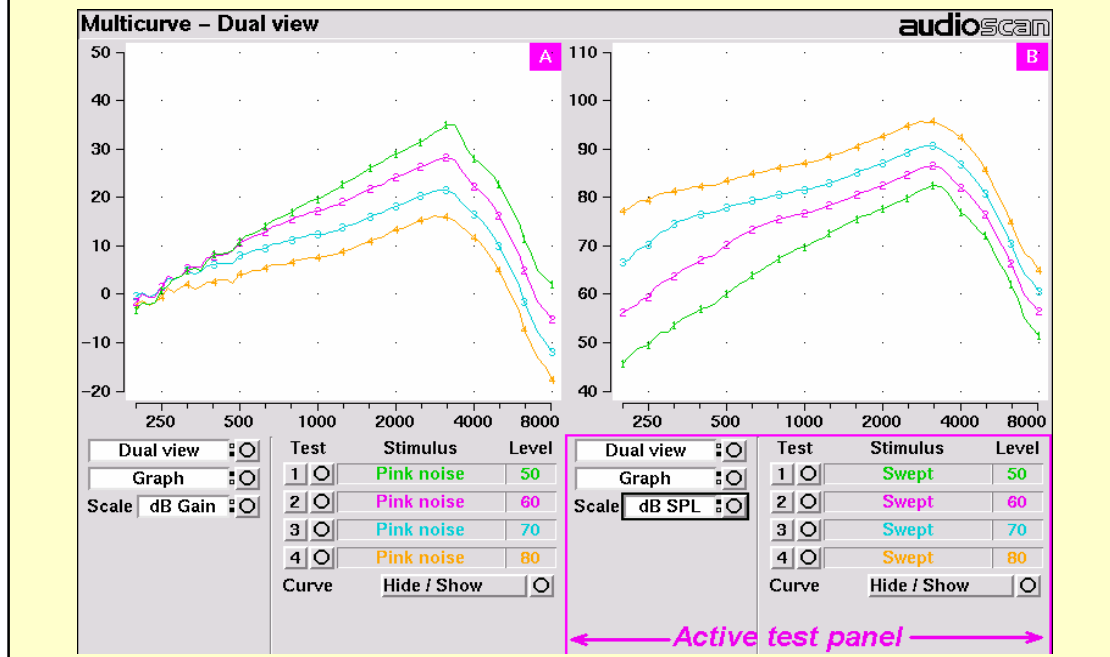

FastFacts 7.2: Test box single or dual view

7.3 Graph or table format

Except for Viewport, Distortion, Input/Output, Battery drain and Manual control screens, you have the option of selecting either a graphic or tabular format.

To change the **data format**, highlight, then <PICK> [Graph] or [Table].

FastFacts 7.3: Graph or table format

7.4 Output or gain scale

On the Multicurve test screen, you have the option of displaying test results in SPL (dB) or gain (dB).

To change the **scale**, highlight, then <PICK> [dB SPL] or [dB Gain]


FastFacts 7.4: Output or gain scale

7.5 Hide or Show test box curves

On screens capable of showing a speech envelope, you may show or hide the speech envelope for clarity of display. You may choose to show the envelope with the LTASS or only the LTASS. Hiding a curve does not erase it. To erase curves use the <Session> key.

To hide or show test curve(s):

- 1) Highlight, then <PICK> Curve [Hide/Show].
- 2) Highlight, then <PICK> the desired test to toggle between the Hide or Show options. The curve will toggle on or off when selected.
- 3) Where available, you may also highlight and <PICK> the envelope to toggle it on or off.
- 4) Press <Continue> to return to the test screen.

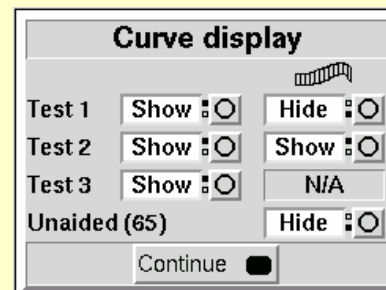

FastFacts 7.5: Hide or Show test box curves

7.6 1996 or 2003 ANSI standard

Automated tests of linear and AGC hearing aids may be conducted according to either the ANSI S-3.22-1996 or ANSI S3.22-2003 test standards. For differences, see [8.2: ANSI S3.22-1996 facts](#) and [8.3: ANSI S3.22-2003 facts](#)

To select the standard to be used:

- 1) Press <Setup, then highlight and <PICK> [ANSI]
- 2) Highlight and <PICK> the version of the ANSI standard you wish to use.
- 3) Press <Continue> to exit

FastFacts 7.6: 1996 or 2003 ANSI standard

7.7 ANSI test frequencies

The Frequencies feature allows hearing instruments to be evaluated at the Special Purpose Average (SPA) frequencies defined by ANSI S3.22, rather than the default High Frequency Average (HFA) frequencies. The choice of SPA (or HFA) frequencies influences the average OSPL90, average full-on gain, reference test gain, bandwidth and equivalent input noise. The frequencies at which distortion is measured are one-half the selected SPA frequencies. You can also select the frequencies at which attack and release times are measured.

To change the averaging frequencies:

- 1) Press <Setup>, then highlight and <PICK> [Frequencies].
- 2) Highlight and <PICK> the HFA/SPA frequencies box, to generate a drop-down list box of alternate test frequencies.
- 3) Highlight the desired HFA/SPA test frequencies and press <PICK> to select.

To change the frequencies for attack and release time measurements:

In the list of available frequencies, highlight and <PICK> the frequencies to be used to measure attack and release time.


Press <Continue> to exit.

FastFacts 7.7: ANSI test frequencies

7.8 Test box calibration facts

Test box microphone calibration establishes a correction curve for the uncalibrated test box reference microphone by comparison with the factory calibrated coupler microphone. It may be performed as often as desired but will be invalidated at 1 am the following day or at 1 am the following Monday, depending on the choice of Daily or Weekly calibration interval. After the calibration has expired, the message window will turn blue and you will be prompted to calibrate whenever you attempt a test that uses the test box microphones.

Note that there are two styles of test box reference microphones. The newer style is larger but responds beyond 16 kHz and has a lower noise floor. The two have different correction curves, as shown below, but will produce the same test results, once calibrated. Your calibration curve should be similar to one of these and should remain stable over time. Irregularities in the calibration curve may be the result of positioning errors. See [7.9: Calibrating test box reference microphone](#).

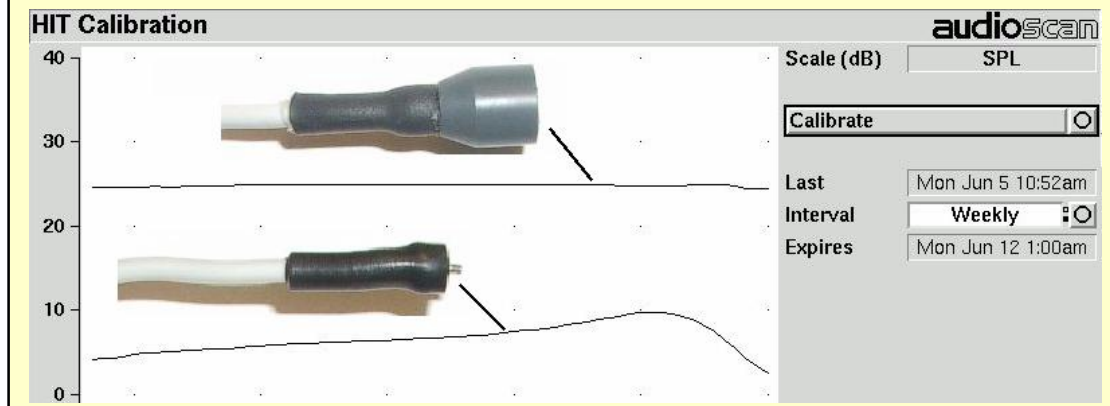

FastFacts 7.8: Test box calibration facts

7.9 Calibrating test box reference microphone

Remove any coupler from the coupler microphone before calibration.

- 1) Position the test box reference microphone opening concentric with the coupler microphone opening and about 1 – 2 mm from as shown below.
- 2) Close and seal the test chamber by tightening the latch at the front of the lid.
- 3) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Calibration].
- 4) Highlight, then <PICK> [Daily] or [Weekly] to select the calibration interval.
- 5) Highlight and <PICK> [Calibrate]. See [7.8: Test box calibration facts](#)

It is recommended that you print a copy of the initial calibration curve and compare it with your daily or weekly curve. Any change in this curve indicates a drift in either the coupler or the reference microphone which needs further investigation. See [7.10: Calibration check for coupler microphone](#) for an additional check.



FastFacts 7.9: Calibration of test box reference microphone

7.10 Calibration check for coupler microphone

Accuracy of test box measurements depends on the calibration of the coupler microphone. This should be checked regularly using a microphone calibrator with an adapter ring. The RE780 (1 1/8") and RE781 (1") rings are designed for Quest CA22 and QC-10/20 respectively. To verify the coupler microphone calibration:

- 1) Calibrate the test box reference microphone. See [7.9: Calibrating test box reference microphone](#)
- 2) Screw the appropriate calibrator ring onto the coupler microphone.
- 3) Insert the coupler microphone with the attached calibrator ring into the calibrator opening.
- 4) Turn the calibrator on and adjust it to present one of the available frequencies and levels (e.g., 94 dB SPL at 1000 Hz).
- 5) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Manual control].
- 6) Highlight and <PICK> the [Level] window, then highlight and <PICK> 0 from the drop-down list.
- 7) Highlight, then <PICK> the [Frequency] window, then highlight and <PICK> the frequency of the calibrator (e.g. 1000 Hz) from the drop-down list.
- 8) Highlight, then <PICK> the [Filter] window, then highlight and <PICK> 1/12th octave from the drop-down list.
- 9) Highlight, then <PICK> [Start test] (which changes to [Stop test]).
- 10) Check that the Coupler microphone level is within 1 dB of the calibrator level setting, +/- the calibrator accuracy. For a 94 dB calibrator with +/- 0.3 dB accuracy, the indicated coupler microphone level should be 94 +/- 1.3 dB.
- 11) To stop the measurement, highlight and <PICK> [Stop test].

FastFacts 7.10: Calibration check for coupler microphone

7.11 Coupling the hearing instrument

The Verifit is supplied with 2 metal couplers, a type HA-1 (ITE) and a type HA-2 (BTE). Note that an **ANSI standard coupler for CICs does not exist**. These must be tested in the HA-1 coupler.

When threading the couplers onto the coupler microphone, **it is very important that they be firmly tightened** to ensure a good acoustic seal. Avoid twisting the coupler microphone cable – turn the coupler not the microphone or unplug the microphone while tightening the coupler.

BTE instruments are coupled to the HA-2 coupler by means of a 10 mm (3/8th “) length of heavy wall #13 earmold tubing. The only purpose of this tubing is to seal the tip of the earhook to the coupler inlet. All of the tubing required by ANSI S3.22 is machined into the metal stem of the HA-2 coupler. The #13 tubing should be inspected regularly for cracks which will cause feedback.

Custom instruments are sealed to the HA-1 coupler with putty so that the end of the eartip is flush with the inside of the coupler opening. Putty should not extend into the coupler cavity or block the sound outlet of the instrument. **Vents should be sealed at the faceplate end**. It is very important that the instrument be well-sealed to the coupler.

Open fit instruments must use the coupler and coupling system specified by the manufacturer. This may involve a hook that replaces the open fit tubing or an adapter tube that may be sealed to the opening of the HA-1 coupler with putty.

RITE (Receiver in the ear) instruments are coupled to the HA-1 coupler using putty to seal the receiver module or soft tip to the coupler opening.


Excess low frequency output is usually due to open vents or a poor seal to the coupler.

For health reasons, hearing instruments should be clean before putty is applied and putty should be replaced frequently.

FastFacts 7.11: Coupling the hearing instrument

7.12 Positioning the hearing instrument

- 1) The hearing instrument should be coupled to the appropriate coupler and a battery or battery pill installed. See [7.11: Coupling the hearing instrument](#) and [2.6: Battery pill use and care](#).
- 2) Place the instrument and coupler in the test chamber so that the microphone opening(s) of the instrument are on the crossbar within the test zone.>>
- 3) For directional instruments, the direction of maximum sensitivity should be oriented toward the left speaker. See [9.6: Directional ITE positioning](#) and [9.7: Directional BTE positioning](#) for special positioning considerations for the directional test.
- 4) Position the test box reference microphone as close as possible to the (front-facing) microphone opening of the hearing instrument without touching it.
- 5) Close and seal the test chamber.

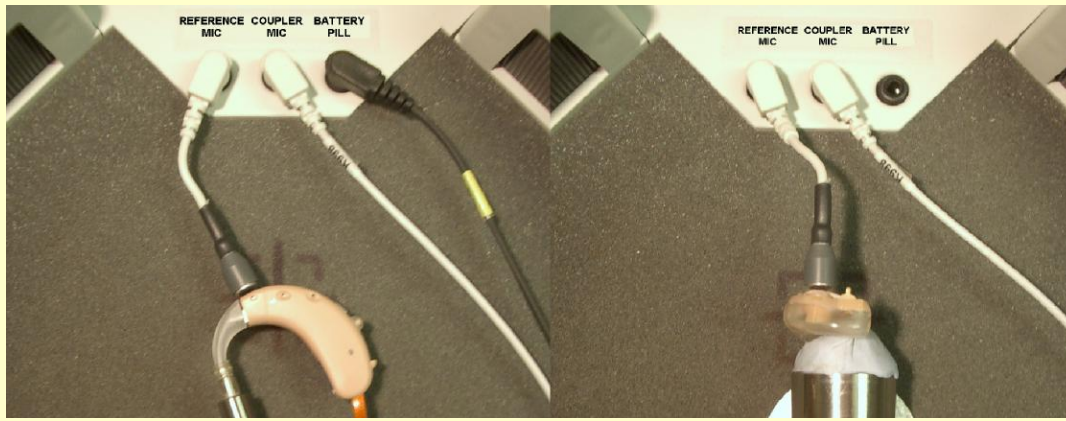

FastFacts 7.12: Positioning the hearing instrument

8 ANSI S3.22 Hearing Aid Tests

8.1 Section overview

These tests follow the procedures of ANSI S3.22-1996 or ANSI S3.22-2003 as selected in Setup. See [7.6: 1996 or 2003 ANSI standard](#) :

Linear: An automated test sequence for linear hearing aids per ANSI S3.22.

AGC: An automated test sequence for AGC hearing aids per ANSI S3.22.

Input/Output: 1 - 4 curves of coupler SPL vs input SPL at .25, .5, 1, 2 and 4 kHz.

Telecoil: Tests for telephone sensitivity and test loop sensitivity using a telephone magnetic field simulator (TMFS) and a large coil in the test chamber floor.

This section covers the following topics:

[8.2: ANSI S3.22-1996 facts](#)

[8.7: ANSI input-output curves](#)

[8.3: ANSI S3.22-2003 facts](#)

[8.8: ANSI telecoil terminology](#)

[8.4: ANSI 1996 Linear and AGC tests](#)

[8.9: ANSI telephone simulator \(TMFS\) test](#)

[8.5: ANSI 2003 Linear and AGC tests](#)

[8.10: ANSI test loop test](#)

[8.6: ANSI test results](#)

[8.11: Telecoil test results](#)

FastFacts 8.1: ANSI S3.22 hearing aid tests

8.2 ANSI S3.22-1996 facts

The Verifit performs selected hearing instrument tests in accordance with ANSI S3.22-1996. This standard expresses some outputs and gains as High Frequency Average (HFA) values. The HFA is the average of values at 1000, 1600 and 2500 Hz. If a hearing aid meets certain conditions, the manufacturer may choose other averaging frequencies called Special Purpose Average (SPA) frequencies. The current averaging frequencies are indicated explicitly on test results where space permits and also appear as tick marks on the frequency axis of graphs. To change the averaging frequencies, see [7.7: ANSI test frequencies](#).

This standard also requires some tests with the gain control full on and some with the gain control in the Reference Test Position (RTP). This requirement applies to both Linear and AGC aids that have a HFA (or SPA) full-on gain greater than the HFA (or SPA) OSPL90 minus 77 dB. Otherwise the RTP is full on. If required, the Verifit will pause in its testing and guide you in setting the gain control to the RTP.

All other controls on the hearing instrument should be set to provide the widest frequency response, the greatest gain and the highest maximum output. AGC controls and/or programmable parameters should be set as designated by the manufacturer.

At full-on gain the **[Linear]** and **[AGC]** test sequences provide a curve of OSPL90 (Output Sound Pressure Level for a 90 dB input SPL), the HFA (or SPA) and peak value of this curve, the frequency of the peak and the HFA (or SPA) gain for a 50 dB input SPL.

With the gain control at RTP, these sequences also provide a frequency response curve with an input SPL of 60 dB (Linear) or 50 dB (AGC), the HFA (or SPA) gain for a 60 dB input SPL, the frequency range, battery drain, equivalent input noise, harmonic distortion, attack and release time (AGC only).

FastFacts 8.2: ANSI S3.22-1996 facts

8.3 ANSI S3.22-2003 facts

ANSI S3.22-2003 has been designated a recognized standard by the FDA but is not yet mandatory. Manufacturers may use either version for reporting test data.

The most significant change from the 1996 version is the requirement for the hearing aid to be set in its most linear mode for the setting of the gain control to Reference Test Setting (changed from Reference Test Position) and for all tests except attack and release and input-output (I/O) curves. These two tests are to be conducted with the AGC function set for maximum effect. The AGC test sequence will pause to allow AGC to be set prior to measuring attack and release time.

Full-on gain is determined with 50 dB input SPL (60 dB was formerly an option) and frequency response curves are run at 60 dB SPL for Linear **and** AGC aids.

The OSPL90 curve, the HFA (or SPA) and peak value of this curve, the frequency of the peak, the HFA (or SPA) full-on gain are determined at full-on gain setting. The response curve, frequency range, Reference Test Gain, battery drain, equivalent input noise, distortion, attack and release times are determined at RTS.

These changes result in more consistent values for reference test gain, equivalent input noise and attack and release time. See [7.6: 1996 or 2003 ANSI standard](#), [8.2: ANSI S3.22-1996 facts](#) and [8.5: ANSI 2003 Linear and AGC tests](#).

FastFacts 8.3: ANSI S3.22-2003 facts

8.4 ANSI 1996 Linear and AGC tests

- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Linear] or [AGC].
- 2) Highlight and <PICK> [Start test]; test setup instructions will be displayed.
- 3) Refer to [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#) and [2.6: Battery pill use and care](#) to prepare the instrument for testing.
- 4) **Turn the instrument ON.**
- 5) Set the gain control (manual or programmable) to maximum with all other controls set to provide the widest frequency response, greatest gain and highest maximum output. AGC controls and/or programmable parameters should be set as designated by the manufacturer.
- 6) Press <Continue> to run the test or press <Cancel> to abort the test. Note that, if required, the Verifit will pause during the test to request that the volume control be adjusted to the Reference Test Position (RTP). Open the test box if necessary and, without moving the hearing instrument, adjust the user gain control until the large vertical arrow on the screen prompt disappears. Press <Continue> to resume the test.

If you are unsure which test sequence to run, try [AGC] first. If attack and release times are all near zero, the instrument is probably linear.

FastFacts 8.4: ANSI 1996 Linear and AGC tests

8.5 ANSI 2003 Linear and AGC tests


- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Linear] or [AGC].
- 2) Highlight and <PICK> [Start test]; test setup instructions will be displayed.
- 3) Refer to [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#) and [2.6: Battery pill use and care](#) to prepare the instrument for testing.
- 4) **Turn the instrument ON.**
- 5) Set the gain control (manual or programmable) to maximum with all other controls set to provide the widest frequency response, greatest gain and highest maximum output. AGC controls and/or programmable parameters should be set to **minimize** AGC action (AGC off or threshold high, compression ratio close to 1).
- 6) Press <Continue> to run the test or press <Cancel> to abort the test. Note that, if required, the Verifit will pause during the test to request that the gain control be adjusted to the Reference Test Setting (RTS). If necessary, open the HIT chamber and, without moving the hearing instrument, adjust the gain control until the large vertical arrow on the screen prompt disappears. Press <Continue> to resume the test.

During the AGC test, the Verifit will pause to request that AGC controls or programmable parameters be set for **maximum** AGC effect. If the manufacturer does not provide settings to produce maximum AGC effect, use settings that result in the highest gain for soft sounds and the lowest gain for loud sounds. This will typically result in the lowest compression threshold, a flat I/O curve and the longest attack/release times. These are not “typical” characteristics but indicate the adjustment range possible for these parameters.

If you are unsure which test sequence to run, try [AGC] first. If attack and release times are all near zero, the instrument is probably linear.

FastFacts 8.5: ANSI 2003 Linear and AGC tests

8.6 ANSI test results

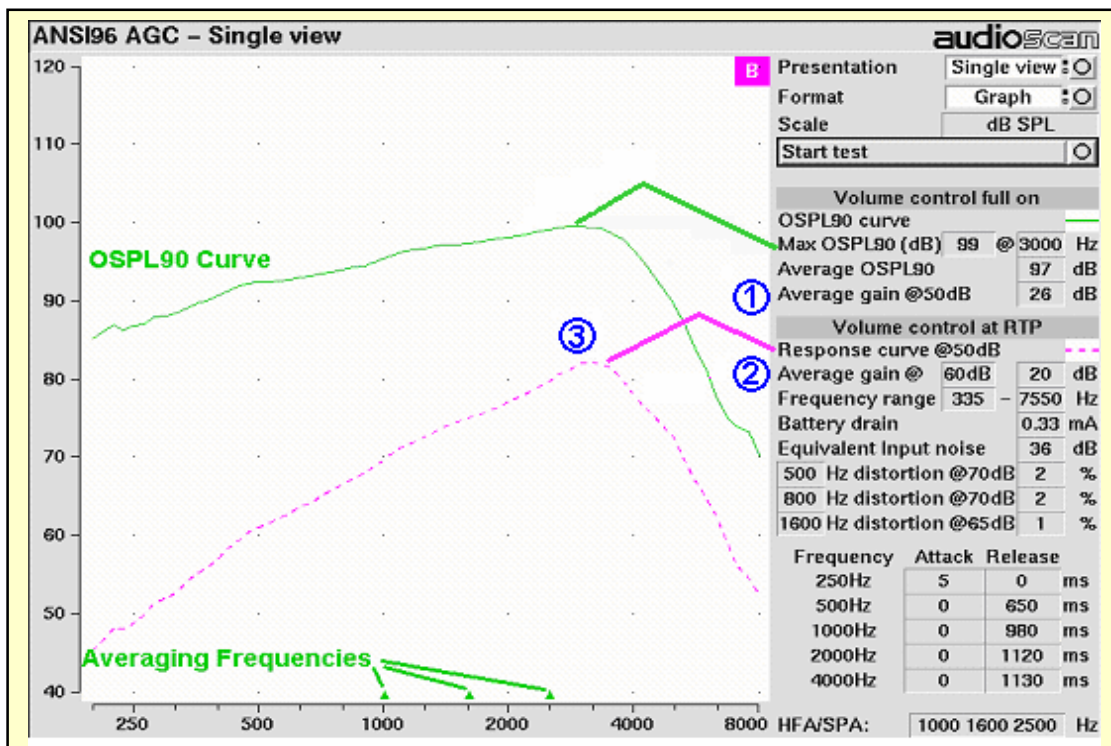

In 1996 tests: Full-on average gain **j** is measured at 50 dB SPL, average gain at RTP **k** is usually (but not always) measured at 60 dB SPL and the frequency response **l** is run at 60 dB SPL for linear aids or at 50 dB SPL for AGC aids.

In 2003 tests: Full-on average gain is measured with a 50 dB input SPL, average gain at RTS is always measured at 60 dB input SPL and the frequency response curve is always run at 60 dB input SPL.

FastFacts 8.6: ANSI test results

8.7 ANSI input-output curves

An input/output curve shows the output level of a hearing instrument as a function of input level and frequency. It is a steady-state pure-tone curve and does not reflect performance with complex signals such as speech.

You may run the tests at any setting of the instrument controls but ANSI S3.22 requires that certain settings be used. See [8.2: ANSI S3.22-1996 facts](#) and [8.3: ANSI S3.22-2003 facts](#). The [Input/Output] test allows you to generate up to 4 separate input/output curves at each of 5 individual frequencies (250, 500, 1000, 2000, 4000 Hz) as provided in ANSI S3.22.

- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Input/Output].
- 2) Refer to [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#) and [2.6: Battery pill use and care](#) to prepare the instrument for testing.
- 3) **Turn the instrument ON**; set AGC controls or programming parameters as in [8.4: ANSI 1996 Linear and AGC tests](#) or [8.5: ANSI 2003 Linear and AGC tests](#)
- 4) Highlight and <PICK> [Curve 1]. Test setup instructions will be displayed.
- 5) Press <Continue> to generate a set of 5 input/output curves.
- 6) To measure another set of 5 input/output curves, highlight and <PICK> another [Curve #].
- 7) To summon a cursor to determine curve values, position the mouse pointer on a graph area and click the left mouse button. Click again to turn it off.

NOTE: Noise in the test area may cause erroneous results at low input levels (particularly at 250 and 500 Hz). Ensure that the test box is well sealed and ambient noise is low.

FastFacts 8.7: ANSI input-output curves

8.8 ANSI telecoil terminology

The Verifit test chamber contains an integrated telephone magnetic field simulator (TMFS) and a telecoil test loop, meeting the requirements of ANSI S3.22, that enable the measurement of coupler SPL produced by a hearing instrument with a telephone pickup coil. ANSI uses the following terms in telecoil testing:

RTG (reference test gain) is the average gain measured with the gain control in the reference test position (1996) or reference test setting (2003). See [8.2: ANSI S3.22-1996 facts](#) and [8.3: ANSI S3.22-2003 facts](#).

TMFS (Telephone magnetic field simulator). A 38.4-mm diameter coil spaced 16.5 mm below a test surfaced driven by a specified test signal. Intended to simulate a telephone handset receiver.

SPLITS (Coupler SPL for an inductive telephone simulator). SPL developed in a 2cc coupler by a hearing instrument with the gain control in the RTP (1996) or RTS (2003) when the input is the magnetic field generated by a TMFS.

HFA (or SPA) –SPLITS. Average of the SPLITS at HFA or SPA frequencies.

STS (1996 - simulated telephone sensitivity) or **RSETS** (2003 - relative simulated equivalent telephone sensitivity) is calculated as HFA (or SPA) SPLITS minus (RTG + 60). A positive STS or RSETS indicates greater output when using the telephone than when using the microphone.

SPLIV (Coupler SPL in a vertical magnetic field). SPL developed in a 2cc coupler by a hearing instrument, positioned as worn, with the user gain control in the RTP (1996) or RTS (2003) when the input is a 31.6 mA/m vertical magnetic field.

HFA (or SPA) – SPLIV. Average of the SPLITS at HFA or SPA frequencies.

TLS (1996 - test loop sensitivity) or **RTLS** (2003 - relative test loop sensitivity) is calculated as HFA (or SPA) SPLIV minus (RTG + 60). A positive TLS indicates greater output when listening in a room loop than when using the microphone.

FastFacts 8.8: ANSI telecoil terminology

8.9 ANSI telephone simulator (TMFS) test

ANSI S3.22 requires that the gain control be set to RTP (1996) or RTS (2003) before running this test. The reference test gain (RTG) is required in order to calculate the STS (1996) or RSETS (2003). The easiest way to meet both requirements is to run a [Linear] or [AGC] test before running this test. See [8.4: ANSI 1996 Linear and AGC tests](#); [8.5: ANSI 2003 Linear and AGC tests](#)

- 1) First, complete an ANSI [Linear] or [AGC] test, then press <Tests>.
- 2) Highlight & <PICK> [Test box measures], then highlight and <PICK> [Telecoil].
- 3) Set the aid to 'T' (telecoil input).
- 4) Highlight and <PICK> [Start TMFS test]. Test setup instructions are displayed.
- 5) Position the aid on the 'T' inscribed on the test chamber floor. BTE aids should lie as flat as possible on the test chamber surface, while custom aids should be oriented with their faceplate parallel to the top surface (if possible) and touching it as shown.
- 6) Press <Continue> to start the test.
- 7) While maintaining the orientation of step 5, move the hearing instrument around on the test chamber surface to maximize the number shown in the 'Maximize Average-SPLITS Value' box.
- 8) Without moving the hearing instrument, press <Continue> to store the Average (HFA-or SPA) SPLITS value and run the SPLITS curve.
- 9) If an RTG has been previously obtained, the STS or RSETS will be shown.



FastFacts 8.9: ANSI telephone simulator (TMFS) test

8.10 ANSI test loop test


ANSI S3.22 requires that the gain control be set to RTP (1996) or RTS (2003) before running this test. The reference test gain (RTG) is required in order to calculate the STS (1996) or RTLS (2003). The easiest way to meet both requirements is to run a [Linear] or [AGC] test before running this test. See [8.4: ANSI 1996 Linear and AGC tests](#); [8.5: ANSI 2003 Linear and AGC tests](#)

- 1) First, complete an ANSI [Linear] or [AGC] test.
- 2) Press <Tests>, then highlight and <PICK> [Test box measures]
- 3) Highlight and <PICK> [Telecoil].
- 4) Set the aid to 'T' (telecoil input).
- 5) Highlight and <PICK> [Start test loop test]. Setup instructions will be displayed.
- 6) Position the hearing instrument on the 'T' inscription on the test chamber floor vertically oriented as worn by a seated individual as shown.
- 7) Without moving the hearing instrument, press <Continue> to run the SPL in a vertical magnetic field (SPLIV) curve or press <Cancel> to abort the test.
- 8) If an RTG has been previously obtained, the TLS or RTLS will be shown.



FastFacts 8.10: ANSI test loop test

8.11 Telecoil test results

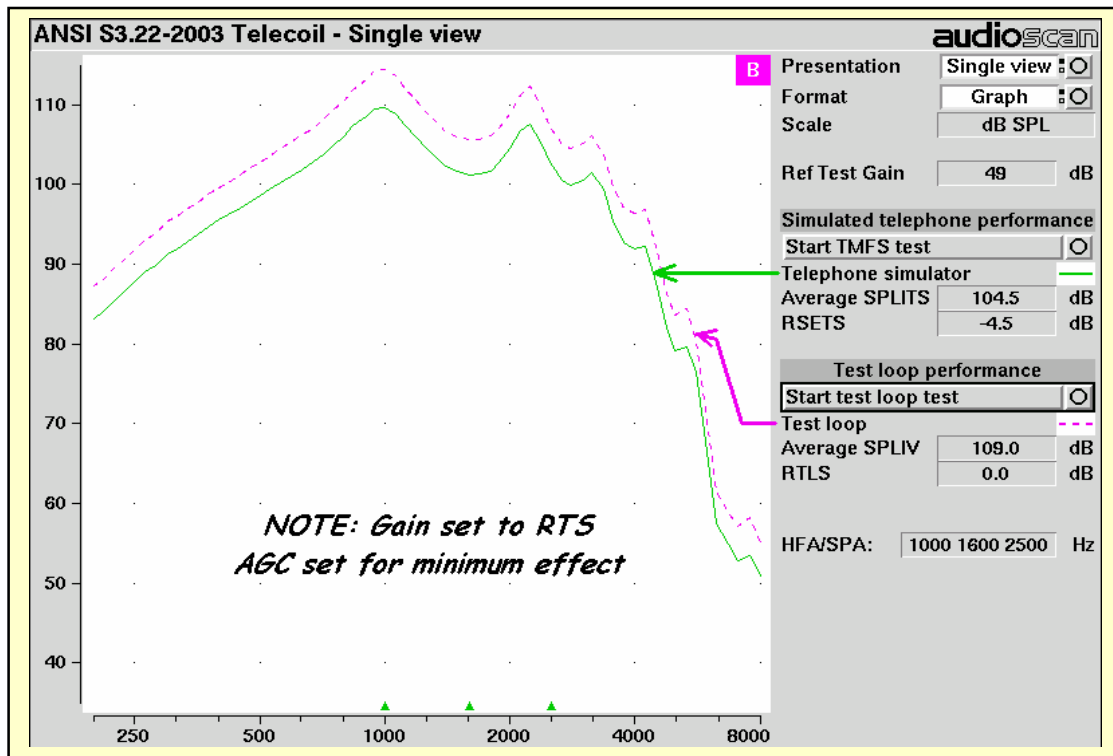

FastFacts 8.11: Telecoil test results

9 Other Test Box Measures

9.1 Section overview

In addition to the tests of ANSI S3.22, the Verifit provides the following:

Noise Reduction: This test shows the amount of noise reduction provided by the hearing instrument.

Feedback Reduction: This test is a before and after comparison of curves with the hearing instrument feedback reduction deactivated vs. activated.

Distortion: Harmonic distortion at 2nd or 3rd harmonics and 2nd plus 3rd with input levels varying from 60 – 90 dB SPL.

Directional: A unique real-time broad-band test signal presented simultaneously from left and right speakers and analysed into two separate response curves.

Multicurve: 1 – 4 curves of coupler SPL or coupler gain for swept tones or pink noise in 1/12th octaves at selected input SPL of 0 or 40 to 90 dB in 5 dB steps. Can be used as a 1/12th octave **spectrum analyzer** by setting stimulus to 0 dB.

Battery drain: A 10 second test of battery current drain with no acoustic input, a 1 kHz 65 dB input SPL, and an average for 90 dB input SPL at the HFA or SPA frequencies selected in Setup.

Manual control: Coupler and test box reference mic. SPL at selected 1/12th octave frequencies and stimulus SPL of 0 and 40 – 90 dB in 5 dB steps. Can be used as a dual channel **sound level meter** with 1/12th octave, A and C filters.

This section covers the following topics:

[9.2: Harmonic distortion](#)

[9.9: Multicurve procedure](#)

[9.3: Noise reduction](#)

[9.10: Multicurve results](#)

[9.4: Feedback](#)

[9.11: Spectral analysis in Multicurve](#)

[9.6: Directional ITE positioning](#)

[9.12: Battery drain](#)

[9.7: Directional BTE positioning](#)

[9.13: Manual control procedure](#)

[9.8: Test box directional procedure](#)

[9.14: Sound level meter using manual control](#)

FastFacts 9.1: Other hearing instrument tests

9.2 Harmonic distortion

This test measures harmonic distortion at 2nd or 3rd harmonics with input levels varying from 60 – 90 dB SPL. It also presents the sum of 2nd and 3rd harmonics as total harmonic distortion (THD). For hearing aids, this differs little from true THD.

- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and pick [Distortion].
- 2) Refer to [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#) and [2.6: Battery pill use and care](#) to prepare the instrument.
- 3) **Turn the hearing instrument ON.**
- 4) Highlight and <PICK> [Start test].
- 5) Highlight and <PICK> [Harmonics], then highlight and <PICK> THD, 2nd or 3rd from the drop-down list.
- 6) Highlight and <PICK> [Input], then highlight and <PICK> the desired input level (60 – 90 dB SPL) from the drop-down list.
- 7) Press <Continue> to run the test. Distortion will be displayed at 1/3 octaves in graphic and in tabular form.

FastFacts 9.2: Harmonic distortion

9.3 Noise reduction

This test captures a response curve before the hearing instrument reacts to the presence of noise; a comparison curve shows the hearing instrument reacting in real-time. The thick curve is the output before the noise reduction is activated and the thin line is the output as the noise reduction takes effect. The test also provides an overall rms noise reduction value in dB.


- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and pick [Noise reduction].
- 2) Highlight and <PICK> Test 1, 2 or 3.
- 3) Highlight and <PICK> a stimulus type from the list box.
- 4) Highlight and <PICK> a stimulus level from the list box.
- 5) Highlight and <PICK> [Start test]
- 6) Highlight and <PICK> [Continue] to capture the comparison curve once the noise reduction function has responded.

FastFacts 9.3: Noise reduction

9.4 Feedback suppression test

This test shows the SPL and the power concentration ratio (PCR) in each 1/3 octave band for a speech signal. Feedback results in a peak in the speech spectrum and in the PCR. Incipient feedback results in a peak in the PCR curve.

- 1) Press <Tests>, then highlight and <PICK> [Test box measures] and highlight and <PICK> [Feedback].
- 2) Highlight and <PICK> Test 1 and select a level of 50 or 60 dB.
- 3) With feedback suppression off, force the instrument into feedback. This may be accomplished by placing the monitoring headphones in the test box as shown. With the test box closed, adjust the monitor volume control until the feedback bar shows red during parts of the speech passage. Then press <Continue>.
- 4) When the test has finished, peaks should be apparent in one or both curves.
- 5) With feedback suppression enabled, highlight and <PICK> Test 2 and select the same level used for Test 1. Then press <Continue>.
- 6) The peaks in the curves of Test 1 should not appear in the curves of Test 2.

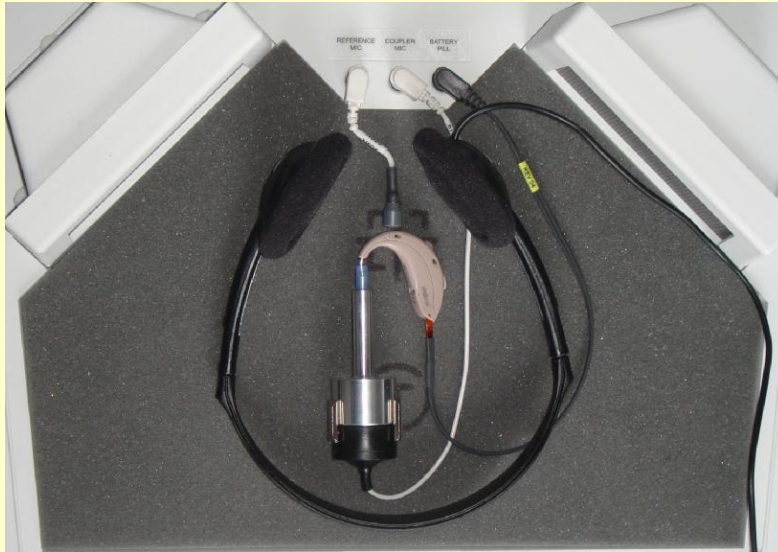

FastFacts 9.4: Feedback suppression test

9.5 Directional function test

Directional characteristics such as polar plots and Directivity Index (DI) are typically measured in a large anechoic chamber using sound from one direction and at one frequency at a time. This means they apply only for a hearing aid set for linear amplification. These are not conditions representative of real use.

The unique Verifit directional test presents over 1000 tones simultaneously at different frequencies from both test chamber speakers. The tones are individually controlled in real time to produce a precise spectrum from each speaker at the test box reference microphone. The coupler SPL is then analyzed into two real-time response curves, labeled L (left) and R (right) to indicate which speaker generated the curve. Since the curves are generated simultaneously, their difference is not influenced by electronic processing such as compression or noise reduction but reflects only the effect produced by the directional microphones in the instrument. The real-time nature of the test allows the operation of adaptive directional systems to be readily visualized.

For testing speech-adaptive instruments, the Verifit presents speech at selectable levels from the left-hand speaker and noise, at a selectable signal-to-noise ratio from the right-hand speaker. At the end of each presentation of the speech passage, the Verifit momentarily presents its unique 1000 tone directional test signal to capture the directional behavior. Once directional performance is observed, the test may be terminated and a final short measure of directional performance will be captured.

Because the Verifit test chamber is small and anechoic only at higher frequencies, results are not expected to agree with data taken in large anechoic chambers. It is a functional test only, providing assurance that the directional microphone array has been correctly assembled and programmed and has not deteriorated due to element drift or the clogging of ports. See also [13: On-Ear Instrument Measures](#)


1.1 Section overview

FastFacts 13.1: On-ear instrument measures

FastFacts 9.5: Directional function test

9.6 Directional ITE positioning


Attach the provided clip to the coupler for stability. Orient the instrument as shown so that the direction of maximum sensitivity is toward one of the speakers and a line joining the microphone ports is at 45° and horizontal. Locate the test box reference microphone at the front microphone opening on the faceplate and seal any vent at the faceplate. Note which speaker (L or R) is in “front” of the hearing instrument for reference. Other positions may be preferable for some instruments.

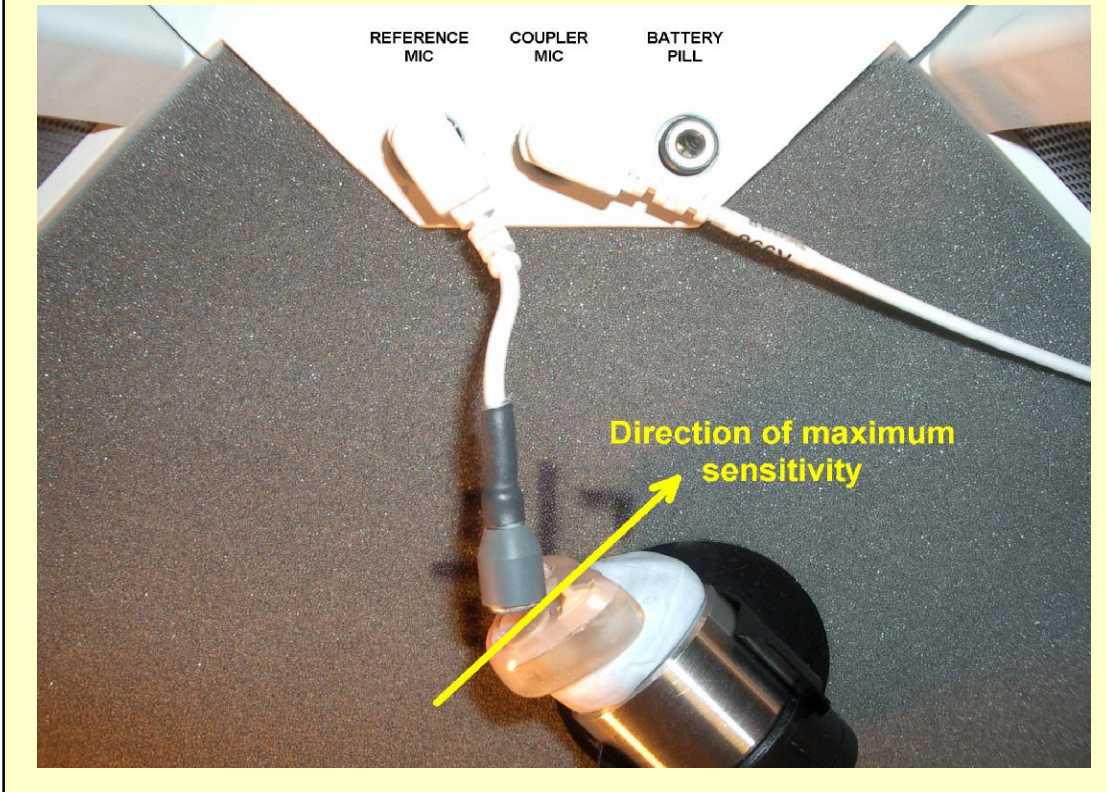

FastFacts 9.6: Directional ITE positioning

9.7 Directional BTE positioning

Use the provided coupler clip to keep the instrument off the floor of the test chamber. The directional BTE instrument should be positioned so that a line joining the microphone openings on the instrument is at 45 ° as shown. Position the test box reference microphone at the front microphone opening on the instrument. Note which speaker (L or R) is in “front” of the hearing instrument for reference. Other positions may be preferable for some instruments.

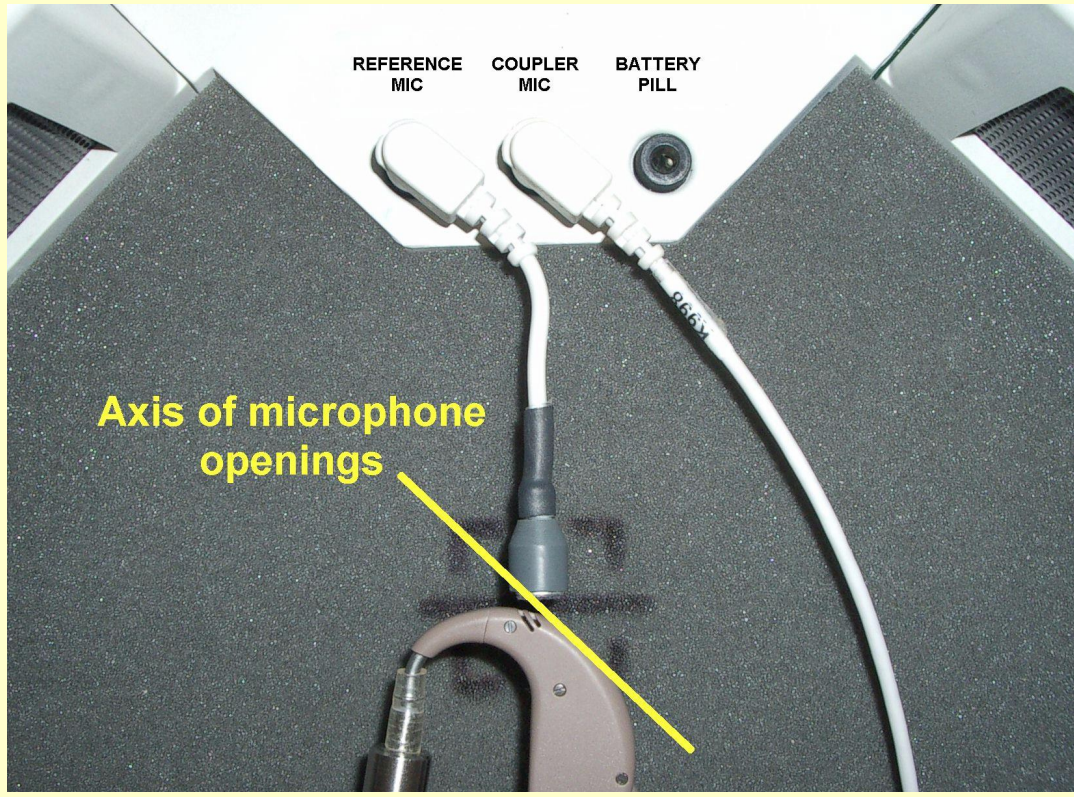

FastFacts 9.7: Directional BTE positioning

9.8 Test box directional procedure

To test directional function in the **test chamber**:

- 1) Mount the hearing aid to the appropriate coupler (see [7.11: Coupling the hearing instrument](#)).
- 2) Position the hearing instrument in the test chamber as shown in [9.6: Directional ITE positioning](#) or [9.7: Directional BTE positioning](#).
- 3) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Directional].
- 4) Highlight and <PICK> Test 1, 2, 3 or 4. This presents test signals from both speakers. Two real-time curves appear on the screen - the heavier one is the response to the left speaker signal and the lighter one the response to the right speaker signal. For a non-directional instrument, the two curves will be nearly identical. Setting the instrument for directional operation should produce a separation of the curves.
- 5) On the Test control poster, highlight and <PICK> the Level window, then highlight and <PICK> a level from the drop-down list. For an **adaptive** directional instrument, a stimulus level between 50 and 70 dB should cause the separation to change over a 15 – 45 second period. For a noise-reducing instrument, both curves may change with time and stimulus SPL. Speech-adaptive directional instruments may appear to be non-directional on this test.
- 6) For **speech-adaptive** directional instruments, highlight and <PICK> [Speech S/N] within the Test control poster and select a signal to noise ratio from the list box. This presents speech from the left and noise from the right test speaker. A 6 dB S/N will usually induce directional performance after 15 – 45 seconds.
- 7) Press <Continue> to capture curves or press <Cancel> to abort the test. Captured curves are labeled **L (left speaker response)** and **R (right speaker response)** followed by the test number (1 - 4). For speech-adaptive directionals you must wait until the aid has become directional before pressing <Continue> Upon pressing continue the Verifit will momentarily present the standard directional test signal and automatically capture the separated curves.

To test directional function in situ, see [13: On-Ear Instrument Measures](#)

FastFacts 9.8: Test box directional procedure

9.9 Multicurve procedure


Multicurve generates 1 to 4 output or gain response curves using either pink noise or swept pure tones at input levels ranging from 40 – 90 dB SPL and 0 dB (off).

- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Multicurve].
- 2) Refer to [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#) and [2.6: Battery pill use and care](#) to prepare the instrument for testing.
- 3) **Turn the instrument ON.**
- 4) Highlight and <PICK> button 1, 2, 3 or 4 in the Test column.
- 5) Highlight and <PICK> the Stimulus window, then highlight and <PICK> either [Pink noise] or [Swept] from the drop-down list.
- 6) Highlight and <PICK> the Level window for the selected Test, then highlight and <PICK> a stimulus level from the drop-down list.
- 7) Press <Continue> to run a sweep or to capture a pink noise curve; press <Cancel> to abort the test.
- 8) To measure another response curve, highlight and <PICK> another test button; to overwrite a curve, highlight and <PICK> its test button.

To change scale from dB SPL to dB Gain, highlight & <PICK> the Scale window.

FastFacts 9.9: Multicurve procedure

9.10 Multicurve results

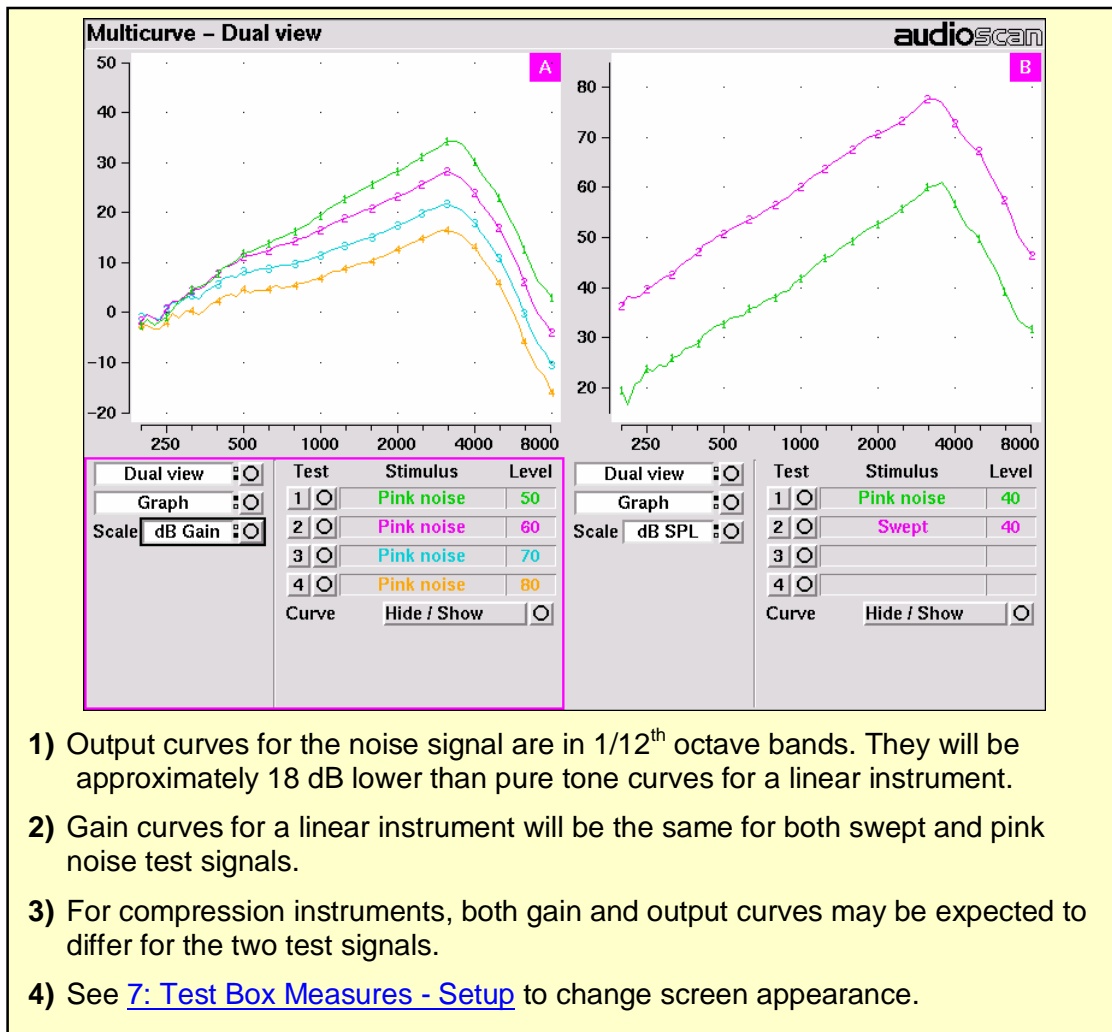

FastFacts 9.10: Multicurve results

9.11 Spectral analysis in Multicurve

Selecting a stimulus level of 0 dB (off) in [Multicurve] presents a spectral analysis of the coupler microphone signal (Scale = dB SPL) or the difference between the coupler and reference microphone spectra (Scale = dB Gain).

If the stimulus type is [Pink noise], the analysis is performed in 1/12th octave bands and displayed in real time. Pressing <Continue> captures the displayed spectrum.


If the stimulus type is [Swept], a swept filter analysis is performed in 1/12th octave bands and displayed at 1/3rd octave frequencies. Pressing <Continue> causes the swept filter analysis to be performed in 1/12th octave bands and displayed at 1/12th octave frequencies.

FastFacts 9.11: Spectral analysis in Multicurve

9.12 Battery drain test

The battery drain test automatically measures the battery current when there is no input signal (quiescent), when the input signal is a 1 kHz tone at 65 dB SPL and the average of the currents when the input signal is 90 dB SPL at the HFA or SPA frequencies (Avg @ 90 dB). Battery life is estimated assuming 80% quiescent and 20% Avg @ 90 dB use. With the hearing instrument set as normally used:

- 1) Install and connect the proper battery pill. See [2.6: Battery pill use and care](#)
- 2) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Battery drain].
- 3) Highlight and <PICK> [Battery type] and select from drop-down list.
- 4) Highlight and <PICK> [Hours per day] and select from drop-down list.
- 5) Highlight and <PICK> [Start test]

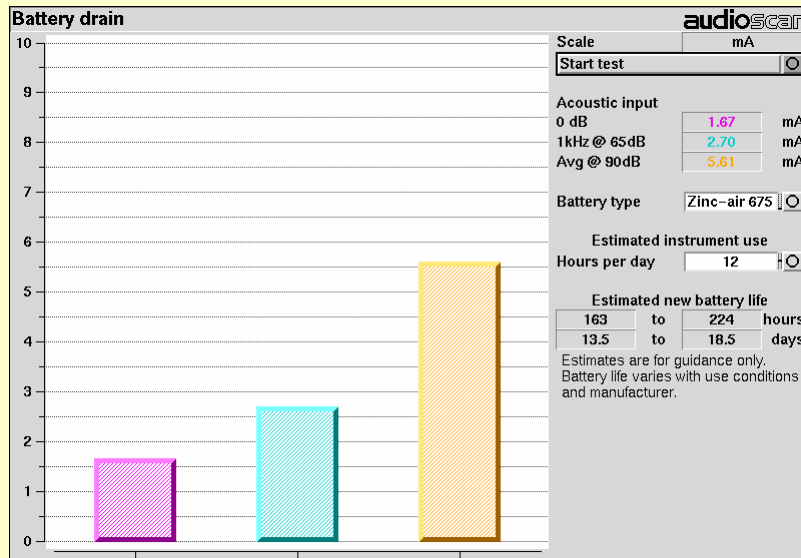


FastFacts 9.12: Battery drain test

9.13 Manual control procedure

Manual control displays the reference microphone SPL, the coupler microphone SPL, their difference in dB (gain) and coupler microphone signal distortion at any selected 1/12th octave frequency and at a selected input level from 40 to 90 dB (in 5 dB steps) and at 0 dB (off).

- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Manual control]
- 2) Highlight and <PICK> the Level window, then highlight and <PICK> the desired stimulus level from the drop-down list.
- 3) Highlight and <PICK> the Frequency window, then highlight and <PICK> the desired frequency from the drop-down list. (Distortion measurement data is available only from 200-4000 Hz.)
- 4) Highlight and <PICK> [Start test] to start the test. [Start test] changes to [Stop Test]. To stop the measurement, highlight and <PICK> [Stop test]. You may change Level and Frequency without stopping the test.

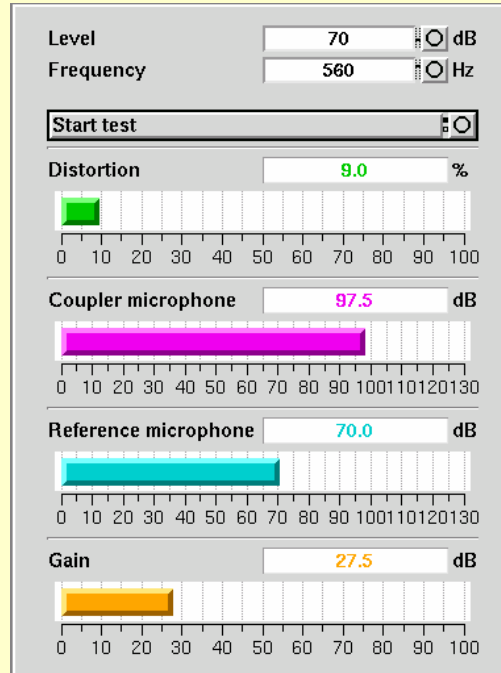

FastFacts 9.13: Manual control procedure

9.14 Sound level meter using manual control

When set for a stimulus level of 0 dB (off), the Manual control mode allows the Verifit to be used as a sound level meter. Sound at the test box reference microphone and the coupler microphone may be analyzed by 1/12th octave, A-weighted or C-weighted filters. A and C-weighted measurements are limited to frequencies above 100 Hz and the averaging time and update rate are 384 ms.

- 1) Press <Tests>, then highlight and <PICK> [Test box measures], then highlight and <PICK> [Manual control].
- 2) Highlight and <PICK> the Level window, then highlight and <PICK> 0 dB stimulus level from the drop-down list.
- 3) Highlight and <PICK> the Filter window, then highlight and <PICK> 1/12th octave, A-weighted, or C-weighted from the drop-down list.
- 4) For 1/12th octave filter, highlight and <PICK> the Frequency window, then highlight and <PICK> the desired frequency from the drop-down list.
- 5) Highlight and <PICK> [Start test] to start the test. [Start test] changes to [Stop Test]. To stop the measurement, highlight and <PICK> [Stop test]. You may change Level and Frequency without stopping the test.

NOTE: A microphone extension cable (VA-130) is available from Audioscan. Standard audio extension cables should not be used. They will substantially increase noise levels.

FastFacts 9.14: Sound level meter using manual control

10 Sensory loss simulator

10.1 Sensory loss simulator description

A pure conductive loss attenuates loud and soft sounds equally, and is easily simulated by just turning down the volume or inserting an earplug. Losses of a sensory nature are more complex and more difficult to simulate.

The most common sensory loss is caused by **outer hair cell** damage and results in loss of audibility for soft sounds but near-normal loudness for loud sounds. **Inner hair cell** damage elevates threshold and reduces loudness for both soft and loud sounds, possibly somewhat more for soft sounds.

This sensory loss simulator (SLS) is intended to allow normal-hearing listeners to hear sounds as if they had the elevated threshold and altered loudness perception caused by outer and inner hair cell damage. It is based on the cochlear hearing loss model described in Moore & Glasberg (2004).

In this simulation, losses less than 58 dB SPL are considered to be due entirely to outer hair cell damage. Any loss greater than this is considered inner hair cell loss.

This SLS does not simulate broadening of the auditory filters or other distortions that may accompany cochlear hearing loss.

For comparison purposes, a simulation of a purely conductive loss is also provided.

To properly experience the simulation, listeners should be within 1m of the sound-field speakers and in a quiet room.

FastFacts 10.1: Sensory loss simulator description

10.2 Sensory loss simulator operation

Listeners should be within 1m of the sound-field speakers and in a quiet room.

- 1) Press <Tests>, then highlight and <PICK> [Sensory loss simulator].
- 2) If an audiogram has previously been entered in Speechmap or Insertion gain, go to step 4. Otherwise, highlight and <PICK> [Audiometry].
- 3) Enter HL threshold values using a mouse or the keypad. Left click the mouse on the audiogram form to enter a point. Click again to delete the point. On the keypad use the arrow keys to change frequency and level and press <PICK> to enter or delete a threshold point.
- 4) Press <Continue> when all points have been entered. See [16.4: Audiometric data entry](#) or [18.2: Assessment data entry](#) for more details.
- 5) Highlight and <PICK> [Start test]. Sound, processed to simulate the entered hearing loss, will be presented via the sound-field speakers.
- 6) Highlight and <PICK> [Stimulus], then highlight and <PICK> a sound to play.
- 7) Highlight and <PICK> [Hearing], then highlight and <PICK> from [Normal, ½ Entered loss, Entered loss].
- 8) Highlight and <PICK> [Type of loss] to toggle between [Sensorineural] and [Conductive] hearing loss simulations.
- 9) Highlight and <PICK> [Cancel] to stop the simulation.

FastFacts 10.2: Sensory loss simulator operation

11 On-Ear Measures - Setup

11.1 Section overview

For accurate, repeatable measurements, the Verifit must be properly set up and calibrated, the client must be properly positioned and the probe tube must be properly positioned in the ear canal.

Real-Ear Unaided Response (**REUR**): SPL, or band SPL, in the open (unaided) ear canal for a given stimulus.

Real-Ear Unaided Gain (**REUG**): Difference, in dB, between the REUR and the stimulus SPL, or stimulus band SPL.

Real-Ear Aided Response (**REAR**): SPL, or band SPL, in the ear canal for a given stimulus, with a working hearing aid in place.

Real-Ear Aided Gain (**REAG**): Difference, in dB, between the REAR and the stimulus SPL, or stimulus band SPL.

Real-Ear Saturation Response (**RESR**): The REAR obtained using a narrow-band signal at a level high enough to saturate the hearing aid (usually 85 - 90 dB SPL).

Real-Ear-to-Coupler Difference (**RECD**): Difference, in dB, between the real-ear SPL and the 2cc coupler SPL produced by the same transducer driven by the same signal.

This section covers the following topics:

[11.2: External sound-field speaker setup](#)

[11.6: Max TM SPL setup](#)

[11.3: On-ear calibration facts](#)

[11.7: ABR nHL to eHL setup](#)

[11.4: Calibration of on-ear probe microphone](#)

[11.8: Positioning the client](#)

[11.5: Calibration check for probe module](#)

[11.9: Positioning the probe tube](#)


FastFacts 11.1: On-ear measures - setup

11.2 External sound-field speaker setup

In some installations, the internal sound-field speakers may not be convenient or may yield speaker overdriven errors because of severe standing waves from nearby walls or shelves. In these cases, an external speaker having a smooth response from 200 - 8000 Hz and an impedance of 4 ohms or greater may be connected to the Front on-ear measure speaker connector on the rear connector panel. The VA-110 speaker is available for this purpose. The speaker should be mounted at ear height, well away from reflective surfaces. See [11.8: Positioning the client](#).

To select this speaker:

- 1) Press <Setup>, then highlight and <PICK> [Speakers].

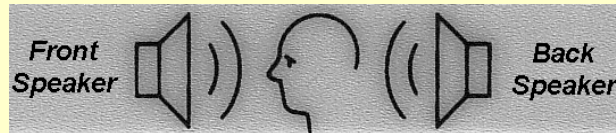

- 2) Highlight and <PICK> the Sound-field speaker button to toggle between [Internal] and [External].



- 3) Press <Continue> to exit Setup

For on-ear directional tests, connect an external speaker (see requirements above) to the back on-ear speaker connector on the rear connector panel. Position the back speaker directly behind and facing the back of the client's head so that front and rear speakers are at the same height and are at the same distance from the center of the client's head. This speaker will be automatically used for the on-ear directional test.

FastFacts 11.2: External sound-field speaker setup

11.3 On-ear calibration facts

The probe microphone and its associated tubing does not have a flat frequency response. The on-ear calibration process compares this response with the flat, factory-calibrated response of the on-ear reference microphone and compensates all subsequent probe-tube measurements for this difference. It may be performed as often as desired but will be invalidated at 1 am the following day or at 1 am the following Monday, depending on the choice of Daily or Weekly calibration interval. After the calibration has expired, you will be prompted to calibrate whenever you attempt a test that uses the on-ear measurement microphones.

It is suggested that a copy of the calibration curves for the probe modules when new be posted near the Verifit for comparison with daily or weekly curves. If there is a drift in calibration with time, the module may need replacement. See [11.5: Calibration check for probe module](#).



Switching left and right probe modules after calibration will result in the wrong calibration curve being applied. Calibration should be repeated if probe modules are switched or if a probe-tube with different dimensions is installed.

FastFacts 11.3: On-ear calibration facts

11.4 Calibration of on-ear probe microphone

Ensure the probe microphone assembly to be calibrated is plugged into the appropriate jack. See [2.4: Microphone connection](#) and [11.3: On-ear calibration facts](#)

- 1) Press <Tests>, then highlight & <PICK> [On-ear measures], then highlight & <PICK> [Calibration].
- 2) Press the enlarged end of a probe tube as far as it will go, into the recessed opening at the top of the probe module.
- 3) Position the open end of the probe tube in front of the reference microphone inlet and press it between the posts as shown.
- 4) Hold the probe module 12.5 - 15 cm (5 – 6 inches) away from and directly in front of the sound-field loudspeakers. Orientation of the probe module is not critical.
- 5) Press <PICK> to calibrate. The resulting curve should be similar in shape to that shown but may be shifted ± 5 dB.
- 6) Press <Left/Right> and repeat steps 2 - 5 for the second probe module.
- 7) To change the calibration interval, highlight & <PICK> [Daily] or [Weekly].

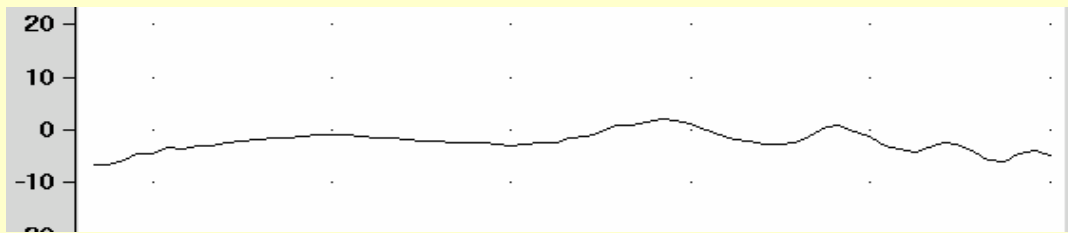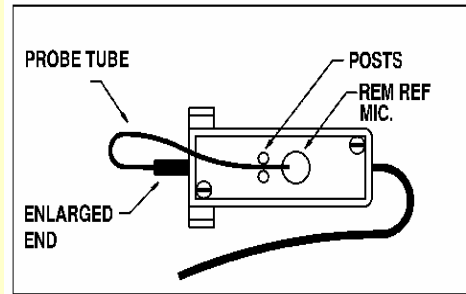


FastFacts 11.4: Calibration of on-ear probe microphone

11.5 Calibration check for probe module

The calibration curve for the probe module is a comparison of the response of the probe microphone (with tube) to the response of the on-ear reference microphone. Any change in the curve indicates a change in one of these microphones (or the probe tube). See [11.4: Calibration of on-ear probe microphone](#). As part of a regular calibration schedule, or to investigate changing calibration curves, the on-ear reference microphone may be checked against a high quality sound level meter.

- 1) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Manual control].
- 2) Highlight & <PICK> [Level], then highlight & <PICK> 70 from the drop-down list.
- 3) Highlight and <PICK> [Frequency], then highlight and <PICK> 2000 Hz from the drop-down list.
- 4) Highlight, then <PICK> [Start test] (which changes to [Stop test]).
- 5) Hold the probe module about 18" (0.5 m) in front of the sound-field speakers.
- 6) Hold the microphone of a calibrated sound level meter next to the on-ear reference microphone as shown. Set the sound level meter to C weighting and slow.
- 7) The sound level meter should read 70 ± 2 dB.
- 8) Repeat with the frequency at 250 Hz.
- 9) To stop the measurement, highlight and <PICK> [Stop test].

The probe module should be replaced if it fails this test.

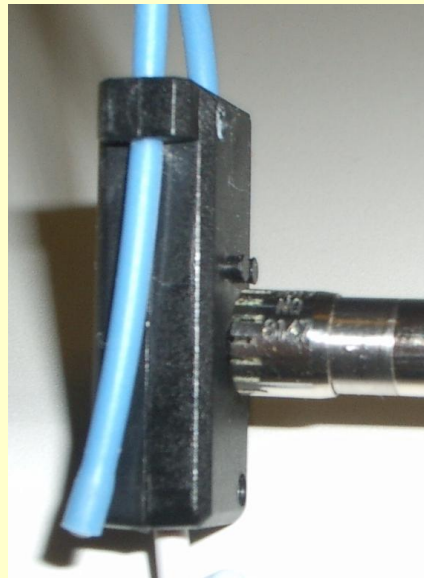

FastFacts 11.5: Calibration check for probe module

11.6 Max TM SPL setup

The Max TM SPL safety feature allows you to set a limit on the SPL, measured by the probe microphone, above which a test terminates. The MAX TM SPL limit is shown on all on-ear measurement screens. The default limit is 120 dB.

To choose a different Max TM SPL setting:

- 1) Press <Setup>, then highlight and <PICK> [Max TM SPL].
- 2) Highlight and <PICK> the current Max TM SPL setting.
- 3) Highlight and <PICK> the desired Max TM SPL setting from the drop-down list.
- 4) Press <Continue> to exit Setup.

This Max TM SPL setting will be in effect until you change it.

If the limit is exceeded during an on-ear test, the Verifit will terminate the test and display the message “**MAX TM SPL EXCEEDED**”. You must then decide to either reduce the maximum output limit of the hearing instrument or increase the Max TM SPL setting following the steps above.

If the MAX TM SPL limit has been disabled, a notice will appear on each real-ear screen. Also, a message reminding you that the MAX TM SPL limit has been disabled will appear when you select an on-ear test.

FastFacts 11.6: Max TM SPL setup

11.7 ABR nHL to eHL setup

Estimated Hearing Level (eHL): An estimate of the behavioral pure tone threshold that accounts for a measured ABR threshold.

Normalized Hearing Level (nHL): (a) Stimulus peak-to-peak equivalent SPL (pe SPL) at ABR threshold relative to the average stimulus pe SPL at ABR threshold of a normal-hearing jury OR (b) stimulus SPL at ABR threshold relative to the average stimulus SPL at behavioral threshold of a normal-hearing jury for the same stimulus. (Note that in this latter case, nHL is equivalent to eHL).

In Speechmap/DSL you have the option of entering ABR thresholds in either eHL or nHL. If eHL is selected, the threshold is treated as if it were a behavioral pure-tone threshold. If nHL is selected, the entered data will first be converted to eHL before SPL threshold and fitting targets are calculated. To do this, appropriate eHL to nHL conversion factors must be applied. You can choose to apply default conversion factors or enter new ones for each client.

This setup allows you to select the **default** conversion factors that are applied in Speechmap/DSL to convert ABR nHL threshold to eHL threshold.

- 1) Press <Setup>, then highlight and <PICK> [ABR eHL to nHL].
- 2) Highlight and <PICK> [DSL defined] or [User defined]
- 3) For [User defined], enter conversion factors using a mouse, the keypad or a keyboard. See [18.2: Assessment data entry](#).

When using the barcode option, both entered nHL data and the nHL to eHL conversion factors are printed as a single barcode. See [3.3: Barcode data input](#).

FastFacts 11.7: ABR nHL to eHL setup

11.8 Positioning the client


Sound reflections from nearby surfaces (including the operator) can cause large measurement errors, especially if the client moves between REUR and REAR measurements. Room noise can increase test time, cause errors and alter the operation of the hearing instrument being evaluated.

- 1) Choose a quiet location and position the client and the sound-field speakers at least 1.5 m (5 feet) away from any hard surfaces.
- 2) Position the client directly in front of, and facing, the front sound-field speaker at a distance of 45 to 90 cm (18" to 36") from the center of the head.
- 3) Position the back speaker (if used) directly behind and facing the back of the client's head so that front and rear speakers are at the same height and are at the same distance from the center of the client's head.

FastFacts 11.8: Positioning the client

11.9 Positioning the probe tube

- 1) Use an otoscope to ensure there is no ear canal obstruction (e.g. cerumen) and to get a sense of ear canal length.
- 2) Install a new probe tube on the probe module. Set the black marker ring approximately 28 mm from end of probe tube for adult females, approximately 30 mm from end of probe tube for adult males and approximately 20-25 mm from end of probe tube for children. These positions may be modified based on the otoscopic examination. (Tip - the probe module body is 28 mm long.)
- 3) Hang the probe module on the ear, as shown. **The reference microphone should face outwards.**
- 4) Adjust the blue cord until the probe module is snug against the head, directly below the earlobe (cheek level). To facilitate this, clip the probe module cable to clothing on the side opposite the test ear and draw the cable snug.
- 5) Hold the front blue cord so that it passes behind the ear canal entrance and pass the end of the probe tube in **front** of the blue cord and into the ear canal.
- 6) Carefully slide the probe tube into the ear canal until the marker ring approaches the intertragal notch. Then let the front blue cord move forward to hold the probe tube in place. **Use an otoscope to verify that the tube is 2 to 5 mm from the eardrum.**

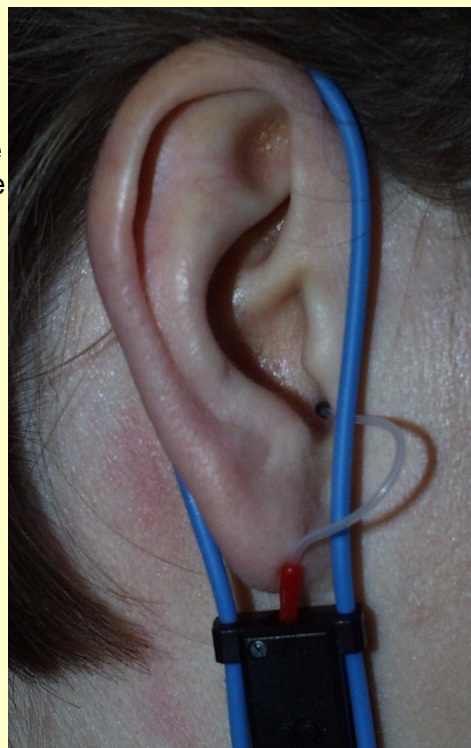

FastFacts 11.9: Positioning the probe tube

12 On-Ear Measures Screen Setup

12.1 Section overview

To accommodate your working preferences, the appearance of On-ear measurement screens may be changed in several ways.

This section covers the following topics:

[12.2: On-ear single or dual view](#)

[12.4: SPL or HL scale](#)

[12.3: Graph, table or 2cc target format](#)

[12.5: Hide or show on-ear curves](#)

FastFacts 12.1: On-ear measures screen setup


12.2 On-ear single or dual view

Most on-ear screens give you the option of selecting either a single, full-screen view or a dual, side by side view. Some data may be dropped from the dual view.

To change the presentation, highlight, then <PICK> [Single view] or [Dual view].

To switch between left and right test panel in single view, press <Left/Right>.

To switch between left and right test panel in dual view, press <Left/Right> or left-click the mouse on a screen button in the inactive test panel.

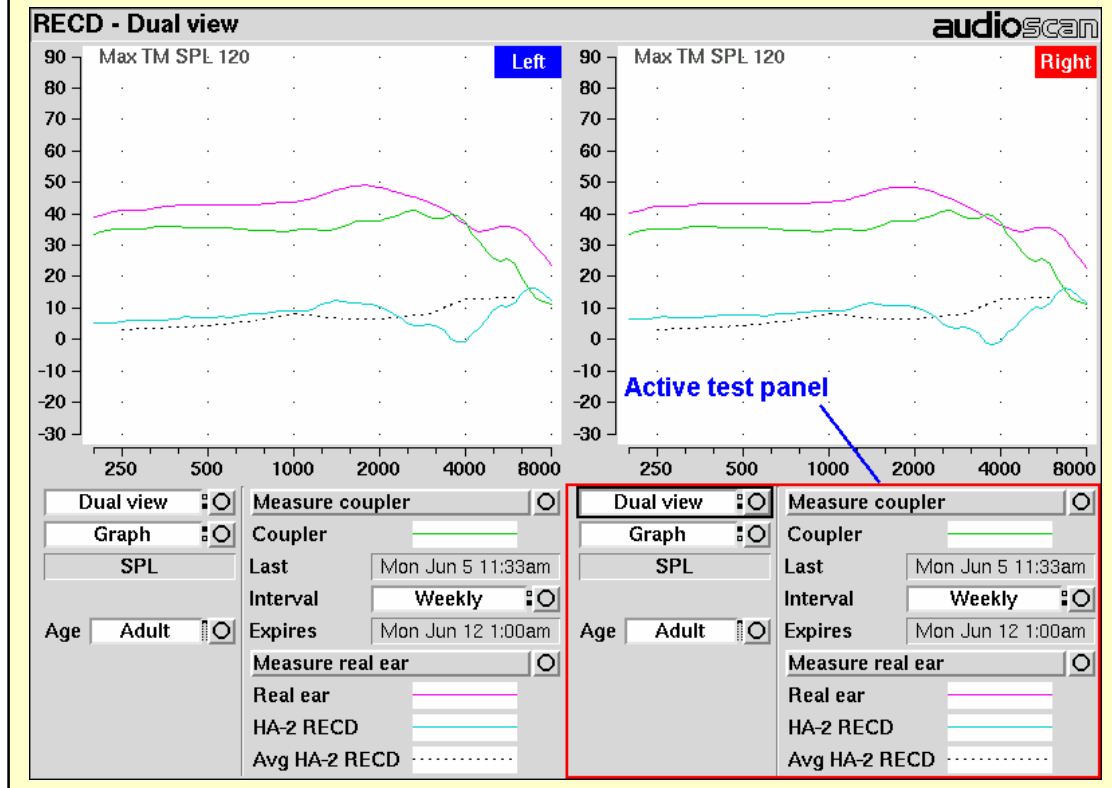

FastFacts 12.2: On-ear single or dual view

12.3 Graph, table or 2cc target format

Most on-ear test screens may be viewed in graphic or tabular format.

- ✓ In RECD and Directional tests, highlight and <PICK> [Format] to toggle between [Graph] and [Table] views.
- ✓ In Speechmap and Insertion gain, highlight and <PICK> [Format], then highlight and <PICK> [Graph], [Table] or [2cc targets] from the drop-down list.
- ✓ In [2cc targets] format, reserve gain and fitting formula may be changed. Highlight & <PICK> the appropriate window, then highlight & <PICK> the reserve gain or fitting formula from the drop-down list.

In Speechmap, if DSL or NAL-NL1 has been selected as a target method, then 2cc coupler targets for user gain, full-on gain and OSPL90 are provided. The full-on gain is just the user gain plus the reserve gain. If no target method has been selected, only 2cc OSPL90 values are indicated. **Target gains are for speech-like signals at a 70 dB input SPL; OSPL90 targets are for 90 dB pure tones.**

On-ear targets are converted to 2cc coupler values by using age-appropriate average or measured (if available) RECD values and the appropriate microphone location effects for the desired style of hearing instrument.

The gain values given assume the hearing instrument is linear for a 70 dB speech input. This assumption is necessary to allow the microphone location effects to be incorporated.

For BTE and body aids, the 2cc coupler values provided assume a fully-occluding earmold with #13 tubing extending to the medial tip of the earmold. If a different type of mold is used, alterations to these values should be made to account for the different acoustic characteristics. For CIC instruments, the 2cc values take into account that the instrument, when deeply inserted into the ear canal, will produce a greater SPL in the ear than that measured in the ITE coupler.

FastFacts 12.3: Graph, table or 2cc target format

12.4 SPL or HL scale

In Speechmap and Insertion gain tests you may view results in either dB HL or dB SPL.

Insertion gain tests view the hearing aid as improving the sound-field threshold. In Insertion gain, the HL scale presents aided results as improved sound-field thresholds while the speech region remains fixed.

Speechmap tests view the hearing aid as increasing the ear canal SPL. Speech gets louder but the threshold remains fixed. On the HL scale in Speechmap, normal ear canal threshold becomes the 0 dB HL line and all data is replotted using this reference. A physical interpretation of the data is difficult and the use of **HL in Speechmap is not recommended**. All instructions given in this guide assume the SPL Speechmap scale.

To change the scale:

Highlight and <PICK> [Scale] to toggle between a dB SPL and dB HL scale.


FastFacts 12.4: SPL or HL scale

12.5 Hide or show on-ear curves

On screens capable of showing a speech envelope, you may show or hide the speech envelope for clarity of display. You may choose to show the envelope with the LTASS or only the LTASS. Hiding a curve does not erase it. To erase curves use the <Session> key.

To hide or show test curve(s):

- 1) Highlight, then <PICK> Curve [Hide/Show].
- 2) Highlight, then <PICK> the desired test to toggle between the Hide or Show options. The curve will toggle on or off when selected.
- 3) Where available, you may also highlight and <PICK> the envelope (right column) button to toggle it on or off.
- 4) Press <Continue> to return to the test screen.

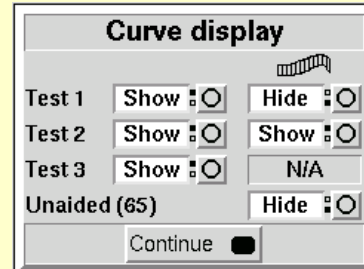

FastFacts 12.5: Hide or show on-ear curves

13 On-Ear Instrument Measures

13.1 Section overview

A number of the tests covered in Test box measures can also be performed with the hearing instrument coupled to the ear of a client rather than to a 2cc coupler. The on-ear results may be expected to differ from the test box results because of the different acoustic environment and the different acoustic load.

This section covers the following topics:

[13.2: On-ear directional test overview](#) [13.5: On-ear noise reduction test](#)

[13.3: On-ear directional testing](#) [13.6: On-ear manual control](#)

[13.4: On-ear feedback test](#) [13.7: Sound level meter using on-ear microphones](#)

FastFacts 13.1: On-ear instrument measures

13.2 On-ear directional test overview

Directional characteristics such as polar plots and Directivity Index (DI) are typically measured in a large anechoic chamber using sound from one direction and at one frequency at a time. This means they apply only for a hearing aid set for linear amplification. These are not conditions representative of real use.

The unique Verifit directional test presents over 1000 tones simultaneously at different frequencies from both front and back sound-field speakers. The tones are individually controlled in real time to produce a precise spectrum from each speaker at the on-ear reference microphone. The ear canal SPL is then analyzed into two real-time response curves, labeled F (front) and B (back) to indicate which speaker generated the curve. Since the curves are generated simultaneously, their difference is not influenced by electronic processing such as compression or noise reduction but reflects only the effect produced by the directional microphones in the instrument. The real-time nature of the test allows the operation of adaptive directional systems to be readily visualized.

For testing speech-adaptive instruments, the Verifit presents speech at selectable levels from the front speaker and noise, at a selectable signal-to-noise ratio from the back speaker. At the end of each presentation of the speech passage, the Verifit momentarily presents its unique 1000 tone directional test signal to capture the directional behavior. Once directional performance is observed, the test may be terminated and a final measure of directional performance will be captured.

Because this measurement is carried out in a real room, results are not expected to agree with data taken in large anechoic chambers. Due to strong reflected signals, measurements made in small reverberant rooms will show small separation between front and back response curves, just as the benefits of directional technology will become less apparent in similar situations. When performed in a sound booth or large acoustically “dead” room, this test can be used to demonstrate and verify directional benefit and to provide assurance that the directional microphone array has been correctly assembled and programmed and has not deteriorated due to element drift or the clogging of ports.

Some hearing aids which use frequency shifting to control feedback may show reversed directional characteristics when tested in this way. In such cases, it is necessary to disable feedback suppression before performing this test.

FastFacts 13.2: On-ear directional test overview

13.3 On-ear directional testing

Best results are obtained using external front and rear sound-field speakers positioned well away from reflective surfaces. See [11.2: External sound-field speaker setup](#). Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.

- 1) Insert the earmold or custom hearing instrument into the ear, being careful not to advance the probe tube further into the ear canal. A lubricant applied in the vicinity of the probe tube will ease insertion and prevent slit leaks.
- 2) Press <Tests> then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Directional].
- 3) Highlight and <PICK> Test 1, 2, 3 or 4. This presents test signals from both speakers resulting in two real-time curves - the heavier one is the response to the left speaker signal and the lighter one the response to the right speaker signal. The two curves will be nearly identical for a non-directional instrument in a sound booth or acoustically “dead” room. Setting the instrument for directional operation should produce a separation of the curves.
- 4) On the Test control poster, highlight and <PICK> the Level window, then highlight and <PICK> a level from the drop-down list. For an **adaptive** directional instrument, a stimulus level between 50 and 70 dB should cause the separation to change over a 15 – 45 second period. For a noise-reducing instrument, both curves may change with time and stimulus SPL. Speech-adaptive directional instruments may appear non-directional in this test.
- 5) For **speech-adaptive** directional instruments, highlight and <PICK> [Speech S/N] within the Test control poster and select a signal to noise ratio from the list box. This presents speech from the front and noise from the back test speaker. A 6 dB S/N will usually induce directional performance after 15 – 45 seconds.
- 6) Press <Continue> to capture curves or press <Cancel> to abort the test. Captured curves are labeled **F** (front speaker response) and **B** (back speaker response) followed by the test number (1 - 4). For speech-adaptive directionals you must wait until the aid has become directional before pressing <Continue> Upon pressing continue the Verifit will momentarily present the standard directional test signal and automatically capture the separated curves.

FastFacts 13.3: On-ear directional testing

13.4 On-ear feedback test

This test shows the SPL and the power concentration ratio (PCR) in each 1/3 octave band for a speech signal. Feedback results in a peak in the speech spectrum and in the PCR. Incipient feedback results in a peak in the PCR curve.

Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.

- 1) Insert the earmold/tip or custom hearing instrument into the ear, being careful not to advance the probe tube further into the ear canal. A lubricant applied in the vicinity of the probe tube will ease insertion and prevent slit leaks.
- 2) Set the instrument as it will be used, with feedback suppression enabled and initialized.
- 3) Press <Tests> then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Feedback].
- 4) Highlight and <PICK> Test 1 and set the Stimulus to Speech-std(1) or (2) and the level to 50 or 60 dB SPL. As the speech passage plays, observe the feedback bar and the lower graph. A yellow Feedback bar and peaks with yellow markers in the lower curve indicate incipient feedback which may impact sound quality. A red Feedback bar and peaks with red markers in the lower curve indicate feedback that may be audible and should be mitigated. Press <Continue> to run the full speech passage and capture average results for the passage.
- 5) Repeat for Test 2 and 3 as needed. These tests may be used to explore the effects of jaw movement and the presence of hands, hats and telephone handsets.

FastFacts 13.4: On-ear feedback test

13.5 On-ear noise reduction test

This test captures a response curve before the hearing instrument reacts to the presence of noise; a comparison curve shows the hearing instrument reacting in real-time. The thick curve is the output before the noise reduction is activated and the thin line is the output as the noise reduction takes effect. The test also provides an overall rms noise reduction value in dB.


Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.

- 7) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and pick [Noise reduction].
- 8) Highlight and <PICK> Test 1, 2 or 3.
- 9) Highlight and <PICK> a stimulus type from the list box.
- 10) Highlight and <PICK> a stimulus level from the list box.
- 11) Highlight and <PICK> [Start test]
- 12) Highlight and <PICK> [Continue] to capture the comparison curve once the noise reduction function has responded.

FastFacts 13.5: On-ear noise reduction test

13.6 On-ear manual control

- 1) Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.
- 2) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Manual control].
- 3) Highlight and <PICK> the Level window, then highlight and <PICK> the desired stimulus level from the drop-down list.
- 4) Highlight and <PICK> the Frequency window, then highlight and <PICK> the desired frequency from the drop-down list. (Distortion measurement data is available only from 200-2500 Hz.)
- 5) Highlight and <PICK> [Start test] to start the test. [Start test] changes to [Stop Test]. To stop the measurement, highlight and <PICK> [Stop test]. You may change Level and Frequency without stopping the test.

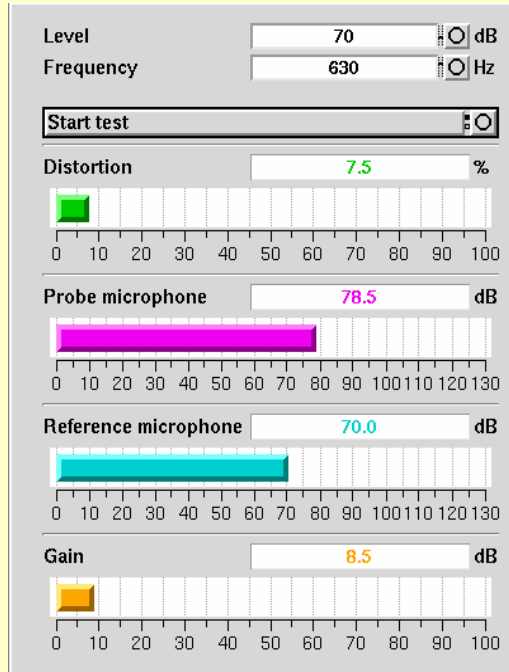

FastFacts 13.6: On-ear manual control

13.7 Sound level meter using on-ear microphones

The manual control feature enables the Verifit to be used as a sound level meter to perform C-weighted, A-weighted or 1/12th octave analysis of sound as measured by the probe and reference microphones. A and C-weighted measurements are limited to frequencies above 100 Hz and the averaging time and update rate are 384 ms.

- 1) Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.
- 2) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Manual control].
- 3) Highlight and <PICK> the Level window, then highlight and <PICK> 0 dB stimulus level from the drop-down list.
- 4) Highlight and <PICK> the Filter window, then highlight and <PICK> 1/12th octave, A-weighted, or C-weighted from the drop-down list.
- 5) For 1/12th octave filter, highlight and <PICK> the Frequency window, then highlight and <PICK> the desired frequency from the drop-down list.
- 6) Highlight and <PICK> [Start test] to start the test. [Start test] changes to [Stop Test]. To stop the measurement, highlight and <PICK> [Stop test]. You may change Level and Frequency without stopping the test.

FastFacts 13.7: Sound level meter using on-ear microphones

14 Occlusion Effect Test

14.1 Section overview


The occlusion effect refers to the unnatural sound of one's own voice that results from occluding the external ear. It is the result of the high SPL produced at the back of the throat during a closed vowel utterance coupling to the closed ear canal via bone conduction. Ear canal levels in a closed ear canal can exceed 90 dB SPL for closed vowels such as "ee". The effect is greatest below 1000 Hz. Clients with mild-moderate low frequency hearing loss may describe their own voice as sounding "hollow" or as if they were talking "in a barrel".

The occlusion effect is measured by comparing the SPL in the ear canal to that outside the external ear while the client vocalizes "ee" with the hearing aid in place and turned off. Differences greater than 10 dB may result in complaints. This occlusion effect measurement is useful in quantifying the extent to which venting or deep insertion is addressing the underlying cause of the complaint.

FastFacts 14.1: Occlusion effect test

14.2 Occlusion effect measurement

- 1) Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube.
- 2) Insert the earmold or custom hearing instrument into the ear, being careful not to advance the probe tube further into the ear canal. A lubricant applied to the earmold or custom instrument shell in the vicinity of the probe tube will make insertion easier and prevent slit leaks around the tube.
- 3) Turn the hearing aid OFF.
- 4) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Occlusion].
- 5) Highlight and <PICK> [Start test]. [Start test] changes to [Stop Test].
- 6) While the client vocalizes "ee" in a normal voice, highlight and <PICK> [Stop test] to stop the test and freeze the screen. A red occlusion bar indicates an occlusion problem, a yellow bar indicates a possible problem and a green bar indicates no problem.
- 7) Increase venting and repeat the test until a green bar is obtained.

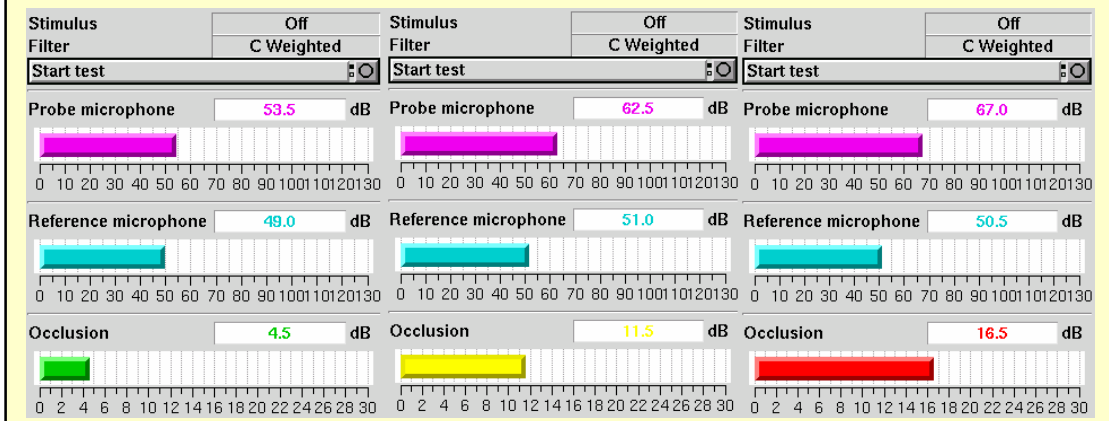

FastFacts 14.2: Occlusion effect measurement

15 RECD (real-ear to coupler difference)

15.1 Section overview

The RECD measurement procedure may be accessed by selecting RECD from the **On-ear tests** list or from the Audiometry poster within **Speechmap**.

This section covers the following topics:

15.2: RECD facts

15.3: RECD coupler response

15.4: RECD on-ear response

15.5: RECD results

FastFacts 15.1: RECD (real-ear to coupler difference)

15.2 RECD facts

Individual RECD values are derived by comparing the real-ear SPL for a given test signal to the 2cc coupler SPL for the same test signal. These RECD values are used to: (1) convert HL threshold and UCL obtained using insert phones to SPL values, (2) simulate real-ear measurements from 2cc coupler measures (test box); and (3) derive 2cc coupler targets from real-ear aided targets.

If you use insert ear phones for audiometry **OR** **if you verify using the test box**, you should **measure RECD** (preferred) or use age-related average RECD.

If you use headphones or sound field for audiometry **AND** you **use on-ear tests** for verification, **RECD is not applied in any way** and need not be measured.

If you use 2cc coupler targets, you should use measured or age-related RECD.

Insert phone audiometry with measured RECD and on-ear verification provides the best fitting accuracy, with test box verification being slightly less accurate. Insert phone HL data with age-related average RECD is next best.

This software provides average RECDs for 1 - 60 months, 6 - 10 years of age and Adult. A sampling of age-related foam-tip **HA-2 RECD** values is shown below.

Age/Freq	250	500	750	1000	1500	2000	3000	4000	6000
1 month	3	8	9	12	15	15	16	20	23
12 months	3	6	8	10	10	11	11	15	17
24 months	3	5	7	9	9	10	10	14	15
36 months	3	5	7	9	8	9	9	13	14
60 months	3	5	7	9	7	8	8	13	13
8 yrs - Adult	3	4	6	8	7	7	8	13	13


Values are from DSL 5.0 but differ from DSL in that values for 119 months have been used for ages >119 months. **DSL 5.0 RECDs differ from DSL 4.1 RECDs.**

FastFacts 15.2: RECD facts

15.3 RECD coupler response

Prepare the system for test box tests. See [2.4: Microphone connection](#), [7.9: Calibrating test box reference microphone](#). Connect the RECD transducer to the RECD jack on the front of the test chamber

- 1) Press <Tests> then highlight and <PICK> [On-ear measures], then highlight and <PICK> [RECD].
- 2) If a valid coupler response exists, go to [15.4:RECD on-ear response](#).
- 3) If no coupler response exists or the measurement interval has been exceeded, connect the RECD transducer to the BTE coupler.

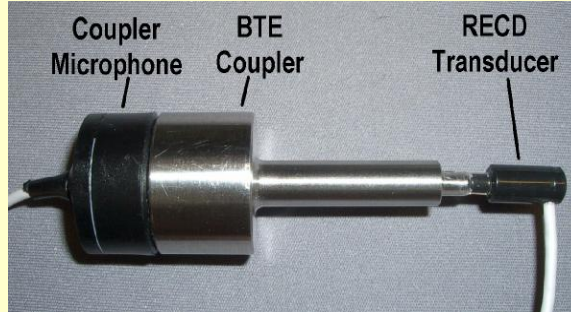

- 4) Highlight and <PICK> [Measure coupler] and follow the setup instructions.
- 5) Press <Continue> to generate the coupler response curve (green). When this curve is stable, press <Continue> to save it.
- 6) Highlight and <PICK> [Daily] or [Weekly] to toggle the measurement interval for the coupler response.


For comparison purposes, an age-related average RECD is shown as a dotted curve on the screen. If you have accessed RECD from the on-ear tests list, you may highlight and <PICK> the Age box to change the age for this reference curve.

FastFacts 15.3: RECD coupler response

15.4 RECD on-ear response

Connect the RECD transducer to the RECD jack on the test chamber. Prepare the system for on-ear tests. See [11.4: Calibration of on-ear probe microphone](#).

- 1) Press <Tests> then highlight and <PICK> [On-ear measures], then highlight and <PICK> [RECD].
- 2) If no coupler response exists or the measurement interval has been exceeded, go to [15.3: RECD coupler response](#)
- 3) If a valid coupler response exists, highlight and <PICK> [Measure real ear].
- 4) Refer to [11.9: Positioning the probe tube](#) and insert the probe tube into the ear canal to within 2 - 5 mm of the eardrum.

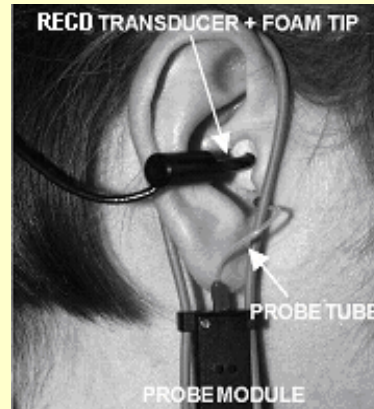


- 5) Insert the foam tip into the ear, being careful not to advance the probe tube further into the ear canal. Insertion depth should be as used with insert phones. **Allow the foam tip to fully expand in the ear.**
- 6) Press <Continue> to generate the real-ear response curve (pink) and the RECD curve (cyan). If the RECD curve is **negative and unstable** in the low frequencies, check the seal of the foam tip. Increase the tip size and/or apply a lubricant to the foam tip to improve the seal. If the RECD curve deviates more than 10 dB from the average in the 4 - 6 kHz region, check for proper probe placement and possible blockage of the tube by the foam tip or cerumen.
- 7) When these curves are stable, press <Continue> to save them.

For comparison purposes, an age-related average RECD is shown as a dotted curve on the screen. If you have accessed RECD from the on-ear tests list, you may highlight and <PICK> the Age box to change the age for this reference curve.

FastFacts 15.4: RECD on-ear response

15.5 RECD results

The RECD measurement screen shows the response of the RECD transducer in the real ear (pink) and in the 2cc coupler (green). The difference between these curves is the RECD (cyan). An age-appropriate average RECD is shown for reference (dotted). The RECD shows a negative RECD below about 1 kHz, often indicating a poor seal between the foam tip and the ear canal. See [15.2: RECD facts](#), [15.3: RECD coupler response](#), [15.4: RECD on-ear response](#).

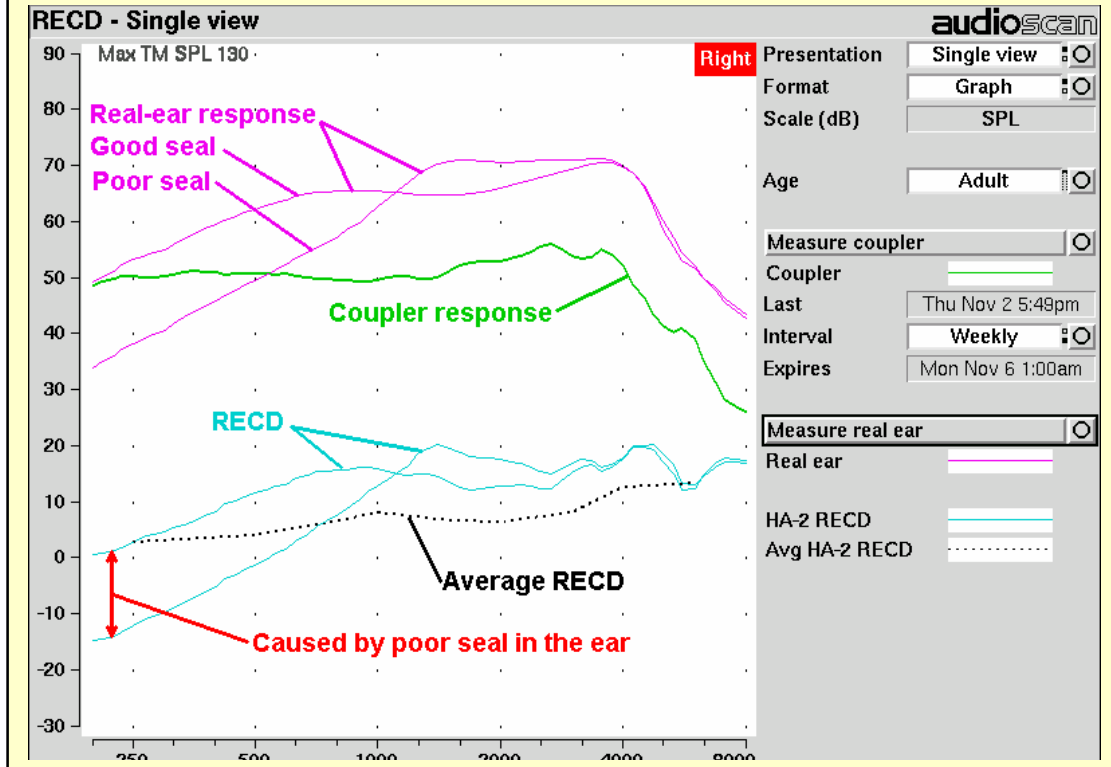

FastFacts 15.5: RECD results

16 Insertion Gain

16.1 Section overview

Insertion gain is the difference between aided and unaided ear canal SPL ($REIG = REAG - REUG$). It inherently assumes a sound-field audiogram. For non-linear hearing aids, REIG is stimulus-dependent and the dependency is specific to each hearing aid. Insertion gain fitting methods ignore these facts. **Viewport or Speechmap is recommended for fitting all linear and non-linear hearing instruments.**

Refer to [11: On-Ear Measures - Setup](#) and [12: On-Ear Measures Screen Setup](#) for terminology and information on preparing the system for use and for proper positioning of the client and the probe tube.

This section covers the following topics:

[16.2: Insertion gain in SPL](#)

[16.6: REAR measurement procedure](#)

[16.3: Insertion gain in HL](#)

[16.7: SII calculation in Insertion gain](#)

[16.4: Audiometric data entry](#)

[16.8: CROS fitting using Insertion gain](#)


[16.5: REUR measurement procedure](#)

FastFacts 16.1: Insertion gain

16.2 Insertion gain in SPL

See [11: On-Ear Measures - Setup](#) and [12: On-Ear Measures Screen Setup](#)

- 1) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Insertion gain].
- 2) Highlight and <PICK> [Instrument] to chose from BTE, ITE, ITC, CIC, Body. This changes the CORFIG used for 2cc targets.
- 3) Highlight and <PICK> fitting method to chose from FIG6, NAL-NL1, NAL-R, POGO II, Berger, Libby, User, None

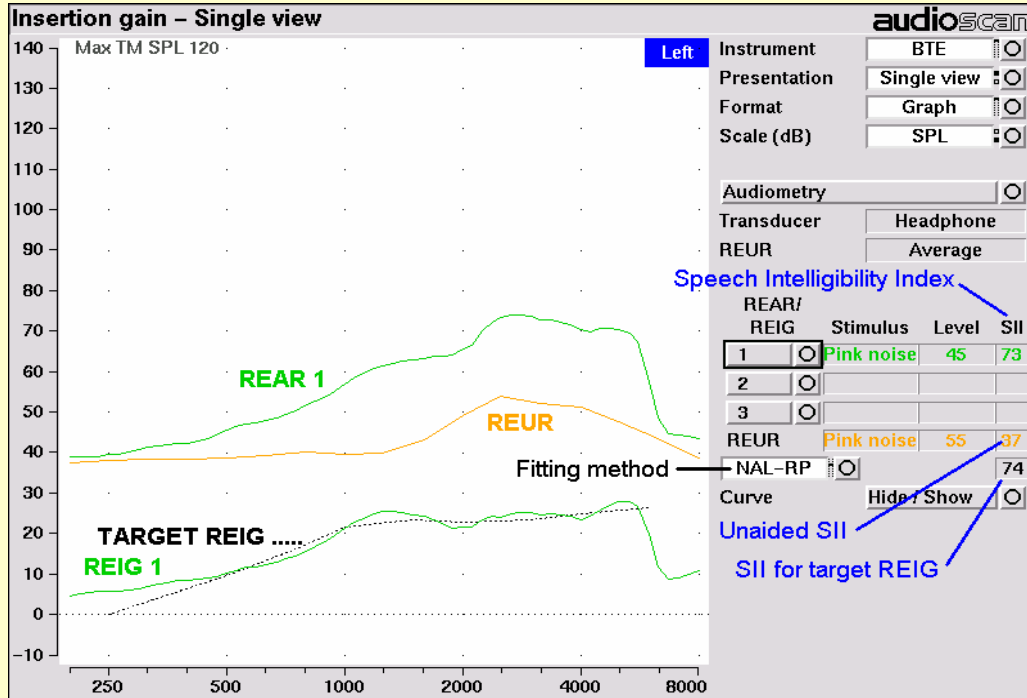

FastFacts 16.2: Insertion gain in SPL


16.3 Insertion gain in HL

See [11: On-Ear Measures - Setup](#) and [12: On-Ear Measures Screen Setup](#)

- 1) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Insertion gain].
- 2) Highlight and <PICK> [Instrument] to chose from BTE, ITE, ITC, CIC, Body. This changes the CORFIG used for 2cc targets.
- 3) Highlight and <PICK> fitting method to chose from FIG6, NAL-NL1, NAL-R, POGO II, BERGER, LIBBY, USER, NONE.

An estimated aided threshold curve is calculated by shifting the unaided threshold curve upward by the REIG. This estimate assumes a) the **unaided threshold is a sound-field threshold** and b) a **linear aid** (i.e. REIG doesn't depend on level).

The REUR curve uses the right-hand SPL scale and is shown for reference only.

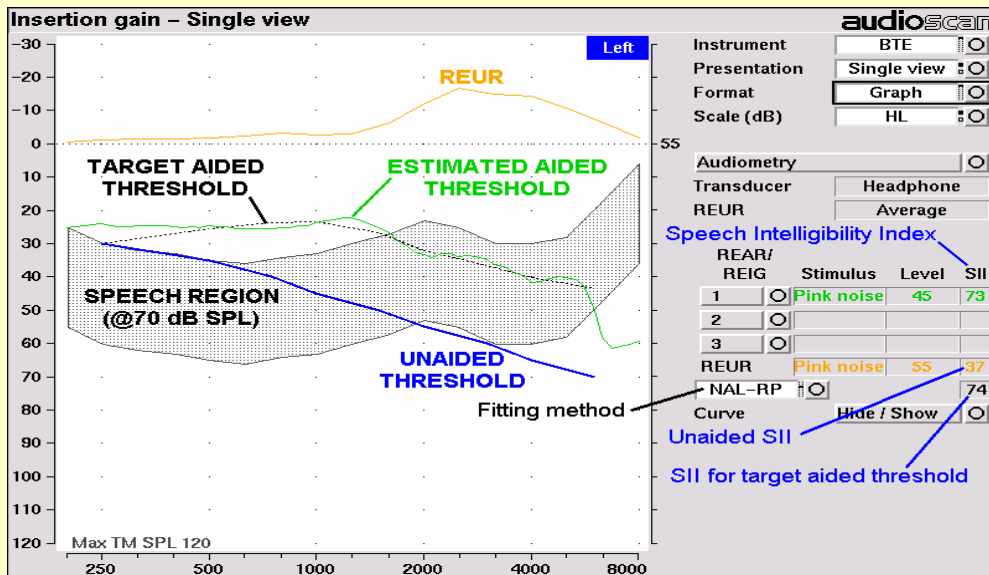

FastFacts 16.3: Insertion gain in HL

16.4 Audiometric data entry


Threshold data and parameters may be entered by scanning a barcode on a previous Verifit or RM500SL printout or manually using a mouse, the keypad or the numeric keypad on an external keyboard. See [2.7: Mouse, keyboard, barcode scanner](#), and [3.2: Input device operation](#).

To enter data from a barcode see [3.3: Barcode data input](#) and:

- 1) Press the barcode trigger and center the red line on the barcode from a distance of 6 - 8". The scanner will beep and the Barcode Entry poster opens.
- 2) Highlight and <PICK> [Done] to accept the data indicated on the poster.

To enter assessment data and parameters manually:

- 1) Highlight and <PICK> [Audiometry].
- 2) Highlight and <PICK> [Transducer], then highlight and <PICK> from the list.
- 3) Highlight and <PICK> [REUR], then highlight and <PICK> to toggle selection.

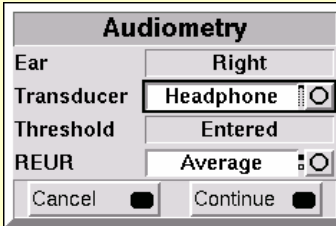

Audiometry	
Ear	Right
Transducer	Headphone
Threshold	Entered
REUR	Average
Cancel	Continue

Measured REUR should be used with a soundfield threshold because individual REUR is part of the threshold. **Average REUR should be used with headphone or insert phone thresholds** because individual REUR isn't part of these thresholds.

- 4) When all selections have been completed, press <Continue>.
- 5) Enter HL threshold values using a mouse or keypad. Left click the mouse on the audiogram form to enter a point. Click again to delete the point. On the keypad use the arrow keys to change frequency and level and press <PICK> to enter or delete a threshold point.
- 6) Press <Continue> when all points have been entered.

FastFacts 16.4: Audiometric data entry


16.5 REUR measurement procedure

Performing a REUR test erases all existing REAR and REIG data associated with the current test ear. REUR should be measured only if Transducer = Soundfield. Otherwise it is more accurate to use average. See [16.4: Audiometric data entry](#).

- 1) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Insertion gain].
- 2) Referring to [11: On-Ear Measures - Setup](#) and [12: On-Ear Measures Screen Setup](#), prepare the system, position the patient and insert the probe tube.
- 3) If REUR is set to Average, highlight and <PICK> [Audiometry] then highlight and <PICK> [REUR] to toggle to [Measure]. Press <Continue> to advance to the Threshold entry poster. You may enter threshold data at this time (see [16.4: Audiometric data entry](#)) or press <Continue> to proceed with the measurement.
- 4) Highlight and <PICK> REUR (below REAR/REIG test 3).
- 5) Press <Continue> to accept the default 55 dB SPL Pink noise stimulus and capture the REUR curve OR highlight and <PICK> [Stimulus] and [Level] to select other values. These choices have no impact on the resulting REIG.
- 6) If there is a notch in the 4000 to 8000 Hz region, try advancing the probe tube 2 mm and repeating step 3. If the notch moves to a higher frequency, repeat this step until the notch moves beyond the highest frequency of interest.

Because it is analyzed in 1/3 octave bands, the Noise stimulus produces a REUR 12 dB below that produced by the Swept stimulus at the same overall level.

The Average REUR used is shown at right. Measured RECD will vary from this.

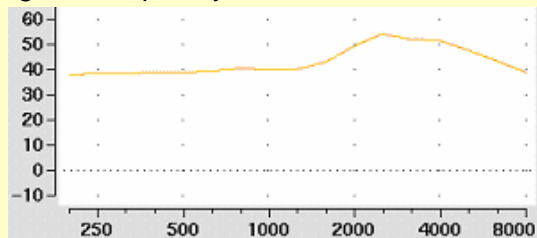

For small children the peak may shift to the right and become sharper. Drainage tubes and perforated ear drums will cause large differences - usually around 1 kHz.

FastFacts 16.5: REUR measurement procedure

16.6 REAR measurement procedure

- 1) Press <Tests>, then highlight and <PICK> [On-ear measures], then highlight and <PICK> [Insertion gain].
- 2) Referring to [11: On-Ear Measures - Setup](#) and [12: On-Ear Measures Screen Setup](#), prepare the system, position the patient and insert the probe tube. (For tight-fitting hearing aids or earmolds, the probe tube may be inserted through a vent.)
- 3) Without disturbing the probe tube, insert the hearing instrument or earmold.
- 4) Referring to [16.4: Audiometric data entry](#), enter threshold data.
- 5) Highlight and <PICK> button 1, 2, or 3 in the REAR/REIG column. This will present a signal from the sound-field speaker and display the REAR Setup box.
- 6) Highlight and <PICK> [Stimulus], then highlight and <PICK> [Pink noise] or [Swept]. Use pink noise when using the NAL-NL1 fitting formula. Use Swept when testing maximum output capabilities.
- 7) Highlight and <PICK> [Level], then highlight and <PICK> a stimulus level from the list box. Use the level shown on the target REIG curves. If no level is shown, use the lowest level required to overcome background noise.
- 8) Adjust the hearing instrument so that the REIG curve (SPL scale) or Aided threshold curve (HL scale) approximates the dotted target curve. For multi-level targets, match each curve using the stimulus level indicated.
- 9) Press <Continue> to complete the test and capture the data.

See [16.2: Insertion gain in SPL](#) and [16.3: Insertion gain in HL](#) for an explanation of screen curves. See [16.7: SII calculation in Insertion gain](#)

NOTE: Because it is analyzed in 1/3 octave bands, the Noise stimulus produces a REAR up to 12 dB below that produced by the Swept stimulus at the same overall level. The REIGs will be similar except at levels which cause non-linear operation of the hearing instrument.


FastFacts 16.6: REAR measurement procedure

16.7 SII calculation in Insertion gain

The Verifit calculates a Speech Intelligibility Index (SII) value for unaided speech at 70 dB SPL (REUR), the target REIG and for each aided test. The calculations use the 1/3 octave band method of ANSI S3.5 –1997 without the 160 Hz band and without masking effects with results expressed as a % by multiplying by 100. Level distortion effects are included but no hearing loss desensitization is applied. An idealized speech region of LTASS +/- 15 dB is used.

The speech recognition associated with a given SII is a function of the test material and the cognitive abilities of the listener. There is considerable individual variability in relating the SII to speech recognition, especially amongst impaired listeners. The following shows expected nominal recognition scores vs SII for normals on the Connected Speech Test (Sherbecoe and Studebaker 2003).

SII (%)	Score (%)
10	1
15	8
20	20
25	45
30	60
35	74
40	84
45	90
50	94
55	96
60	98
70	99
80	100
90	100

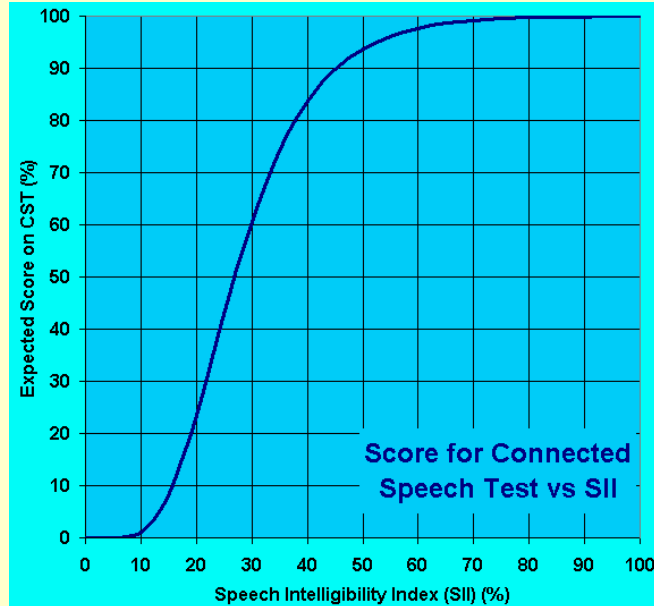


FastFacts 16.7: SII calculation in Insertion gain

16.8 CROS fitting using Insertion gain

See [11: On-Ear Measures - Setup](#), and [12: On-Ear Measures Screen Setup](#).

- 1) Place a probe module on each ear and insert the probe tube into the good ear only. Position the CROS instruments on (in) the ears and turn them ON.
- 2) Position the speaker at 45° to the good ear and set Instrument to [BTE].
- 3) Select REAR 1 and record a response using [Pink noise] at 55. It should be like a normal REUR. If not, the ear may not be fully open.
- 4) Position the speaker at 45° to the poor ear.
- 5) Set Instrument to [CROS].
- 6) Select REAR 2 and obtain a response using [Pink noise] at 55. Adjust the response to match that obtained in step 3.
- 7) Position the speaker directly in front of the client. Select REAR 3 and use [Pink noise] at 55 to check for a smooth response. Repeat with Instrument set to [BTE]. An irregular response may be the result of reflections from nearby objects (including the operator) or it may indicate phasing problems in the aids.

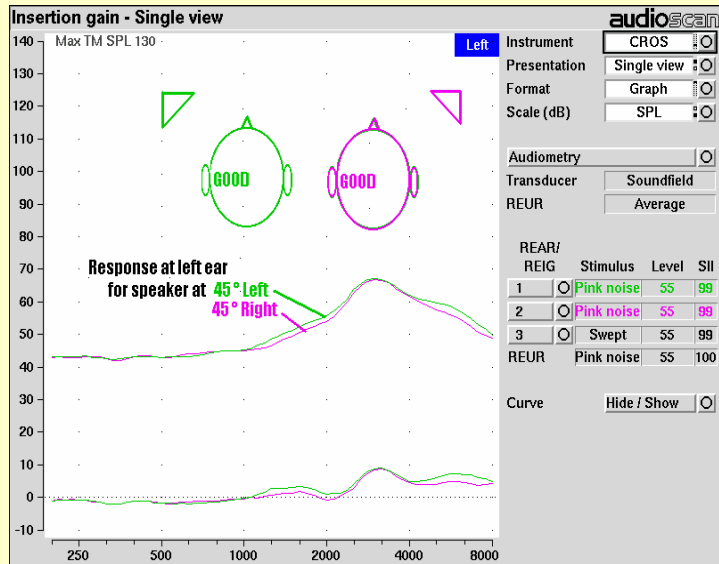

FastFacts 16.8: CROS fitting using Insertion gain

17 Speechmap®

17.1 Section overview

Speechmap presents hearing threshold, UCL and amplified speech spectra as SPL in the ear canal. The fitting goal is to make amplified speech audible and comfortable while avoiding discomfort for loud sounds.

Speechmap may be accessed from the Tests menu as either a Test box measure or an On-ear measure. See [17.10: On-ear or Test box mode](#) for details.

Speechmap is normally used in SPL and this guide describes only this usage. See [12.4: SPL or HL scale](#) for details.

This section covers the following topics:

[17.2: Speechmap facts](#)

[17.7: Using Speechmap](#)

[17.3: DSL 5.0 in Speechmap](#)

[17.8: Screen tour - unaided screen](#)

[17.4: DSL 5.0 changes](#)

[17.9: Screen tour - aided screen](#)

[17.5: NAL-NL1 in Speechmap](#)

[17.10: On-ear or Test box mode](#)

[18.6: Camfit in Speechmap](#)

[17.11: SII calculation in Speechmap](#)

Related sections are [18: Speechmap Fitting Procedures](#) and [19: Speechmap Technical Details](#)

FastFacts 17.1: Speechmap

17.2 Speechmap facts

Speechmap is a trademarked hearing instrument fitting environment introduced by Audioscan in 1992. It presents hearing threshold, UCL and amplified speech spectra as SPL in the ear canal. The fitting goal is to make amplified speech audible and comfortable while avoiding discomfort for loud sounds. Originally speech was simulated using amplitude-modulated tonal signals. The Verifit replaces these simulated speech signals with real speech, both recorded and live. For details of the test signals and analysis methods, see [19: Speechmap Technical Details](#).

Speechmap departs from insertion gain target methods in presenting amplification goals in the context of the residual auditory area, not as an isolated target line, and in the use of real speech, not tones or broad-band noise. This is extremely important for non-linear hearing aids, analog or digital, because their processing is signal-dependent. It also makes provision to correct audiometric data and 2cc coupler measurements for age-related or individually-measured acoustic differences between real ears and the couplers used for calibration and testing.

The Speechmap environment provides two hearing instrument verification options: **on-ear measures** and **test box measures**. See [17.10: On-ear or Test box mode](#) for details.

FastFacts 17.2: Speechmap facts

17.3 DSL 5.0 in Speechmap

The **Desired Sensation Level (DSL)** method is a systematic eardrum sound pressure level (SPLogram) approach to hearing instrument fitting that seeks to ensure amplified speech will be audible and comfortable while loud sounds will not be uncomfortable. It takes into account individual acoustic factors in audiometric and electroacoustic data. DSL was developed by the National Centre for Audiology (NCA), London, Ontario. Audioscan has implemented DSL within its Speechmap fitting system since 1994.

In 2005, DSL underwent an extensive revision resulting in DSL 5.0. It was further revised in 2007 to version 5.0a. See [17.4: DSL 5.0 changes](#) for details of the changes.

The Audioscan version of DSL 5.0 may differ from others in the following ways:

- 1) DSL 5.0 uses RECD values for ages > 119 months from a different source than for ages ≤ 119 months; Audioscan has used the RECD values for a 119 month old for ages > 119 months so all values come from the same study.
- 2) Input-output curve targets and compression threshold targets are not provided. Such steady-state parameters are of little value in estimating amplified speech levels. Matching amplified speech to the LTASS targets at levels from 50 to 75 dB SPL provides much better assurance that fitting goals are being met.
- 3) There is no provision to input the number of compression channels. This may be useful when the signal used for verification is not the signal for which the targets were developed. It is irrelevant when using real-speech signals to match speech targets and when using narrow-band signals to match narrow-band maximum output targets.
- 4) The term Uncomfortable Level (UCL) is used rather than Upper Limit of Comfort (ULC) in referring to the ear canal SPL that should never be exceeded.
- 5) Broadband output limiting targets are not provided.

FastFacts 17.3: DSL 5.0 in Speechmap

17.4 DSL 5.0 changes

The Audioscan version of DSL 5.0 includes provision for **bone conduction** and **ABR** threshold, **binaural** targets, targets for speech levels from **50 to 75 dB SPL**, **new targets** for children, **targets for adults** and **new RECD** default values for children (1 month to 10 years) and adults. The **UWO child's spectrum** has been removed and an **“own voice”** spectrum has been added. As a result of a re-analysis of published data, the **vocal effort effects for Soft and Average** speech levels have been made the same. In addition, the target symbol has been elongated to indicate that the fit-to **target is a range, not a point**.

The children's LTASSa targets for 70 dB speech are lower than those in previous versions of DSL. These differences are functions of the hearing loss and age (where average RECDs are used). Revisions made in version 5.0a reduce these differences at 250 Hz and for more severe losses. The differences will be more significant when the individual RECD values have not been measured for the child being fitted.

Audioscan assumes no responsibility for the validity of these changes; that responsibility rests with the National Centre for Audiology.

FastFacts 17.4: DSL 5.0 changes

17.5 NAL-NL1 in Speechmap

The **NAL-NL1** procedure seeks to amplify speech such that all bands of speech are perceived with equal loudness while maximizing speech intelligibility and ensuring that the wearer perceives speech to be no louder than that which a normal hearing person would perceive. Although NAL-NL1 states its goals for speech, it derives insertion gain targets assuming noise as a verification signal. Because it is more accurate to verify a non-linear fitting using the signal for which the procedure was developed, Audioscan converts the NAL-NL1 insertion gain targets to LTASS targets for amplified speech in the Speechmap environment, resulting in **Speechmap/NAL-NL1**. The conversions use the same adult average RECD and REUG as used in DSL. The RESR targets are from Hearing Aids (Dillon, Thieme Publishing, 2001, p274).

An insertion gain version is available by selecting [Insertion gain] from the on-ear measures test menu. The pink noise stimulus should be used.

FastFacts 17.5: NAL-NL1 in Speechmap

17.6 Camfit in Speechmap

Camfit is an approach to fitting hearing aids developed by Brian Moore at the University of Cambridge, UK. It specifies the level-dependent gain required to correct abnormal loudness as modelled by his cochlear hearing loss simulation. It provides two approaches to correcting for the hearing loss:

Camfit restoration aims to amplify sounds that are soft, comfortable and loud for a person with normal hearing so that they are soft, comfortable and loud for a hearing aid wearer. This is the stated goal of the IHAFF (International Hearing Aid Fitting Forum) fitting method.

Camfit equalisation aims to amplify speech to produce the same loudness in each critical band. It has been argued that this is likely to give the highest intelligibility for a given overall loudness. This is the rationale behind the NAL fitting methods and will generally produce similar targets.

FastFacts 17.6: Camfit in Speechmap

17.7 Using Speechmap


The Speechmap fitting environment provides a number of distinctly different stimuli. Speech and noise are analyzed in 1/3 octave bands and (except for pink noise) data are presented as 3 curves. The top curve is the level exceeded 1% of the time (speech peaks), the lower curve is the level exceeded 70% of the time (speech valleys) and the middle curve is the average. Starting a test causes the signal to be presented continuously - the speech signals run in a loop - and the curves are updated frequently to show any changes you make to the hearing instrument. Pressing <Continue> restarts the speech passage and the curves are then calculated for the entire passage, producing an accurate display of the speech region and LTASS, and an accurate SII calculation. When using live speech, a <Freeze curve> button captures short-term speech features. See [17.11: SII calculation in Speechmap](#) and [19: Speechmap Technical Details](#).

Before starting, please see [11: On-Ear Measures - Setup](#), [17.2: Speechmap facts](#), [17.10: On-ear or Test box mode](#).

- ✓ **To access Speechmap**, press <Tests>, then highlight and <PICK> [On-ear measures] or [Test box measures], then highlight and <PICK> [Speechmap].
- ✓ **To change screen setup**, see [12: On-Ear Measures Screen Setup](#), and [17.10: On-ear or Test box mode](#).
- ✓ **To set up hearing instrument type**, highlight & <PICK> [Instrument], then highlight & <PICK> BTE, ITE, ITC, CIC, RITE (receiver in the ear), Open, Body, CROS or FM. This changes microphone location effects and the RECD used in test box measures and 2cc targets, and the reference microphone used for FM and CROS.
- ✓ **To enter audiometric data**, see [18.2: Assessment data entry](#).
- ✓ **To run a test**, highlight and <PICK> one of the 4 Test buttons. Then highlight and <PICK> [Stimulus] to select the stimulus type. Highlight and <PICK> [Level] to select stimulus level. Press <Continue> to run the test and display the data. See, [17.8: Screen tour](#) & [18: Speechmap Fitting Procedures](#).


FastFacts 17.7: Using Speechmap

17.8 Screen tour - unaided screen

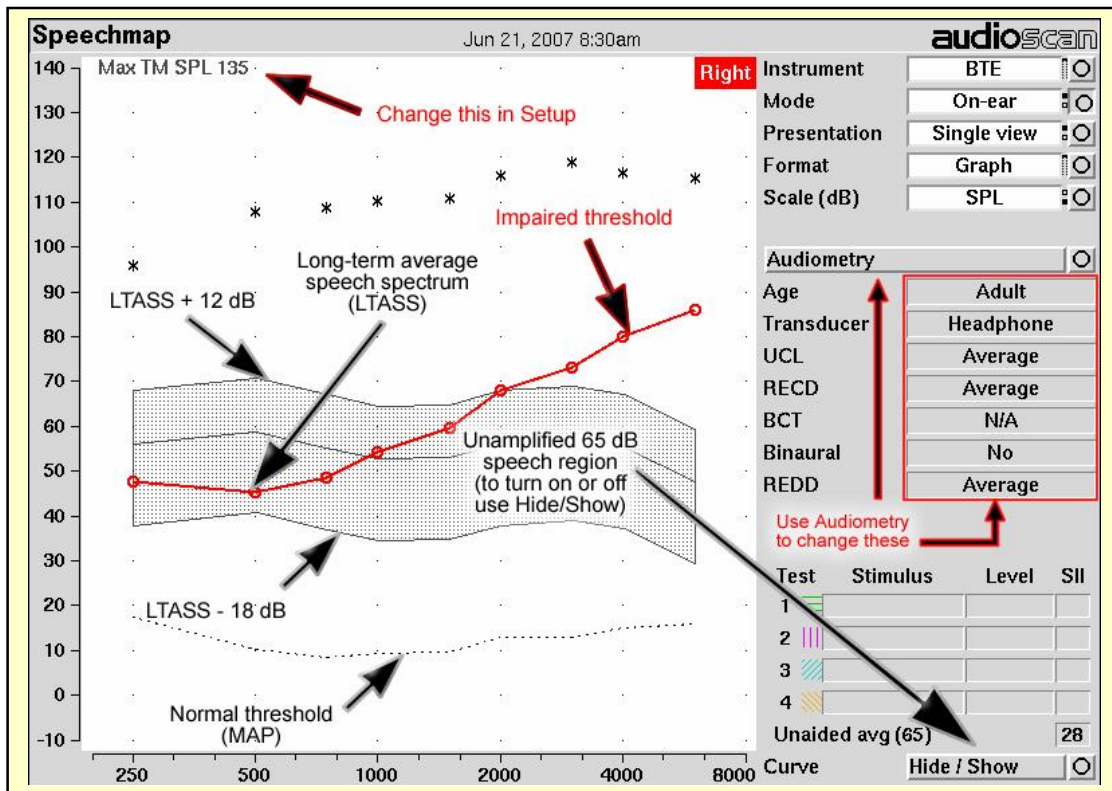

FastFacts 17.8: Screen tour - unaided screen

17.9 Screen tour - aided screen

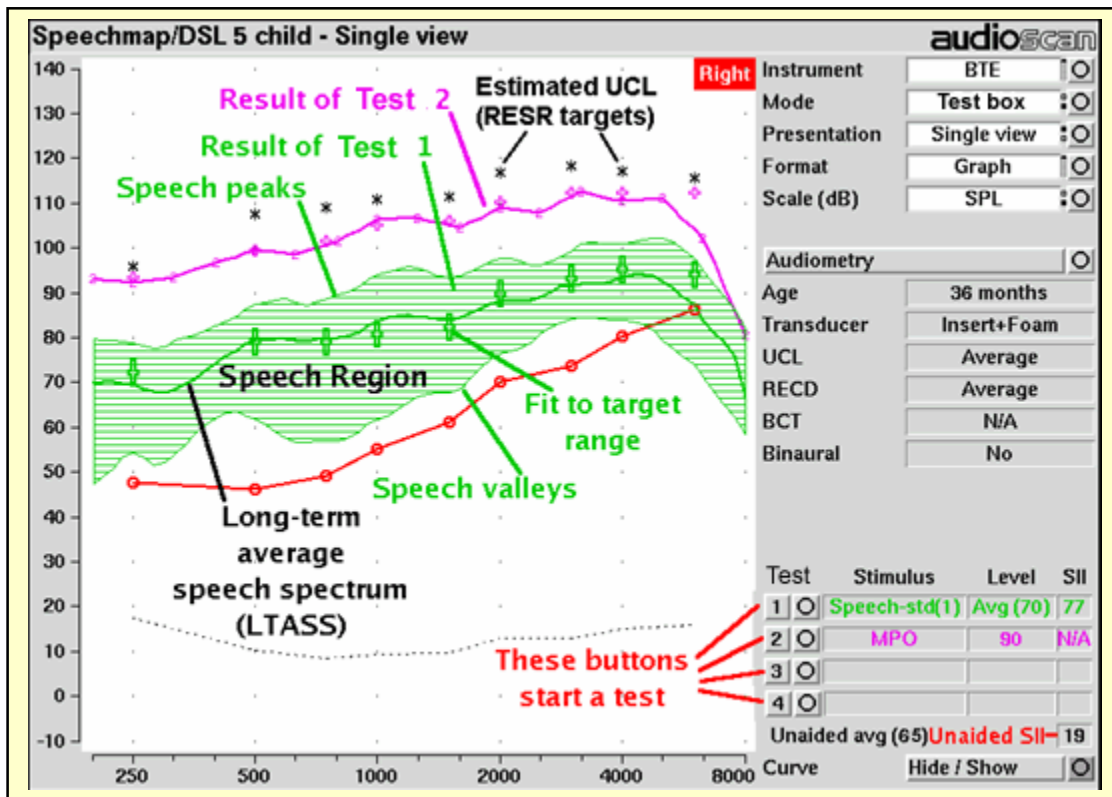

FastFacts 17.9: Screen tour - aided screen

17.10 On-ear or Test box mode

The Speechmap environment provides two hearing instrument verification options: **On-ear measures** and **Test box measures**. Using on-ear mode, the stimulus is controlled by the on-ear reference microphone below the ear and SPL in the ear canal is measured by the probe microphone. In test box mode, levels measured in the 2cc coupler are converted to estimated ear canal levels using the RECD values (age-appropriate average, entered, or measured). In addition, the stimulus delivered to the hearing instrument microphone is modified to include average microphone location effects for the hearing instrument type indicated. The screen display for both modes is the same, showing SPL, measured or simulated, in the ear canal.

For On-ear mode, press <Tests>, then highlight and <PICK> On-ear measures and highlight and <PICK> Speechmap. Review [11: On-Ear Measures - Setup](#).

For Test box mode, press <Tests>, then highlight and <PICK> Test box measures and highlight and <PICK> Speechmap. Review [7.9: Calibrating test box reference microphone](#), [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#).

To switch modes, highlight and <PICK> [Mode].

It is suggested that hearing instruments be preset in test box mode with fine tuning done in on-ear mode.

Note that on-ear mode should be used for vented or open fittings. Vent effects depend on the vent, the ear and the hearing aid settings and cannot be reliably predicted from 2cc coupler measurements.


FastFacts 17.10: On-ear or Test box mode

17.11 SII calculation in Speechmap

The Verifit calculates a Speech Intelligibility Index (SII) value (in %) for unaided speech at 65 dB SPL and for each aided test. The calculations use the 1/3 octave band method of ANSI S3.5 –1997 without the 160 Hz band and without masking effects with results expressed as a % by multiplying by 100. Level distortion effects are included but no hearing loss desensitization is applied. An idealized speech region of LTASS +/- 15 dB is used.

The speech recognition associated with a given SII is a function of the test material and the cognitive abilities of the listener. There is considerable individual variability in relating the SII to speech recognition, especially amongst impaired listeners. The following shows expected nominal recognition scores vs SII for normals on the Connected Speech Test (Sherbecoe and Studebaker 2003).

SII (%)	Score (%)
10	1
15	8
20	20
25	45
30	60
35	74
40	84
45	90
50	94
55	96
60	98
70	99
80	100
90	100

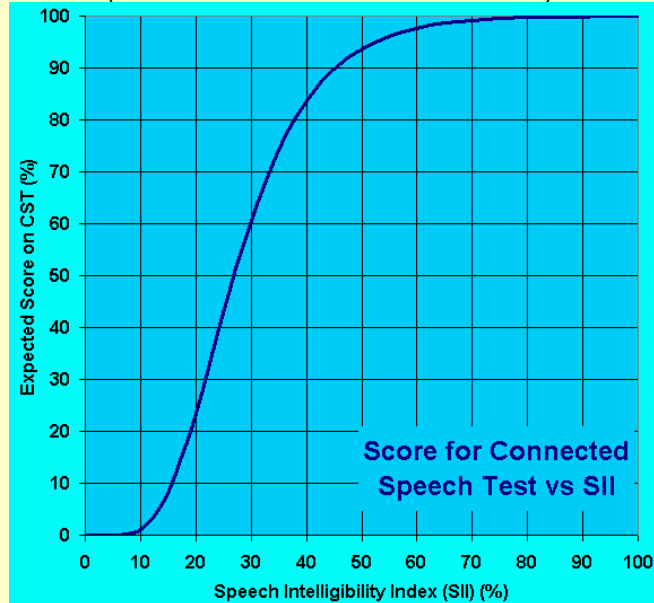

FastFacts 17.11: SII calculation in Speechmap

18 Speechmap Fitting Procedures

18.1 Section overview

The procedures in this section assume a familiarity with [7: Test Box Measures - Setup](#), [11: On-Ear Measures - Setup](#), [12: On-Ear Measures Screen Setup](#) and [17: Speechmap®](#).

This section covers the following topics:

- | | |
|--|---|
| 18.2: Assessment data entry | 18.7: Fitting to targets for mid-level speech |
| 18.3: Assessment data choices - 1 | 18.8: Open fittings in Speechmap |
| 18.4: Assessment data choices – 2 | 18.9: FM fitting and verification |
| 18.5: Fitting to targets for soft speech | 18.10: CROS fitting in Speechmap |
| 18.6: Fitting to targets for loud sounds | |

Related sections are [17: Speechmap®](#) and [19: Speechmap Technical Details](#)

FastFacts 18.1: Speechmap fitting procedures

18.2 Assessment data entry

Threshold data and parameters may be entered by scanning a barcode on a previous Verifit or RM500SL printout or manually using a mouse, the keypad or the numeric keypad on an external keyboard. See [2.7: Mouse, keyboard, barcode scanner](#), and [3.2: Input device operation](#).

To enter data from a barcode see [3.3: Barcode data input](#) and:

- 1) Press the barcode trigger and center the red line on the barcode from a distance of 6 - 8". The scanner will beep and the Barcode Entry poster opens.
- 2) Highlight and <PICK> [Done] to accept the data indicated on the poster.

To enter assessment data manually:

- 1) Highlight and <PICK> [Audiometry].
- 2) Highlight and <PICK> an item to toggle its setting or generate a list box. Age can be changed only if Targets = [DSL child]. See [18.3: Assessment data choices - 1](#) and [18.4: Assessment data choices – 2](#)
- 3) Press <Continue> to proceed to data entry.
- 4) Enter HL threshold using a mouse, keypad or keyboard. Left click the mouse on the audiogram to enter a point. Click again to delete the point. On the keypad the arrow keys change frequency and level and <PICK> enters or deletes a threshold point. On a keyboard enter HL values on the numeric keypad.
- 5) Press <Continue> when all points have been entered.
- 6) Repeat the process for Bone conduction, UCL and RECD if [Enter] has been selected for any. If RECD is set to [Measure], see [15: RECD \(real-ear to coupler difference\)](#).

Audiometry	
Ear	Right
Targets	NAL-NL1
Age	Adult
Transducer	Headphone
Threshold	Entered
Bone conduction	N/A
UCL	Average
RECD	Average
Binaural	Yes
REDD	Average
Cancel [X] Continue [X]	

FastFacts 18.2: Assessment data entry

18.3 Assessment data choices - 1

The following audiometric assessment parameters can be specified in the Speechmap Audiometry setup box. Defaults are the last-used choices if test setup memory has been turned on. See [4.4: Saving test setup](#).

- 1) **Target:** Select fitting targets from [DSL adult, DSL child, NAL-NL1, Camfit restoration, Camfit equalization, None]. See [17.3: DSL 5.0 in Speechmap](#), [17.5: NAL-NL1 in Speechmap](#), [17.6: Camfit in Speechmap](#).
- 2) **Age:** This is permitted for [DSL child] target only. Select client's age in months from [1 months to 60 months] or years from [6 years to 10 years]. For children older than 10 yrs, previously fit using DSL, choose Adult.
- 3) **Transducer:** Type of audiogram. Select from [Headphone, Insert + Foam, Insert + Mold, Soundfield, ABR (eHL), ABR (nHL)]. **If Insert + Foam or Mold** is selected, the appropriate RECD will be used to convert entered HL values to SPL values. **If Soundfield** is selected appropriate real-ear unaided gain (REUG) values @ 45 degrees azimuth, will be used to convert HL values to SPL values. **If ABR (eHL)** is selected, entered thresholds will be treated as behavioral pure tone thresholds. **If ABR (nHL)** is selected, nHL to eHL conversion factors may be selected from [Default, Enter]. In either case, an insert earphone is assumed for sound delivery and the appropriate RECD will be applied. Note that, in some facilities, nHL is defined in such a way that it is effectively eHL; in these cases, ABR (eHL) should be selected. See [11.7: ABR nHL to eHL setup](#) and [18.4: Assessment data choices – 2](#).
- 4) **Bone conduction:** Select from [N/A, Enter]. If [Enter] is selected, an entry screen will be provided after the air conduction thresholds have been entered.

Continued in: [18.4 Assessment data choices – 2](#)

FastFacts 18.3: Assessment data choices - 1

18.4 Assessment data choices – 2

Continued from: [18.3 Assessment data choices - 1](#)


- 5) **UCL:** Uncomfortable Level. Select from [Enter, Average]. If measured UCL values are entered, these values will be converted to real-ear SPL and used as real-ear saturation response (RESR) targets. Note that DSL 5.0 has changed this term to Upper Limit of Comfort (ULC). Regardless of the acronym, it is a narrow-band SPL that should never be exceeded.
- 6) **RECD:** Select from [Measure, Enter, Average]. If Average is selected, age-appropriate real-ear to coupler difference (RECD) values will be used. If Measure is selected, you will be required to measure the client's RECD values or accept values from the RECD test. If Enter is selected, **HA-2** RECD values may be entered and are required **at all 9 audiometric frequencies**.
- 7) **Binaural:** Select from [Yes, No]. Determines if fitting targets will be adjusted for binaural summation effects.
- 8) **REDD:** Real-ear-to-Dial Difference. [Average] (this cannot be changed). REDD values are similar for adults and children. If Headphone is selected as the transducer type, adult average values will be used to transform dB HL audiometry to dB SPL.
- 9) **ABR nHL to eHL:** Normalized HL to estimated HL conversion factors. Permitted only if [Transducer] = ABR (nHL). Select from [Default, Enter]. If [Enter] is selected, an entry screen will be provided after threshold data have been entered. Otherwise default factors will be used. These may be set to DSL values or to values that you have entered in Setup. Note that, in some facilities, nHL is defined in such a way that it is effectively eHL; in these cases, ABR (eHL) should be selected and no conversion factors will be applied. See [11.7: ABR nHL to eHL setup](#) and [18.3: Assessment data choices - 1](#).

FastFacts 18.4: Assessment data choices - 2

18.5 Fitting to targets for soft speech

See [17.10: On-ear or Test box mode](#), [17.7: Using Speechmap](#) and [18.2: Assessment data entry](#) for setup and audiometric data entry.

- 1) Select [Audiometry] to choose the target method and enter audiometric data.
- 2) Select Test 1 and set stimulus to [Speech-std(1)] or [Speech-std(2)] and level to 50. Elongated + symbols show the target range for 50 dB speech LTASS.
- 3) Adjust frequency shaping and gain for soft sounds so that the middle curve (the LTASS) falls within the target range, especially between 500 - 4000 Hz. Press <Continue> to run the complete passage. Repeat as necessary.

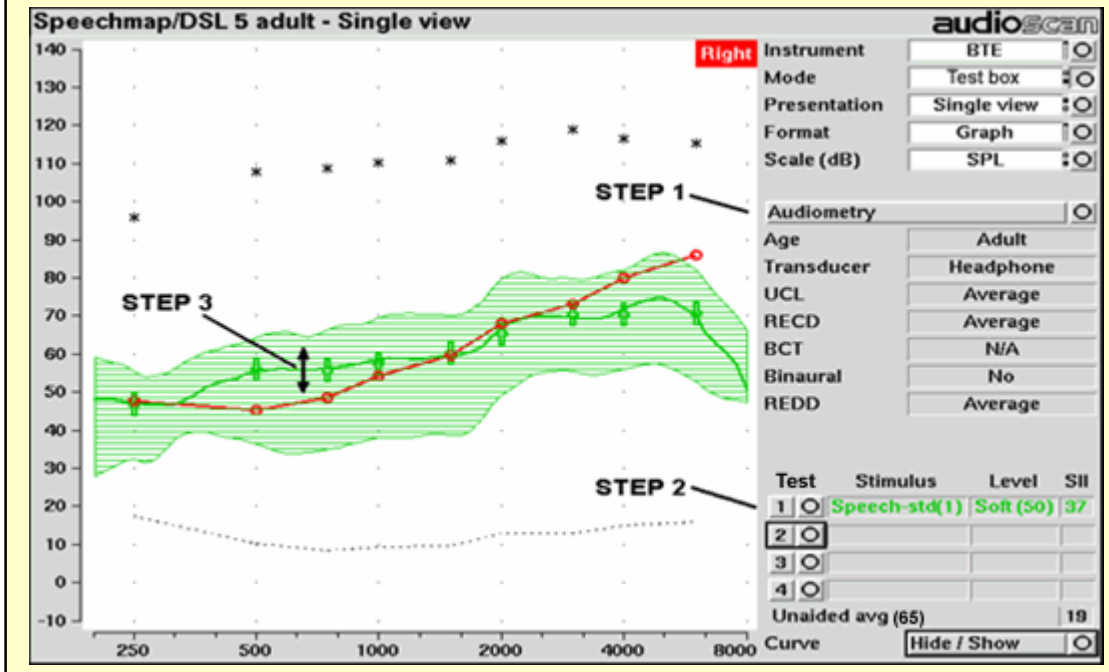


FastFacts 18.5: Fitting to targets for soft speech

18.6 Fitting to targets for loud sounds

- 4) Select Test 2 & set stimulus to [MPO]. In **test box mode**, a 90 dB tone sweeps continuously. In **on-ear mode**, the tone sweeps when you press <Continue>.
- 5) In **test box mode**, adjust high-level gain and/or output limiting (e.g., MPO, PC) so the curve matches the + REAR90 targets (DSL) or approaches but doesn't exceed the * RESR targets (NAL-NL1) at all frequencies. Press <Continue> to record the MPO curve. In **on-ear mode**, advise the client to point to the screen if a sound becomes too loud during the test (see [11.6: Max TM SPL setup](#)). Press <Continue> to run and record a single sweep. Adjust the gain and output for loud sounds as for test box measures, factoring in client response. Repeat as necessary.


When you start a new test, the speech envelope of the previous test is automatically hidden. To redisplay, highlight and <PICK> Hide/Show, then highlight and <PICK> the envelope display (right column) button for the test.

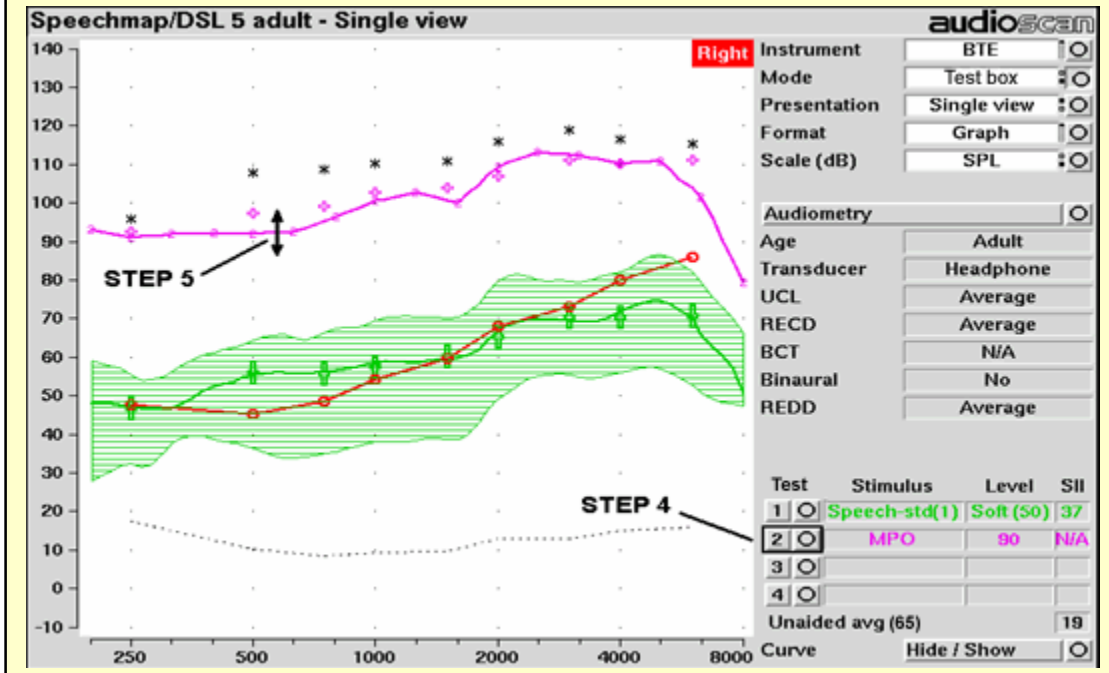

FastFacts 18.6: Fitting to targets for loud sounds

18.7 Fitting to targets for mid-level speech

- 6) Select Test 3 and set stimulus to [Speech-std(1)] or [Speech-std(2)] and level to 70. Elongated + symbols show the target range for 70 dB speech LTASS.
- 7) Adjust gain for mid-level sounds so that the middle curve (the LTASS) falls within the target range. Press <Continue> to run the complete passage. Repeat as necessary. Achieving this goal may require adjusting gain for soft speech, which will reduce its audibility. This may be a necessary compromise with some hearing instruments and for clients with a very narrow dynamic range.

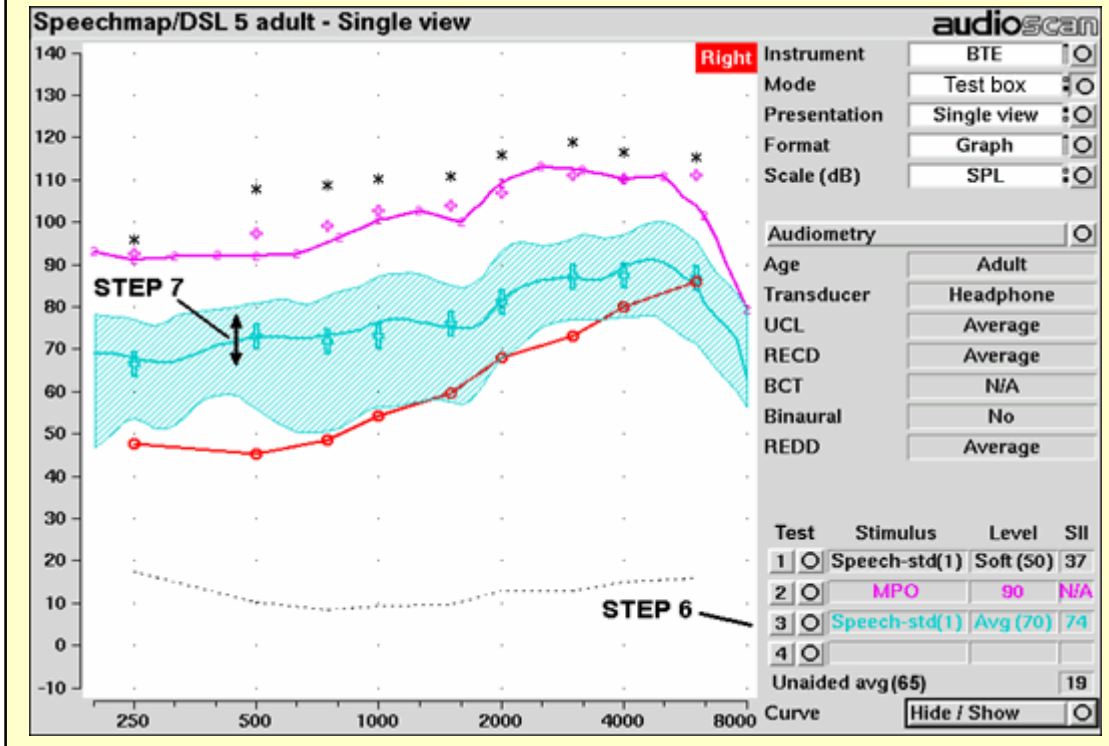

FastFacts 18.7: Fitting to targets for mid-level speech

18.8 Open fittings in Speechmap

On-ear mode should be used for open and vented fittings. There are no special real-ear SPL targets for open fittings - it doesn't matter how the sound reaches the TM, the perception is the same. However, the sound that escapes from a vent or open fitting may be detected by the on-ear reference microphone, interfering with the sound-field equalization. For this reason, the hearing instrument must be muted during sound-field equalization.

- 1) Position the client and probe tube and enter audiometric data. See [11: On-Ear Measures - Setup](#), [17: Speechmap®](#), [18: Speechmap Fitting Procedures](#).
- 2) Highlight and <PICK> Instrument and select Open from the list.
- 3) Highlight and <PICK> Test 1 - 4. If this is the first Open test in a session, you will be prompted to mute or turn off the hearing instrument and select the Equalize button on the setup poster. Otherwise, you are reminded to equalize whenever the client or nearby objects are moved. After equalization, unmute or turn the instrument on and follow the fitting procedures in [18.5: Fitting to targets for soft speech](#) to [18.7: Fitting to targets for mid-level speech](#).

FastFacts 18.8: Open fittings in Speechmap

18.9 FM fitting and verification

FM Offset or FM Transparency method in Speechmap (after Platz 2004)

It is assumed that the hearing aid has been set up for optimal audibility and maximum output. See other sections of [18: Speechmap Fitting Procedures](#).

- 1) Before attaching the FM receiver, couple the hearing aid to the 2cc coupler in the test box and select Speechmap from the Test box measures menu. Select the appropriate hearing Instrument type (typically BTE).
- 2) Highlight and <Pick> Test 1 and choose Speech-std(1) or (2) at a Level of 65 dB. Press <Continue> to run and record the speech passage.
- 3) Attach the FM receiver to the hearing aid and set the FM receiver to the default setting. The FM transmitter should be ON and set to MUTE. Repeat step 2 but use Test 2 instead of Test 1. If the results of Test 1 & Test 2 are not similar, the presence of the FM receiver has changed the intended performance of the hearing aid. Where possible, make programming adjustments to compensate - being careful not to make adjustments that will change Test 1 results.
- 4) Move the hearing aid, attached to the coupler and microphone, outside the test box at least 12" away from the FM transmitter*. Put FM transmitter microphone, set to Omni, within 2mm of the reference microphone in the test box.
- 5) With the FM transmitter unmuted, repeat step 2. **Note that the hearing aid is outside the test box during this test - Quiet please!** The results should be within +/- 2 dB of the results from step 3. If not, change FM settings and repeat.

* A coupler mic. extension cable is available. See [2.4:Microphone connection](#).

Real-ear verification of FM-only open fittings in Speechmap (based on draft AAA Clinical Practice Guidelines – 2007)

Open fittings must be verified in On-ear mode.


- 1) Enter Audiometry data and select FM from the Instrument list box.
- 2) Position the transmitter microphone as in step 4 above. Place the probe tube in the ear canal and couple the FM-only device to the ear canal as in normal use.
- 3) At maximum volume setting, select the MPO stimulus. Adjust the receiver settings to meet targets. See [18.6: Fitting to targets for loud sounds](#).
- 4) At use volume setting, select Speech-std (1) or (2) and choose FM chest (84 dB SPL) or FM boom (93 dB SPL) for the level. Adjust the FM receiver gain/volume settings to meet targets in the 1000 – 4000 Hz range.
- 5) Repeat step 3 at use volume setting.

FastFacts 18.9: FM fitting and verification

18.10 CROS fitting in Speechmap

See [17.7: Using Speechmap](#).

- 1) Place a probe module on each ear and insert the probe tube into the good ear only. Position the CROS instruments on (in) the ears and turn them ON.
- 2) Position the speaker at 45° to the good ear and set Instrument to [BTE].
- 3) Select Test 1 and record a response using [Pink noise] at Soft (55). It should be like a normal REUR. If not, the ear may not be fully open.
- 4) Position the speaker at 45° to the poor ear.
- 5) Set Instrument to [CROS].
- 6) Select Test 2 and obtain a response using [Pink noise] at Soft (55). Adjust the response to match that obtained in step 3.
- 7) Position the speaker directly in front of the client. Select Test 3 and use [Pink noise] at Soft (55) to check for a smooth response. Repeat with Instrument set to [BTE]. An irregular response may be the result of reflections from nearby objects (including the operator) or it may indicate phasing problems in the aids.

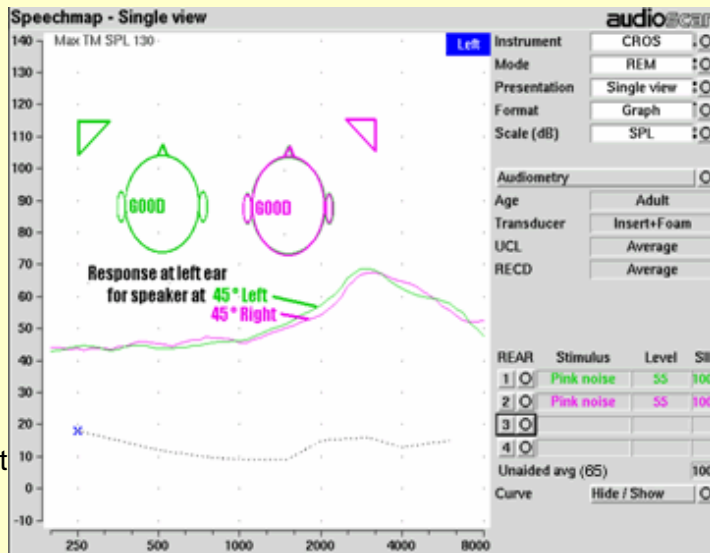

FastFacts 18.10: CROS fitting in Speechmap

19 Speechmap Technical Details

19.1 Section overview

Speechmap provides a variety of stimuli for the testing of non-linear hearing aids with a range of digital processing features.

This section covers the following topics:

[19.2: Speechmap stimuli](#)

[19.4: Microphone location effects](#)

[19.3: Stimulus levels](#)

[19.5: Speech signal analysis](#)

For additional details, see [26: Appendix 2: AudioNote 2.1 VERIFIT and RM500SL Test Signals and Analysis](#)

FastFacts 19.1: Speechmap technical details

19.2 Speechmap stimuli

The Speechmap fitting environment provides a number of well-controlled, distinctly different stimuli. These include three varieties of speech - standard (previously “shaped”), female, child, ICRA noise (digitally distorted speech) and pink noise at levels of 50 to 75 dB SPL in 5 dB steps and a tone burst at 85 or 90 dB SPL.

The two standard speech signals are by the same male talker. The spectrum has been filtered to closely match the Cox & Moore (1988) long-term average speech spectrum (LTASS). The ICRA noise, female and child speech signals are presented as recorded with no spectral shaping. Live speech may also be used.

As a result of reanalysis of published data, the previous distinction between “soft” and “average” vocal effort has been eliminated. Consequently, the speech and ICRA signals are presented unfiltered for levels from 50 to 70 dB SPL. At 75 dB SPL, the spectrum is filtered to represent a “loud” vocal effort. An “own voice” filter is also available at the 75 dB SPL level. This is intended to produce a spectrum and level at a BTE microphone location similar to that produced by the voice of an average wearer.

A tone burst (MPO) stimulus provides 128 (changed from 64) ms bursts at 1/3 octaves with a level of 90 dB SPL in the test box and 85 dB SPL for on-ear mode.

In test box mode, all signals are filtered to include BTE, ITE, ITC, CIC, Body microphone location effects. The selected overall rms level is established prior to filtering.

For fitting FM systems, the stimuli may be modified to represent the level (84 dB SPL) and spectrum at a chest microphone location and the level (93 dB SPL) and spectrum at a boom microphone location. These are presented in the test box in both on-ear and test box modes.

The overall SPL, 1/3rd octave band SPL, vocal effort and microphone location effects are given in the following tables for the various broad-band signals available.

FastFacts 19.2: Speechmap stimuli

19.3 Stimulus levels

1/3 octave band SPL						Add to band SPL for			
Hz	Speech			ICRA Noise	Pink Noise	Loud	Own Voice	FM Chest	FM Boom
	Standard	Female	Child						
200	56.8	54.9	32.3	52.9	53	-2.2	.5	9.0	19.2
250	56.0	53.1	51.5	55.9	53	3.3	10.7	12.0	22.2
315	53.0	48.4	57.3	56.9	53	9.6	11.0	16.6	26.0
400	57.0	55.8	54.1	57.5	53	6.1	10.0	13.3	22.8
500	58.0	57.3	53.2	57.8	53	7.7	10.1	13.2	25.3
630	55.0	58.1	61.2	54.3	53	12.4	10.5	21.0	29.1
800	52.5	54.6	56.5	52.8	53	14.4	10.8	26.4	32.1
1000	51.0	52.0	52.7	50.4	53	15.0	10.5	26.4	32.7
1250	50.5	51.0	45.4	51.3	53	14.8	5.5	22.0	33.6
1600	48.0	51.1	41.1	48.5	53	15.5	4.0	22.1	33.9
2000	45.0	51.2	39.7	47.0	53	16.2	6.0	21.0	33.0
2500	44.0	49.3	35.5	44.7	53	14.8	6.0	18.2	31.9
3150	42.5	46.0	28.8	43.3	53	14.5	2.0	14.4	30.4
4000	42.0	40.0	31.2	41.1	53	13.2	-0.5	11.8	29.0
5000	40.0	36.1	30.6	39.6	53	9.2	-1.0	12.4	27.8
6300	41.5	43.5	32.6	33.5	53	4.2	0.0	11.4	26.2
8000	41.5	39.9	35.2	28.2	53	2.7	-4.0	11.0	25.8
Overall	65.0	65.0	65.0	65.0	65	75.0	75.0	84.0	93.0

LTASS for standard speech is from Cox & Moore (1988). Loud spectrum is from ANSI S3.5-1997. Own voice spectrum is from Cornelisse et al (1991) adjusted to a 1 m reference.

FastFacts 19.3: Stimulus levels

19.4 Microphone location effects

Microphone Location Effects					
Add to all stimuli in test box mode					
Hearing instrument type					
Frequency	BTE	ITE	ITC	CIC	BODY
200	0.5	0.3	0.0	0.9	3.0
250	0.5	0.5	0.3	0.8	3.0
315	0.8	0.8	0.3	0.7	3.0
400	1.1	1.0	0.7	0.5	3.0
500	1.2	1.8	0.0	0.4	4.0
630	1.1	2.0	0.1	0.4	3.0
800	0.9	2.0	0.4	0.4	2.0
1000	0.3	1.5	1.2	0.6	0.0
1250	0.6	0.3	-1.6	1.1	0.0
1600	2.5	-0.3	-1.9	2.0	-4.0
2000	4.1	3.8	2.1	3.3	-4.0
2500	3.5	5.0	4.8	5.0	-3.0
3150	2.8	3.3	3.5	6.9	-2.0
4000	3.7	4.3	6.4	8.3	0.0
5000	-1.2	4.3	6.6	7.6	0.0
6300	1.6	-0.4	-1.8	4.2	0.0
8000	3.3	1.0	-1.9	-5.6	0.0

Microphone location effects are added after the overall rms level has been set

FastFacts 19.4: Microphone location effects

19.5 Speech signal analysis

One of the most-used measures of a speech signal is the long-term average speech spectrum (LTASS). This is a 1/3 octave spectrum averaged over a sufficiently long portion of the speech material to provide a stable curve. In practice a 10 second average meets this requirement and, for this reason, all Verifit passages are at least 10 seconds long.

The dynamic nature of speech is often characterized by the distribution of short-term levels in each 1/3 octave band. These levels are determined by calculating a spectrum for each of a series of short time periods within the passage. Historically, time periods of 120, 125 or 128 ms have been used. The Verifit uses a 128 ms time period, resulting in 100 levels (or samples) in each 1/3 octave band for a 12.8 second passage. The level in each band that is exceeded by 1% of the samples (called L1, the 1st or 99th percentile) has historically been referred to as the speech peak for that band. The curve for these 1% levels is approximately 12 dB above the LTASS. The level in each band that is exceeded by 70% of the samples (called L70, the 70th or 30th percentile) has historically been called the valley of speech for that band. The curve for these 70% levels is approximately 18 dB below the LTASS. The region between these two curves is often called the speech region, speech envelope or speech “banana”. The speech envelope, when derived in this way, has significance in terms of both speech detection and speech understanding. Generally, speech will be detectable if the 1 % level is at or near threshold. The Speech Intelligibility Index (SII) is maximized when the entire speech envelope (idealized as a 30 dB range) is above (masked) threshold. This will not be an SII of 100% (or 1) because of loudness distortion factors, but higher SII values will not produce significantly higher scores on most test material. The speech-reception threshold (SRT) is attained when the LTASS is at threshold (approximately - depending on test material and the individual)


FastFacts 19.5: Speech signal analysis

20 Viewport

20.1 Section overview

Viewport combines four Verifit Test box or On-ear tests into a single, comprehensive diagnostic and fitting suite. The four tests are Speechmap, Directional, Noise Reduction, and Feedback. The tests in Viewport have been optimized and preset to make the fitting/testing process as efficient as possible. The stand-alone tests offer more flexibility should it be required. The preset tests may be customized to suite practice preferences (see [20.2: Viewport setup](#)).

The Audibility test is a simplified version of Speechmap. For more details, see [17: Speechmap](#)®, [18: Speechmap Fitting Procedures](#)

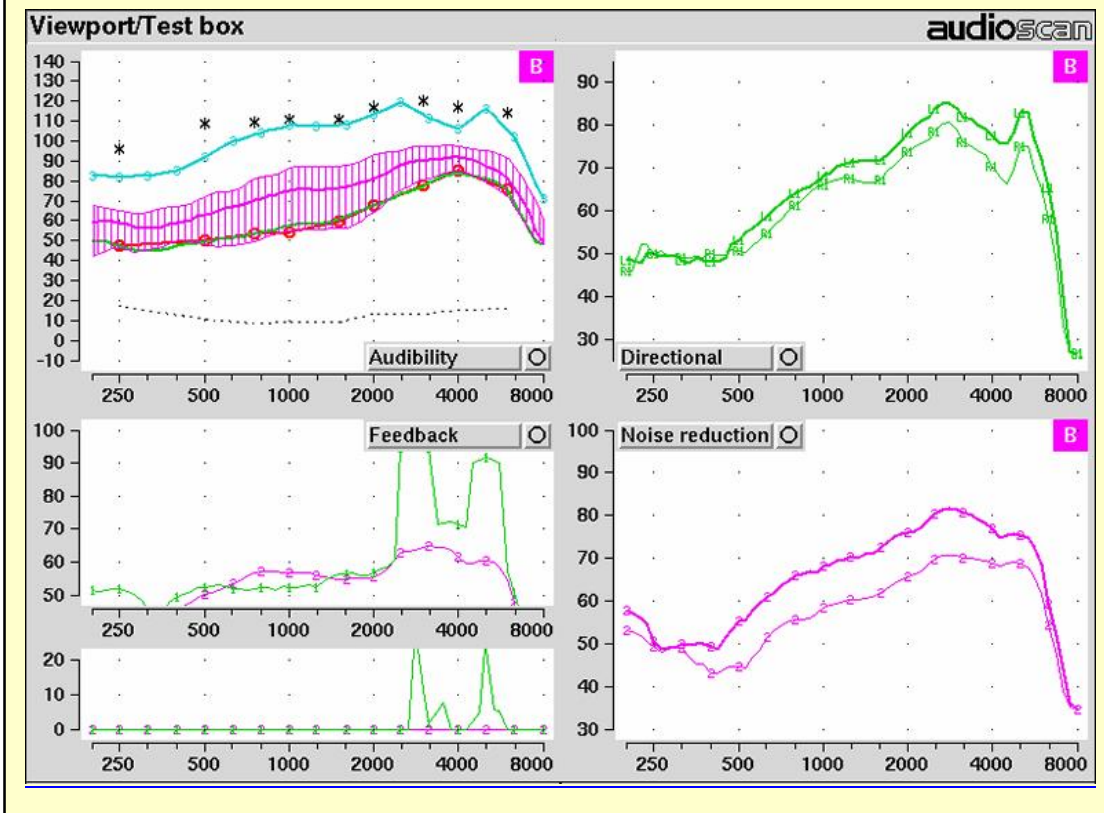

FastFacts 20.1: Viewport

20.2 Viewport setup

The stimulus type and level used in each of the four tests that make up the Viewport suite can be preset. These settings will be remembered until they are changed or reset to default values.

To change Viewport settings:

- 1) Press <Setup>, then highlight and <PICK> [Viewport].
- 2) Highlight and <PICK> the Stimulus, Level or SNR list buttons and then highlight and <PICK> one of the items in the list box.
- 3) Highlight and <PICK> [Reset to defaults] to restore factory settings.
- 4) Press <Continue> to save the changes and exit.

FastFacts 20.2: Viewport setup

20.3 Accessing Viewport tests

- 1) Press <Tests>, then highlight and <PICK> [Viewport].
- 2) Highlight and <PICK> [Test box] to prefit a hearing instrument and verify digital feature operation; highlight and <PICK> [On-ear] to perform on-ear fitting and digital feature verification.

If you have already performed separate Speechmap, Directional, Noise reduction or Feedback tests in Test box or On-ear measures, the results of those tests will appear in the appropriate Viewport quadrants. However only results of Speechmap Tests 1 - 3 will appear in Audibility and only results of Tests 1 and 2 from Directional, Noise reduction and Feedback tests will appear in their respective quadrants.

FastFacts 20.3: Accessing Viewport tests

20.4 Viewport Audibility test

- 1) On the Viewport screen, highlight and <PICK> [Audibility]. Existing results from Speechmap tests 1 – 3 and any threshold and assessment selections (e.g. target, age, transducer etc.) in Speechmap will automatically transfer to the Audibility screen. You may work with these, scan a threshold barcode or enter a new threshold which applies default assessment selections (no fitting targets, adult age, AC headphone, average UCL and RECD).
- 2) To enter a new threshold, highlight and <PICK> [Audiometry]. Enter HL threshold using a mouse or the keypad. Left click the mouse on the audiogram to enter a point. Click again to delete the point. On the keypad the arrow keys change frequency and level and <PICK> enters or deletes a threshold point. Press <Continue> when all points have been entered.
- 3) For Test box measures, position the hearing aid in the test box. See [7.11: Coupling the hearing instrument](#), [7.12: Positioning the hearing instrument](#).
- 4) For On-ear measures, position the client, probe module and probe tube and insert the earmold or hearing aid into the test ear. See [11.8: Positioning the client](#), [11.9: Positioning the probe tube](#).
- 5) Highlight and <PICK> Test 1 to present speech at 50 dB SPL (default). The middle curve should hover about the (red or blue) threshold curve. If it does not, adjust gain (or low-level gain) until it does. Press <Continue> and wait for the test to complete. Repeat as needed to get the middle curve near threshold.
- 6) Highlight and <PICK> Test 2 to present speech at 65 dB SPL (default). The lower curve should hover at or just below the threshold curve. If it does not, adjust high-level gain (or compression ratio or threshold) until it does. Press <Continue> and wait for the test to complete. Repeat as needed. **Running a new test hides the envelope for previous speech tests.** To redisplay, highlight and <PICK> Hide/Show and select the envelope (right column) button.
- 7) Highlight and <PICK> Test 3 to present tones at 90 (Test box) or 85 (On-ear) dB SPL. Press <Continue> (in On-ear mode no tones are presented until you press <Continue>). The curve should be near but below the * markers at all frequencies. If it is not, adjust the maximum output settings until it is.
- 8) Press <Continue> to close [Audibility] and move to [Directional].

FastFacts 20.4: Viewport Audibility test

20.5 Viewport Test box directional test

- 1) On the Viewport Test box screen, highlight and <PICK> [Directional].
- 2) Position the hearing aid in the test box. See [7.11: Coupling the hearing instrument](#), [9.6: Directional ITE positioning](#), [9.7: Directional BTE positioning](#), [9.8: Test box directional procedure](#).
- 3) Highlight and <PICK> Test 1. This will introduce speech at 70 dB SPL (default) from the left speaker and noise at 64dB SPL (default) from the right speaker. Two very active curves will appear on the screen as the speech passage is played. At the end of the speech passage, a brief noise signal is presented from both speakers. The output of the hearing aid is analyzed to produce two response curves. The heavier one is the response to the signal from the left speaker while the lighter curve is the response to the signal from the right speaker.
- 4) If the instrument is omni-directional, the two curves will be nearly identical. If the instrument is directional the curves will separate indicating different amplification for sound coming from different directions.
- 5) If the directional response of the instrument is speech adaptive, the curves will start out being similar but will separate after 2 or 3 repetitions of the speech passage. Press <Continue> to capture the directional curves and end the test.
- 6) Highlight and <PICK> Test 2. This is like Test 1 but without the speech. If the directional response of the instrument is noise-adaptive, the curves will separate after 15 – 45 seconds. A speech-adaptive directional instrument will remain omni-directional and no separation will be observed. Press <Continue> to capture the directional curves and end the test.
- 7) Press <Continue> to close [Directional] and move to [Noise reduction].

Some hearing aids which use frequency shifting to control feedback may show reversed directional characteristics when tested in this way. In such cases, it is necessary to disable feedback suppression before performing this test.

FastFacts 20.5: Viewport Test box directional test

20.6 Viewport On-ear directional test

Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube. Insert the earmold/tip or custom hearing instrument into the ear, being careful not to advance the probe tube further into the ear canal. A lubricant applied in the vicinity of the probe tube will ease insertion and prevent slit leaks.

- 1) On the Viewport On-ear screen, highlight and <PICK> [Directional]..
- 2) Highlight and <PICK> Test 1. This will introduce speech at 70 dB SPL (default) from the front sound-field speaker and noise at 64dB SPL (default) from the back sound-field speaker. Two very active curves will appear on the screen as the speech passage is played. At the end of the speech passage, a brief noise signal is presented from both speakers. The output of the hearing aid is analyzed to produce two response curves. The heavier one is the response to the signal from the front speaker while the lighter curve is the response to the signal from the back speaker.
- 3) If the instrument is omni-directional, and the room is large and acoustically “dead”, the two curves will be nearly identical. If the instrument is directional the curves will separate indicating different amplification for sound coming from different directions.
- 4) If the directional response of the instrument is speech adaptive, the curves will start out being similar but will separate after 2 or 3 repetitions of the speech passage. Press <Continue> to capture the directional curves and end the test.
- 5) Highlight and <PICK> Test 2. This is like Test 1 but without the speech. If the directional response of the instrument is noise-adaptive, the curves will separate after 15 – 45 seconds. A speech-adaptive directional instrument will remain omni-directional and no separation will be observed. Press <Continue> to capture the directional curves and end the test.

- 6) Press <Continue> to close [Directional] and move to [Noise reduction].

Some hearing aids which use frequency shifting to control feedback may show reversed directional characteristics when tested in this way. In such cases, it is necessary to disable feedback suppression before performing this test.

FastFacts 20.6: Viewport On-ear directional test

20.7 Viewport Noise reduction test

- 1) On either the Viewport Test box or On-ear screen, highlight and <PICK> [Noise reduction].
- 2) Leave the setup in the test box or on-ear as used for previous Viewport tests.
- 3) Highlight and <PICK> Test 1. This will present a continuous noise at 70 dB and capture the hearing aid output within the first second as a heavy curve. A real-time output curve (light) will continue to be displayed and will drop as the noise reduction feature of the instrument takes effect. A measure of overall noise reduction is shown (NR). Press <Continue> to capture the curves and end the test.
- 4) Test 2 is the same as Test 1 but with a different type of noise.
- 5) Press <Continue> to close [Noise reduction] and move to [Feedback].

FastFacts 20.7: Viewport Noise reduction test

20.8 Viewport Test box feedback test

- 1) On the Viewport screen, highlight and <PICK> [Feedback].
- 2) For **Test box** tests, connect the monitor headphones and position them, with the hearing aid, in the test box as shown in [9.4: Feedback suppression test](#). Set the volume control on the monitor headset for minimum gain (rotate toward rear of test box) and close the test box.
- 3) Highlight and <PICK> Test 1. With feedback suppression off, force the instrument into feedback by rotating the monitor volume control toward the front of the test box until the feedback bar shows red during parts of the speech passage. Then press <Continue>.
- 4) When the test has finished, peaks should be apparent in one or both curves.
- 5) Without changing the setting of the monitor volume control, enable feedback suppression and highlight and <PICK> Test 2. Allow the speech to run for a few seconds, then press <Continue>.
- 6) When the test has finished, the peaks in the curves of Test 1 should not appear in the curves of Test 2.
- 7) Press <Continue> to close [Feedback] and return to the Viewport screen.

FastFacts 20.8: Viewport Test box feedback test

20.9 Viewport On-ear feedback test

Refer to [11: On-Ear Measures - Setup](#) to prepare the system for use and for proper positioning of the client and the probe tube. Insert the earmold/tip or custom hearing instrument into the ear, being careful not to advance the probe tube further into the ear canal. A lubricant applied in the vicinity of the probe tube will ease insertion and prevent slit leaks.

- 1) Set the instrument as it will be used, with feedback suppression enabled and initialized.
- 2) Highlight and <PICK> Test 1. As the speech passage plays, observe the feedback bar and the lower graph. A yellow Feedback bar and peaks with yellow markers in the lower curve indicate incipient feedback which may impact sound quality. A red Feedback bar and peaks with red markers in the lower curve indicate feedback that may be audible and should be mitigated. Press <Continue> to run the full speech passage and capture average results for the passage.
- 3) Test 2 is the same as Test 1 but uses live speech (default).
- 4) The effects of jaw movements and the presence of hands, hats and telephone handsets may be explored using these tests.
- 5) Press <Continue> to close [Feedback] and return to the Viewport screen.

FastFacts 20.9: Viewport On-ear feedback test

21 Troubleshooting

21.1 Section overview

This section covers common problems encountered when using the Verifit. If you require further assistance, note your model and serial number (located next to the power switch) and contact your local distributor, or Audioscan at 519-268-3313 800-265-2093 (USA only), 519-268-3256 (fax) or service@audioscan.com

PLEASE DO NOT SHIP YOUR UNIT BACK TO THE FACTORY. In most cases we can courier you a part that you can quickly and easily replace yourself.

This section covers the following topics:


- | | |
|--|---------------------------------------|
| 21.2: Self test failures | 21.8: Invalid test box calibration |
| 21.3: Test box high distortion or noise | 21.9: No on-ear ref. mic. detected |
| 21.4: Test box curves inconsistent | 21.10: Invalid on-ear calibration |
| 21.5: Test box curves differ from specifications | |
| 21.6: Test box speaker overdriven | 21.11: Sound-field speaker overdriven |
| 21.7: No test box reference mic. detected | 21.12: Barcode scanner malfunction |

FastFacts 21.1: Troubleshooting

21.2 Self test failures

Any failure during power-on self test (P.O.S.T.) is indicated on-screen and further use of the instrument is denied. Try the following steps to resolve the failure:

- 1) To access Self test, press <Setup>, then highlight and <PICK> [Self test].
- 2) If you received a Max out/in or a Routing failure, remove any hearing aids from the test chamber, unplug the monitor headphones (if used) and rerun self test.
- 3) Check that the cables from the rear display unit to the test chamber and that the test box reference microphone are firmly plugged in. Rerun self test.
- 4) Unplug all microphones (On-ear & Test box) from the test chamber. Rerun self test. If self test passes, reconnect the microphones, one by one, running self test each time until a failure occurs. If the failure returns, display and print the test results and contact Audioscan for help.

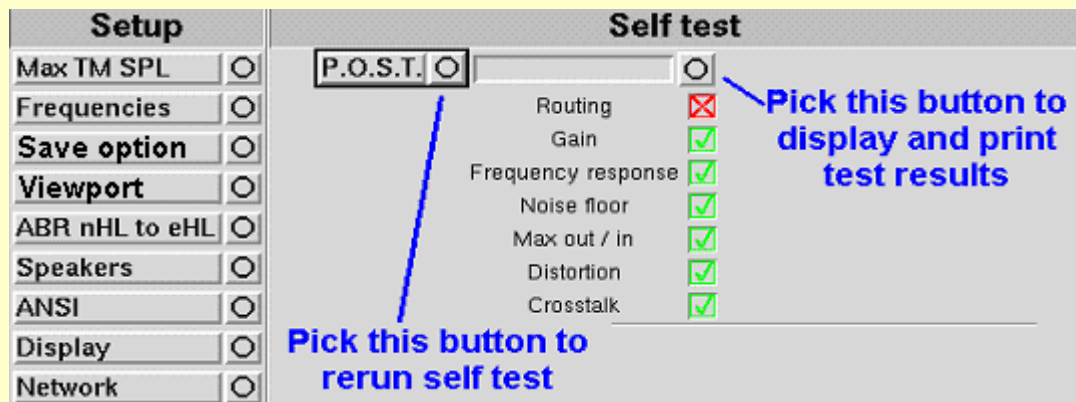

FastFacts 21.2: Self test failures

21.3 Test box high distortion or noise

High distortion and/or noise numbers in an ANSI test

- ✓ There may be a leak between the hearing instrument under test and the coupler it is connected to. For BTE type instruments, check the integrity of the #13 tubing at the end of the coupler where it joins with the ear hook. Look for cracks. ITE type instruments need to have a good seal between the hearing instrument and the coupler. In addition, all vents need to be closed from the faceplate side of the hearing instrument.
- ✓ Ambient room noise may be leaking into the test chamber. Air conditioners, computer fans and similar noisemakers can yield artificially high noise figures. Check the sealing of the test chamber. Try turning off suspected noise sources to identify the cause.
- ✓ Vibration may be coupled into the test chamber from other equipment on the same table or wooden floor. Computers, fans and air conditioners may be the source of the vibration. Try turning off suspected sources to identify the cause. Relocate the source or the Verifit test chamber or mount the test chamber on a foam pad.

FastFacts 21.3: Test box high distortion or noise

21.4 Test box curves inconsistent

The test box curves keep changing from test to test.

- ✓ The battery may be close to end of life or starved for air (zinc-air cells) because of a tight seal on the battery compartment. Try using the Verifit battery pill.
- ✓ The test box reference microphone (the white one in the sound chamber) is incorrectly positioned. In order to control sound pressure levels precisely at the microphone inlet of the hearing instrument, the test box reference microphone must be positioned as close as possible to the hearing instrument's microphone without actually touching it.
- ✓ The noise or feedback reduction features are attempting to reject the test signal. These need to be disabled prior to tests with tones or noise.
- ✓ The instrument is on the verge of feedback due to a poor seal to the coupler, unplugged vent or a crack in the tubing on the BTE coupler.

FastFacts 21.4: Test box curves inconsistent

21.5 Test box curves differ from specifications

The OSPL90 and response curves don't look like the manufacturer's.

- ✓ Check that all controls or program settings are the same as those used by the manufacturer.
- ✓ Check that vents on earmolds or ITE instruments are plugged at the faceplate.
- ✓ Check that ITE instruments are **well sealed** to the coupler.
- ✓ When putting a hearing instrument into the ITE coupler, be sure to keep the canal portion of the instrument flush with the inside of the 2cc volume.
- ✓ Ensure that the test box is sealed tightly.
- ✓ Ensure that the test box microphone calibration was performed correctly.
- ✓ Check that the coupler is screwed tightly onto the coupler microphone.

FastFacts 21.5: Test box curves differ from specifications

21.6 Test box speaker overdriven

Test box speaker overdriven! or Test box right/left speaker overdriven!

The first message indicates that the left test chamber speaker is unable to produce the SPL required for a test. The second message will appear during the directional test if the right/left speaker is unable to produce the required SPL.

- ✓ Check that the two cables running between the test chamber and the main unit are securely connected.
- ✓ Interchange the cables connecting the test chamber to the rear display unit.

FastFacts 21.6: Test box speaker overdriven

21.7 No test box reference mic. detected

No test box reference mic. detected!

This message may be displayed when starting a hearing instrument test. It indicates that the Verifit is not measuring any sound at the test box reference microphone. The test box reference microphone is the white microphone located in the test chamber.

- ✓ Ensure that the test box reference microphone is secure. Unplug the white test box reference microphone. Plug the microphone back in again, ensuring that it 'clicks' into place. Try the calibration procedure again.
- ✓ Interchange the cables connecting the test chamber to the rear display unit.
- ✓ Check the two 15 pin connectors on the rear of the test chamber. Look carefully at the pins using an otoscope to make sure they are all the same length and are not bent. If any have been pushed back into the connector, you should contact Audioscan.

FastFacts 21.7: No test box reference mic. detected

21.8 Invalid test box calibration

Invalid test box Calibration!

This message may be displayed during the test box calibration procedure. It indicates that the calibration curve is significantly different from what is expected. It may be accompanied by a missing test box reference microphone error.

- ✓ Check that there is no coupler on the coupler microphone and that the microphones are correctly positioned for test box calibration.
- ✓ Check that the coupler microphone is securely plugged into the correct socket.
- ✓ Follow the suggestions in [21.7: No test box reference mic. detected.](#)

FastFacts 21.8: Invalid test box calibration

21.9 No on-ear ref. mic. detected

No (right or left) on-ear reference mic. detected.

Check connection on (right or left) probe microphone.

- ✓ Check that a probe microphone assembly is securely plugged into the PROBE socket of the side (Right or Left) you are trying to calibrate.
- ✓ Try switching right and left probe microphone assemblies.
- ✓ Try switching the right and left cables connecting the rear display unit to the test chamber. Carefully observe the pins in the test chamber connector for any that are bent or pushed in.

FastFacts 21.9: No on-ear ref. mic. detected

21.10 Invalid on-ear calibration

Invalid On-Ear Calibration!

This message may be displayed during the on-ear calibration procedure. It indicates that the calibration curve is significantly different from what is expected. It may be accompanied by an on-ear reference microphone error.

- ✓ Check that a probe microphone assembly is securely plugged into the PROBE socket of the side (Right or Left) you are trying to calibrate.
- ✓ Try using a new probe tube. Ensure it is correctly attached and positioned.
- ✓ Try switching right and left probe microphone assemblies.
- ✓ Try switching the right and left cables connecting the rear display unit to the test chamber. Carefully observe the pins in the test chamber connector for any that are bent or pushed in.

FastFacts 21.10: Invalid on-ear calibration

21.11 Sound-field speaker overdriven

Sound-field speaker overdriven! or External sound-field speaker overdriven!

This message may occur in or-ear directional or on-ear unaided or aided tests if:

- ✓ No external front speaker is connected but one has been selected in Setup. See [2.8: External display, printer, speaker](#), [11.2: External sound-field speaker setup](#).
- ✓ No back speaker has been connected but an on-ear directional test is being attempted. See [11.2: External sound-field speaker setup](#).
- ✓ The test environment is too reverberant for the distance between the client and the speaker(s). Move the on-ear speaker(s), internal or external, and client away from walls and hard objects. Move the client closer to the speaker(s).
- ✓ The speaker or connecting cable (external speakers only) is defective. If this is the case, you should not hear any sound coming from the internal or external speaker(s) during any test. Try a different speaker or cable.



FastFacts 21.11: Sound-field speaker overdriven

21.12 Barcode scanner malfunction

If there is no red scanning beam when the trigger is pulled, tug on the cable where it enters the scanner. If it comes loose easily, push it back in until it clicks in place.

If the scanning beam stays on when the trigger is released or if the barcode scanner fails to read known good barcodes, try the following:

- 1) Unplug the scanner from the Verifit, then reconnect it. If the problem persists...
- 2) Print this page on an external printer (see [2.8: External display, printer, speaker](#)) or from the electronic user's guide on the Verifit software CD (see [1.3: Electronic user's guide](#)).
- 3) Scan these two barcodes, starting with the top one. This will reset the scanner to its default settings.

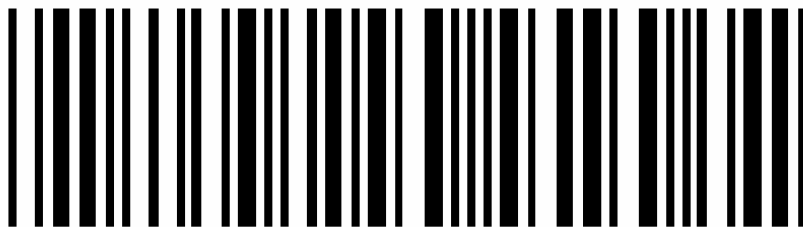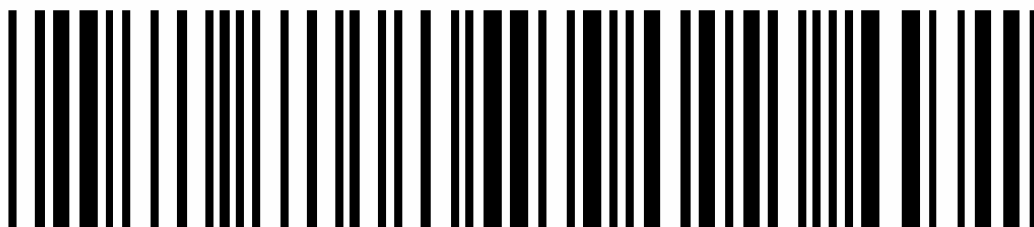

FastFacts 21.12: Barcode scanner malfunction

22 Technical Specifications

Specifications subject to change without notice

Storage & Transportation

Temperature - 20°C to +60°C
Relative humidity (non-condensing).....5% to 95%
Atmospheric pressure..... 500 - 1060 hPa

General

Power source:.....100 – 240 V, 50 – 60 Hz, 250 VA
Fuse: 2A type T, 250 V
Overall dimensions
Rear main unit 14.5x16x6.5 in. (36.8x40.6x16.5 cm)
Test chamber 14x14.4x4 in. (35.6x36.6x10.2 cm)
Weight 17.6 lb (8 kg)
Display type fluorescent backlit active color
Display size 12.1" diagonal
Resolution 800x600 (SVGA)
Printer.....200 dots per inch thermal line printer
Paper width 3" (80mm)
Headphone monitor amplifier 1 watt into 16 ohms
Power amplifiers 2 @ 5 watts each
Stimulus Channels 2
Measurement channels 2
Connectors..... 1 - mouse (PS2-6 pin)
1 - QWERTY keyboard (PS2-6 pin)
1 - external monitor (15D HD)
1 - parallel printer (25D)
2-USB
1 - Ethernet (RJ45)
1 - RS232 serial (9D)
2 - external speaker jacks (RCA)
2 - expansion audio jacks (1/4" mono)
1 - monitor headset (3.5 mm st)
1- RECD transducer(3.5mm st)
2 - real-ear mic.(3.5mm st)
1 - test chamber ref. mic.(3.5mm st)
1 - coupler microphone (3.5mm st)
1 - battery substitute (3.5mm st)

Test Box

Working Space 11x5x1.5 in (28x12.7x3.8 cm)
Speaker 2 - 2x3 in (5x7.5 cm) independent
Test box isolation @ 1kHz >25 dB
Induction Coils per ANSI S3.22-2003
1 - Telephone Magnetic Field Simulator (TMFS)
1 - 23 cm diameter test loop
Battery Simulator per ANSI S3.22 2003
Frequency Range 200 - 8000 Hz
Coupler microphone noise floor (200 – 8000 Hz) <40 dB SPL
Test Stimuli tone, tone burst, pink noise,
dual-direction noise,calibrated or live speech
Test stimulus levels 40 to 90 dB SPL in 5 dB steps
Test stimulus levels (inductive) 31.6mA/m(ANSI S3.22)
Test stimulus distortion..... <2% at 90dB SPL
<0.5% at 70 dB SPL
Test stimulus accuracy at reference mic. for tones
(200 - 2000Hz) ±1.5 dB SPL
(2000-8000 Hz) ±2.5 dB SPL
Equalization method real-time pressure method
Analysis frequencies per octave 12
Analysis filter bandwidth 1/12 octave
Measurement accuracy at 1 kHz ±1db
Measurement accuracy re 1 kHz ±1 dB (200-5000 Hz)
±2.5 dB (5000-8000 Hz)
Measurement range 30 - 140 dB SPL
Harmonic distortion measurement 2nd, 3rd
or 2nd plus 3rd
Harmonic distortion range 200 to 4000 Hz
Harmonic distortion accuracy ±1%
Battery drain range 0 - 20mA
Battery drain accuracy ±5%
Battery drain resolution. ±0.01 mA

ANSI S3.22 - 1996 and 2003 tests available

- a OSPL90
- a Full-on Gain
- a Reference Test Gain
- a Frequency Response
- a Frequency Range
- a Maximum OSPL90
- a Harmonic Distortion
- a Attack & Release time
- a Equivalent Input Noise
- a Input/Output Curves
- a Coupler SPL - Telephone Simulator
- a Simulated Telecoil Sensitivity
- a Battery Drain

Other tests available

- a Speechmap® real-speech audibility measures
- a Real-time adaptive directional verification
- a Coupler SPL vs frequency
- a Coupler gain vs frequency
- a Spectral analysis
- a Noise reduction verification
- a Feedback suppression verification
- a Distortion vs frequency
- a Manual measurement of output, gain and distortion
- a VIEWPORT® audibility, directional, noise reduction and feedback suppression verification

On-Ear

Speakers 2 - 2x 3 in. (5x7.5 cm)
Probe modules .. 2 - each containing probe & ref. microphones
Probe microphone tube Silicone 1 mm diameter x 75 mm
Probe microphone noise floor (200 – 8000 Hz) .. < 45 dB SPL
Frequency Range 200 to 8000 Hz
Test Stimuli..... frequency-modulated tone, tone burst,
pink noise, dual-direction noise, calibrated or live speech
Freq. modulation sawtooth ±3% over 128 ms
Stimulus levels for tones 40 - 85 dB SPL in 5 dB steps
Test stimulus accuracy at reference mic. for tones
(200 - 2000 Hz) ±1.5 dB SPL
(2000- 8000 Hz) ±2.5 dB SPL
Equalization Method real-time modified pressure method
(stored for open fittings)
Frequencies per octave (swept tones) 12
Frequencies per octave (tone burst) 3
Analysis bandwidth (speech, noise) 1/3 octave
Measurement accuracy at 1 kHz ±1 dB
Measurement accuracy re 1 kHz ±1.5 dB (200-5000 Hz)
±2.5 dB (5000-8000Hz)
Measurement Range 20-135 dB SPL (200-2500 Hz)
30-140 dB SPL (2500-8000Hz)

ANSI S3.46 - 1997 tests available

- a Real-Ear Unaided Response
- a Real-Ear Aided Response
- a Real-Ear Occluded Response
- a Real-Ear Insertion Gain

Other tests available

- a Speechmap® real-speech audibility measures
- a Real-time adaptive directional verification
- a On-ear harmonic distortion
- a On-ear spectral analysis
- a On-ear noise reduction verification
- a On-ear feedback suppression verification
- a Manual measurement of output, gain and distortion
- a VIEWPORT® audibility, directional, noise reduction and feedback suppression verification

Fitting methods available

- a Speechmap with DSL5.0a, NAL-NL1, CAMFIT
- a Insertion gain with NAL-RP, NAL-NL1, Fig6, Pogoll, Berger, Libby

Sensory Loss Simulator

Simulation types Linear conductive
Non-linear outer hair cell cochlear loss
Simulation bands 65

23 Glossary

AGC, Automatic Gain Control, the means by which gain is automatically controlled by the level of the signal being amplified. (ANSI S3.22)

AGC-I, input-controlled AGC (see "input compression").

AGC-O, output controlled AGC (see "output compression").

AGRAM, Abbreviation for audiogram.

ANSI, American National Standards Institute.

Attack Time, the time required for HA output to reach +/- 3dB of its steady state level in response to an input step from 55 to 90dB SPL. (ANSI S3.22)

ASP, Automatic Signal Processing, In hearing aids, level dependent frequency response shaping.

AVC, Automatic Volume Control, slow-acting AGC that attempts to maintain a constant average signal level.

BSPL, Band Sound Pressure Level, the SPL within a restricted frequency band.

BTE, Behind-The-Ear (hearing aid).

Compression, a type of AGC in which an incremental change in the input level produces a smaller incremental change in the output level.

CR, Compression Ratio, the ratio of incremental change in input SPL to the resulting incremental change in output SPL. Used to characterize steady state AGC action; can be a function of input level, duration and frequency.

CROS, Contralateral Routing Of Signals; a type of hearing aid in which the microphone is at the ear opposite to the one receiving the amplified sound.

CORFIG, Coupler Response for Flat Insertion Gain; the transformation added to real-ear insertion gain to obtain 2-cc coupler gain. Assumes linear amplification.

Coupler, a device used to acoustically couple an earphone or hearing aid to a measurement microphone and to provide an acoustic load.

DSL, Desired Sensation Level Method, a systematic eardrum (SPLogram) approach to the hearing aid fitting process that seeks to ensure amplified speech will be audible and comfortable while loud sounds will not be uncomfortable.

Directional hearing aid, a hearing aid for which the gain is dependent on the direction of sound incidence when measured under free-field conditions (ANSI S3.22).

Equalization, process of controlling the SPL (or band SPL) of the stimulus, as a function of frequency, at the field reference point (ANSI S3.46).

Expansion, a type of AGC in which an incremental change in the input level produces a larger incremental change in the output level.

FM, frequency modulation, a method of transmitting information in which the frequency of a radio wave (carrier) is varied (modulated) by the information (signal). Also, a method of reducing standing waves in a sound field in which the frequency of a test tone (carrier) is varied by a low frequency triangle or sine wave.

FM System, an assistive listening device consisting of a microphone, an fm radio transmitter and an fm receiver. The microphone and transmitter are worn by the speaker and the receiver is worn by the listener.

FOG, Full On Gain, gain for a 50 dB input SPL with the gain control set to maximum (ANSI S3.22)

Gain (in dB), output sound pressure relative to input sound pressure, expressed in decibels; gain = $20\log_{10}$ (output sound pressure / input sound pressure); also, gain = (output SPL) - (input SPL).

Gain control, a manually or electronically operated control for the adjustment of overall gain (ANSI S3.22).

HA-1 Coupler, a coupler having a volume of 2 cubic centimetres with direct access to the cavity. Used for testing in-the-ear and in-the-canal hearing aids; can also be used to test behind-the-ear hearing aids with a custom earmold attached. Aka the ITE coupler.

HA-2 Coupler, a coupler having a volume of 2 cubic centimetres with access through a rigid tube. When used for testing BTE hearing aids, the rigid tube consists of 18 mm of 3 mm tubing plus 25 mm of 2 mm tubing (ANSI S3.22). When used for calibrating insert earphones, the 2 mm tubing is not used (ANSI S3.6). Aka the BTE coupler.

Harmonic, a component of a tone complex that is an integer multiple of the lowest frequency component (fundamental).

Harmonic Distortion, the addition of harmonic components to a signal. The rms value of a harmonic component of a signal as a percentage of the rms value of the fundamental. If less than 20%, the rms value of the total signal may be used instead of the fundamental.

HFA, High Frequency Average, the average of values in dB at 1000, 1600, and 2500 Hz. (ANSI S3.22).

HL, Hearing Level, the hearing threshold referenced to the threshold of normal hearing, expressed in dB.

HI, Hearing instrument.

Input Compression, a form of AGC in which the signal level is regulated before the volume control.

Input / Output function, steady state, single-frequency plot of the coupler SPL on the ordinate as a function of input SPL on the abscissa with equal decibel scale divisions on each axis (ANSI S3.22).

ITE, In-The-Ear (hearing aid).

Kneepoint (compression threshold), the point on an input/output curve at which the slope digresses from unity indicating the signal level at which a non-linear process begins to take effect.¹

LCD, liquid crystal display. A thin, planar information display.

Linear Region, that portion of the input-output function which is a straight line at 45 degrees.

Linear Amplification, amplification having the same gain for all input levels until the maximum output of the device is reached.

LTASS, Long-term average speech spectrum, the rms level of a speech passage in 1/3rd octave bands, averaged over the entire passage.

Loop System, an assistive listening device that uses magnetic induction to carry a signal from a microphone/amplifier to a wire loop. The signal is picked up from the loop by a hearing aid set to the telecoil position or by a receiver with built-in telecoil, volume control, and earpiece. Loops are either worn around an individual's neck or can encircle a room, such as a classroom, providing an excellent signal-to-noise ratio.¹

MAF, minimum audible field, SPL of a tone at the threshold of audibility measured in a free sound field for a subject listening with both ears and facing the sound source. The SPL is measured with the subject removed from the field at the midpoint of an imaginary line joining the centers of the ear canal openings.

MAP, minimal audible pressure, SPL of a tone, measured or inferred at the tympanic membrane, at the threshold of audibility.

MPO, Maximum Power Output, the maximum SPL that a hearing aid can deliver in response to a steady narrow band input. The stimulus may be pulsed (a tone burst) to avoid discomfort and to obtain a worst-case maximum output.

Modulation, the process of varying a characteristic of one signal with another.

NAL, National Acoustic Laboratories (Australia)

NCA, National Centre for Audiology (Canada)

Nonlinear Region, that portion of the input-output function which is not a straight line at 45 degrees.

OSPL90 (output SPL for 90-dB input SPL), The SPL developed in a 2cc coupler when the input SPL is 90 dB, with the gain control of the hearing aid full-on (ANSI S3.22).

Oscillation, in audiology, refers to the undesired tonal output of an amplifier behaving unstably as a result of feedback; whistling.¹

Output Compression, a form of AGC in which the signal level is regulated at the output.

OLC, Output-limiting Compression (compression limiting), a form of AGC with a high kneepoint and high compression ratio.

PC, Peak-clipping, a method of limiting MPO by truncating the electrical signal supplied to the output transducer. The resulting acoustic signal may not be truncated but will be distorted.

Probe microphone, microphone (which may include an extension tube) having a sound inlet which is small enough to be acoustically unobtrusive when inserted into the ear canal (ANSI S3.46).

Pure tone, a signal containing one, and only one, frequency; a sinusoidal acoustic signal.¹

REAG, Real-Ear Aided Gain, difference, in dB as a function of frequency, between the SPL (or BSPL) at a specified measurement point in the ear canal and the SPL (or BSPL) at the field reference point, for a specified sound field, with the hearing aid in place and turned on (ANSI S3.46).

REAR, Real-Ear Aided Response, the SPL (or BSPL) as a function of frequency, at a specified measurement point in the ear canal for a specified sound field, with the hearing aid in place and turned on (ANSI S3.46).

RECD, Real-Ear-to-Coupler Difference, difference, in dB as a function of frequency, between the SPL (or BSPL) produced by an insert transducer at the eardrum and in a 2cc coupler. HA-1 or HA-2 coupler should be specified.

REDD, Real-Ear-to-Dial Difference, difference, in dB as a function of frequency, between the SPL (or BSPL) measured at the eardrum and the audiometer dial setting that produced it. It is specific to the client, headphone and audiometer used to generate it.

REIG, Real-Ear Insertion Gain, difference, in dB as a function of frequency, between the REAG and the REUG taken with the same measurement point and the same sound field conditions. $REIG = REAG - REUG$. ANSI S3.46.

REOG, Real-Ear Occluded Gain, difference, in dB as a function of frequency, between the SPL (or BSPL) at a specified measurement point in the ear canal and the SPL (or BSPL) at the field reference point, for a specified sound field, with the hearing aid in place and turned off (ANSI S3.46).

REOR, Real-Ear Occluded Response, SPL (or BSPL), as a function of frequency, at a specified measurement point in the ear canal, for a specified sound field, with the hearing aid in place and turned off (ANSI S3.46). The REOR is used to evaluate the seal of an earmold. It is not related to the required gain of a hearing aid.

REUG, Real-Ear Unaided Gain, difference, in dB as a function of frequency, between the SPL (or BSPL) at a specified measurement point in the ear canal and the SPL (or BSPL) at a field reference point, for a specified sound field, with the ear unoccluded (ANSI S3.46).

REUR, Real-Ear Unaided Response, SPL (or BSPL), as a function of frequency, at a specified measurement point in the ear canal, for a specified sound field, with the ear canal unoccluded (ANSI S3.46).

Reference microphone, microphone used to measure the stimulus level in the measurement process or to control it in the equalization process.

Release Time, the time required for HA output to fall to within +/-4dB of its steady-state level for an input step from 90 to 55dB SPL. (ANSI S3.22)

REM, Real-Ear Measurement.

RTG, Reference Test Gain, the HFA gain for a 60 dB input SPL with the gain control at RTS (ANSI S3.22).

Reference, Sound-field, the calibration point of a sound-field measurement.

RSETS, Relative Simulated Equivalent Telephone Sensitivity, difference in dB between the HFA-SPLITS (SPA-SPLITS) and the RTG plus 60 in the 'M' (microphone) mode. $RSETS = HFA-SPLITS - (RTG+60)$

RTS, Reference Test Setting of the gain control, for a 60 dB input SPL, the setting of the gain control required to produce an HFA gain within 1.5 dB of the HFA-OSPL90 minus 17 dB. If the full-on HFA gain is less than the HFA-OSPL90 minus 17 dB, the RTS is the full-on setting of the gain control (ANSI S3.22).

Saturation, in a hearing aid, the limiting of output caused by the output circuitry and transducer reaching their maximum capability.

SPA, Special Purpose Average, the average of values in dB at three third octave frequencies separated by 2/3 of an octave (ANSI S3.22). See also HFA.

Spectrum, the BSPL as a function of frequency for a broad-band signal.

SII, Speech Intelligibility Index, a quantity calculated from measures of speech, noise and hearing threshold that is highly correlated with the intelligibility of speech (ANSI S3.5).

Speech Region, a statistical range of short-term spectra present in a speech passage. The short-term spectra are typically 120 - 130 millisecond averages in 1/3 octaves. The statistical range is typically from the 70th to the 99th percentile.

SPL, Sound Pressure Level, rms sound pressure relative to 20 uPa, expressed in decibels. $SPL = 20\log_{10} (\text{rms sound pressure} / 20\text{uPa})$.

SPLITS, coupler SPL for an inductive telephone simulator, SPL developed in a 2cc coupler by a hearing aid with the gain control at the RTS when the input is the magnetic field generated by a TMFS (ANSI S3.22).

TMFS, Telephone magnetic-field simulator, a device for producing a magnetic field of consistent level and geometric shape when driven by a specified current (ANSI S3.22).

THD, Total Harmonic Distortion, the power in all harmonic components as a percentage of the power in the fundamental. If less than 20%, the power of the total signal may be used instead of the fundamental. For most hearing aids, the inclusion of harmonics above the third will not significantly alter the THD (ANSI S3.22).

Transducer, a device which transforms energy from one form to another. For example, a microphone or earphone.

UCL, Uncomfortable Listening Level, the level for a specified stimulus that is judged to be definitely uncomfortable.

VGA, video graphics adapter.

VC, Volume Control, a user-operated gain control.

Warble Tone, a frequency-modulated sine wave. The modulating signal is often a low frequency triangle wave.

1. Valente, M: *Strategies for Selecting and Verifying Hearing Aid Fittings*. New York: Thieme Medical Publishers, Inc., 1994.

ANSI S3.5, *Methods for Calculation of the Speech Intelligibility Index*, 1997

ANSI S3.6, *Specification for Audiometers*, 1996

ANSI S3.22, *Specification of Hearing Aid Characteristics*, revised 2003.

ANSI S3.46, *Methods of Measurement of Real-Ear Performance Characteristics of Hearing Aids*, 1997.

24 References

AAA Clinical Practice Guidelines: *Remote Microphone Hearing Assistance Technologies for Children and Youth Birth-21 Years*. Supplement A: Fitting and Verification Procedures for Ear-level FM. Draft 2007.

American National Standards Institute. (1997). *American National Standard Methods of Measurement of Real-Ear Performance Characteristics of Hearing Aids. ANSI S3.46-1997*. New York: American National Standards Institute.

American National Standards Institute. (1996). *American National Standard Specification of Hearing Aid Characteristics. ANSI S3.22-1996*. New York: American National Standards Institute.

Bagatto, M., Moodie, S. T., Scollie, S., Seewald, R., Moodie, K.S., Pumford, J. & Liu, R. (2005). *Clinical protocols for hearing instrument fitting in the desired sensation level method*. *Trends in Amplification*, 9-4, 199-226.

Bentler, R.A. & Pavlovic, C.V. (1989). Transfer functions and correction factors used in hearing aid evaluation and research. *Ear and Hearing*, 10(10), 58-63.

Byrne, D. (1977). The speech spectrum - Some aspects of its significance for hearing aid selection and evaluation. *British Journal of Audiology*, 11, 40-46.

Cole, W.A. & Sinclair, S.T. (1998). The Audioscan RM500 Speechmap/DSL fitting system. *Trends in Amplification*, 3(4):125-139.

Corliss, E. L. R. (1990) The ear as a mechanism of communication. *Journal of Audio Engineering Society*, 38(9):640-652.

Cornelisse, L.E., Gagne, J-P & Seewald, R.C. (1991). Ear level recordings of the long-term average spectrum of speech. *Ear and Hearing*, 12(1), 47-54.

Cornelisse, L.E., Gagne, J-P & Seewald, R.C. (1991). Long-term average speech spectrum at chest-level microphone location. *Journal of Speech, Language, Pathology and Audiology*, 15(3):7-12.

Cornelisse, L.E., Seewald, R.C., & Jamieson, D.G. (1994). Wide-dynamic-range compression hearing aids: The DSL[i/o] approach. *The Hearing Journal*, 47(10), 23-29.

Cornelisse L.E., Seewald R.C., & Jamieson D.G. (1995). The input/output (I/O) formula: A theoretical approach to the fitting of personal amplification devices. *Journal of the Acoustical Society of America*, 97(3), 1854-1864.

Cox, R.M. & Moore, J.N. (1988). Composite speech spectrum for hearing aid gain prescriptions. *Journal of Speech and Hearing Research*, 31:102-107.

- Dunn, H. K., & White, D. S. (1940). Statistical measurements on conversational speech. *Journal of the Acoustical Society of America*, 11:278-288.
- Feigin, J.A., Kopun, J.G., Stelmachowicz, P.G. & Gorga, M.P. (1989). Probe-tube microphone measures of ear-canal sound pressure levels in infants and children. *Ear and Hearing*, 10(4), 254-258.
- Fikret-Pasa, S. & Revit, L.J. (1992). Three experiments to validate new probe-measurement practices in prescribing customized hearing aids. *Journal of Speech and Hearing Research*, 35:384-400.
- Hawkins, D.B., Cooper, W.A., & Thompson, D.J. (1990). Comparisons among SPLs in Real Ears, 2cc and 6cc couplers. *Journal of the American Academy of Audiology* 1:154-161.
- Hawkins, D.B., Walden, B.E., Montgomery, A.A., & Prosek, R.A., (1987). Description and validation of an LDL procedure designed to select SSPL90. *Ear and Hearing* ,8:162-169.
- Killion, M. C., & Monser, E. L. (1980). "Corfig: Coupler response for flat insertion gain." In G. A. Studebaker and I. Hochberg (Eds.),. *Acoustical Factors Affecting Hearing Aid Performance*. Baltimore: University Park Press.
- Lewis, D.E., Feigin, J.A., Karasek, A.E., & Stelmachowicz, P.G. (1991). Evaluation and assessment of FM systems. *Ear and Hearing*, 12:268-280.
- Martin, H.C., Munro, K.J., & Langer, D.H. (1997). Real-ear to coupler differences in children with grommets. *British Journal of Audiology* 31:63-69.
- Moodie, K.S. (1996). A practical approach to hearing aid selection. *BSA News* 18:6-8.
- Moodie, K.S., Seewald, R.C. & Sinclair, S.T. (1994). Procedure for predicting real-ear hearing aid performance in young children. *American Journal of Audiology*, 3(1), 23-31.
- Moore, Brian C.J. & Glasberg, Brian R. (2004). A revised model of loudness perception applied to cochlear hearing loss. *Hearing Research*, 188 (2004), 70-88.
- Mueller, H.G. & Bright, K.E. (1994). Selection and verification of maximum output. In Valente M (ed.): *Strategies for Selecting and Verifying Hearing Aid Fittings*. New York: Thieme Medical Publishers, 38-63.
- Pavlovic, C. V. (1991). Speech recognition and five articulation indexes *Hearing Instruments* 42(9):20-23.
- Pearsons, K.S., Bennett, R.L., & Fidell, S. (1977). *Speech levels in various noise environments, Project Report On Contract 68 01-2466*. Washington, DC: US Environmental Protection Agency.
- Platz, Rainer, (2004). *SNR Advantage, FM Advantage and FM Fitting, Proceedings from the 1st International FM Conference*. www.phonak.com.
- Scollie, S.D., Seewald, R.C., Cornelisse, L.E. & Jenstad, L.M. (1998). Validity and repeatability of level-independent HL to SPL transforms. *Ear and Hearing* 19(5):407-413.
- Scollie, S.D., Seewald, R.C., Moodie, K.S., Dekok, K. (2000). Preferred listening levels of children who use hearing aids: Comparison to prescriptive targets. *JAAA* 11:230-238.

- Scollie, S., Seewald, R., Cornelisse, L., Moodie, S. T., Bagatto, M., Larnagaray, D., Beulac, S. & Pumford, J. (2005). *The Desired Sensation Level Multistage Input/Output Algorithm*. Trends in Amplification, 9-4, 159-197.
- Seewald, R.C. (1994a). Current issues in hearing aid fitting. In J-P Gagne and N. Tye-Murray (Eds.), *Research in Audiological Rehabilitation: Current Trends and Future Directions*. *Journal of the Academy of Rehabilitative Audiology Monograph, XXVII*.
- Seewald, R.C. (1994b). Fitting children with the DSL method. *The Hearing Journal, 47(9)*, 10,48-51.
- Seewald, R.C., Moodie, K.S., Sinclair, S.T. & Cornelisse, L.E. (1996). Traditional and theoretical approaches to selecting amplification for infants and young children. In Bess, F.H., Gravel, J.S. & Tharpe, A.M. (eds.): *Amplification for Children with Auditory Deficits*, Nashville: Bill Wilkerson Center Press, 161-191.
- Seewald, R.C. & Ross, M. (1988). Amplification for young hearing-impaired children. In M.C. Pollack (Ed.), *Amplification for the Hearing-Impaired*, Third Edition (pp. 213-271). Orlando: Grune & Stratton.
- Seewald, R.C., Ross, M. & Spiro, M.K. (1985). Selecting amplification characteristics for young hearing-impaired children. *Ear and Hearing, 6(1)*, 48-53.
- Seewald, R.C., Ross, M. & Stelmachowicz, P.G. (1987). Selecting and verifying hearing aid performance characteristics for young children. *Journal of the Academy of Rehabilitative Audiology, 20*, 25-38.
- Seewald, R., Moodie, S. T., Scollie, S. & Bagatto, M. (2005). *The DSL method for pediatric hearing instrument fitting: Historical perspective and current issues*. Trends in Amplification, 9-4, 145-157.
- Seewald, R.C. & Moodie, K.S. (1992). Electroacoustic considerations. In M. Ross (ed), *FM Auditory Training Systems: Characteristics, Selection, & Use*. Timonium, MD: York Press.
- Sherbecoe, R.L & Studebaker, G.A. (2003) Audibility-Index predictions of normal-hearing and hearing-impaired listeners' performance on the Connected Speech Test. *Ear and Hearing, 24-1*, 71-88.
- Sinclair, S.T., Moodie, K.S. & Seewald, R.C. (1997). Pediatric Hearing Instrument Fitting: *Phonak Video Focus #2 Booklet*, Stofa, Switzerland: Phonak A/G.
- Sinclair, S.T. Seewald, R.C. & Cole, W.A. (1996). A new real-ear fitting system: Because no two ears are alike. *Hearing Review 3(6)*:46-48.
- Skinner, M. W. (1988). *Hearing Aid Evaluation*. Englewood Cliffs: Prentice-Hall Inc.
- Smith, D. A. 1990. Technological report: Acoustometry. *Hearing Instruments 41(11)*:40-44.
- Valente, M., Potts, L.G., Valente, M., Vass, W & Goebel, J. (1994). Intersubject Variability of real-ear sound pressure level: Conventional and insert earphones. *Journal of the American Academy of Audiology, 5*, 390-398.
- Valente, M: *Strategies for Selecting and Verifying Hearing Aid Fittings*. New York:Thieme Medical Publishers, Inc.,1994.
- Zelisko, D.L.C., Seewald, R.C. & Whiteside, S. (1992). Comparing three procedures for predicting the ear canal SPL at LDL. *ASHA, 34(10)*.

25 Appendix 1: Manufacturer Disclosure Statement for Medical Device Security

Manufacturer Disclosure Statement for Medical Device Security--MDS ²					
Device Category *	Manufacturer *		Document ID	Document Release Date	
Hearing Aid Analyzer	Etymonic Design Inc.				
Device Model	Software Revision	Software Release Date	Other		
Verifit / RM500SL	See Page 1 of Users Guide	See page 1 of Users Guide			
Manufacturer or Representative Contact Information	Name	Title	Department		
	Chris Stokes-Rees	Technical Services Representative			
	Company Name	Telephone Number	E-mail		
	Etymonic Design Inc.	800-265-2093	chris@audioscan.com		
MANAGEMENT OF ELECTRONIC PROTECTED HEALTH INFORMATION (PHI) As defined by HIPAA Security Rule, 45 CFR Part 164				Yes No	Note #
1. Can this device transmit or maintain <i>electronic Protected Health Information</i> (ePHI)?				N/A	
2. Types of ePHI data elements that can be maintained by the device:				Yes	1
a. Demographic (e.g., name, address, location, unique identification number)?				No	
b. Medical record (e.g., medical record #, account #, test or treatment date, device identification number)?				No	
c. Diagnostic/therapeutic (e.g., photo/radiograph, test results, or physiological data with identifying characteristics)?				No	
d. Open, unstructured text entered by device user/operator?				Yes	1
3. Maintaining ePHI: <i>Can the device</i>					
a. Maintain ePHI temporarily in volatile memory (i.e., until cleared on by power-off or reset)?				Yes	2
b. Store ePHI persistently on local media?				No	
c. Import/export ePHI with other systems?				Yes	3
4. Mechanisms used for the transmitting, importing/exporting of ePHI: <i>Can the device</i>					
a. Display ePHI (e.g., video display)?				Yes	
b. Generate hardcopy reports or images containing ePHI?				Yes	1
c. Retrieve ePHI from or record ePHI to removable media (e.g., disk, DVD, CD-ROM, tape, CF/SD card, memory stick)?				Yes	1
d. Transmit/receive or import/export ePHI via dedicated cable connection (e.g., IEEE 1073, serial port, USB, FireWire)?				No	
e. Transmit/receive ePHI via a network connection (e.g., LAN, WAN, VPN, intranet, Internet)?				Yes	
f. Transmit/receive ePHI via an integrated wireless connection (e.g., WiFi, Bluetooth, infrared)?				No	
g. Other					
ADMINISTRATIVE SAFEGUARDS				Yes No	Note #
5. Does manufacturer offer operator and technical support training or documentation on device security features?				N/A	
6. What underlying operating system(s) (including version number) are used by the device? Linux kernel 2.4.22				Yes	4
PHYSICAL SAFEGUARDS				Yes No	Note #
7. Are all device components maintaining ePHI (other than removable media) physically secure (i.e., cannot remove without tools)?				Yes	
8. Does the device have an integral data backup capability (i.e., backup onto removable media such as tape, disk)?				No	
9. Can the device boot from uncontrolled or removable media (i.e., a source other than an internal drive or memory component)?				Yes	5
TECHNICAL SAFEGUARDS				Yes No	Note #
10. Can software or hardware not authorized by the device manufacturer be installed on the device?				N/A	
11. Can the device be serviced remotely (i.e., maintenance activities performed by service person via network or remote connection)?				No	
a. Can the device restrict remote access to specific devices or network locations (e.g., specific IP addresses)?				No	
b. Can the device log provide an audit trail of remote-service activity?				No	
c. Can security patches or other software be installed remotely?				No	
12. Level of owner/operator service access to device operating system: <i>Can the device owner/operator?</i>					
a. Apply device manufacturer-validated security patches?				No	
b. Install or update antivirus software?				No	6
c. Update virus definitions on manufacturer-installed antivirus software?				No	6
d. Obtain administrative privileges (e.g., access operating system or application via local root or admin account)?				No	
13. Does the device support user/operator specific ID and password?				No	
14. Are access sessions terminated after a predetermined length of inactivity (e.g., auto logoff)?				No	
15. Events recorded in device audit log (e.g., user, date/time, action taken): <i>Can the audit log record</i>					
a. Login and logout by users/operators?				No	7
b. Viewing of ePHI?				No	7
c. Creation, modification or deletion of ePHI?				No	7
d. Import/export or transmittal/receipt of ePHI?				No	7
16. Does the device incorporate an emergency access ("break-glass") feature that logs each instance of use?				No	
17. Can the device maintain ePHI (e.g., by internal battery) during power service interruptions?				No	
18. Controls when exchanging ePHI with other devices:					
a. Transmitted only via a physically secure connection (e.g., dedicated cable)?				No	
b. Encrypted prior to transmission via a network or removable media?				No	8
c. Restricted to a fixed list of network addresses (i.e., host-based access control list)?				No	9
19. Does the device ensure the integrity of the ePHI data with implicit or explicit error detection/correction technology?				No	

Manufacturer Disclosure Statement for Medical Device Security--MDS²

RECOMMENDED SECURITY PRACTICES

If possible, do not create ePHI. ePHI can only be created if a QWERTY keyboard is connected to the device(s). Not providing a keyboard is an effective way of restricting the creation of ePHI.

In the event that ePHI *must* be produced, the device(s) are capable of outputting ePHI in several different forms as disclosed in this document. Any data containing ePHI should be de-identified before storing the data, whether it is to hard copy, network file, or USB. If de-identification is not possible, physical access to the device or hard copies of data should be restricted to authorized personnel and saving to USB sticks should be prohibited.

EXPLANATORY NOTES (from questions 1 – 19)

IMPORTANT: Refer to Instructions for the Manufacturers Disclosure Statement for medical Device Security for the proper interpretation of information provided in this form.

1. The device(s), if equipped with a QWERTY keyboard, allows the operator to enter free-form text when saving or printing data. This data can be stored and retrieved from a USB stick or network file.
2. If an operator enters a header for printing, that header will remain on the machine until power is cycled or it is overwritten. Clearing a session does not erase header information an operator may have entered for printing.
3. The device(s) can import and export data and headers that may include ePHI, but only in the event that PHI was entered by the operator. The device(s) can import/export via network connection or transfer via USB stick.
4. This document along with available phone support constitutes our support for HIPAA compliance.
5. The Verifit device boots from a removable CD. The RM500SL device boots from a USB memory stick. The devices can boot from removable media only if it contains software supplied by the manufacturer.
6. There is no virus protection software available for the device(s).
7. The device(s) do not use any password protection and there is no device audit log.
8. The device(s) do not encrypt data and cannot save to encrypted media such as an encrypted USB. Any PHI produced by the device(s) should be de-identified appropriately.
9. The device(s) can connect to any networked computer to store and retrieve data, provided the user name and password is entered. Once the username and password is entered it is maintained by the device(s).

26 Appendix 2: AudioNote 2.1 VERIFIT and RM500SL Test Signals and Analysis

For additional details of the signals and their analysis, see the appended AudioNote.

Within the European Community:

EU Authorised representative:


PC Werth Ltd. Audiology House

45 Nightingale Lane, London SW12 8SP UK

Council Directive 93/42/EEC for Medical Devices.

CE 0088



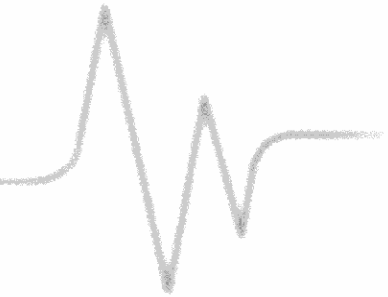

VERIFIT and RM500SL Test Signals and Analysis

Bill Cole BAsC, PEng

1 Introduction

The Verifit and RM500SL are full-duplex dual-channel audio measurement systems designed for the testing and fitting of all types of hearing instruments and many assistive listening devices. They provide an acoustic test chamber for the testing of devices coupled to standard 2 cm³ couplers as well as sound-field speaker(s) for real-ear measurement (REM) of hearing aid performance. The Verifit test chamber contains two loudspeakers for the functional testing of directional hearing instruments. Test signals are delivered to the device under test via the test chamber loudspeaker(s), the test chamber telephone magnetic-field simulator (TMFS), the test chamber magnetic field test loop (Verifit only) or the sound-field loudspeaker(s). Device output signals are measured in the 2 cm³ coupler via the coupler microphone or in a real ear via the probe microphone. Data from the 2 cm³ coupler may be viewed as coupler SPL, coupler gain or as estimated SPL in the ear canal (simulated real-ear measurements, S-REM). Data from the probe microphone may be viewed as ear canal SPL, as ear canal SPL re normal hearing threshold (i.e. dB HL), as insertion gain or as ear canal SPL in the context of an auditory area (Speechmap[®]).

Some useful facts about broad-band signals

- Ø **Overall SPL** is the SPL in a band containing all significant frequency components of the signal.
- Ø **Spectrum level** is the SPL in a band 1 Hz wide.
- Ø **Band SPL** is the SPL in a restricted frequency range. If the signal is uniform in the band, band SPL = Spectrum level + 10*log(bandwidth).
- Ø A **spectrum** is the band SPL, or spectrum level, in a series of adjacent bands.
- Ø For a broad-band signal, the overall SPL is greater than the band SPL and the band SPL is greater than the spectrum level. For a pure tone, the overall SPL, the band SPL and the spectrum level are the same.
- Ø A **white noise** signal has a spectrum level that is independent of frequency (i.e. constant SPL per Hertz).
- Ø A **pink noise** signal has a spectrum level that is inversely proportional to frequency (i.e. constant SPL per octave).
- Ø A **fractional octave** band (e.g. 1/3 octave) has a bandwidth that is proportional to frequency.
- Ø A pink noise signal has a flat spectrum when analyzed in fractional-octave bands.
- Ø A **critical band** is a band within which the loudness of a continuously-distributed signal of constant SPL is independent of bandwidth.

2 The test signals

Four types of test signals are currently available. These are tonal signals, pseudo-random noise, digitized real speech signals and the ICRA distorted speech signal. Test signals are generated in real time by the digital signal processor (DSP) or played from 16 bit binary audio files stored in the on-board flash memory. In the Verifit, these audio files are up-loaded from the internal compact disc (CD) drive at power up. In order to provide a repeatable acoustic signal to the device under test, a reference microphone is used in conjunction with a digital control loop to maintain the desired band level at each frequency. Live audio (speech, music etc.) may also be used as a test signal but it is not controlled for spectrum or level.

2.1 Narrow-band signals

2.1.1 Pure tones

Pure-tones are used in the ANSI hearing aid tests, in Multicurve 2 cm³ frequency response tests (gain or output), in 2 cm³ and real-ear single frequency (manual) tests, in real-ear insertion gain tests and in tone-burst maximum output (MPO) tests. Frequencies used are 1/12 octave except for the MPO test which uses 1/3 octave frequencies. Levels from 40 to 90 dB SPL are available at the reference microphone. Control of the level at each frequency is maintained by measuring the frequency response of the signal path to the reference microphone 256 ms before each test using a 256 ms, 50 – 55 dB SPL, multi-tone complex or a 256 ms chirp, depending on the test to follow. Drive levels at each frequency are then set to achieve the desired band levels at the reference

microphone.

2.2 Broad-band signals

2.2.1 Pink Noise signal

The pink noise signal is available for Multicurve 2 cm³ coupler gain and output frequency response measurements and for Insertion gain and Speechmap real-ear measurements. It is a pseudo-random signal composed of 1024 simultaneous tones summed to provide a crest factor of 12 dB. The spectrum of the pink noise signal is controlled by the reference microphone in conjunction with a digital feedback loop with a frequency resolution of 1/12 octave and a response time of about 1/3 of a second. Pink noise was selected as a test signal because it has equal energy per octave, producing a flat spectrum when analyzed in 1/12 or 1/3 octave bands. Figure 1 shows the 1/3 octave spectrum of the pink noise signal and the noise signal specified in ANSI S3.42-1997. Note that the ANSI S3.42-1997 spectrum represents speech peaks not the long-term average.

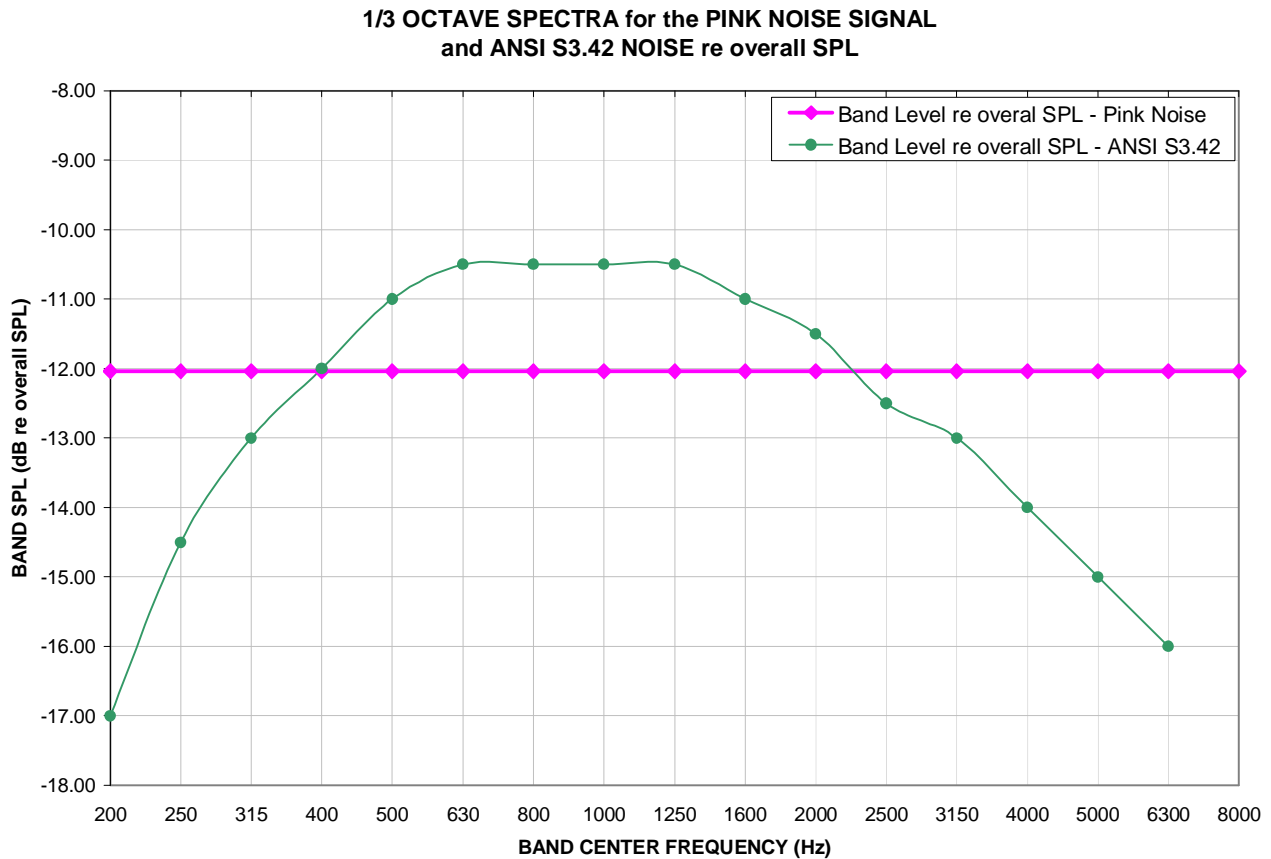


Figure 1: 1/3 octave spectra for the pink noise signal and the noise signal specified in ANSI S3.42-1997.

2.2.2 Dual-direction pink noise signal (Verifit only)

This unique test signal is available in the dual-source Verifit test chamber and for real-ear measurements by adding an additional sound-field speaker. It is similar to the pink noise signal described previously except that the component tones are presented simultaneously from two speakers, half from each. The level of each tone is independently controlled at the reference microphone. Two frequency response curves are generated simultaneously -one for each speaker. This provides a real-time measure of the functioning of the directional features of hearing aids that is independent of compression or noise reduction algorithms. Measurement methods which sequentially measure response from different directions work only with these features disabled.

2.2.3 Real-speech signals

Real-speech signals are provided in Speechmap for both REM and S-REM measurement modes. Four different

digitized speech passages and the single-talker ICRA distorted speech are provided. Each has a duration of 10 – 15 seconds and may be presented as a single passage or in a continuous loop. In order to provide a repeatable speech signal to the device under test, the signal path must be equalized prior to the presentation of the speech signal. This is accomplished by presenting a 896 ms pink noise burst at the selected speech level, 256 ms prior to the start of each speech passage and adjusting a digital filter to provide a flat response at the reference microphone.

The two “Speech-std” signals are by the same male talker, filtered to provide the long-term average speech spectrum (LTASS) recommended by Cox & Moore (1988) for average vocal effort. This is the LTASS assumed in the Desired Sensation Level (DSL) method of hearing aid fitting. The “female” and “child” signals have been chosen to provide a range of spectra and are presented “as recorded” without any processing. Overall SPLs of 50, 55, 60, 65, 70 and 75 dB (at the reference microphone) are available. Soft and Average levels (50 - 70 dB SPL) have the same spectrum. A Loud vocal effort filter is applied to the 75 dB level signals. This is shown in Figure 2.

The ICRA distorted speech signal is a recording of an English-speaking talker that has been digitally modified to make the speech largely unintelligible. The resultant signal has many of the properties of real speech but has a harsh sound and lacks harmonic structure. The latter may be significant for hearing aids which use this property of speech to control noise reduction schemes. The LTASS is similar to the “Speech-std” signals up to 5 kHz.

The test systems provide a simulated real-ear mode (S-REM) in which measurements made in the 2 cc coupler in the test chamber are converted to estimated real ear SPL. In this mode, the effects of hearing aid microphone location need to be included in the test signals so spectra are further altered to include the microphone location effects shown in Figure 3.

LOUD VOCAL EFFORT EFFECT re nominal band SPL

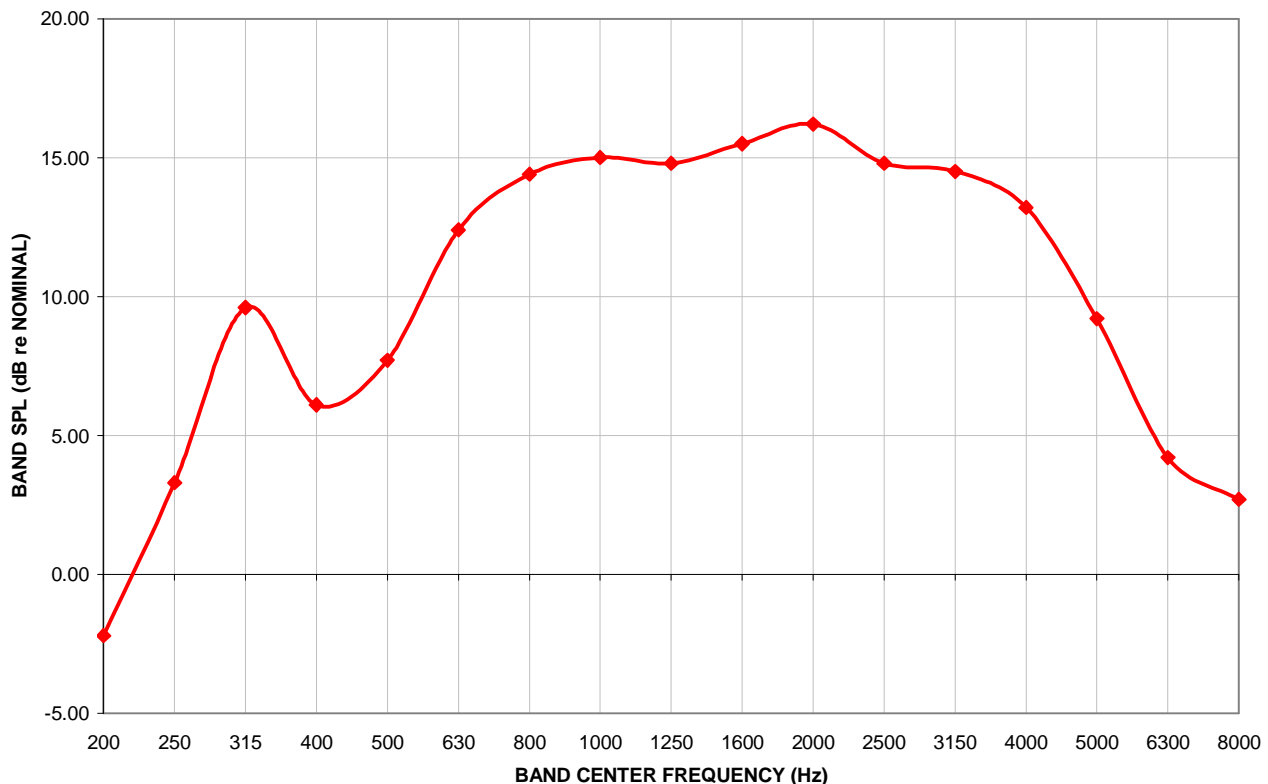


Figure 2: Loud vocal effort effect in dB re nominal band levels. Nominal band levels are band levels for an overall SPL of 65 dB. This shaping is applied to the 75 dB SPL speech and ICRA signals.

In addition to these well-controlled and repeatable signals, live speech may be used as a test signal. It will, of course, be unequalized and at an uncontrolled level.

MICROPHONE LOCATION EFFECTS re nominal band SPL

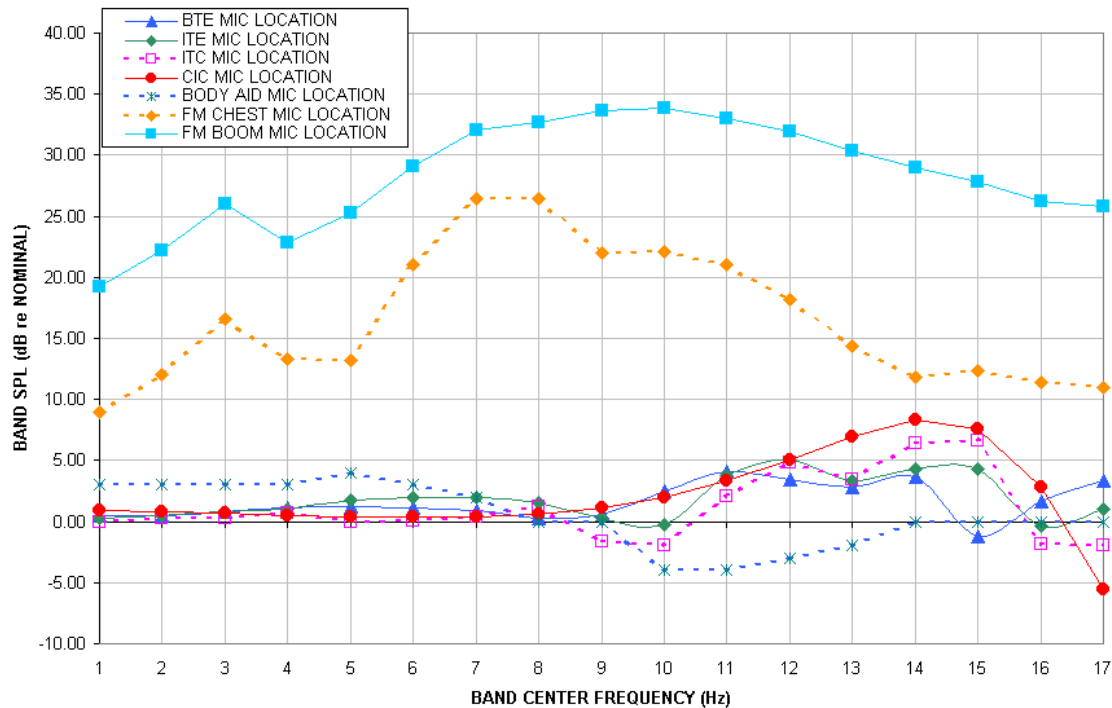


Figure 3: Microphone location effects in dB re nominal band SPL. Nominal band levels are band levels for an overall rms level of 65 dB SPL and average vocal effort in a free sound field.

3 Analysis of broad-band signals

Broad-band signals contain energy at many frequencies simultaneously. Such signals are usually analyzed in a series of narrow frequency bands to produce a spectrum. The auditory system functions on a logarithmic frequency scale and analyzes broad-band signals in critical bands which approximate 1/3 octave bands (Figure 4). Using 1/3 octave bands for analysis of broad-band signals allows measured levels to be compared more readily to narrow-band behavioral measures, such as threshold. For this reason, analysis of broad-band signals in all Real-Ear Measurements (including S-REM in Speechmap) is in 1/3 octave bands. Other analyzers use narrower analysis bands, sometimes having constant bandwidth. As shown in Figure 4, this can result in significantly underestimating the audibility (or comfort or discomfort) of a complex signal.

It should be noted that, in all Real-Ear Measurements, 1/3 octave band SPL is displayed (and reported when saving data to a file) at 1/12 octave frequencies which provides curve smoothing. When calculating overall rms from these data, the SPL at 225 Hz and subsequent 1/3 octave increments should be used.

Analysis of broad-band signals in all Hearing Instrument Tests is in 1/12 octave bands and the band SPL is displayed at 1/12 octave frequencies.

3.1 Pink noise signal analysis

All band levels are calculated with an averaging time of 128 ms.

For a linear hearing aid, a pink noise stimulus will result in a response curve that has the same shape as one obtained using a swept pure tone. However, it must be remembered that, while a swept tone has the same band SPL as overall SPL, the band SPL for a noise signal is significantly lower than the overall SPL. Consequently, for a linear aid, output curves obtained using 1/12 octave analysis will be about 18 dB lower than the output curves obtained using a swept tone at the same overall SPL. As long as the hearing aid is linear, the gain obtained will be the same for both signals. Figure 5 shows output (A) and gain (B) for a linear hearing aid, obtained using a swept tone (1) and pink noise (2) with a 60 dB overall SPL.

**Analysis Bandwidth Used by Various Analyzers
To Analyze Complex Signals**

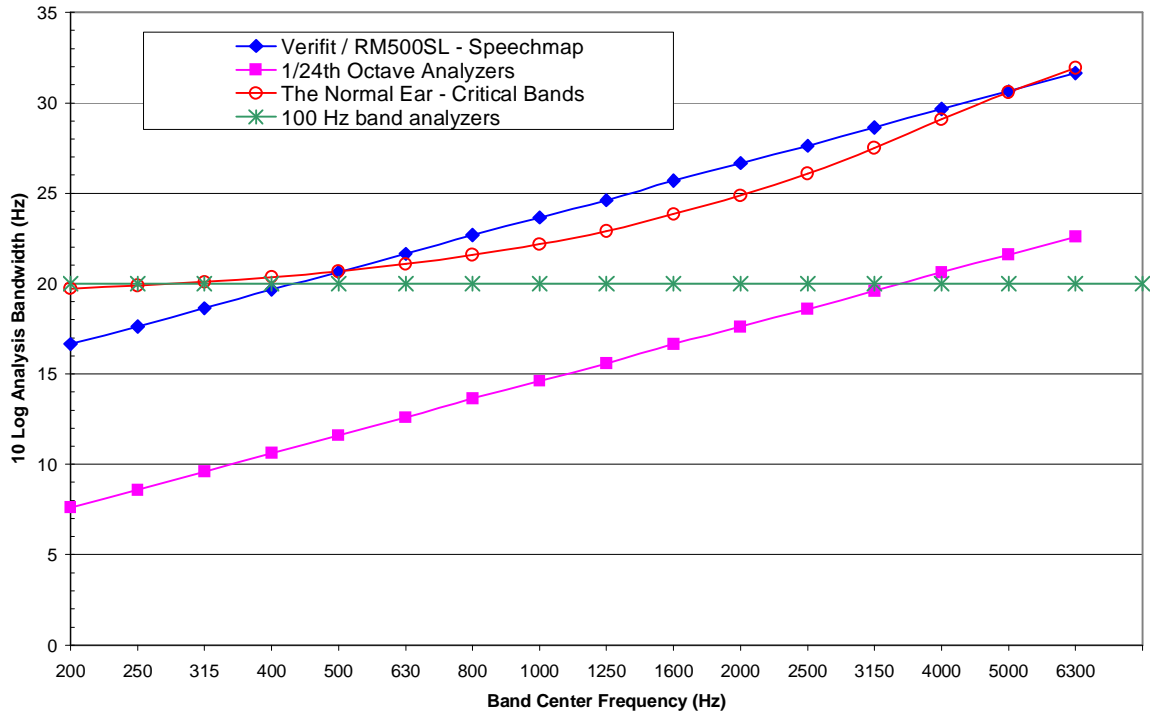


Figure 4: Analysis bandwidths of some real-ear analyzers and critical bands for a normal ear. If the analysis bandwidth is less than a critical band, aided response curves shown to be at threshold (or UCL) will actually be well above it.

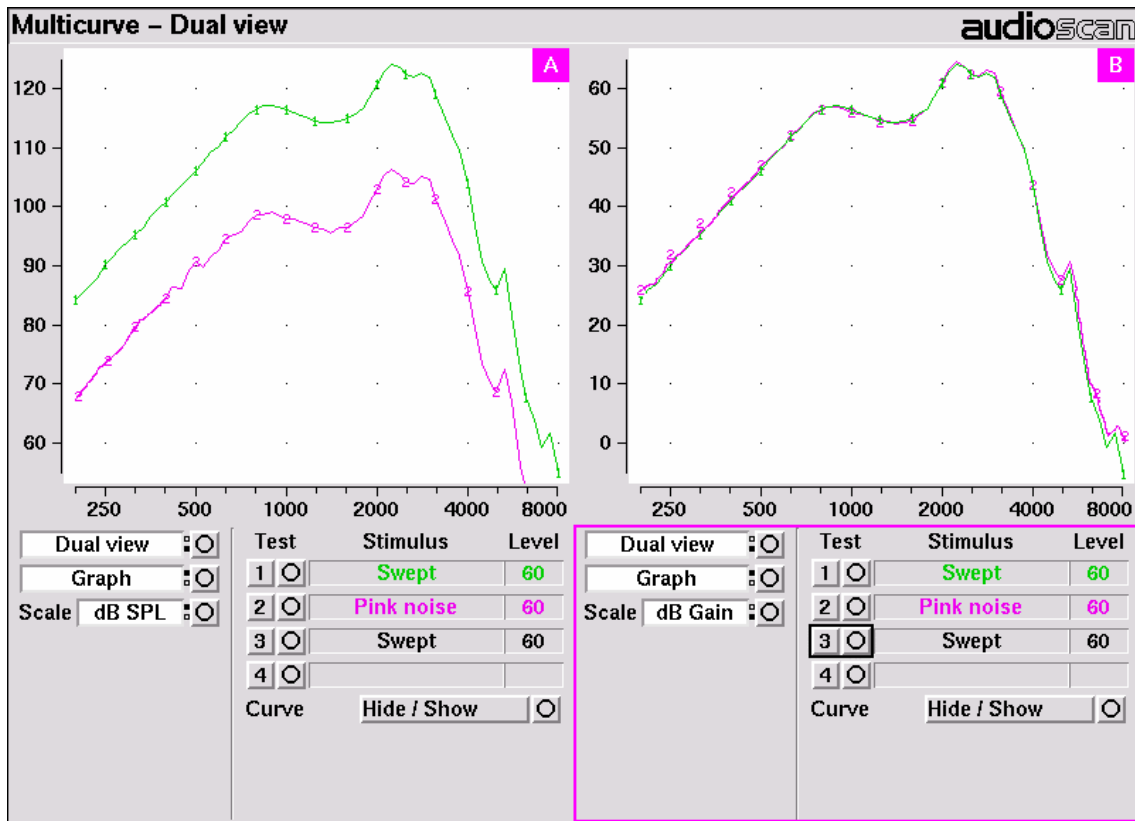


Figure 5: Output (left) and Gain (right) curves for a linear hearing aid, generated using a swept tone (1) and pink noise analyzed in 1/12 octave bands (2). (Dual view not available in RM500SL)

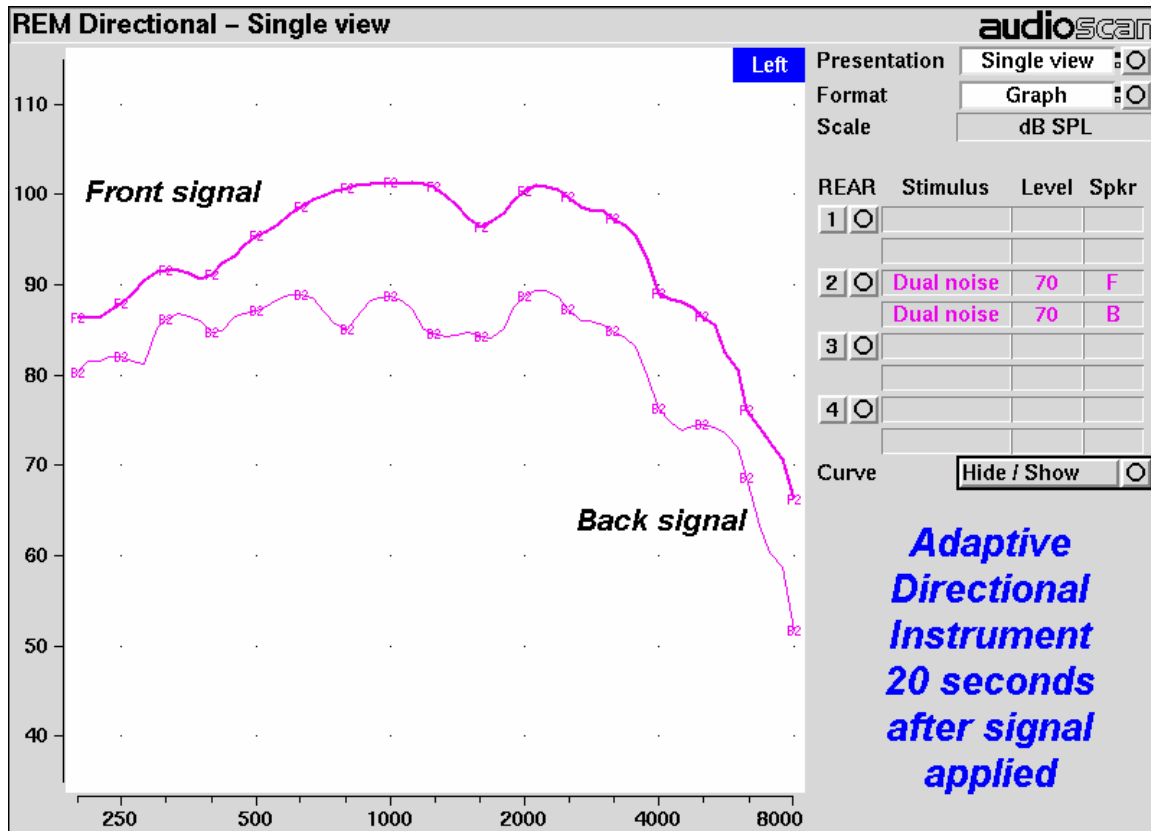


Figure 6: A real-ear directional test on an adaptive instrument with active compression and noise reduction. (Directional test not available in RM500SL)

3.2 Real-speech signal analysis

One of the most-used measures of a speech signal is the long-term average speech spectrum (LTASS). This is a 1/3 octave spectrum averaged over a sufficiently long portion of the speech material to provide a stable curve. In practice a 10 second average meets this requirement and, for this reason, all Speechmap passages are at least 10 seconds long.

The dynamic nature of speech is often characterized by the distribution of short-term levels in each 1/3 octave band. These levels are determined by calculating a spectrum for each of a series of short time periods within the passage. Historically, time periods of 120, 125 or 128 ms have been used. The Verifit and RM500SL use a 128 ms time period, resulting in 100 levels (or samples) in each 1/3 octave band for a 12.8 second passage. The level in each band that is exceeded by 1% of the samples (called L1, the 1st or 99th percentile) has historically been referred to as the speech peak for that band. The curve for these 1% levels is approximately 12 dB above the LTASS. The level in each band that is exceeded by 70% of the samples (called L70, the 70th or 30th percentile) has historically been called the valley of speech for that band. The curve for these 70% levels is approximately 18 dB below the LTASS. The region between these two curves is often called the speech region, speech envelope or speech “banana”. The speech envelope, when derived in this way, has significance in terms of both speech detection and speech understanding. Generally, speech will be detectable if the 1 % level is at or near threshold. The Speech Intelligibility Index (SII) is maximized when the entire speech envelope (idealized as a 30 dB range) is above (masked) threshold. This will not be an SII of 100% (or 1) because of loudness distortion factors, but higher SII values will not produce significantly higher scores on most test material. The speech-reception threshold (SRT) is attained when the LTASS is at threshold (approximately - depending on test material and the individual). These scenarios are shown in Figures 7 - 9 which follow.

It should be noted that analysis methods which use shorter time periods produce higher peak levels and significantly different speech envelopes. In order to produce results that can be directly compared to measures of threshold (and UCL), the analysis time period needs to approximate the integration time of the ear. Although this varies with frequency and individuals, a value between 100 - 200 ms is likely. The Verifit and RM500SL use a

128 ms analysis time period as an approximation because it also has considerable historic support.

Because the spectrum of a speech signal varies with time, it is necessary to average measurements over several seconds to obtain a stable, repeatable result. However, the effects of adjustments to hearing aid parameters need to be seen quickly in order to be useful for optimization. To resolve these conflicting needs, speech signals are first presented in a repeating loop with a sound-field equalization test prior to the start of each passage. A running calculation of the short-term speech envelope and average is performed on 2 - 3 seconds worth of data and is displayed in real time so that the effects of adjustments to hearing aid parameters are easily viewed. Pressing the Continue key causes the passage to restart and run in its entirety. The envelope and average are calculated over the full passage to provide stable and repeatable data.

When using live speech as a test signal, a "Freeze curve" function is available to capture the short-term spectra for examination and counseling purposes.

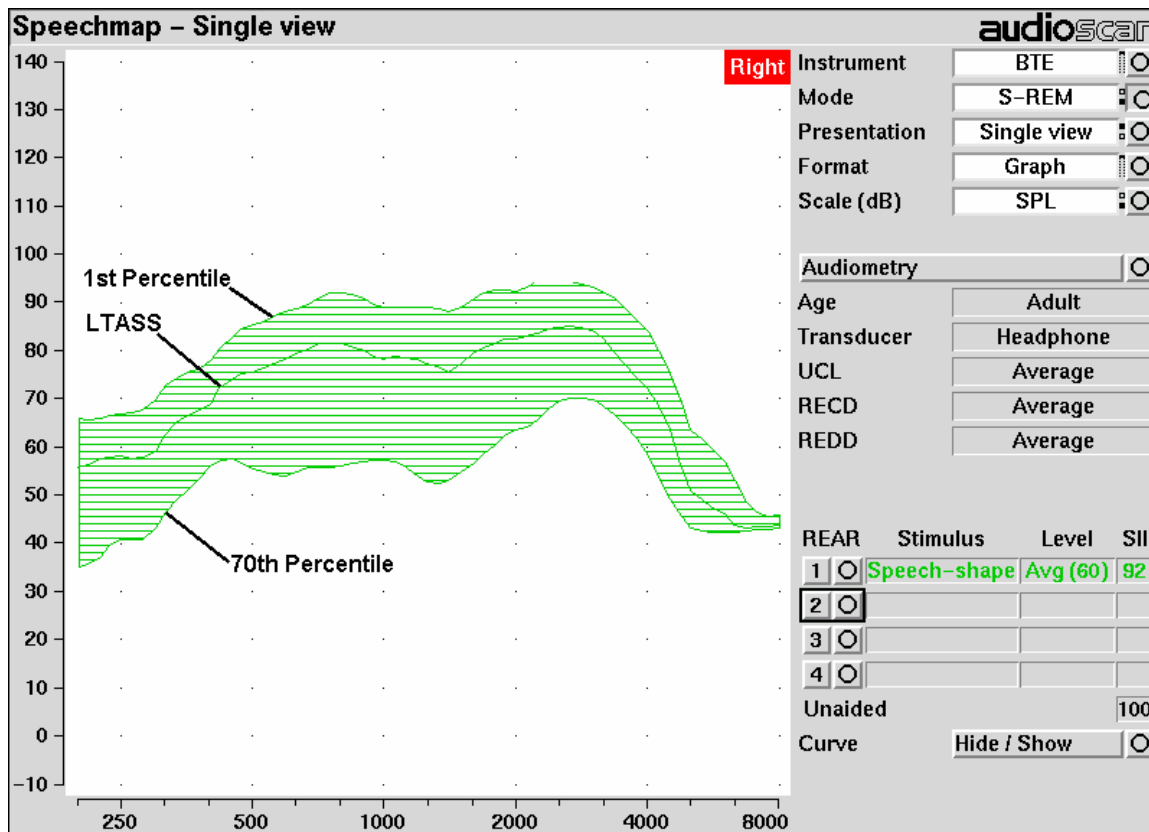


Figure 7: Example of speech envelope and LTASS

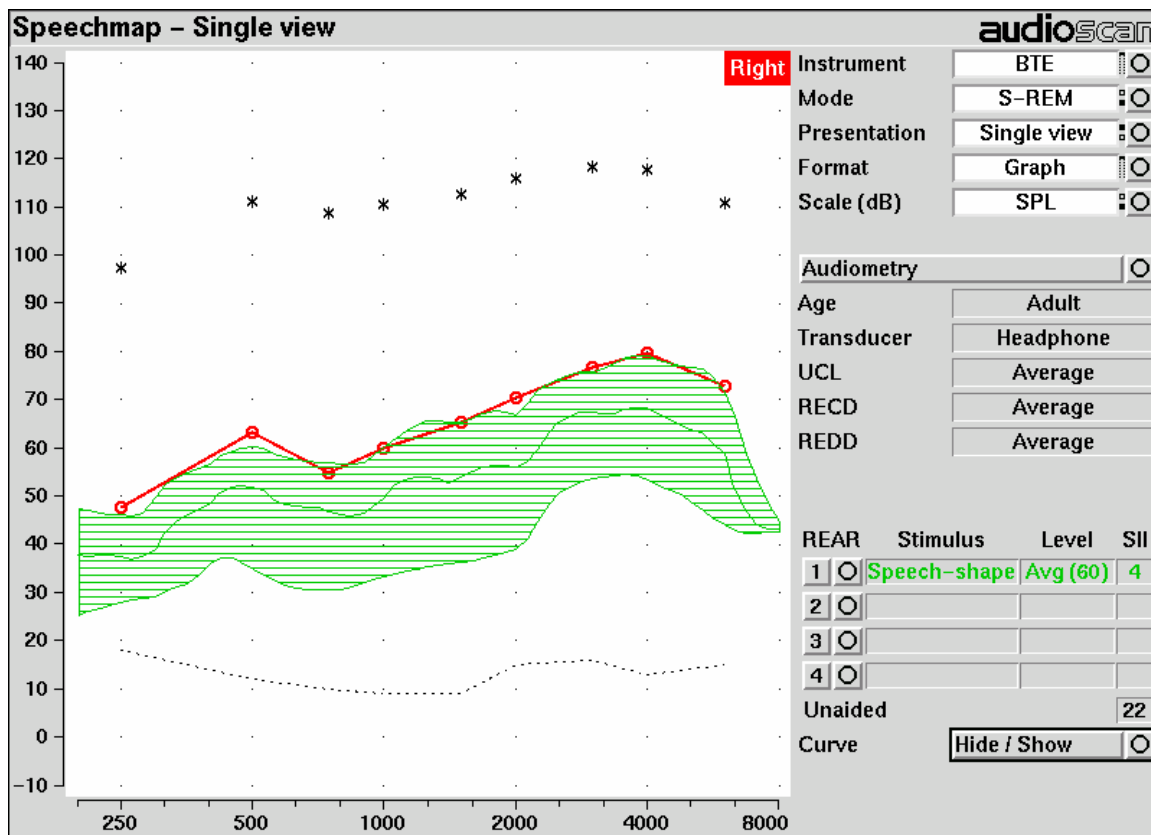


Figure 8: Speech is detectable but not understandable if the 1st percentile is at threshold

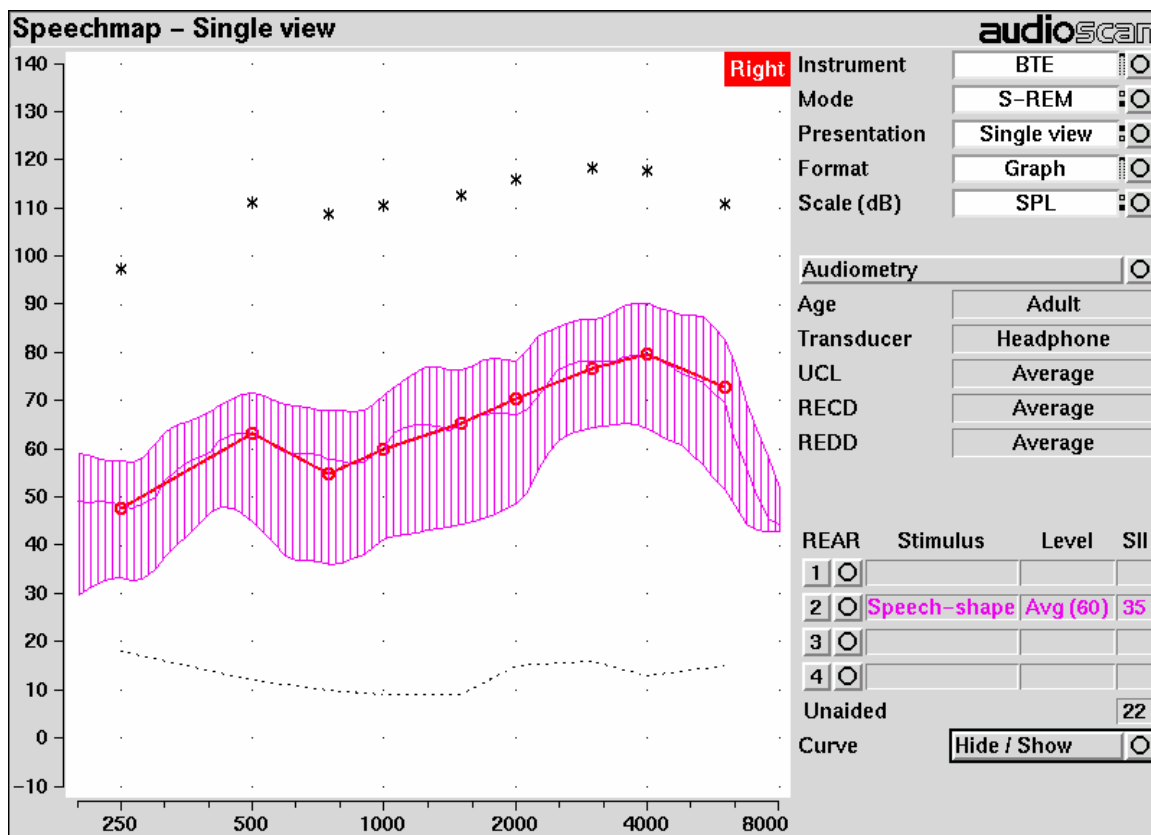


Figure 9: When LTASS is at threshold, SII \approx 33% which corresponds approximately to SRT

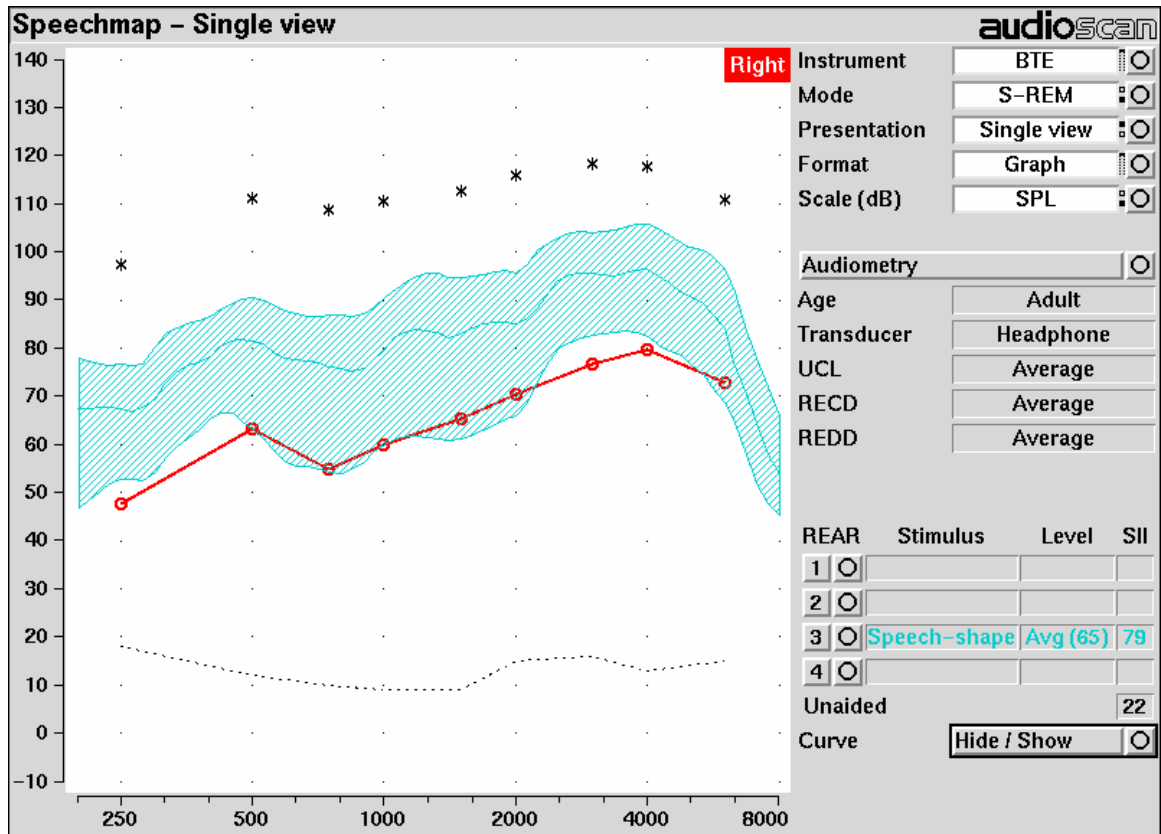


Figure 10: The highest SII is obtained when the entire speech envelope is above threshold

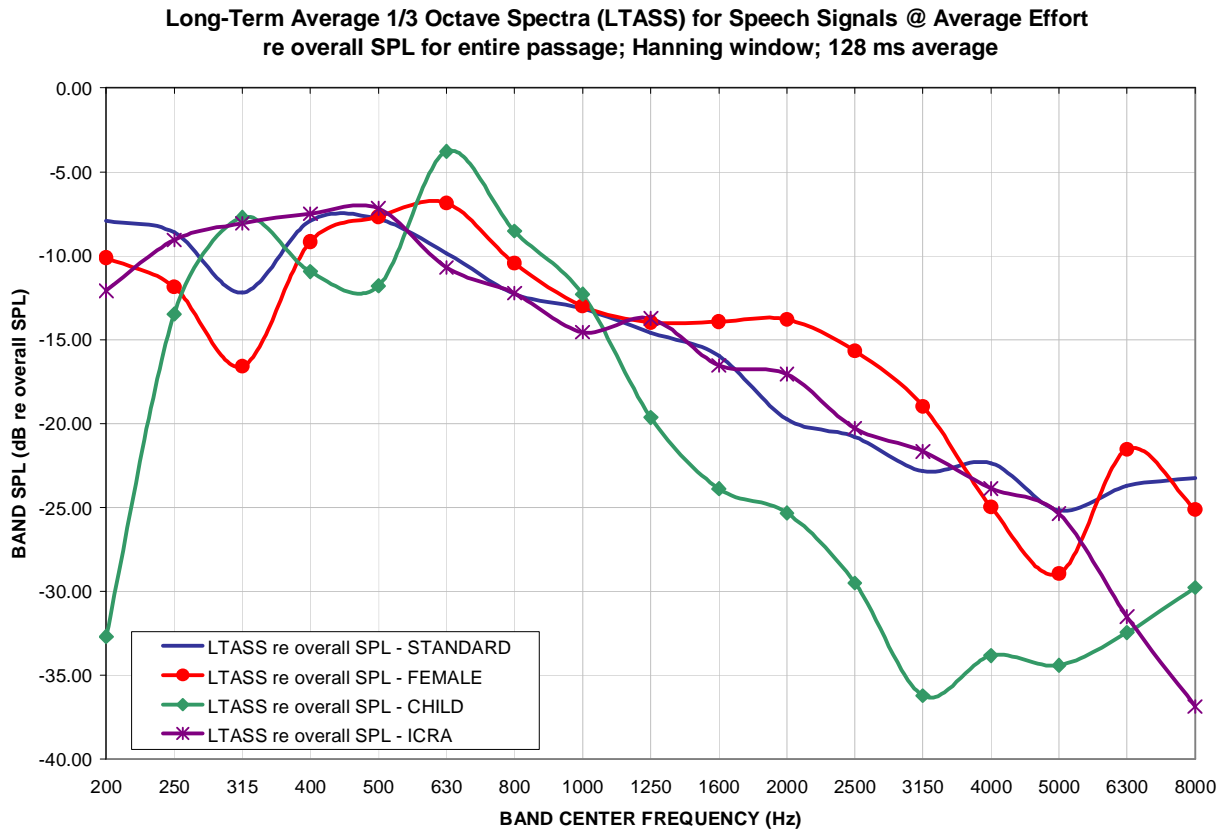


Figure 11: LTASS for Speechmap speech signals at average vocal effort

1st Percentile of 1/3 Octave Band SPLs for Speech Signals @ Average Effort]
re LTASS for entire passage; Hanning window; 128 ms average

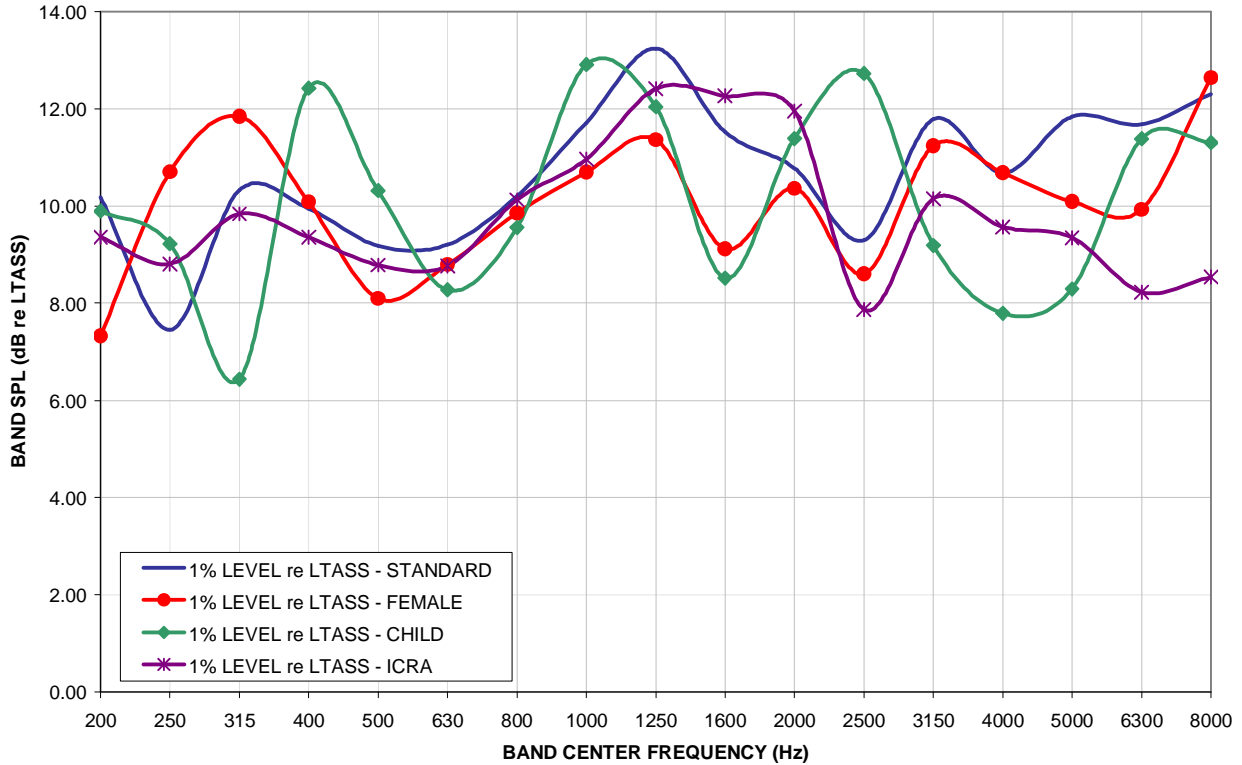


Figure 12: Level of first percentile for Speechmap speech signals at average vocal effort

70th Percentile of 1/3 Octave Band SPLs for Speech Signals @ Average Effort]
re LTASS for entire passage; Hanning window; 128 ms average

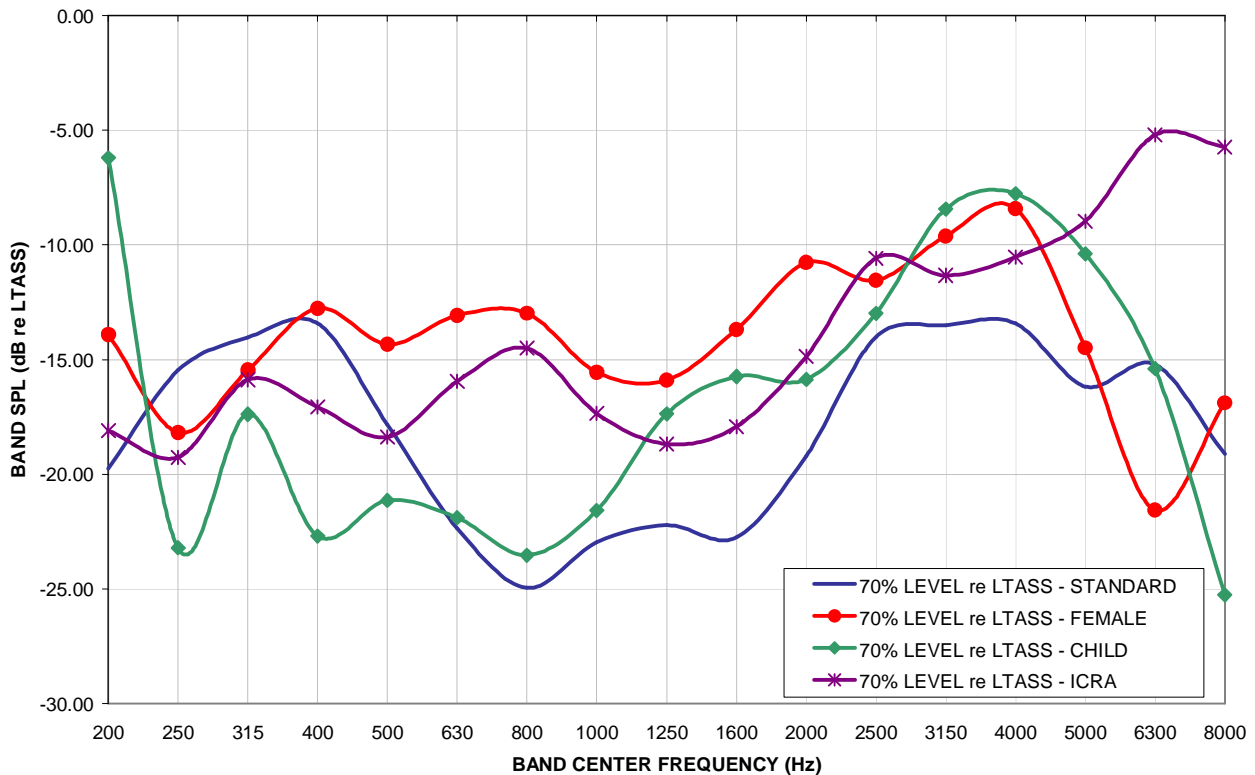


Figure 13: Level of 70th percentile for Speechmap speech signals at average vocal effort

**Dynamic Range for Speech Signals @ Average Effort]
re LTASS for entire passage; Hanning window; 128 ms average**

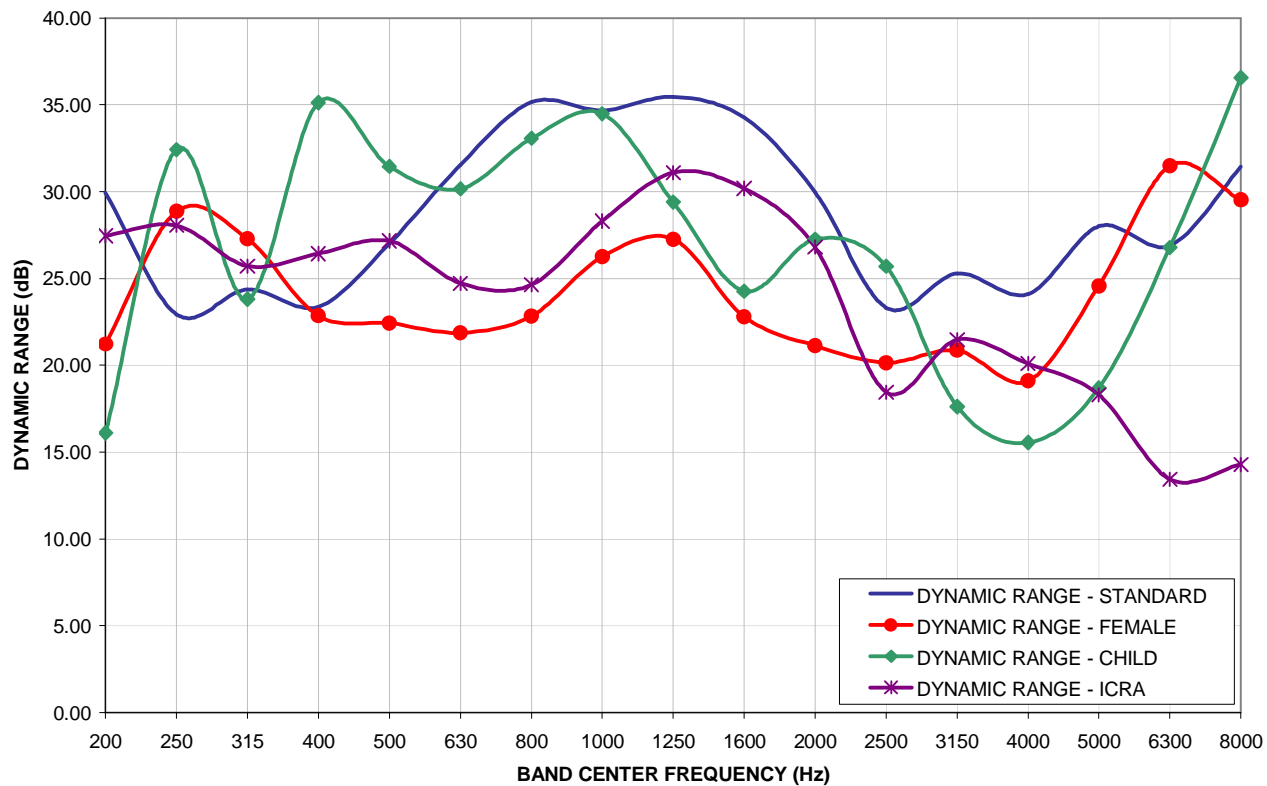


Figure 14: Dynamic range of Verifit speech signals at average vocal effort